B-Max Lampada polimerizzante senza fili
B-Max Wireless Curing light

Lampada polimerizzante Mod. B-Max

B-Max Curing Light

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B-Max is a latest generation **curing light** with solutions and features that are unique to the market.

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High power adjustable up to 1600 mW/cm², for rapid and safe curing.

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Besides looking good, it has been designed for easy ergonomic use.

Vari programmi / Various curing programs

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Free selection from 6 programs with different powers, suitable for your every need.

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The battery is a separate unit and can be removed easily. When necessary, it can be easily changed by the operator without technical assistance.

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Multi-function graphic display, with warning messages and treatment data information.

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The long life battery allows many curing operations to be carried out.

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B-Max has been designed and tested for use on all dental compounds.

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B-Max has a simple-to-read information menu, with a rich selection of messages, and interacts with the user through graphs, instructions and written messages. An exclusive and unique solution.

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B-Max is a led high tech curing light, light, easy to handle, wireless.

B-Max exploits the new led technology, this ensures a focused, cold and with low infrared outlet high light emission with 450/480 nm spectrum.

B-Max ensures a light emission with 1600mW power.

B-Max includes a wireless handpiece with a wide colour display, long lasting lithium battery, easy to move. Control panel for time setting, program setting, start of cycle.

B-Max has 6 programs that can be selected with different power, **B-Max** is supplied with shaped support with built-in battery charger and led indicator.

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
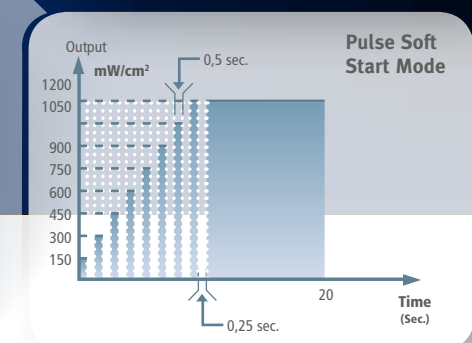
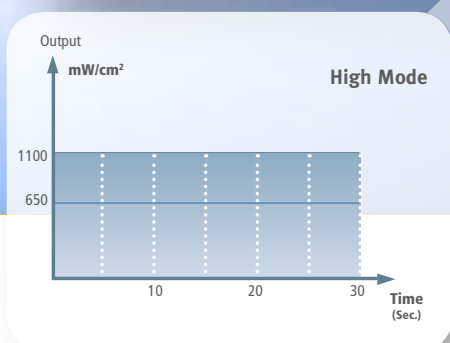
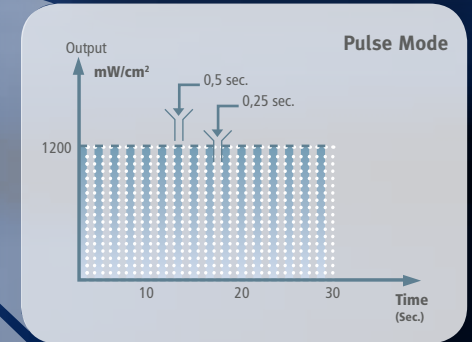
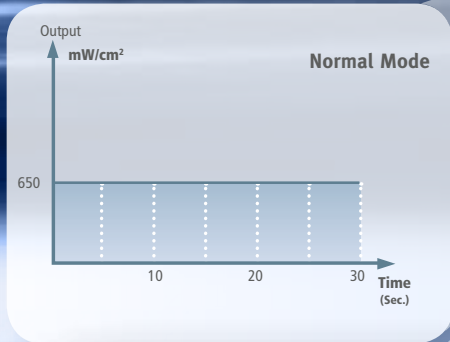
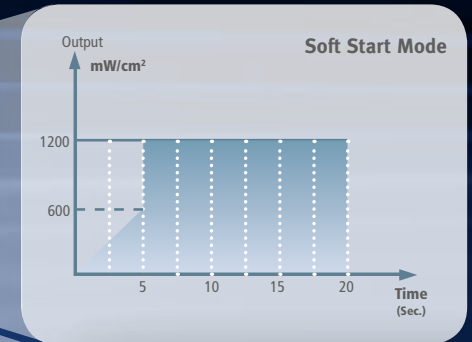
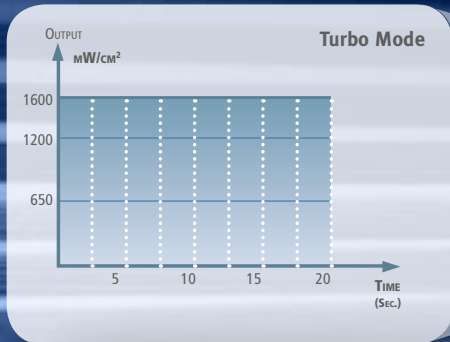
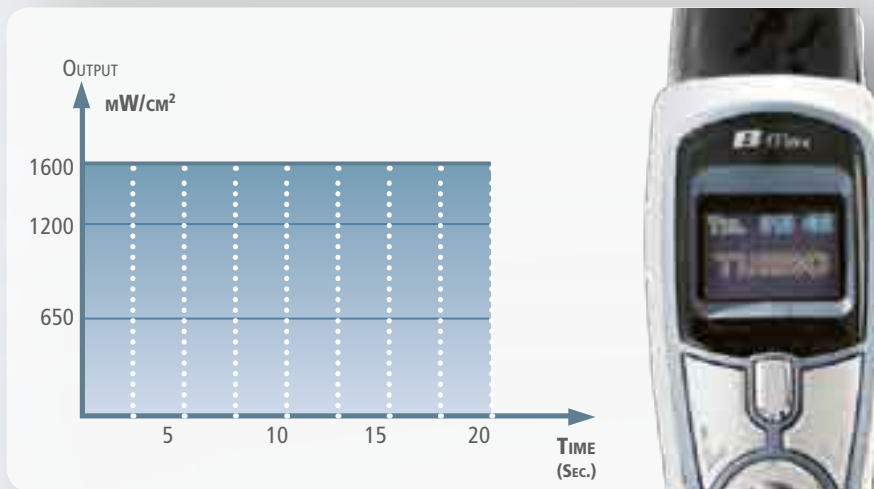
The selection of programs can be made through the buttons placed on the control panel. B-MAX has 6 programs that can be selected according to user's needing.

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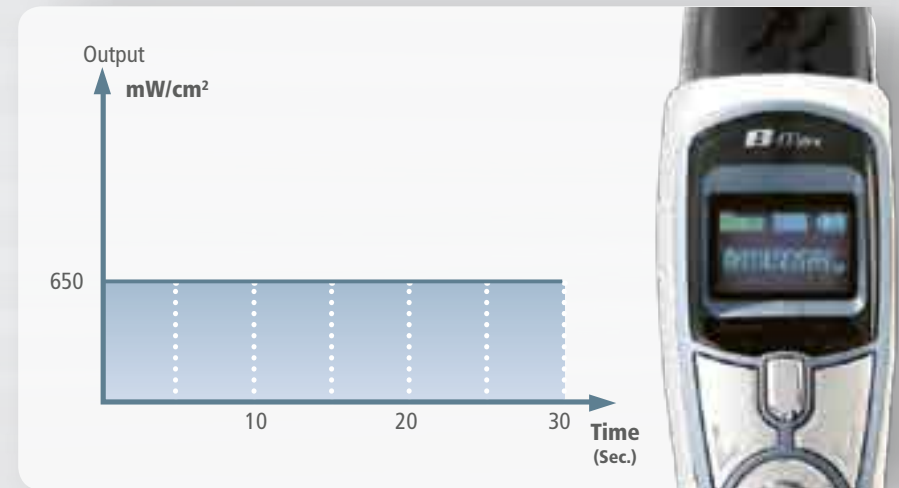PROGRAM TURBO: immediate output of max power: 1600 mW/cm², the length of the emission can be fixed to 5"/10"/20".

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

PROGRAM NORMAL: power output higher than 650 m W/cm², the length of the emission can be fixed to 10"/20"/30".

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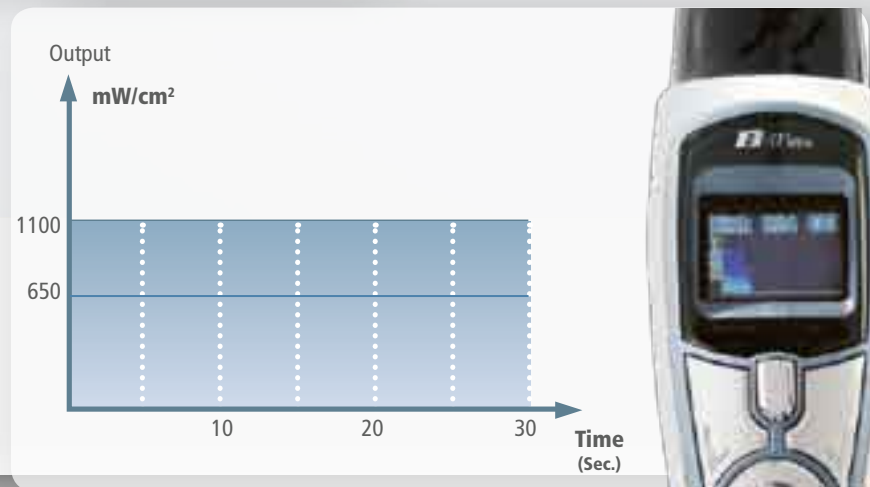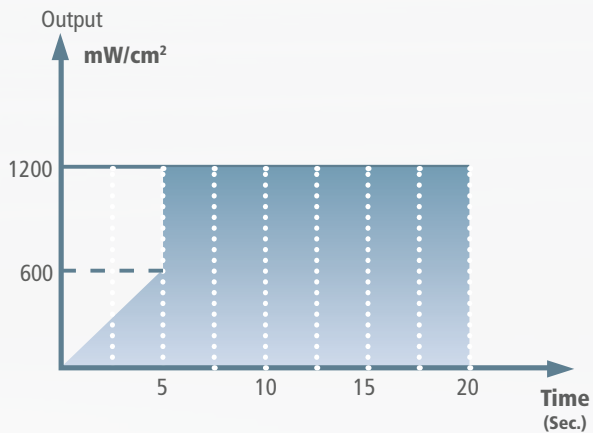
PROGRAM HIGH: power output higher than 1200 mW/cm², the length of the emission can be fixed to 10"/20"/30".

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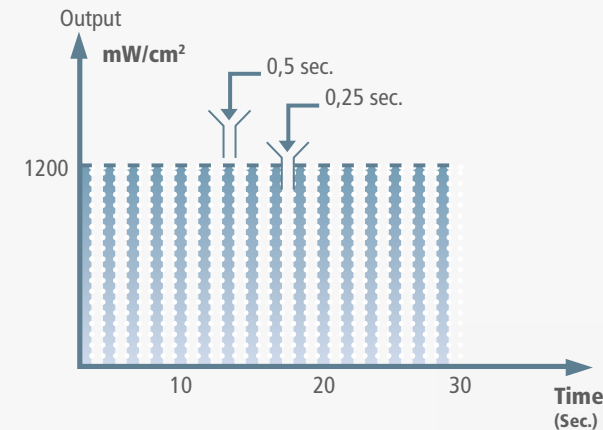PROGRAM SOFT/ START:
power output raises from 0 m W/cm2 to 650 m W/cm2 in 5" Pulse mode, then maintains output power 1200 mW/cm2 during the following 15", fixed time 20".

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
PROGRAM PULSE: power output reaches 1200 mW/cm2 in 10"/20"/30", output sequence is 0.25".

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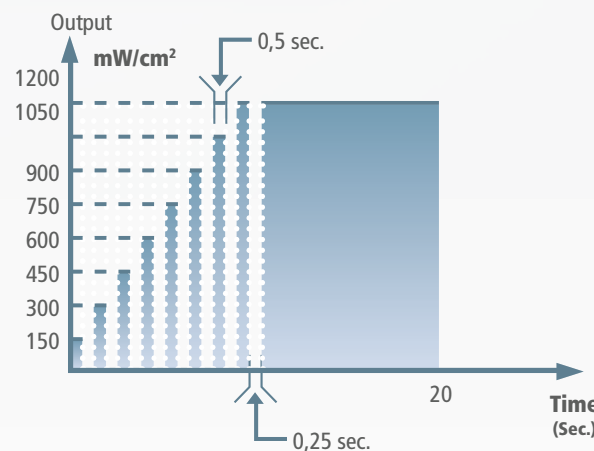PROGRAM SOFT/ START: power output raises from 0 m W/cm2 to 650 m W/cm2 in 5" Pulse mode, then maintains output power 1200 mW/cm2 during the following 15", fixed time 20".

Strada Cavalli, 4 • 43038 Sala Baganza (PR) Italy
Tel./Ph +39 0521 8380 • Fax +39 0521 833391
info@tecnogaz.com

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All rights reserved. Variations can be done without notice. Tecno-Gaz S.p.A. is to be considered not responsible for damages caused by the lack or the wrongness of the information here mentioned.