AREE OPERATIVE OPERATIONAL AREAS

SMART BOOK

- *Rischio biologico • Biological Risk*
- *Primo Soccorso • First Aid*
- *Sedazione cosciente • Conscious sedation*
- *Illuminazione chirurgica • Surgical lighting*
- *Varie studio • Various - cabinet*
- *Sbiancamento professionale • Professional whitening*

Tecno-Gaz Group

Quality&Know-How
Made in Italy

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Quality and reliability

Tecno-Gaz S.p.A. operates in a number of different fields with a strong tendency towards, and specialization in, the world of dentistry, focusing on the production of equipment and devices to improve the operational practices of professionals in the field through innovative solutions and products. **For more than 30 years** we have been setting ourselves the goal of finding more and more appropriate solutions to the demands of doctors and those who work with them, as well as creating products that guarantee safety and performance and can contribute towards improving **professional qualities**.

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Link immediati per Smart-Phone • Smart-Phone direct links

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Why the sterilization process

When evaluating risks within the medical environment, biological risk is considered the main factor to be analysed and brought under control. The environment and practices of medicine represent a risk for the transmission of pathogens, particularly those that are transmitted parenterally, if rigorous prevention methods to protect patients and operators are not applied. Within the medical environment, the transmission of agents of contamination can happen through:

- airborne pathways
- water contaminated by bio-film and back-flow of revolving instruments
- contaminated surgical instruments

The concept of prevention must be considered a very real cultural and structural project to be constructed with skill and care. **With Tecno-Gaz you can have the most complete, rational and safe plan for instrument sterilization;** no-one can count on solutions like ours, researched to make every activity safe and to guarantee maximum results. **Tecno-Gaz S.p.A.** can offer you solutions for fighting airborne infections and for the management and treatment of water, the latter being an issue of absolute importance for every medical facility.

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Tecno-Gaz is able to offer a complete plan for managing every aspect linked to prevention within your facility.

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MultiSteril Fast
pagina / page 8

Unika
pagina / page 12

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Sterilization test and archiving system

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Unique Advantages

- making traditional sterilization more effective
- saving you time and money
- providing a maximum legal guarantee

Biological Risk

How to prevent infections in medical field

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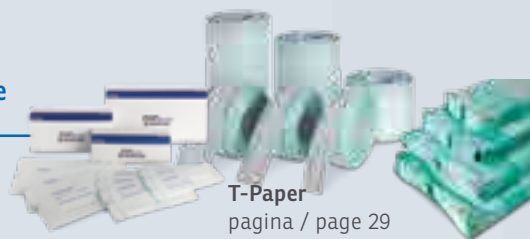
Frame and click! Use your mobile phone to get more information about this issue.

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EN

Clean is a new device created for instrument cleaning. The aim of this device is to help operators during the instrument cleansing phase, avoiding prolonged handling, thus making this phase simpler, more efficient and safer. **CLEAN is a simple device that includes the following:**

- a compartment for storing and placing the instruments to be treated.
- a central compartment complete with a coiled bristle to clean the instruments.

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EN

Artica vessels have been expressly studied to assure simple and controlled disinfection. All vessels are equipped with integrated timer support, total lid for easy liquid substitution and partial lid for the insertion of the instruments, avoiding the leakage of odours. Artica vessels are all supplied with plastic tray, removable on demand. **Artica line is composed by two models:**

- Artica 1 conceived for average loads, particularly suitable for surgeries.
- Artica 2 conceived for large loads, perfect for hospitals or big centres.

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EN

The ultrasonic tub is designed to **wash instruments used in the medical, dental, cosmetic, veterinary fields of application and all those areas which require an ultrasonic washing**. The cleaning device involving ultrasound (via cavitation) allows you to thoroughly clean places otherwise difficult to reach by manual cleaning, without any risk resulting from unsafe handling of contaminated instruments.

Heating

The device proceeds to the heating phase of the detergent. It stops automatically when the preset temperature is reached. Always consider that during the ultrasonic cleaning the temperature of the detergent tends to rise independently.

Degassing (DEGAS)

The device starts activating the ultrasonic phases intermittently. This process reduces the gas dissolved in the liquid, thus improving the efficiency of the subsequent washing.

Ultrasonic cleaning

Washing phase through ultrasonic cavitation.

SWEEP

Ultrasounds are modulated in frequency, thus reducing the negative effects of standing waves, increasing the distribution of ultrasonic energy within the washing liquid. The cleaning is more efficient (**improved cavitation**).

Discharge

The tub is emptied by opening the appropriate valve.

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EN

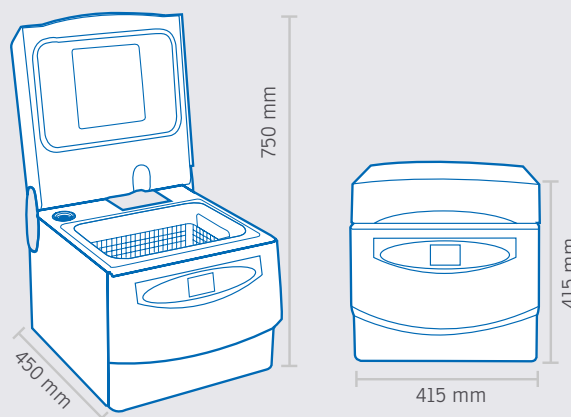
Tecno-Gaz S.p.A. has been able to create **a new and revolutionary device that can automatically carry out all the major preparation phases for the sterilization cycle, such as disinfection, cleansing, rinsing, drying, without human intervention, without any professional risk and with a reduction in the overall time taken.** A device that is unique in the world and indispensable for every facility that carries out sterilization.

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The aspects that Multisteril Fast can help you manage correctly

Assistants' expenditure of time

Assistants spend a lot of time for the preparation phase to tools sterilization, because of multiple and complex manual procedures. **Multisteril Fast does not require human presence.**

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Risk factor

The manipulation of tools and potential infectious materials is a very important factor and it must be kept under strict control, both for operators' protection and the responsibility of the owner of the studio. **Multisteril Fast cancels the risk.**

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Management protocol

There is the sterilization if the procedure of preparation of materials is done properly, if not, the whole process is completely nullified. **Multisteril Fast performs correctly all the cycles.**

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Certified disinfection

Certificate issued by University of Dresden attesting cleaning **capacity of Multisteril Fast in combination with Multisteril CD cleaner.**

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Spaces

Space is another important problem, in fact, sterilization rooms often have small size and rationalization of space assumes a considerable value. **Multisteril Fast can do all the process filling 40 cm.**

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Overall costs

Cost is an important component which must be brought under control. **With Multisteril Fast every cycle costs about 2 Euros.**

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Liquid
concentrate for
the non
protein-fixing
cleaning and
disinfection

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Liquid concentrate for the non
protein-fixing cleaning and disinfection

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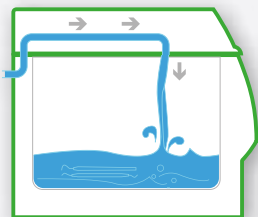
Support for 2 surgical trays
(article code refers
to support only)

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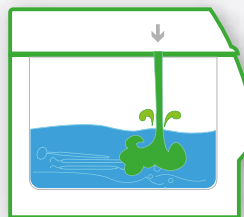
Loading the water

The water enters the tank thanks to the direct connection to the mains water supply.

Stand-by

To improve cleansing, an initial standby phase has been created, which allows the tub to be filled with water and liquid detergent/disinfectant, so that the instruments sit better in the solution, **ensuring perfect cleansing, even when using bi-component cements.**

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
Disinfectant/detergent entry and liquid activation

A special peristaltic pump present inside the device automatically **introduces the percentage of disinfectant/detergent previously outlined in the parameters of the cycle.**

Fasi del processo | Phases of the process

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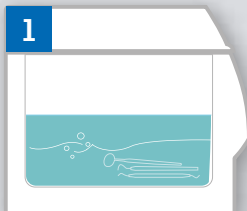Heating

The device moves on to heating and to a **first static decontamination of the instruments.**

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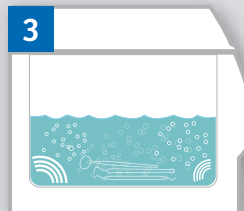
Degasification

This process eliminates the gases dissolved in the water avoiding interference with the ultrasounds and improving efficiency.

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
Ultrasonic wash

At the end of the degasification phase the ultrasonic cleansing begins, setting off the cavitation, which allows for the removal of chemical and organic residue present on the materials immersed in the tank, **destroying the bacterial load by more than 95%. The length of this procedure can be programmed by the operator for between 5 and 30 minutes.**

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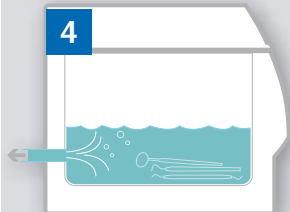
Waste with pump

At the end of the cleansing phase, **MultiSteril automatically empties all the liquids present in the tank.** Once the tank is completely empty the following rinse phase begins.

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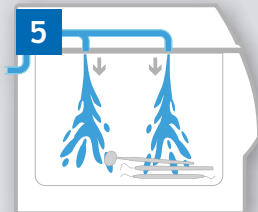
Rinse

To remove all chemical residue from the instruments, **the rinse phase begins, taking place through the pressurised intake of water through the special nozzles in the lid.** The water taken in is then automatically removed through the mesh with a special pump that permit to make the process faster. At the end the drying phase is launched.

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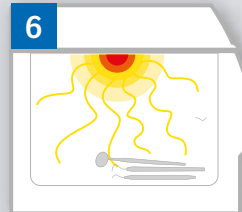Drying

To guarantee a proper drying of the materials **the machine generates a flow of hot air, produced by a fan and a resistor located on the lid of the device**, which lightly touches the objects and takes away the dampness in the form of gas.

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EN

Demonstrating yet again its great attention to growing operating needs, Tecno-Gaz has produced a unique and extremely rational mechanical device for drying the materials to be sterilized. This device allows the obligatory drying to be performed and to avoid any handling of instruments especially sharp ones.

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EN

ONE is a new generation sealer that has been created with the aim of reducing envelope packaging time, monitoring the quality of the welding, and is able to seal all types of paper for sterilization, including folded paper.

One assures:

- Sealing band of 12 mm.
- Sealing bar 32 cm.
- Great easiness of paper introduction.
- Active electronic control over the couple of resistors.
- Visibility of the area of cutting.
- Signal of possible bad sealing.
- Introducible in any sterilizing piece of furniture.

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EN

Tecno Seal is a roller sealing machine with photocell automatic feed. Tecno Seal could be equipped with built-in printer (TA002TAU) to print on pouches packaging and expiry dates as well as symbols during the sealing phase. This solution allows to have constant control on the state of sterility of instruments. On one side of the machine there is an electronic control that allows the management of the parameters of sealing.

- Automatic photocell feed system (10 mt/min).
- Complete with integrated printer (Art. TA002TAU)
- Automatic printing of seal/expiry date.
- Electronic card for control/management of parameters of use.

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EN

The tracking of the instruments is a basic procedure for the handling of sterilized materials. Tracking guarantees the recording of the "path" taken by each and every instrument, from preparation to sterilization, until it is used. This is made possible thanks to a data cross checking management system. Tracking is an activity that forms an integral part of the sterilization cycle procedure and should be implemented after the wrapping phase and before the autoclave is loaded. **This procedure is indispensable in order to guarantee maximum legal protection to the operators and to the health director.**

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Printer for traceability with barcodes (compatible with commercial scanners). It is compatible and programmable directly from the menu of autoclaves ONYX B.

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Moduls specifically devised for recording sterilization cycles. CD containing files with printable forms.

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320 mm / 400 mm*

Optional

Stampante integrata / Built-in printer

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* modello Andromeda Vacuum Plus Xp (camera 20 lt.)
* model Andromeda Vacuum Plus Xp (20 lt. chamber)

* modello Hydra Evo Plus (camera 18 lt.)
* model Hydra Evo Plus (18 lt. chamber)

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EN

Autoclave is the key device within each medical structure, therefore it is fundamental choosing a well-known and referenced product which can rely on a widespread network of technical support. Onyx B is the new range of Tecno-Gaz autoclaves, a world leading Company for many years.

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Double load... Same time

Using standard size autoclave about 4 kg instruments can be sterilized, while with Onyx B you can sterilize from 5 kg to 8 kg of instruments. **With 1 double load you are saving 50% of your time.**

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SD card

SD CARD system allows **constant machine updating** by a simple download on it and then by inserting it into the autoclave system. **An extraordinary option in order to have constantly efficient and updated device.**

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Cycles for all needs

Flexibility is fundamental to rationalise working and create operative efficiency. **Onyx B autoclaves offer a wide range of cycles all studied to help you every day.**

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Best management of instruments

Having big load autoclave helps you to rationalize instruments flow and increases your operative efficiency. Efficiency means profit, Onyx B autoclaves are advantageous for this too.

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Energy saving

Onyx B autoclaves have low energetic consumption as well as unbeatable consumption/ load ratio. **Economic saving and benefit for the environment.**

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EN

Big load

New regulations and new sterilization concept have developed sterilization practice in all medical and dental facilities along with wider range of instruments and spread of professional facilities which developed the need of autoclaves able to sterilize large loads. **Onyx project is born with the aim to satisfy these need adopting specific and new solutions enabling to manage easily also partial loads.**

FL System - Full Load System

To avoid manual loads with possibility of water pouring off, Onyx B autoclaves throughout their automatic clean water tank automatic system are using a level sensor control pump with automatic lock. **Frontal loading, no opening cover or removing system, enabling simple placement of the machine.**

Proper trays for any type of load

Trays of different lengths enables you to insert instruments inside bulky envelopes, to place load more rationally enabling sterilization of **different loads following the correct guidelines for load placement.**

PHD System

Preheating system with stand-by for minimum temperature **maintenance and consequent overall sterilization cycles time reduction.**

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Water net connection

The autoclaves Onyx B are **autonomous** as equipped with double tank of pure water and exhausted water, they are **prepared for the connection to the water supply through osmosis system mod. PURA Plus.**

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Inspectionable tanks

In case of insertion of unsuitable liquids, autoclave tanks are inspectionable and cleanable as per technical normative regulations. **Onyx B autoclaves are supplied with pure and used water tanks.** Each tank is supplied with probes indicating levels.

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Overnight cycles

Automatic overnight cycles with anti-condensation system and automatic switch off.

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ID system

Authentication intergrated system of the operator which is protected by password and permits at the end of the cycle to associate operator to sterilized load having visual convalidation of the operator himself. The same authentication is also reported by the procedure of load traceability.

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Reduced maintenance

Onyx B autoclaves allow you to make big loads thus reducing the number of cycles. This means **extending autoclave life over 50% reducing maintenance costs and amortizing the cost of the product over the time.**

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EN

7 sterilisation cycles

Onyx B is born with the aim of rationalizing the work of sterilization operators, therefore the autoclaves are equipped with cycles which have been expressively conceived for improving time management and operator's needs within outpatient clinics.

4 trays
extra4 trays
extraCarico
5/7/8 kgLoad
5/7/8 kg7 cicli
operativi7 sterilisation
cycles3 cicli
test3 test
cycles

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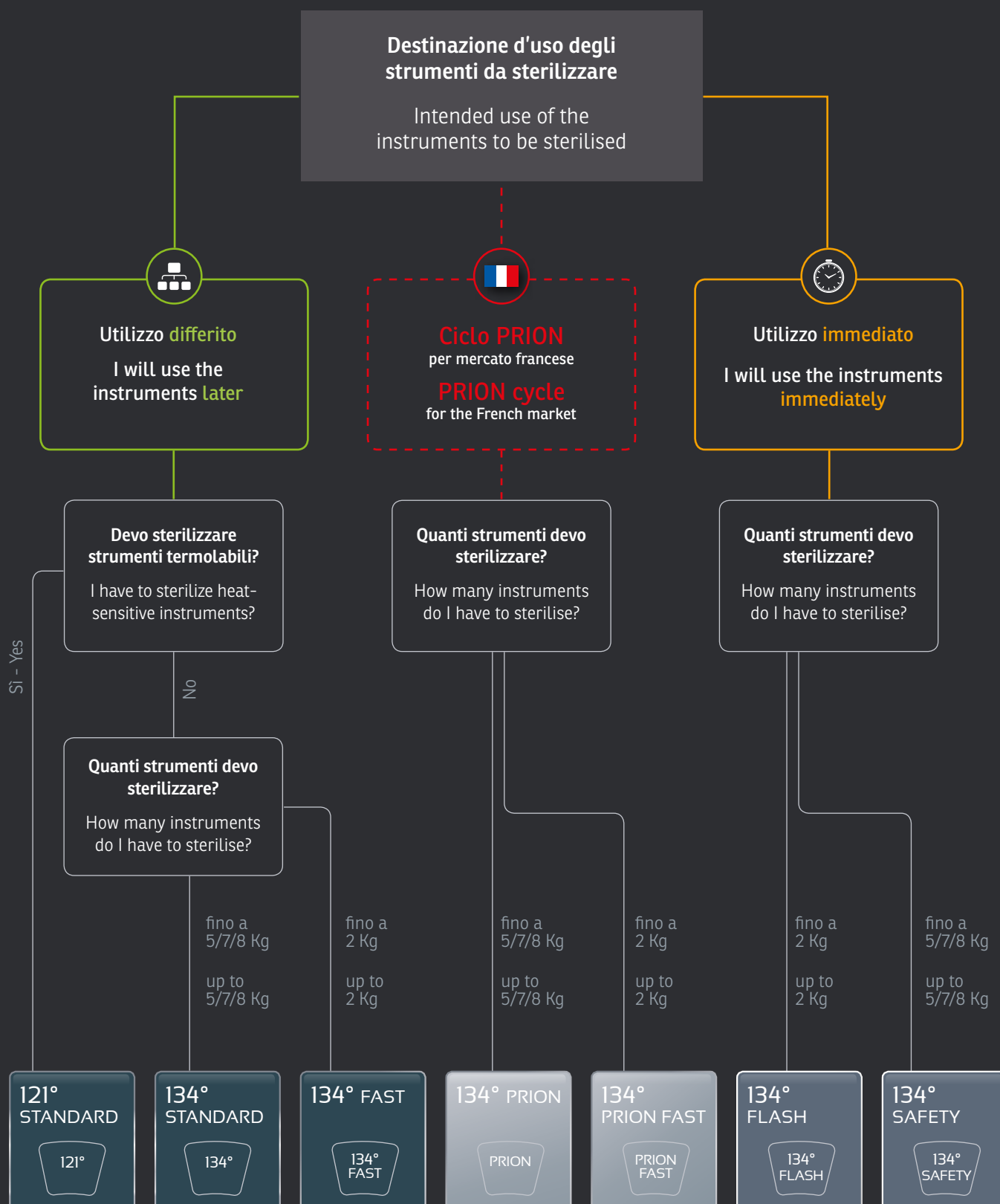
Completely automatic cycles

All the cycles are completely automatic, **all the phases are managed by the advanced card equipped with powerful microprocessor**. Each and every phase (vacuum phase, preheating, exposure time, drying) is monitored, backed up and managed automatically. There is no human involvement in the carrying out of the cycles.

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EN

Europa B Evo is an autoclave that has been developed jointly by the Tecno-Gaz research office and important external development facilities. Europa B Evo is indispensable for all medical and dental facilities and in all fields where safe sterilization of all materials is required: hollow, porous, solid, wrapped and unwrapped. Europa B Evo continues the success that Tecno-Gaz has achieved over the years, in fact, thousands of professionals use Tecno-Gaz autoclaves and are very satisfied.

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P.I.D. System

P.I.D. means Proportional Integral Derivate and represents the apex of closed loop controls. The structure of this technologically advanced control is simple and flexible. P.I.D. ensures a constant control and monitoring of the whole operating system of the autoclave, acting automatically in case of anomalies or incorrect values in technical and electronic parameters. **This device behaves like a "real technician" with real-time diagnostic and intervention capability. This solution reduces anomalies and warnings, granting maximum reliability and minimising the machine inactivity times.**

Safety on the door

We have applied three safety devices on the autoclave door:

- **The first device is a mechanic and electromagnetic one:** it triggers when the sterilization cycle begins and completely blocks the door opening. This device can be removed at the end of the cycle by pushing the device-unlocking button, but only if positive pressure is near zero.
- **The second device is an electrical one:** it triggers automatically when an attempt to force the door open is detected.
- **The third device is dynamic** and acts on the structure of the gasket.

Safety devices in place

- Certified safety valve.
- Ped on sterilization chambers.
- Insulation of the chamber with certified ceramic material.

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Barometric alignment

Automatic barometric alignment is provided from the first time the device is switched on, without any technical intervention.

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Overnight cycles

Automatic overnight cycles with anti-condensing system and automatic switch-off, with memory effect.

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Provision for mains water supply

The osmosis system mod. Pura is an extremely compact piece of equipment for osmosis, created to be directly connected to the autoclave, guaranteeing a constant supply.

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Built-in printer

Built-in printer for the registration of cycles carried out. The printer includes:

- date, time
- cycle number
- serial number of the device
- minute by minute cycle data
- outcome of the cycle

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EN

Andromeda Vacuum XP is provided with a powerful vacuum pump which, at the beginning of the cycle, ejects the airpockets which are into the sterilization's chamber, and, at the end of the sterilization's phase, grants a very good drying. A sophisticated electronic device grants a constant monitoring of pressure, temperature and time. It can work also without the presence of the operator, as all the functions are automatic. Andromeda Vacuum XP has an electromagnetic block device on the door. The total automatism of each phase of the cycle prevents from any possible human error, also from inexperienced personnel. Stainless steel chamber, uniformly heated by a band heating element. Twin tank for clean and used water. Andromeda Vacuum XP is provided with 5 cycles of sterilization: 121° C and 134° C for unwrapped and wrapped instruments, and a "flash" cycle for the fast sterilizations. Andromeda Vacuum XP is supplied together with the printer built-in-into the drive board.

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EN

Hydra Evo is steam autoclave for the sterilization.

- **2 sterilization cycles:** possibility of choosing two cycles at 134°C or 121°C.
- **Automatic phases** of preheating, sterilization cycle and drying.
- **Closure system with an operator safety pivot.** Air pockets automatic ejection system: thermodynamic system for the ejection of the "cold" air pockets.
- **Stainless steel chamber:** the sterilization chamber is made entirely of stainless steel and is heated by a band shaped heating element which guarantees uniformity of heat.
- **4 aluminium trays.**
- **Electronic control with digital display**, allowing the digital display of times and temperature.
- **Control of the sterilization process**, through microprocessor management with a constant measurement system.
- **Operation control, blocking the autoclave** in the event that there is a lack or an inadequate quantity of water in the chamber.

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EN

The Pura Plus osmosis system is extremely compact and it's designed for connection directly to the autoclave, to guarantee a constant supply of pure water. The system can be installed in a number of positions to facilitate mounting and the optimum organisation of the available space. Pura Plus is fitted with a series of osmotic membranes and filters, with an average duration of about 800 cycles, depending on the original hardness of the water. The cartridge should therefore need replacing about once a year, or after 800 cycles. The overall cost of the filters is very reasonable, and the cost per litre is therefore almost negligible. When the Pura Plus system is not producing water, it is not pressurized, and this avoids all kinds of problems caused by the accidental breakage of couplings and fitting.

Why an autoclave osmosis system?

The osmosis system is recognised as the best method, ensuring treated water free from impurities and hard particles in order to protect the workings of the autoclave. The system allows for a direct connection with the autoclave, ensuring service and a reduction in costs relating to the purchase of distilled water.

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Devices for supplying autoclaves

This is a simple, safe and rational system that ensures optimum quality distilled water, thanks to a boiling procedure with forced recondensation. The use of Distiller will avoid the purchase and storage of cumbersome water barrels. Distiller is supplied complete with accessories, filters and other equipment.

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3 phial biological incubator, **with spy operation and master switch, connected to mains electricity, 220 V~.**

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EN

Biological control test for autoclaves

Our biological incubator is a product developed for the **incubation of biological indicators**. The incubator consists in a frame, an ABS cover, an aluminium heating unit with 3 slots for test tubes/biological indicators and an electronic card for automatic control of heating unit temperature.

What are these used for?

This can be used by **all those who own an autoclave**.

Why choose a Tecno-Gaz incubator?

Because they are **certified incubators**.

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Periodicità

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EN

15 phial biological incubator, **with spy operation and master switch**. Dual temperature selector 37°C for ethylene oxide, 57°C for steam, connection to mains electricity, 220 V~.

What are these used for?

Biological test suitable for all types of steam **autoclaves**. It is used to assess the ability of the autoclave to destroy all micro-organisms including spores. **Requires the use of spores in phials and a biological incubator**.

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EN

What are these used for?

Physical test, only suitable for autoclaves that are able to sterilize porous loads. **It is used to assess the ability of steam penetration in porous bodies. Requires the use of spores in phials and a biological incubator.**

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EN

What are these used for?

Tests suitable for all types of autoclaves. They are used to assess the ability of colour change in the autoclaves through the parameters of the machine time temperature - pressure.

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EN

What are these used for?

It is used to assess the ability of steam penetration in hollow bodies.

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EN

Why an archiving system?

An archiving system serves to manage all tests carried out in an **organised way**, facilitating the work of the operators in charge. This system also allows for availability during the time that the test is being carried out. An organic, rational and ordered method.

What are these used for?

STERILDOCS teaches the operator **constant regularity** and an organised method, ensuring a continuous use of control tests.

STERILDOCS is a container featuring the following sections:

- DVD-operating protocol support section;
- Written protocol instruction section including sterilization tests;
- Practice and equipment data sheet section;
- Section where all the practice and autoclave data are recorded. The paper used is watermarked and does not allow any modifications or corrections;
- Filing section. The filing section is divided into sectors and is identified by a specific graphical structure that enables user-friendly filing of each test providing evidence for data, cycles, etc.
- Filing is simple, rational and orderly and allows you to manage all the tests of each autoclave in an organized way.

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EN

Tecno-Gaz spa has introduced a new series of sterilization bags and reels called T paper. These devices are designed for hospital, medical, dental, cosmetic, veterinary and podology uses and, in general, for all the personnel that carries out steam or gas sterilization processes.

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Buste autosigillanti • Self-sealing bags

Lunghezza film Film length	Pz./conf. - Pcs./pack	Codice / Code
90x230 mm	200	Art. CS011Z01
140x260 mm	200	Art. CS012Z01
190x330 mm	200	Art. CS013Z01
300x420 mm	200	Art. CS014Z01

Carta standard Standard paper

Lunghezza rotolo Reel Length	200 mt. standard 100 mt. (a richiesta/on demand)
Colore materiale plastico Colour of plastic material	verde standard/green standard azzurro (a richiesta) blue (on demand)
gr/m ²	60 gr/m ² 70 gr m ² (a richiesta/on demand)

Sterilair Pro

STERILAIR PRO (a parete/ wall mounted) ART. SA210ZSA
SUPPORTO stativo / stand mount support ART. SA002ZSA

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EN

Safeguard your professional image and your prestige, by protecting your health and that of you colleagues and patients. SterilAir PRO is the first air sterilization system. SterilAir PRO is an air sterilizing appliance which guarantees the elimination of all microorganisms including spores. This device can run continuously also in the presence of people, ensuring maximum safeguarding of the operators. SterilAir PRO is simple, silent and ergonomic.

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Functioning

its functioning is based on a system of forced ventilation closed loop. The air drawn into the module first passes through a filter, mechanically positioned, within the inlet entrance. Here the coarser pollutants are blocked, preventing the fouling of the germicide lamps. Then the air is forced to pass in direct contact with mercury vapor tubes that, thanks to the emission of the UV-C, carry out the maximum germicidal action. The high reflectivity shielding action of the housing concentrates the reflections of the UV-C radiation. The air is subsequently expelled through the nozzle output, thus leading to an effective microbial killing.

The system

The great advantage consists in the absolute lack of danger for the operator since there is the minimum emission of UV-C radiations from the unit. The device can effectuate a continuous air disinfection treatment (120 m³/h). It's possible to carry out a demolition of microbiologic pollution of the air in every room, during every productive phase without any damage for operator's health. After about three hours from installation 90% of demolition of bacterial charge in the air.

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EN

WATER SAFETY devices are based on an innovative double wall hollow microfiltration technique. The large filtering surface assures a long life to used products. Double wall filter assures a sterile filtration maintaining unaltered the properties of water.

Filtering kit Legionella protection is composed by:

Art. 1FLAA0001 Pre-filter

Dimensions: length 140 mm, diameter 47 mm

Art. 1FLAA0002 Legionella protection filter

Dimensions: length 187 mm, diameter 40 mm

Art. 1FLAA0003 Connection with gaskets

(for inline installation)

The kit can supply water to one dental unit and must be replaced after one year / 5000 liters

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Maximum filtering

WATER SAFETY system is composed by a pre-filter, to **eliminate the impurities of the water and protect the succeeding filter**, and by WATER SAFETY.

Special materials

All filters are **100% tested to maintain the integrity and the air-tight**, granting a continuous operational efficiency. WATER SAFETY have been constructed with **special materials which allow to maintain unaltered the physical performances**, even in presence of high temperature water and chlorine. **Duration of filters is approximately 5000 liters**, but in any case they should be replaced every year.

Suitable for any type of dental unit

Thanks to its reduced dimensions WATER SAFETY **can be easily installed inside every type of dental unit**.

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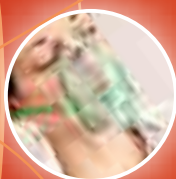
Protect your facility

First aid products are life-saving devices, therefore equipping yourself with these items is a legal, ethical and professional duty. Having knowledge, medication and emergency devices also means protecting yourselves, your reputation and that of your profession, as well as **being ready and prepared to save a person's life**. Tecno-Gaz S.p.A. is not offering you first aid devices, but rather **a programme and a project that is well-designed and suitable for surgery**. The devices and solutions are the result of careful operational analysis, carried out by specialists and professionals in the field. **Choosing a Tecno-Gaz device means setting out on an organic and complete path to safety.**

First Aid

How to prevent emergencies in the field of dentistry

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EN

VITAL TEST is a professional pulse oximeter that is designed for use in a medical environment. Simple and multi-functional it can be supplied with a special support, designed to be fixed to a unit, wall or stand. This option allows for constant control of the instrument and allowing it to become a convenient instrument for constant use.

Principal Characteristics

- wave-length or decimal reading of the values on the display
- adult-child reading selection
- on/off sound and adjustable volume
- SpO₂ min / max alarms
- Pr min / max alarms
- patient identification
- data reading

On request **Vital Test** can be supplied **programmed with an interface to be connected to a computer**, for the registration and printing of all parameters measured.

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EN

An important device as oxygen forms the basis for keeping cells alive. The cylinder guarantees supply for up to 1 hour with 15 litres per minute. The cylinder must have a minimum of 5 liquid litres (1.000 litres of gas).

Tecno-Gaz can provide the AIR-OX resuscitator fitted with:

- an **oxygen cylinder** with 5 litres (1.000 litres of gas) complete with an adaptor and litre dose measurer (*cylinder available only for export market*)
- oxygen therapy **mask**
- resuscitation **balloon** with mask
- various **accessories** (mouth-opener, tongue-puller, Guedel pipes, tubes).

AIR OX • ART. 3003-E

AIR OX (senza bombola con carrello / without bottle and with trolley)
ART. **KT017ZDM**

AIR OX (senza bombola senza carrello / without bottle and without trolley)
ART. **KT016ZDM**

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EN

Tecno-Gaz's MD monitor is a professional instrument with many operational and memory functions, fitted with accessories and circuits. The monitor has a dual function: diagnosis and monitoring. Allowing the **measurement of various vital parameters**, which can be measured singly or at the same time.

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Battery and power supply status indicator LED

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5 kg

Here are some of its functions:

- Multiparametric monitor, with numerical and trace display.
- Operates with a battery or from the mains
- Complete with built-in printer
- Outlet for mains connection for remote management
- Adjustment and set-up of parameters
- 10 inch monitor

5 Parameters:

- ECG 5 channels 2 traces
- Systolic/diastolic/average pressure
- Heartbeat
- Pulse oximeter
- Respiratory control
- Dual sound temperature

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EN

Resuscitation set supplied in a practical, shockproof, polypropylene case without bottle, complete with:

- Resuscitator bag
- Oronasal mask
- Mouth opener
- Tongue puller
- 3 Guedel airways
- Ox tube

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EN

What is it useful for?

Semi-automatic defibrillator enables defibrillation of heart muscle walls safely since it is equipped of sensors which are able to recognize cardiac arrest due to **arrhythmia, ventricular fibrillation and ventricular tachycardia**. As for above mentioned cases, defibrillator automatically determines if discharge is necessary by selecting energy level. Operator does not have possibility to force the discharge if the device informs it is not necessary.

- **Selector adult child:** It permits to treat pediatric patients through a simple selection on the device (it does not need double plates, adaptors, keys etc...)
- **Guide voice:** for a simple and assisted usage.
- **Sd board:** data storage for legal protection (OPTIONAL).
- **Soft cover:** To protect from hits and scratches (STANDARD).
- **LiMnO2 long life battery:** 5 years, 200 discharges (STANDARD).
- **Battery indicator:** to constantly check battery level.
- **Critical components self test:** to maintain device working efficiently.
- **Impedance calculation:** o verify the integrity of DAE and patient contact.
- **Display screen:** to read instructions in case it is not possible to hear guide voice.
- **Biphasic shock:** discharge which goes through heart in both ways.
- **Heart on software:** useful to archive data concerning DAE usage and for firmware update.
- **Pre-connected electrodes:** to reduce time interventions.
- **Battery:** 200 discharges.
- **Irda port:** to permit pc-defibrillator communication.
- **Variable energy:** adult (<25kg) from 185 to 200j/child (<25kg) from 45 to 50j.

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Software AED Event Review
Art.1Z12A0006

Software AED Event Review
Guida Utente / User manual
Art.1Z12A0007

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EN

Any emergency requires paying maximum attention to both organizational and operating aspects. With this view rationalization of the equipment is an essential aspect. Tecno-Gaz S.p.A. has created a "Mobile First Aid Centre (E-POINT)". **During any emergency speed is the prime factor**. For this reason being able to use a mobile centre that is ready for use is undoubtedly the ultimate operating solution.

This solution enables users to have a structure that features:

- a resuscitation set
- a defibrillator
- a multi-parameter monitor
- a pulse-oximeter

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Removable compartment containing resuscitation kit

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Wall-mounted conscious
sedation equipment

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Conscious sedation

The controlled administration of oxygen and nitrous oxide by inhalatory route **induces a relaxed feeling, reducing emotional stress. Anxiety and the perception of pain are reduced**, increasing the amenability of the patient and facilitating **collaboration with the doctor**, while maintaining the vital reflexes unaltered.

Conscious sedation

In order to treat all types of patients and cases

Master Flux Plus Electronic
(versione mobile / mobile version)
pagina / page 42

Circuiti e maschere

Circuits and masks

Accessori per Master Flux Plus
Accessories for Master Flux Plus
pagina / page 45

Master Flux Plus e Master Flux Plus Electronic

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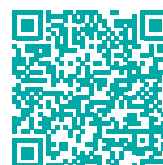
Master Flux Plus and Master Flux Plus Electronic

Are the unique conscious sedation machines entirely researched and developed in Italy.

All parts are manufactured by Tecno-Gaz and are subject to scrupulous individual inspection. To give the machine a positive visual and psychological impact on the patient, the components are enclosed in an elegant unit which contains the oxygen and nitrous oxide bottles with relative reducers and attachments.

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EN

Why conscious sedation?

The complexity of operations and increased invasiveness have resulted in significant lengthening of sessions and require a new way of approaching the patient. **As a result of conscious sedation, it is possible to decrease surgical stress, increase patient compliance and, consequently, productivity.**

Why Master Flux Plus

Master Flux PLUS is the unique conscious sedation machine entirely researched and developed in Italy. All parts are manufactured by Tecno-Gaz and undergo scrupulous individual inspection. To prevent the machine from having a negative visual and psychological impact on the patient, it has been placed in an elegant unit with the capacity to contain the oxygen and nitrous oxide bottles and their relative reducers and attachments. The unit is painted RAL 9002 white, is equipped with wheels and laminated instrument tray and has a compartment to house 5-10-14 litres gas bottles. On the front there is a space for the flow control box, complete with accessories. Master Flux Plus has passed the stability tests laid down by directives.

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IntelliFlux supplies gas
through two centre pipes,
which do not obstruct
operations.

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Rear elastic support to
secure the mask to the
patient's face.

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No gas dispersion in dental clinic environment

Many particular specifications
of IntelliFlux masks contribute
to prevent dispersion such as:

- Backflow valve
- Adaptable connector to dental unit aspirators
- New easy-breathe membrane
- Face-adaptable, single use silicone shape

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EN

- **Electronic system for conscious sedation**
- Graphic interface with colour display
- Mains and battery (*4 h*) power supply
- Safety systems with sound and visual alarms

Safety devices

The MF ELECTRONIC flowmeter box is a sophisticated electronic engineering project equipped with a proportioner-driven delivery system. An electronic method has been chosen, in which the flow can be precisely adjusted with the touch of a button. This has allowed us to obtain a high precision product which is very reliable and simple to use. Master Flux Plus Electronic is easy to use. Its technical features make it extremely adaptable. Its effective safety devices avoid every possibility of error, and therefore any negative consequences for the patient. The displayed graphic contents are easy to read.

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Minimum oxygen concentration There is always a percentage of 30% oxygen present, thereby safeguarding against the risk of hypoxia and anoxia. The pressure reducers connecting the flowmeter box to the cylinders are mounted inside the cabinet, out of the sight of the patient; the mounting design is safe with no risk of exchanging the two gas reducers to both the cylinder, and to the flowmeter box.

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Possibility to record new patients with progressive numbers and to retrieve performance data (time and amount) subsequently

Intelliflux

CIRCUITO E MASCHERINE / CIRCUIT AND MASKS

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Silicon profile suitable for any type of face. No signs and great comfort. It allows to reduce N₂O emissions in the environment.

EN

New nose masks

Nose masks have a rigid transparent polycarbonate centre shell, which enables visibility and prevents any torsion, thus guaranteeing a perfect hold. The adaptive cushioning and front forks enable perfect, stable positioning, thus preventing any dispersion or leakages of gas. All parts of the nose mask are autoclaved at 134° and the centre connection contains the non-return valves.

"Y" Connection

Allows the proper flow of gas mixture preventing that the exhaled gases can return to power supply circuit. The small chamber eliminates the risk of CO₂ permanence.

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Gas in entrata con valvola antiritorno
Inlet gas with non-return valves

ACCESSORI / ACCESSORIES

versione mobile e a parete
mobile and wall-mounted versions

Art. MF835/6/7/8ZMF

Maschere in varie misure
Masks in various sizes

Misura / Size 1
Art. MF835ZMF

Misura / Size 2
Art. MF836ZMF

Misura / Size 3
Art. MF837MF

Misura / Size 4
Art. MF838MF

RICAMBI / PARTS

Supporto frontale siliconico misura unica e profilo nasale in varie misure
One-size silicone head cushion and silicone mask pads in various sizes

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EN

Solution for centralized system

This version has an extendable arm, separate couplings for hooking up to the gases, flow control box and the usual accessories. **The wall-mounted version means that a Master Flux Plus art. 1601/AS can be installed in each workstation, with the gases being delivered from a centralized point.** If the machine does not need to be used simultaneously in all workstations, simply make provision for the extendable arm and connections next to each dentist's chair and move the flow control box from one workstation to another. This solution guarantees flexibility and economic savings.

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Flow control box

The flow control box is a sophisticated project of mechanical engineering with an automatic proportioning service system. The mechanical method allows "direct" gas management: i.e. it is not subject to other electronic or electrical "controls". We have thus created a high precision product that is highly reliable and easy to use. User-friendly, **Master Flux displays technical characteristics that make it extremely versatile and with effective safety devices that absolutely do not allow for errors to be committed that would negatively impact the patient. The silk-screen drawing is simple and easy to read.**

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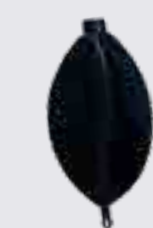
Unique Advantages

Tecno-Gaz can propose you a complete range of lighting support suitable for all operative exigency:

- Surgical chair
- Operating theatre
- Ambient

Surgical lighting

Lighting project in operatory field

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EN

EndoLed

During endoscopy surgery it is possible to activate EndoLed in order to get environmental lighting. This function enables to **light only one module on and to regulate light intensity and color temperature.**

Cold light

The increase of thermic radiation on operator field is proportional to the quantity and to the power of used Leds. **Tecno-Gaz, thanks to indirect light concept, is using only few Leds.**

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As for specific surgeries such as:

- thoracic surgery
- addominal surgery
- caesarean section

it is necessary to have enlarged lighting field with higher lighting intensity. Additional optics named E-View (Extended-View) enables to increase illuminated field till 32 cm with **light intensity on external board of 6000 lux keeping light intensity unchanged.**

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Indirect light, main feature of PentaLed lamps, guarantees the surgeon a cold deep and comfortable light thanks to the use of a **trajectory studied in order to reflect all lighting rays created by Led with no dispersion at all on operator field.** The result enable to:

- reduce the number of Leds used.
- reduce temperature on operator field.

Thanks to indirect light principle, surgeon and assistants are never bothered by light and can operate with no eyestrain nor dazzling.

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Nature inspired all man discoveries! Sun is the main light source and its rays have monochromatic light emission, that is thorough uniform coloring. **Starting from this idea, Tecno-Gaz has projected its lamps using only white Leds in order to avoid any alteration of color temperature on operator field and undesired shadow effects.**

Orion 40DS

LAMPADIE CHIRURGICHE A LED / LED SURGICAL LAMPS

A PARETE | WALL-MOUNTED • ART. LC101LRD

A SOFFITTO | CEILING • ART. LC102LRD

A STATIVO | ON TROLLEY • ART. LC103LRD

A SOFFITTO CON SUPPORTO TELECAMERA
CEILING WITH CAMERA SUPPORT • ART. LC106LRD

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Stands for ceiling fixing created according to the height of the ceiling.

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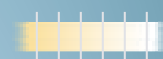
Autoclavable handle, for adjusting the spotlight

EN

Why Orion 40DS

It is especially recommended for operating theatres where the surgeon needs a small lamp to avoid interference with other overhead equipment. **Excellent for oral and maxillofacial surgery and aesthetic plastic surgery.** It has **30 elliptical reflectors split** into 6 modules, each containing 5 LEDs, providing **130.000 Lux**. An aluminium ring runs around the dome for easier positioning. **The E-View system** lets the surgeon adjust the extent of the lit field to get the right type of light for each surgical procedure.

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Diametro illuminato / Enlightened diameter: 15 cm


This function allows to replace the lightening generated by traditional lamps for dental unit. This function permits to light only two modules and set the intensity and colour temperature. **Focus On Mouth Spot** (compared to hospital surgical lamps). Enlightened diameter 15 cm (20 cm with extended view) perfect for the spot of the mouth in dental field, prevents glare typical of hospital lamps with larger focus.

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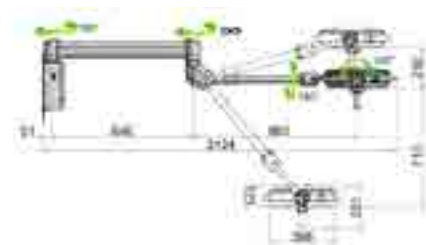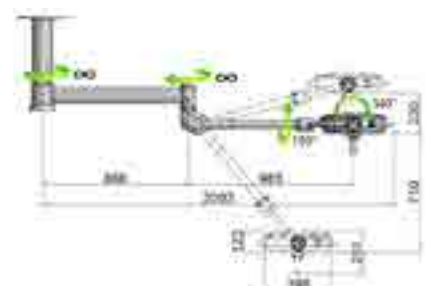
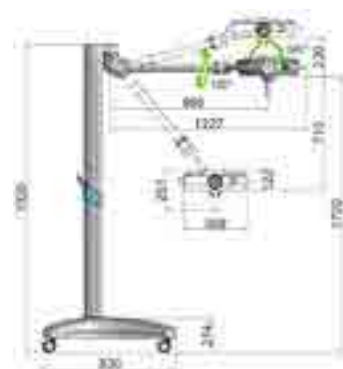
 Soffitto con supporto telecamera
 Ceiling with camera support

Orion 60

LAMPADE CHIRURGICHE A LED / LED SURGICAL LAMPS

A STATIVO | ON TROLLEY • ART. LC101LRB
A PARETE | WALL-MOUNTED • ART. LC102LRB
A SOFFITTO | CEILING 60+40 • ART. LC103LRB
A SOFFITTO | CEILING 60+60 • ART. LC104LRB
A SOFFITTO CON SUPPORTO TELECAMERA
CEILING WITH CAMERA SUPPORT • ART. LC109LRC

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EN

Why Orion 60

81 elliptical reflectors split into 9 modules, each containing 9 LEDs. The lamp lens is **designed to guarantee perfectly focused lighting**, i.e. without the need to focus the light every time the dome is moved. The light is pre-focused at different distances (from 80 cm to 200 cm). The **E-View system** lets the surgeon adjust the extent of the lit field to get the right type of light for each surgical procedure. The light structure of the lamp makes it easy to handle and yet still very stable.

The controls on the membrane keyboard activate the following functions:

- **colour temperature** (5000°K and 4500°K)
- **brightness control (up to 160 Klx)**
- **lit field adjustment**
- **switching on/off**
- **EndoLed**, a special type of light that facilitates endoscopic operations. The surgeon can select the exact brightness and colour temperature required.

The modular construction of the electronic board that feeds the LEDs guarantees continuous light, even in the unlikely event that a LED or board component should fail. **Smooth 360° rotation** with slip-ring contacts for power transmission.

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

Art. LC103LRB
Orion 60 + Orion 40
versione a soffitto | Ceiling version

Art. LC104LRB
Orion 60 + Orion 60
versione a soffitto | Ceiling version

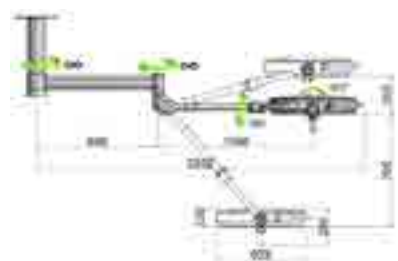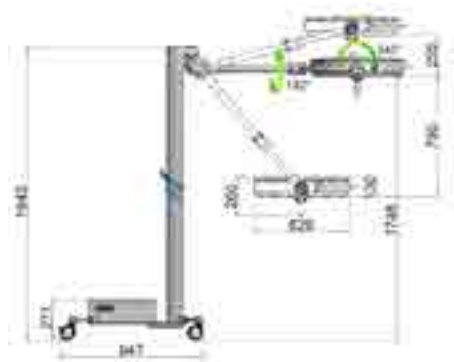Art. LC102LRB
Versione a parete / Wall-mounted version

Art. LC101LRB
Versione a stativo / On trolley version

Art. LC109LRB
Soffitto con supporto telecamera
Ceiling with camera support

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A SOFFITTO | CEILING 80+40 • ART. LC102LRC

A SOFFITTO | CEILING 80+60 • ART. LC003LRC

A SOFFITTO CON SUPPORTO TELECAMERA
CEILING WITH CAMERA SUPPORT • ART. LC107LRC

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EN

Why Orion 80

It has a central module like the Orion 60 lamp and 4 side petals, each fitted with 6 LEDs radiating outwards.

This configuration offers two great advantages:

- **Increased scialytic effect**
- **Increased depth of field** thanks to the double focalization of O-Focus.

Thanks to the double focalisation O-Focus. The central and side modules are designed to provide lighting at two different focal distances and thus guarantee greater depth of light and constant light intensity without losing focus.

The **O-Deep** module is fitted at the centre of the lamp: **this has 9 LEDs with special parabolic dishes designed to reflect the light at depths**. O-Deep means the surgeon can enjoy excellent 3D lighting, especially in cavities. The E-View system lets the surgeon **adjust the extent of the lit field** to get the right type of light for each surgical procedure.

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Soffitto con supporto telecamera
Ceiling with camera support

Slim

LAMPADA CON TECNOLOGIA LED / LAMP WITH LED TECHNOLOGY

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EN

Why a LED lamp:

The numerous and undisputed advantages of LED technology are the reason for its rapid development in all fields. SLIM is an LED lamp for outpatient and diagnostic use. The clear and cold light of SLIM 5.000 °K, faithfully reproduces colours ensuring optimum detail definition, important for the various outpatient specialties.

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Stands for ceiling fixing
created according to the
height of the ceiling.

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Luminosity adjuster

Lighting intensity is electronically adjustable with a potentiometer from 20,000 Lux up to maximum intensity. The surgeon can set the desired lighting intensity corresponding to the type of procedure.

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Autoclavable handle, for
adjusting the spotlight

A SOFFITTO | CEILING • ART. 370S2
VERSIONE A PARETE | WALL-MOUNTED VERSION • ART. LC001Z17
VERSIONE A STATIVO | ON TROLLEY VERSION • ART. 372S2
VERSIONE PER RIUNITO | UNIT VERSION • ART. 371S2

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Art. LC001Z17
Versione a parete / Wall-mounted version

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Sphera
pagina / page 61

Overall project

As well as its many product lines, Tecno-Gaz offers a series of small pieces of equipment designed to improve and complete operators' everyday work. All the devices have been created in order to improve performance and to rationalise surgery times and methods.

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EN

Formed from unbreakable material, it features a simple and rapid mounting device for radiographs. Its design allows it to be used either wall mounted or on the workbench. Illumination is by means of a fluorescent lamp (6500° K) fitted to an unbreakable opaline diffuser.

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EN

Ray is a dark room with a modern design that is ideal for the manual development of small films.

Ray is an ergonomic, practical and attractive device, that has been created for ease of use thanks to its special shape. **Ray has been created to be conveniently placed against a wall**, taking up very little space. **Its large window allows for a wide field of vision** and the compartments for the hands have been designed at an angle that enables natural movement.

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Ray will be equipped with:

- **1 support** with container holder
- **4 containers** for liquids (developing-washing- fixing-washing)
- **2 x-ray plates holders.**

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EN

Heating anesthetics and irrigations liquids improves the patient's treatment and comfort, reducing pains and unpleasant sensations. Tecno-Gaz has realized SPHERA a double temperature device, electronically controlled with microprocessor and visual signals. **Warmed composites** are more mouldable and improve the performances of photo polymerization. **SPHERA is realized with 4 housings for syringes, 1 housing for composites, 2 housings for tips.**

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SPHERA is equipped with a double heating:

- 37°C for irrigation liquids or composites
- 55°C for anesthetics

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EN

B-Max is a curing light with LED technology. Light, easy to use, wireless. B-Max exploits the new led technology, this ensures a focused, cold and with low infrared outlet high light emission with 450/480 nm spectrum. B-Max ensures a light emission with 1600mW power.

B-Max is supplied with shaped support with built-in battery charger and led indicator.

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Batteries

Life: used in High Mode: 500-600 times, 10 seconds/treatment

Art. SLPA 188

Schermo antirifless
Anti-glare screen

Art. SLPA 190

Base Carica-batterie
Battery charger base

Art. SLPA 186

Adattatore AC
AC Adaptor

Art. SLPA 187

OPTIONAL

Fibra ottica 11mm
Optical fiber 11mm
Art. SLPA 191

Alta Potenza / High Power

Vari programmi / Various curing programs

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Overall project

Whitening is a very wide-spread technique and, in order to obtain real results without causing damage, **quality products and devices must be used**. Unfortunately, a lack of knowledge of the techniques often pushes people towards products and devices that are not suitable. **Tecno-Gaz markets products that have been researched and created at its own plants in Italy, and each product has appropriate scientific support**. We are able to offer you a very interesting range of efficient products.

Added value for your surgery

Whitening is a beauty treatment that is widely appreciated **by patients**, therefore being able to carry out this service in the best possible way is undoubtedly an excellent professional opportunity.

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Art. MTGD00034

Portadepliant in plexiglass (1 pz.) / Plexiglass brochure holders (1 pcs)

EN

DeaLux uses a wave length of between 480 and 520 nm and the lamp has a special screen that allows **absorption of ultraviolet and infrared light to prevent heating of the pulp and mucous membranes, so avoiding or reducing their sensitivity.** The light is controlled and conveyed by a special optical fibre screen that ensures concentration of the rays to allow maximum performance of the equipment and reduction of treatment times. Dea Lux has been designed to take up very little room. It has a solid, stable and balanced base **with wheels so that it is easy to move around.** The lamp has a **full rotation system so that it can be positioned as desired.**

Dati tecnici • Technical data	Dea Lux
LxA / WxH	450x1400 mm
Diametro / Diameter	Ø 500 mm
Peso / Weight	15 kg
Alimentazione monofase / Single-phase power supply	230 V-ca.
Frequenza / Frequency	50 Hz
Potenza max. assorbita / Max. absorbed power	50 VA
Fusibile / Fuse	F 1 A
Emissione luminosa / Light emission	Dea Lux
Lunghezza d'onda della radiazione Radiation wavelength	495 nm
Potenza luminosa massima irradiata Maximum radiated light power	30.000 mw
Dispositivo medicale / Medical device	Classe / Class: 1 93/42/CEE
Sicurezza elettrica / Electric safety	Classe/Class: 1 Tipo B CEI EN 60601-1
Tipo protezione / Type of protection	PE

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All rights are reserved. Any variations on images and subjects can be done without notice. Tecno-Gaz S.p.A is not to be considered responsible for damages caused by lack or wrong information here mentioned. Product images are only for advertising purposes.

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Tecno- Gaz S.p.A.