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# DEA LUX

Ref. 4460S

**CE**

ITALIANO

ENGLISH

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Declaration of conformity	2
Warranty condition	3
Loss of warranty	4
Delivery terms	5
General warnings and safety rules	6
Useful contacts and addresses	6
Symbols	7
Product description	8
Constructive specifications	10
Positioning	11
Installation	11
Description of machine operation	13
Adoption of personal protective equipment (PPE)	14
Demolition and dismantling	15
Maintenance and troubleshooting	15
Commercial components, spare parts and relevant documentation	16

**DECLARATION OF CONFORMITY**

( according to ISO/IEC 17050-1)

This device compliance to Directive 93/42/EEC .

The original declaration of confirmity is provided in attached to the manual.

**WARRANTY CONDITION**

- 1) **DURATION:** the product is covered by a 12-month (twelve) guarantee.
- 2) **EXCLUSION:** the guarantee excludes:
  - a) faults originating from non-performance of scheduled maintenance and due to negligence or improper use of the product by the User;
  - b) periodic checks and maintenance;
  - c) repair or replacement of parts subject to wear that are fragile or have an unpredictable life, unless they prove faulty at the time of their delivery;
  - d) faults caused by personnel or originating from technical personnel transfer and transportation;
  - e) faults or damage originating from improper use or operating errors;
  - f) faults or damage originating from contamination in the water and air infeed systems, extraordinary chemical agents or electrical events;
  - g) faults or damage originating from the use of detergents, disinfecting agents, sterilizing fluids or processes that are not described explicitly in the instruction manual for operation and maintenance;
  - h) natural color change of plastic components.
- 3) **INSTALLATION AND TESTING REPORT:** an essential prerequisite for guarantee validity is the return of the installation and testing report after all its sections have been filled in by the User and the Fitter. The report must be returned within 15 days after installation, otherwise the guarantee becomes null and void.
- 4) **LIMITS:** the guarantee entitles its holder to repairs or replacement of faulty components free of charge. The right to replace the entire machine is excluded. As for components either applied or added to TECNO-GAZ products or by third parties, having their own guarantee certificate, the conditions, limits and exclusions indicated in the relevant certificates apply.

Under no circumstances shall carrying out of one or more than one repair during the guarantee term change the guarantee expiry date.
- 5) **CLAIMS:** in the event of the Purchaser making a claim relating to the application of the guarantee and the quality or conditions of the delivered equipment, the said Purchaser shall in no way suspend and/or delay payments.
- 6) **DISPUTES:** if a dispute arises on the application and interpretation of this Certificate of guarantee, the Court of PARMA (Italy) shall be the court of jurisdiction, regardless of the location in which the purchase agreement was stipulated.

7) DEROGATIONS: any special derogations from the guarantee terms described in this section shall not imply the granting to the Purchaser of any right and shall be considered as granted for a specific case only.

8) MISCELLANEOUS: for any other issue, which is not covered by this Certificate of Guarantee, the rules described in the Civil Code and existing relevant Laws in the Republic of Italy apply.

## **LOSS OF WARRANTY**

The guarantee becomes null and void in the event that:

- a) the equipment presents damage caused by a fall, exposure to flames, liquid spillage, natural events, bad weather conditions or by any other cause that is not due to manufacturing defects;
- b) installation has not been performed in line with TECNO-GAZ instructions and by authorized personnel;
- c) the equipment is repaired, modified or tampered with by the Purchaser or by non-authorized third parties;
- d) when asking for an intervention under guarantee, the product serial number has been removed, deleted, counterfeited, etc.;
- e) the installation and testing report is not returned suitably filled in and signed within 15 days from installation;
- f) the Purchaser suspends and/or delays payment of amounts due, for any reason whatsoever, relating to the purchase and/or maintenance of the equipment;
- g) scheduled periodic maintenance or any other instruction envisaged in the manual for operation and maintenance is not carried out.

**DELIVERY TERMS**

1. All goods delivered must be packaged in their original packaging. If the goods are delivered in a non-genuine packaging, a charge will be applied to restore the original packaging.
2. All goods must be delivered CARRIAGE PAID.
3. The client shall ask a written authorization to TECNO-GAZ sales dept. before returning the goods by means of the "Form for Client's Returned Goods Management" (CM-M-002) in compliance with the requirements of TECNO-GAZ Quality Management System (ref. CM-P-003, CM-I-009).
4. The document accompanying the returned goods shall show:
  - the TECNO-GAZ product code/item no.;
  - the number of the TECNO-GAZ purchase invoice or shipment document;
  - the product serial number and batch number;
  - a description of the goods returned;
  - the number of the Form for Client's Returned Goods Management" (CM-M-002) or a copy of the same attached.
5. Returned goods are considered as accepted only after they have been examined. If they prove non-conforming, they may be sent back to the consignor at TECNO-GAZ full discretion.
6. Charges will be applied to restore and test the returned product.
7. Parts replaced under guarantee must be returned, carriage paid, to TECNO-GAZ.
8. If the parts are not returned, their cost will automatically be debited to the recipient.
9. TECNO-GAZ does not accept goods returns from end users.
10. Goods sent to TECNO-GAZ for repair are managed as returned goods.

## GENERAL WARNINGS AND SAFETY RULES

-  Make sure that the equipment is supplied with the correct voltage as shown in the plate.

---

-  Do not remove the plate.

---

-  Make sure that the machine is equipped with a grounding system.

---

-  Use a dry cloth to clean the machine.

---

-  Before any operation is performed, disconnect the power cable from the socket.

---

-  Use original spare parts only.

---

*Non-compliance with the rules above relieves TECNO-GAZ of all responsibility.*

## USEFUL CONTACTS AND ADDRESSES

For technical service please apply to the dealer that invoiced the device, or directly to:

TECNO-GAZ SpA

Str. Cavalli n°4 – Sala Baganza – Parma – ITALIA.

Tel. +39 521 8380 - Fax +39 521 833391

e-mail : [info@tecnogaz.com](mailto:info@tecnogaz.com)

<http://www.tecnogaz.com>

## SYMBOLS

Pursuant to DIRECTIVE 2002/96/EC, this symbol shows that the product must not be disposed of as urban waste at the end of its operating life.

General warning and information for the user.

Pay attention to the instructions provided alongside this symbol.

Conformity with the Directive 2004/108/CE and Directive 2006/95/CE and further amendments.

Alternate current.

Warning: high voltage. Risk of electrocution!

Read the instruction manual carefully.

**Fg.**

Figure

**s**

Seconds (time unit).

**W**

Watt (Power unit).

**Hz**

Hertz (Frequency unit).

**mm**

Millimeters (Length unit).

**A**

Ampere (Electric Current unit).

**V**

Volt (Electric Voltage unit).

**Pa**

Pascal (Atmospheric Pressure unit).

**°C**

Degrees Centigrade (Temperature unit).

**kg**

Kilograms (Weight unit).

**PRODUCT DESCRIPTION**

Classification of medical devices pursuant to Directive 93/42/EEC (Annex IX)

- Duration: Temporary
- Invasive devices: Non-invasive
- Active medical device: Active

Due to the above characteristics the device falls under class I (in accordance with Rule 1)

Classification in accordance with general regulations (standard CEI EN 60601)

- According to the type of protection against electrical hazards: Class I equipment
- According to the type of protection against ingress of water: Ordinary equipment
- According to the degree of safety in use in the presence of a flammable anaesthetic mixture with air or oxygen or nitrous oxide: Device not suitable for use with what is described above.

The dental whitening lamp is for use in the odontoiatry.

Dental whitening is the only and sole purpose of device

The lamp emits an intense light radiation.

The operator who uses the device must have the necessary qualifications and experience.

The operator must have received the necessary training in its correct use and must know the position of all the controls:

- for positioning the lamp
- for switching the lamp on and off.

	<p>Read the instructions for use carefully before installing and using the device.</p> <p>Illegitimate use of the device and operator negligence can cause damage to persons, the environment and property; the manufacturer cannot be held responsible for said events.</p> <p>The operator who uses the device must have been correctly trained.</p>
---	--

The dental whitening lamp consists of (fig.1):

- base with 4 castors (A,B), which is a useful support for moving and positioning the device; with rear brake (A).
- A central plastic body that is ergonomic and easy to clean (C).
- Plastic lamp on steel shelf (D)
- Flexible arm, provided with double flexible hose for easy and precise positioning (E,F).
- 2 pairs of protective glasses (for operator and patient).

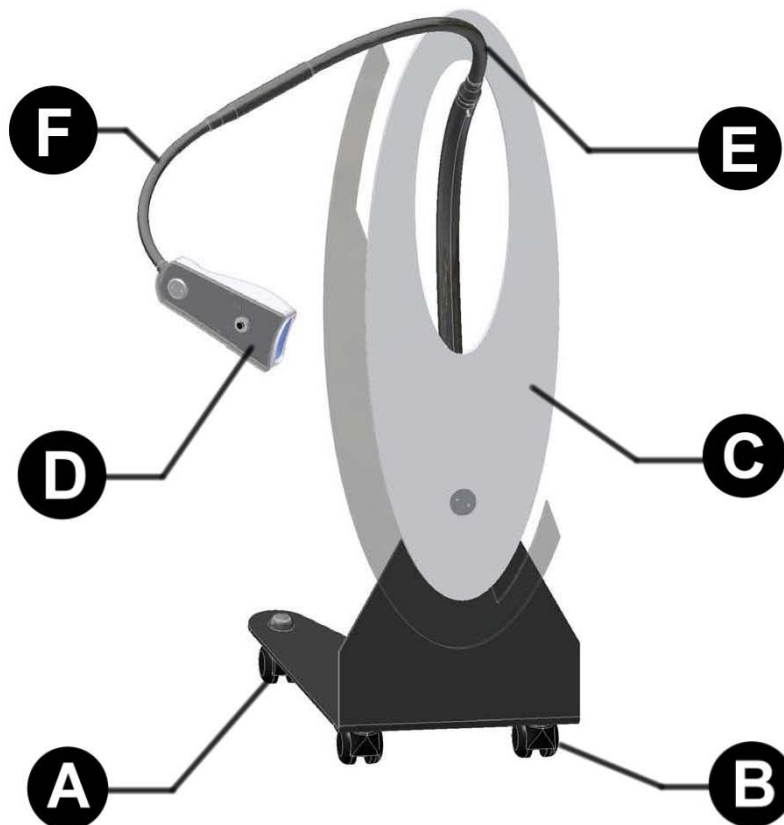
fig.1

CONSTRUCTIVE SPECIFICATION

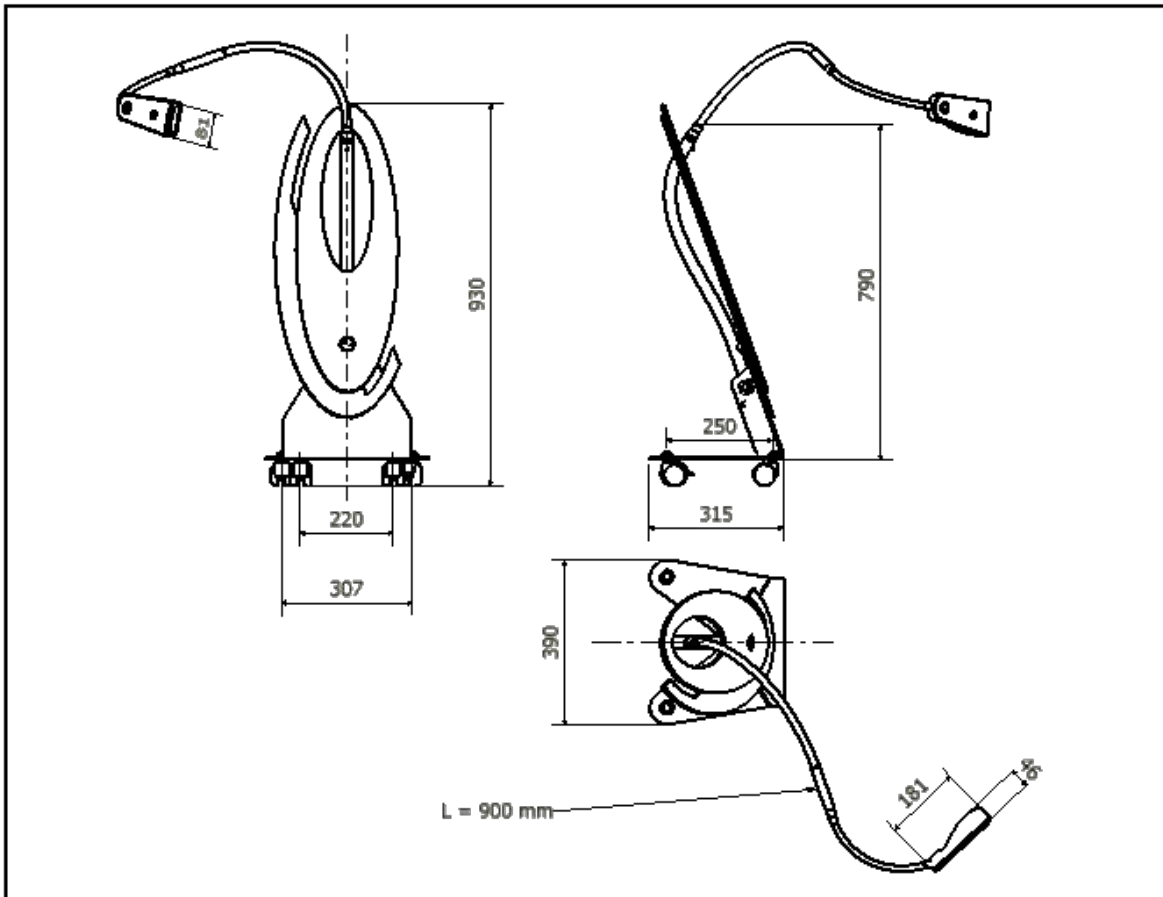
fig.2

<b>Sizes (X,Y,Z)</b>	1000 x 390 x 315 mm
<b>Weight</b>	15 kg
<b>Packaging sizes (X,Y,Z)</b>	1000x480x375 mm
<b>Supply voltage</b>	230V~ ± 10%
<b>Number of phases</b>	1
<b>Supply frequency</b>	50 Hz
<b>Maximum absorbed power</b>	50 W
<b>Fuse</b>	F 1 A
<b>Radiation wavelength</b>	495 nm
<b>Output power</b>	30 W

## POSITIONING

Temperature	- 20 to + 45 °C
Humidity	10 to 100 %
Pressure	500 to 1060 hPa

Before installing, check the integrity of the device; if there are signs of transit damage, please notify the dealer. **WARNING!** Do not switch on the device.

The device must be placed far away from instruments that generate strong heat emissions and electromagnetic radiations that may damage the installed electronics.

Failure to comply with the above will terminate the guarantee.

Disposal of the packaging components must be carried out in compliance with the current regulations in the country where the device is used.

## INSTALLATION (FOUNDATION)

Because of the characteristics of the device is important to follow certain precautions:

**The device should not be used in potentially explosive atmospheres.**

**Remove packaging, and check the status of the unit.**

**If any signs of damage to vehicles, alert the manufacturer.**

**Packaging must be preserved throughout the warranty period.**

**The manufacturer does not accept returns without original packaging.**

The device is put in a cellophane bag and then arranged in a corrugated cardboard box and prevented from moving about with polystyrene foam.

It must be transported following the rules for the transportation of fragile goods.


The original packaging should be kept for the duration of the guarantee period, since only in this container is it possible to avoid damage in transit (fg.3).

**Place the unit on a horizontal surface, away from heat sources.**

**Connect the power cord through the special connector on tap on the rear lamp fig. 4.**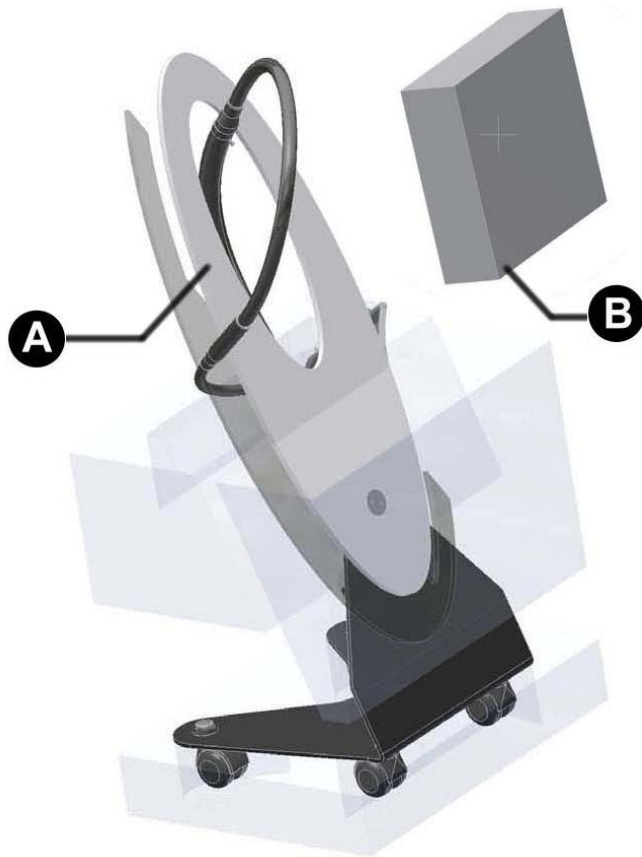

A. 4460S Lamp complete (fig.3)  
(body, arms, instruction booklet,  
accessories for installation)

B. 4465S Kit   
For the bleaching of teeth

fig. 3

**Connection and putting into service:**

The electrical connection must be made in accordance with current technical regulations.

Make sure the measured mains voltage value corresponds to that written on the machine's identification plate.

Make sure the electrical systems are efficiently earthed.


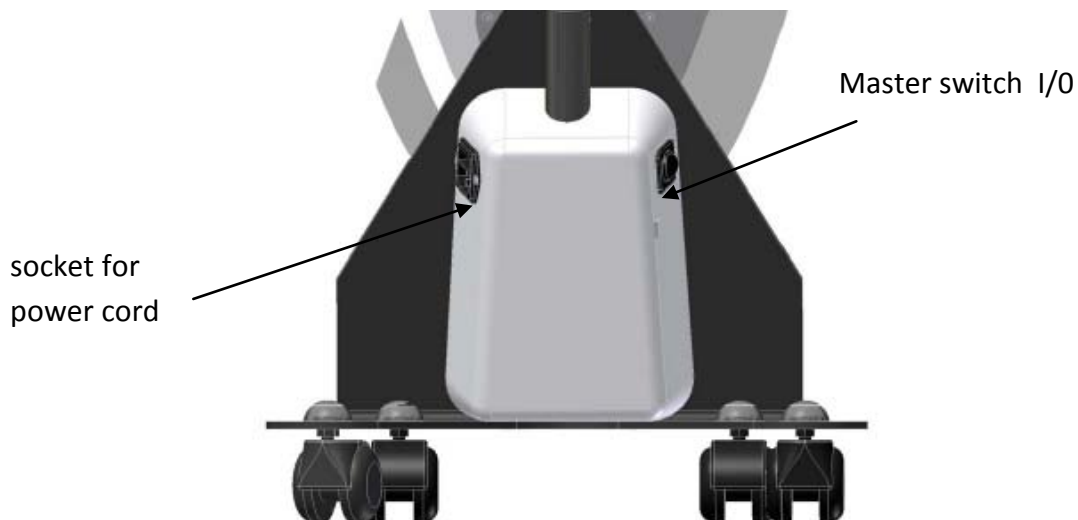
fig.4

**DESCRIPTION OF MACHINE OPERATION**

The lamp is a dental whitening device to be used with manufacturer certified whitening products.

Other uses are not envisaged and are not allowed.

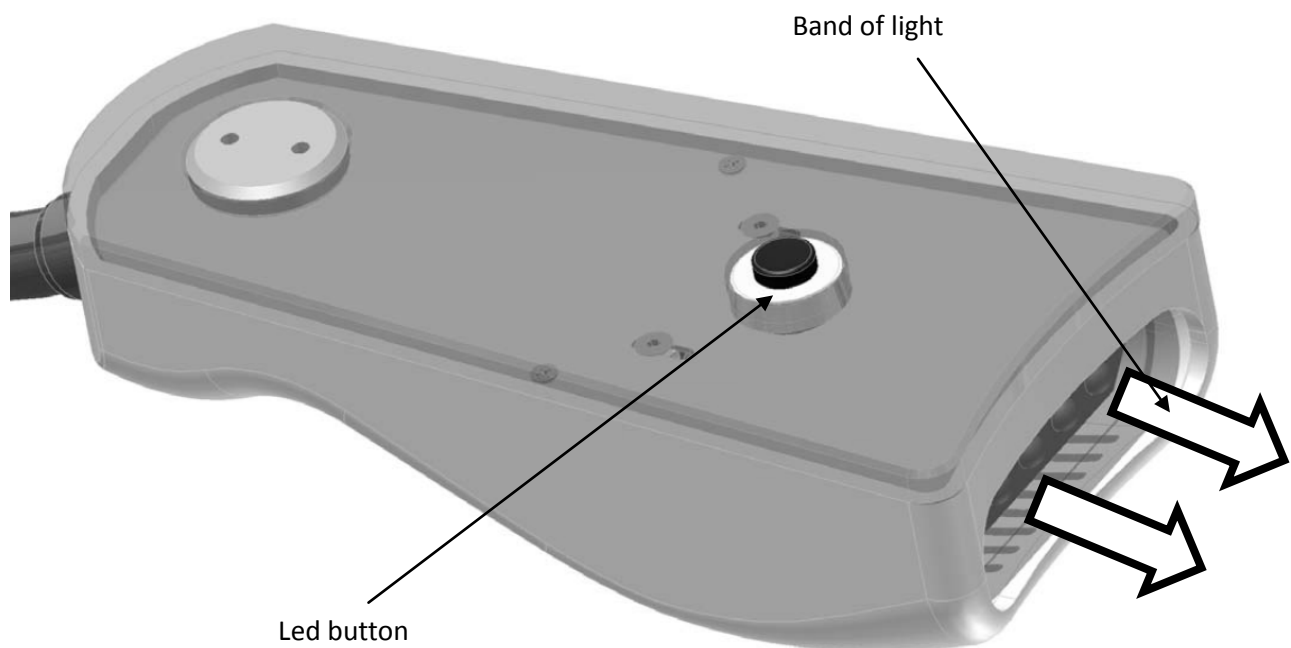**fg.5**

The lamp is a dental whitening device and its innovative LED technology solves the problems of heat emission, light power variations and lamp deterioration.

To obtain maximum efficiency, the light is brought to the outside of the lamp by optical fibres.

The device is easy to use and position and therefore ideal for any work situation.

Position the lamp near the patient's mouth and adjust its height and distance through the flexible hose, so that the mouth is invested by light beam (fg.5).

The lamp should be placed at 2-10 cm from the patient's mouth.

Put the master switch on the side, in position I (fg.4), the device is now ready to be used with whitening product .

- Turn on the lamp through the button on the lamp head: fg.5.
- If you want to stop treatment, press Another time the button on the lamp head: fg.5.
- If nothing is the lamp turns off when the cycle ends and begins a set period (10') of further cooling led off.

Operating and environmental limits:

Operating temperature	5 ÷ 30	°C
Humidity	10 ÷ 90	%
Pressure	500 ÷ 1060	hPa.

## ADOPTION OF PERSONAL PROTECTIVE EQUIPMENT (PPE)

Before switching on the bleaching lamp, the operator, patient, and in general all persons present during the treatment, must put on the special glasses (PPE - personal protective equipment) supplied with the device.

The intensity of the emitted light can cause irreparable damage to the retina and it is therefore necessary to use special PPE (personal protective equipment).

Check that the lamp is positioned correctly.

## DEMOLITION AND ELIMINATION

Directive on waste from electrical and electronic equipment (WEEE)

Pursuant to DIRECTIVE 2002/96/EC, this symbol shows that the product must not be disposed of as urban waste at the end of its operating life.

The product must be taken to centers specialized in separate collection of electric and electronic equipment or to a dealer, when a similar product is purchased.

The machine holder shall be responsible for bringing it to collection centers.

For further information about collection systems, address the local waste disposal service.

**Correct disposal of the equipment, which is no longer used, prevents negative consequences for the environment and human health.**

**Abuses will be prosecuted by law.**

## MAINTENANCE AND TROUBLESHOOTING

The device requires no maintenance, except regular cleaning.

For cleaning, please refer to the health and hygiene regulations.

- Switch off the machine and pull the plug out of the electrical socket.
- If the machine has just been used, waiting for the cooling: shutdown fan after using
- Use a sponge dampened with soapy water and dry with a clean dry cloth.
- Do not use harsh detergents, which could irremediably alter the appearance of the plastic parts of the machine.
- Do not spray directly onto the machine, as this would risk damaging the electrical part inside it.

**DO NOT DO MAINTENANCE OR INSIDE REPAIR TO THE DEVICE.**  
**DISMANTLE THE DEVICE MEANS BREAKING IT.**

<b>Defect</b>	<b>Cause</b>	<b>Intervention</b>
Pushing START button the device does not work	Lack Power	Control plug fit correctly in sockets
	Fuse/s burned	Replace the fuse/s FA1 as tag

### **Diagnostics and troubleshooting**

If the device does not work properly or not at all, contact the manufacturer directly or an authorized service centre.

It is not possible to work directly on the device.

The only work allowed is the following:

- Cleaning and disinfection (MAINTENANCE AND TROUBLESHOOTING)
- Checking the efficiency of the LEDs: Reduction in the light power, as a consequence of LEDs that are not working or power supply problems, reduces the whitening process.

## **COMMERCIAL COMPONENTS, SPARE PARTS AND DOCUMENTATION**

The commercial components that can be supplied directly to the customer are the following:

<b>Withenings DEA LUX products</b>	<b>Cod.4465S</b>
<b>COSMETICBRIGHT</b> Active hydrogen peroxide to 6% Template already filled (single-treatment) Disposable.	<b>Cod. CM41046 (10 p.zi)</b>
<b>PROTECTIVE GLASSES</b>	<b>Cod. CM61046 (2 p.zi)</b>
<b>BLEACHING GEL</b> Booster whitening gel	<b>Cod. CM61102 ( 10 p.zi)</b>
Cleaning Kit <b>KUMAPAN</b>	<b>Cod. CM61100 ( 10 p.zi)</b>

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**GB**

This manual must always be kept with the product, in complying with the Directives of European Community. TECNO-GAZ reserves the right to modify the enclosed document without notice. TECNO-GAZ reserves the property of the document and forbids others to use it or spread it without its approval.

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**TECNO-GAZ S.p.A.**