*MANUALE UTENTE
USER MANUAL
MANUEL DE L'UTILISATEUR
ANWENDERHANDBUCH*

UNIKA

PLUS

Art. 2045-S-2

CE

ITALIANO

ENGLISH

FRANÇAIS

DEUTSCH

ML	2 - 13
ITALIANO	14 - 20
ENGLISH	21 - 27
FRANÇAIS	28 - 34
DEUTSCH	35 - 41

1.	Avvertenze generali e informazioni al destinatario General warning and information for the user Avertissements généraux et informations pour le destinataire Allgemeine Hinweise und Informationen für dem Empfänger
1.1	Dichiarazione di Conformità / Declaration of Conformity / Déclaration de conformité Konformitätserklärung

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Conforms to the legal requirements of the Machines directive 98/37/CE, the Low Voltage Directive 2006/95/CEE, the Electromagnetic Compatibility Directive 2004/108/CEE and their later modifications, amendments and additions.

The original declaration of confirmity is provided in attached to the manual.

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The "UNIKA PLUS" model has been inspected in accordance with the requirements of the directive 98/37/CEE and later modifications and additions.

The fact that this inspection has been made can be seen from the plate on the machine which bears the CE mark shown in the conformity declaration which accompanies this booklet.

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GUARANTEE CONDITIONS

- 1) **DURATION:** the product is covered by a 12-month (twelve) guarantee.
- 2) **EXCLUSION:** the guarantee excludes:
 - a) faults originating from non-performance of scheduled maintenance and due to negligence or improper use of the product by the User;
 - b) periodic checks and maintenance;
 - c) repair or replacement of parts subject to wear that are fragile or have an unpredictable life, unless they prove faulty at the time of their delivery;
 - d) faults caused by personnel or originating from technical personnel transfer and transportation;
 - e) faults or damage originating from improper use or operating errors;
 - f) faults or damage originating from contamination in the water and air infeed systems, extraordinary chemical agents or electrical events;
 - g) faults or damage originating from the use of detergents, disinfecting agents, sterilizing fluids or processes that are not described explicitly in the instruction manual for operation and maintenance;
 - h) natural color change of plastic components.
- 3) **INSTALLATION AND TESTING REPORT:** an essential prerequisite for guarantee validity is the return of the installation and testing report after all its sections have been filled in by the User and the Fitter. The report must be returned within 15 days after installation, otherwise the guarantee becomes null and void.
- 4) **LIMITS:** the guarantee entitles its holder to repairs or replacement of faulty components free of charge. The right to replace the entire machine is excluded. As for components either applied or added to TECNO-GAZ products or by third parties, having their own guarantee certificate, the conditions, limits and exclusions indicated in the relevant certificates apply.
Under no circumstances shall carrying out of one or more than one repair during the guarantee term change the guarantee expiry date.
- 5) **CLAIMS:** in the event of the Purchaser making a claim relating to the application of the guarantee and the quality or conditions of the delivered equipment, the said Purchaser shall in no way suspend and/or delay payments.
- 6) **DISPUTES:** if a dispute arises on the application and interpretation of this Certificate of guarantee, the Court of PARMA (Italy) shall be the court of jurisdiction, regardless of the location in which the purchase agreement was stipulated.
- 7) **DEROGATIONS:** any special derogations from the guarantee terms described in this section shall not imply the granting to the Purchaser of any right and shall be considered as granted for a specific case only.
- 8) **MISCELLANEOUS:** for any other issue, which is not covered by this Certificate of Guarantee, the rules described in the Civil Code and existing relevant Laws in the Republic of Italy apply.

LOSS OF THE GUARANTEE

The guarantee becomes null and void in the event that:

- a) the equipment presents damage caused by a fall, exposure to flames, liquid spillage, natural events, bad weather conditions or by any other cause that is not due to manufacturing defects;
- b) installation has not been performed in line with TECNO-GAZ instructions and by authorized personnel;
- c) the equipment is repaired, modified or tampered with by the Purchaser or by non-authorized third parties;
- d) when asking for an intervention under guarantee, the product serial number has been removed, deleted, counterfeited, etc.;
- e) the installation and testing report is not returned suitably filled in and signed within 15 days from installation;
- f) the Purchaser suspends and/or delays payment of amounts due, for any reason whatsoever, relating to the purchase and/or maintenance of the equipment;
- g) scheduled periodic maintenance or any other instruction envisaged in the manual for operation and maintenance is not carried out.

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DELIVERY TERMS

1. All goods delivered must be packaged in their original packaging. If the goods are delivered in a non-genuine packaging, a charge will be applied to restore the original packaging.
2. All goods must be delivered CARRIAGE PAID.
3. The client shall ask a written authorization to TECNO-GAZ sales dept. before returning the goods by means of the "Form for Client's Returned Goods Management" (CM-M-002) in compliance with the requirements of TECNO-GAZ Quality Management System (ref. CM-P-003, CM-I-009).
4. The document accompanying the returned goods shall show:
 - a. the TECNO-GAZ product ode/item no.;
 - b. the number of the TECNO-GAZ purchase invoice or shipment document;
 - c. the product serial number and batch number;
 - d. a description of the goods returned;
 - e. the number of the Form for Client's Returned Goods Management" (CM-M-002) or a copy of the same attached.
5. Returned goods are considered as accepted only after they have been examined. If they prove non-conforming, they may be sent back to the consignor at TECNO-GAZ full discretion.
6. Charges will be applied to restore and test the returned product.
7. Parts replaced under guarantee must be returned, carriage paid, to TECNO-GAZ.
8. If the parts are not returned, their cost will automatically be debited to the recipient.
9. TECNO-GAZ does not accept goods returns from end users.
10. Goods sent to TECNO-GAZ for repair are managed as returned goods.

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For technical service please apply to the dealer that invoiced the device, or directly to:
TECNO-GAZ SpA
Str. Cavalli n°4 – Sala Baganza–Parma – ITALIA. Tel. +39 521 8380 - Fax +39 521 833391
E-Mail : info@tecnogaz.com
<http://www.tecnogaz.com>

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
Pos. 5

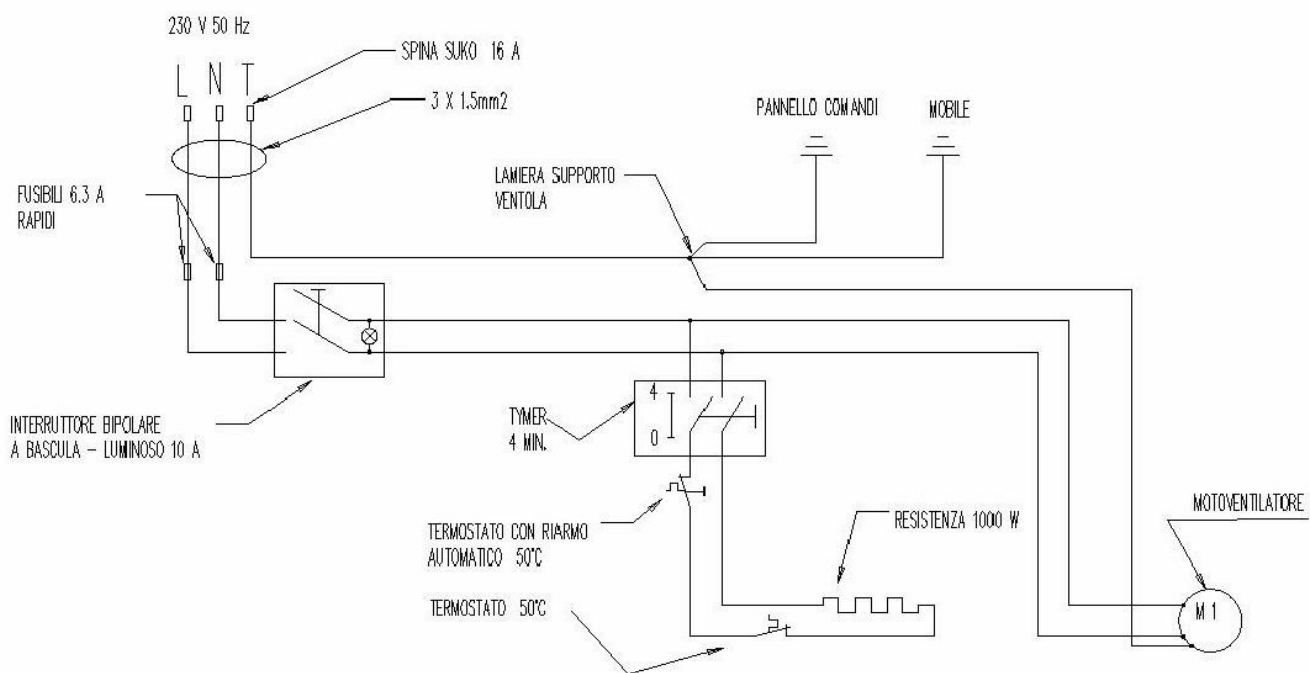FIG. B

INDEX		
Pr.	CHAPTERS	Pg.
1.	Avvertenze generali e informazioni al destinatario	2
1.1	Declaration Of Conformity	2
1.2	Mark	3
1.3	Warranty	4
1.4	Delivery terms	8
1.5	General Warnings And Safety Rules	10
1.6	Useful Contacts And Addresses	11
2.	Organization's Presentation Relating To The Manual Contents And Consultation	12 - 13
2.1	Symbols	12 - 13
3.	Specifications	22
4.	Packaging Transport And Storage	22
5.	Operating Safety Instructions	22
6.	Using The Machine	22
7.	Operating Principle	23
8.	Description Of The Machine	23
9.	Installation	23
10.	Operating Instructions	23
11.	Accessories	24
12.	Potentially Harmful Operations	24
13.	Situations That Could Arise During Operation	24
14.	Ordinary Maintenance	24
15.	Extraordinary Maintenance And Repairs	25
16.	Scrapping And Disposal	25
17.	Summary Of Critical Instructions	25
18.	Spare Parts	26
19.	Picture - Wiring Diagram	27

3	SPECIFICATIONS
----------	-----------------------

MECHANIC:		ELECTRICAL:	
LENGTH	200 mm	GROUNDING SINGLE-PHASE POWER SUPPLY	230 V
HEIGHT	200 mm	FREQUENCY	50 Hz
WIDTH	304 mm	FUSE CURRENT	6 A
WEIGHT	4.5 Kg	INPUT POWER	1 kW
MATERIAL	INOX 430	PROTECTION TYPE	PE
MAX TEMPERATURE	50 °C	CLASS	1

4	PACKAGING TRANSPORT AND STORAGE
----------	--

The device is contained in a plastic bag, then placed inside a corrugated board box and stabilised by means of expanded polystyrene.

It must be transported in compliance with the regulations on fragile goods. No specific problems emerge during its storage; up to three boxes can be piled up and no expiration date is envisaged. The original package must be stored during the whole life of the warranty, since it is the only container that can prevent any damage during transport.

If the instructions above are not complied with, the warranty is void.

The packaging components must be disposed of pursuant to the regulations in force in the country where the machine is used.

5	OPERATING SAFETY INSTRUCTIONS
----------	--------------------------------------

Operating the machine does call for means of personal protection; nevertheless, it must be borne in mind that the machine is equipped with an electric system and that it produces heat during operation.

To avoid injuries or damage, the instructions included in the present manual must be strictly complied with.

6	USING THE MACHINE
----------	--------------------------

The device must be used only for the purpose of drying surgical instrumentarium which has been washed or rinsed in water.

Do not insert flammable material or material that can be altered by heat, objects treated with flammable liquids, nor objects or substances that can produce toxic emissions when exposed to heat.

Any improper use of the machine or negligence on the part of the operator may lead to injuries or damage the environment and objects; in such cases MANUFACTURER does not shoulder any responsibility.

7	OPERATING PRINCIPLE
----------	----------------------------

To dry the instrumentation, the machine generates a flow of hot air that flows on the objects and removes their moisture as vapour.

One fan locate in the top part of the machine convey the air into electric resistance; that heat the air which is blown into the instrumentalism below.

The water remaining on the surface of the instrumentation evaporates and is conveyed outside by the air flow.

8	DESCRIPTION OF THE MACHINE
----------	-----------------------------------

The device dryer is an AISI 430 stainless steel compact case (see picture A, B) supported by four rubber feet. The front includes control panel, with a luminous main scale switch and a mechanic-electrical timer to adjust treatment time (see picture A pos. 1, 2).

The door to the drying chamber is located under the front control panel (see picture A pos. 3).

The air intakes for the motor-fan is on the side of the machine.

The back includes the fuses and the mains lead (see picture B, pos. 5, 6); the moisture discharge grid is in the bottom part.

The drying chamber contains a removable tray, upon which the objects to be dried are laid.

9	INSTALLATION
----------	---------------------

The machine must be placed on a stable flat surface, far from other equipment and not inserted in narrow spaces since the machine produces heat and emits water vapour, albeit in small quantity.

It must be connected to an electric socket supplying 1 KW power and 230 V voltage and equipped with a safety cut-out.

10	OPERATING INSTRUCTIONS
-----------	-------------------------------

Place the objects on the dedicated tray after having washed them with water and let them drip thoroughly. The objects must be spread on the tray net and not be touching or overlapping, to allow the air to flow on their surface, heat them, remove moisture and flow out into the chamber below.

Insert the tray with the objects into the drying chamber, close the door and turn on the main switch (see picture A pos. 1).

Fan starts to work immediately.

Activate the resistance, that heats the air, setting the drying time (max.4 min) with the timer hand-grip (fig.A pos.2).

At the end of the set time, the resistance stops automatically and only the fun will work.

If you switch off the main switch the fan stops too.

Open the door and check if the items are dried.

If the drying time was not sufficient, turn the objects upside down and repeat the operation.

Objects characterized by a cavity retaining water must be dripped thoroughly before undergoing the treatment and disassembled as much as possible before being placed on the tray.

Flat objects tend to remain humid on the surface touching the tray net; therefore, to achieve a correct drying, they need to be turned upside down and undergo a new treatment.

11	ACCESSORIES
-----------	--------------------

The device does not need any specific accessory to be operated; the tray is the only accessory coming with the machine and was designed specifically to achieve an optimal air flow.

12	POTENTIALLY HARMFUL OPERATIONS
-----------	---------------------------------------

As described under par.6, potentially harmful operations are due to negligence or incapacity on the operator's part or improper use of the machine.
Damage may derive also from cleaning and maintenance operations not complying with the instructions described under par. 13, 14; in particular, the mains lead must be disconnected from the electric network each time maintenance operations are carried out.

13	SITUATIONS THAT COULD ARISE DURING OPERATION
-----------	---

Since the machine's principle of operation is very simple, it rarely malfunctions.
When the machine is switched on using the main switch and the switch does not light up, this means that there is no electricity. If the plug is properly inserted, then there is a fuse problem (see fig B pos 5). The fuses should therefore be replaced and, if they blow again, technical assistance should be sought.
When the switch lights up and the heating element does not heat when the timer is activated, this means that either the heating element is damaged or the timer is not working. Switch off the machine and seek technical assistance.

Another situation that could arise is that the heating element no longer switches itself off automatically. This is because the timer is broken. Also in this case, technical assistance service must be sought.

In certain situations the temperature of the machine may rise too high, e.g. during the summer, or if the machine is kept on for long periods, or if it is installed in a tight cabinet with too little space around it to permit sufficient ventilation.

In such cases, the heating element may automatically disconnect from the **thermostat** in order to safeguard against the machine overheating. In this case, the air produced by the machine will be cold.
Since the thermostat has an automatic resetting mechanism, **it resets itself automatically**, after the machine has cooled down. So to restore it to normal operating conditions, simply **let the machine cool down** for a few minutes.

Repairs must be carried out by MANUFACTURER or by authorised personnel. Unauthorised operations are considered as tampering and automatically entail the annulment of the warranty and MANUFACTURER does not shoulder any responsibility for damage following such unauthorised operations.

14	ORDINARY MAINTENANCE
-----------	-----------------------------

Ordinary maintenance consists in normal cleaning operations to be carried out when necessary according to the state of the machine.

Before starting cleaning, the mains lead must be disconnected from the electric supply network.

To clean the tray and the drying chamber, bear in mind that the accumulated dirt is calcareous and therefore calls for passing a sponge soaked with scale remover on the surface, waiting for the limestone to be removed, rinsing with the sponge and finally drying.

The external surface can be simply cleaned by means of a moist cloth; no abrasive agent must be used. Do not flood the machine, nor use water jets; the electrical and mechanic components would be damaged and the operator may be involved in electric shock hazards.

15	EXTRAORDINARY MAINTENANCE AND REPAIRS
-----------	--

The machine does not require any extraordinary or periodical maintenance; it contains no components requiring greasing, nor seals subject to wear.

The electric resistances do not need any maintenance either, unless they are damaged following specific events.

16	SCRAPPING AND DISPOSAL
-----------	-------------------------------

The machine drier is machine fully manufactured in AISI 304 stainless steel and equipped with fully inert electromechanical components.

Scrapping must be entrusted upon firms specialising in the recovery of metals. Do not dump the machine in unattended places or in instable positions.

17	SUMMARY OF CRITICAL INSTRUCTIONS
-----------	---



- 01 STORE THE ORIGINAL PACKAGING.
- 02 USE THE MACHINE ONLY TO DRY HEAT-RESISTANT INSTRUMENTARIUM.
- 03 DO NOT INSERT FLAMMABLE OBJECTS.
- 04 DO NOT INSERT OBJECTS WASHED WITH FLAMMABLE SUBSTANCES.
- 05 DO NOT INSERT OBJECTS WASHED WITH SUBSTANCES WHICH EMIT TOXIC GASES WHEN HEATED.
- 06 PLACE THE MACHINE FAR FROM OTHER APPLIANCES.
- 07 DO NOT INSERT THE MACHINE IN NARROW PLACES (BUILT-IN).
- 08 REMEMBER THAT THE MACHINE PRODUCES WATER VAPOUR.
- 09 DO NOT CLEAN THE MACHINE BY MEANS OF WATER JETS OR BY FLOODING.
- 10 DO NOT TAMPER INSIDE THE MACHINE.
- 11 IF REPAIRS ARE NEEDED, CONTACT THE DELIVERY DEPARTMENT OR MANUFACTURER DIRECTLY
- 12 IN CASE OF SCRAPPING, SEND THE MACHINE TO SPECIALISED FIRMS.
- 13 STORE THE PRESENT MANUAL IN GOOD CONDITIONS, AVAILABLE TO THE OPERATOR.

18	SPARE PARTS
-----------	--------------------


PICTURE REFERENCES, WIRING DIAGRAM AND SPARE PARTS		
POS.	DESCRIPTION	CODE
1	10A LUMINOUS BIPOLAR SCALE SWITCH	CEQ0027
2	4 MIN TIMER	CEQ0033
3	TRAY	SKUA026
4	DOOR	SKUA007
5	3 x 1mm CABLE WITH SHUKO 16A PLUG	CE06008
6	FUSE HOLDER	CEX0047
7	RAPID 6 A FUSES 5x20	CEF0010
8	MOTOR-FAN	CEE0019
9	RESISTANCE 1 kW	CEE0017
10	THERMOSTAT WITH AUTOMATIC RESET 50°C	CEB0011

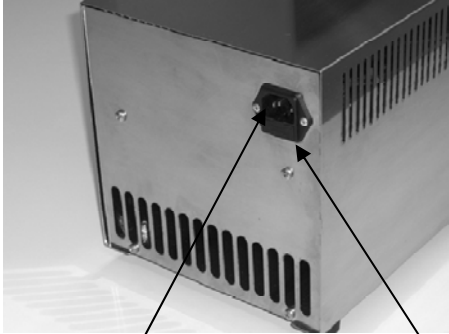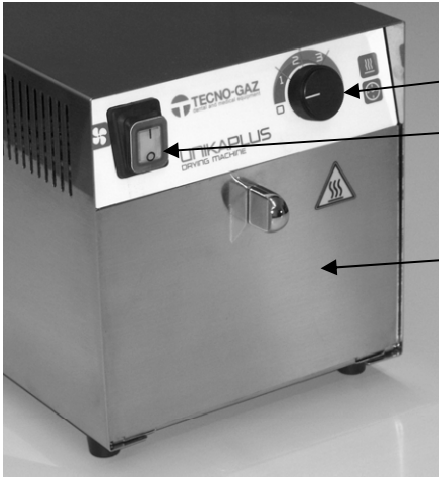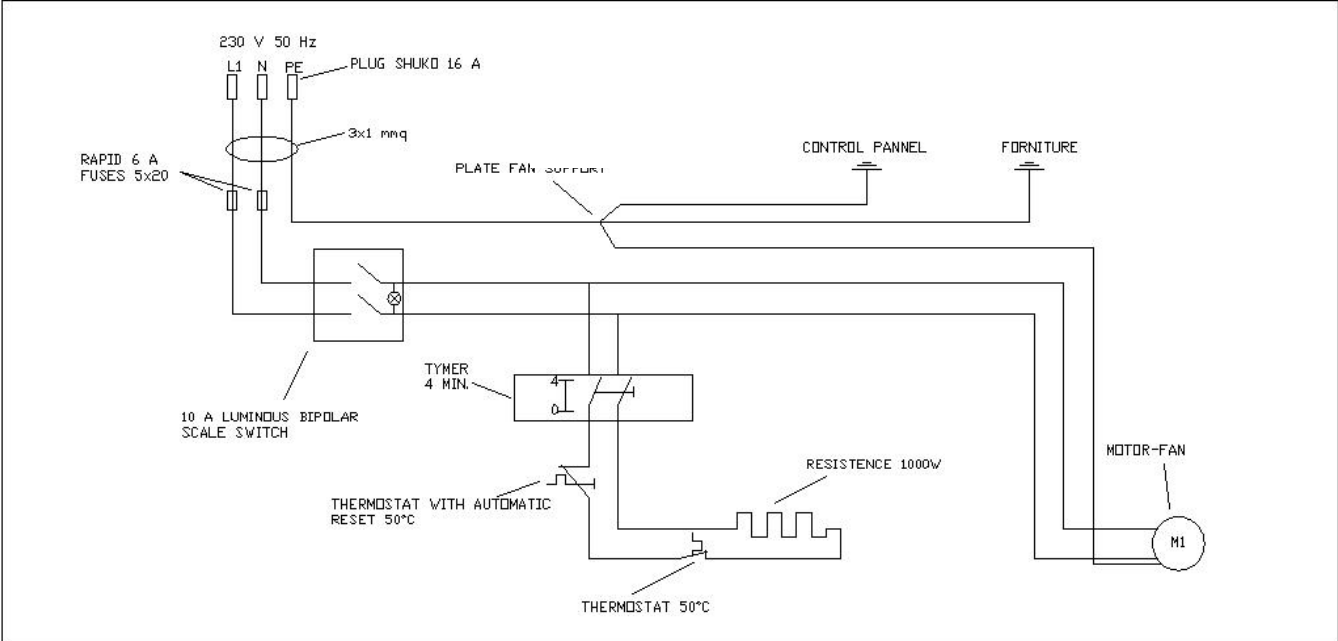19 SPARE PARTS AND PICTURE REFERENCES

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TECNO-GAZ S.p.A.