TECNO-GAZ
i n d u s t r i e s

**MANUALE UTENTE
USER MANUAL
MANUEL DE L'UTILISATEUR
ANWENDERHANDBUCH
MANUAL DEL USUARIO**

B-MAX

Art. 4700S3

ITALIANO

ENGLISH

FRANÇAIS

DEUTCH

ESPAÑOL

MLP0001

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Declaration of conformity	2
Warranty condition	3
Loss of warranty	4
Delivery terms	5
General warnings and safety rules	6
Useful contacts and addresses	6
Symbols	6
Product presentation and scope of product	8
Composition	8
Technical specification	9
Positioning	9
Installation	9
Demolition and dismantling	10
Instructions for the user	11
Adoption of personal protective equipment (PPE)	17
Maintenance and troubleshooting	18
Diagnostics and troubleshooting	19

DECLARATION OF CONFORMITY

This device compliance to Directive 93/42/CEE and modification.

The original declaration of conformity is provided in attached to the manual.

WARRANTY CONDITION

- 1) DURATION: the product is covered by a 12-month (twelve) guarantee.
- 2) EXCLUSION: the guarantee excludes:
 - a) faults originating from non-performance of scheduled maintenance and due to negligence or improper use of the product by the User;
 - b) periodic checks and maintenance;
 - c) repair or replacement of parts subject to wear that are fragile or have an unpredictable life, unless they prove faulty at the time of their delivery;
 - d) faults caused by personnel or originating from technical personnel transfer and transportation;
 - e) faults or damage originating from improper use or operating errors;
 - f) faults or damage originating from contamination in the water and air infeed systems, extraordinary chemical agents or electrical events;
 - g) faults or damage originating from the use of detergents, disinfecting agents, sterilizing fluids or processes that are not described explicitly in the instruction manual for operation and maintenance;
 - h) natural color change of plastic components.
- 3) INSTALLATION AND TESTING REPORT: an essential prerequisite for guarantee validity is the return of the installation and testing report after all its sections have been filled in by the User and the Fitter. The report must be returned within 15 days after installation, otherwise the guarantee becomes null and void.
- 4) LIMITS: the guarantee entitles its holder to repairs or replacement of faulty components free of charge. The right to replace the entire machine is excluded. As for components either applied or added to TECNO-GAZ products or by third parties, having their own guarantee certificate, the conditions, limits and exclusions indicated in the relevant certificates apply.
Under no circumstances shall carrying out of one or more than one repair during the guarantee term change the guarantee expiry date.
- 5) CLAIMS: in the event of the Purchaser making a claim relating to the application of the guarantee and the quality or conditions of the delivered equipment, the said Purchaser shall in no way suspend and/or delay payments.
- 6) DISPUTES: if a dispute arises on the application and interpretation of this Certificate of guarantee, the Court of PARMA (Italy) shall be the court of jurisdiction, regardless of the location in which the purchase agreement was stipulated.
- 7) DEROGATIONS: any special derogations from the guarantee terms described in this section shall not imply the granting to the Purchaser of any right and shall be considered as granted for a specific case only.
- 8) MISCELLANEOUS: for any other issue, which is not covered by this Certificate of Guarantee, the rules described in the Civil Code and existing relevant Laws in the Republic of Italy apply.

LOSS OF WARRANTY

The guarantee becomes null and void in the event that:

- a) the equipment presents damage caused by a fall, exposure to flames, liquid spillage, natural events, bad weather conditions or by any other cause that is not due to manufacturing defects;
- b) installation has not been performed in line with TECNO-GAZ instructions and by authorized personnel;
- c) the equipment is repaired, modified or tampered with by the Purchaser or by non-authorized third parties;
- d) when asking for an intervention under guarantee, the product serial number has been removed, deleted, counterfeited, etc.;
- e) the installation and testing report is not returned suitably filled in and signed within 15 days from installation;
- f) the Purchaser suspends and/or delays payment of amounts due, for any reason whatsoever, relating to the purchase and/or maintenance of the equipment;
- g) scheduled periodic maintenance or any other instruction envisaged in the manual for operation and maintenance is not carried out.

DELIVERY TERMS

1. All goods delivered must be packaged in their original packaging. If the goods are delivered in a non-genuine packaging, a charge will be applied to restore the original packaging.
2. All goods must be delivered CARRIAGE PAID.
3. The client shall ask a written authorization to TECNO-GAZ sales dept. before returning the goods by means of the "Form for Client's Returned Goods Management" (CMM002) in compliance with the requirements of TECNO-GAZ Quality Management System (ref. CMP003, CMI009).
4. The document accompanying the returned goods shall show:
 - the TECNO-GAZ product ode/item no.;
 - the number of the TECNO-GAZ purchase invoice or shipment document;
 - the product serial number and batch number;
 - a description of the goods returned;
 - the number of the Form for Client's Returned Goods Management" (CM-M-002) or a copy of the same attached.
5. Returned goods are considered as accepted only after they have been examined. If they prove non-conforming, they may be sent back to the consignor at TECNO-GAZ full discretion.
6. Charges will be applied to restore and test the returned product.
7. Parts replaced under guarantee must be returned, carriage paid, to TECNO-GAZ.
8. If the parts are not returned, their cost will automatically be debited to the recipient.
9. TECNO-GAZ does not accept goods returns from end users.
10. Goods sent to TECNO-GAZ for repair are managed as returned goods.

GENERAL WARNINGS AND SAFETY RULES

-  Make sure that the equipment is supplied with the correct voltage as shown in the plate.
-  Do not remove the plate.
-  Make sure that the machine is equipped with a grounding system.
-  Use a dry cloth to clean the machine.
-  Before any operation is performed, disconnect the power cable from the socket.
-  Use original spare parts only.

Non-compliance with the rules above relieves TECNO-GAZ of all responsibility.

USEFUL CONTACTS AND ADDRESSES

For technical service please apply to the dealer that invoiced the device, or directly to:
TECNO – GAZ S.p.A.

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Pursuant to DIRECTIVE 2002/96/EC, this symbol shows that the product must not be disposed of as urban waste at the end of its operating life.

General warning and information for the user.

Pay attention to the instructions provided alongside this symbol.

Conformity with the Directive 2004/108/CE and Directive 2006/95/CE and further amendments.

PE

External protection.

Protection GROUNDING SYSTEM.

	(Operating) GROUNDING SYSTEM.
	Alternate current.
	Warning: high voltage. Risk of electrocution!
	Read the instruction manual carefully.
	Applied Part
	Push the button.
POS.	Position (number identifying a component in the figures or diagrams).
Fg.	Figure
Ω	Ohm (Electric Resistance unit).
s	Seconds (time unit).
W	Watt (Power unit).
Hz	Hertz (Frequency unit).
mm	Millimeters (Length unit).
A	Ampere (Electric Current unit).
V	Volt (Electric Voltage unit).
Pa	Pascal (Atmospheric Pressure unit).
°C	Degrees Centigrade (Temperature unit).
kg	Kilograms (Weight unit).

PRODUCT PRESENTATION AND SCOPE OF PRODUCT

Dental whitening lamp " B-MAX " is intended for the dental sector.

The sole purpose of " B-MAX " is to polymerize photohardening dental materials.

The lamp emits intense light radiation.

The operator who uses the " B-MAX " must be suitably qualified to do so.

The operator should have received the necessary training in using the lamp correctly and must be familiar with the position of all the controls:

- for positioning the lamp
- for switching on and off and for the setting functions.

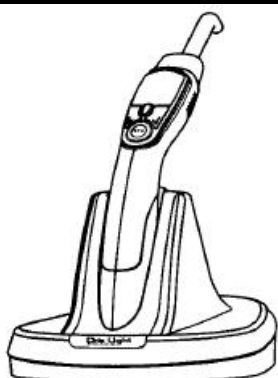
B-MAX should be used only to polymerize photohardening dental materials with photoinitiator in the wavelength band between 410 and 490 nm.

Do not use with other types of material or for applications outside the oral cavity.

If you have any doubts about the type of material, you should contact the manufacturer of the compound.

Illegitimate use of the instrument and operator negligence can cause injury to people and damage to the environment and property; TECNO-GAZ will not be held liable for any such injury or damage.

COMPOSITION

1

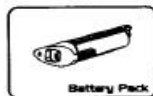
2

3

4

5

6

Fig. 1

B MAX has been made with the most advanced and up-to-date technologies.

B MAX generates a light with a maximum power of 1600mW/cm². The minimum treatment time is 5 seconds.

The dentist can select the most suitable program for his/her treatment.

List of components

1 Hand piece

2 Battery charger stand

3 Power supply unit

4 Battery

5 Optical fibre

6 Protective screen

TECHINICAL SPECIFICATION

HAND PIECE DIMENSIONS	L 182 W 95 H 22 (mm)
STAND DIMENSIONS	L 170 W 180 H 145 (mm)
HAND PIECE WEIGHT	170 gr.
STAND WEIGHT	240 gr.
OPERATING VOLTAGE	6V DC
BATTERY	3.7 V al litio
LIGHT SOURCE	Led ad alta potenza
WAVE FREQUENCY	420 ÷ 490 (nm)
MAXIMUM POWER	1600 mW/cm2 (+/- 10%)

POSITIONING

The components of B-MAX are placed in a cellophane bag and then arranged in a corrugated cardboard box with internal packing.

The device must be transported observing the rules for fragile goods.

There are no particular problems with storage; up to five boxes can be stacked and there is no expiry date for the goods.

The original packaging should be kept for the entire guarantee period, since it is only with this packaging that damage during transport can be avoided.

Failure to comply with the above instructions will terminate the guarantee.

The packaging components should be disposed of in compliance with the current standards in the country where the instrument is used.

INSTALLATION

Before installing the device, check that it is intact; notify the dealer if there are signs of transport damage. **WARNING!** Do not switch on the device.

B MAX must be placed on a stable surface, away from instruments that generate strong emissions of heat and electromagnetic radiation that can damage the installed electronics.

It must be connected to a 230-volt electric socket with ground fault interrupter. After the device has been connected, the LED on the battery charger stand lights up.

Because of the characteristics of the device is important to follow certain precautions:

The device should not be used in potentially explosive atmospheres.

Remove packaging, and check the status of the unit.

If any signs of damage to vehicles, alert the manufacturer.

Packaging must be preserved throughout the warranty period.

The manufacturer does not accept returns without original packaging.

It must be transported following the rules for the transportation of fragile goods. The original packaging should be kept for the duration of the guarantee period, since only in this container it is possible to avoid damages during transport.

Place the unit on a horizontal surface, away from heat sources.

Connection and putting into service:

The electrical connection must be made in accordance with current technical regulations.

Make sure the measured mains voltage value corresponds to that written on the machine's identification plate.

CHARGE THE LAMP WITH BATTERY CONNECTED FOR AT LEAST SIX CONTINUOUS HOURS. FAILURE TO FOLLOW THESE INSTRUCTIONS COULD LIMIT THE PERFORMANCE OF THE LAMP.

DEMOLITION AND DISMANTLING

Directive on waste from electrical and electronic equipment (WEEE)

Pursuant to DIRECTIVE 2002/96/EC, this symbol shows that the product must not be disposed of as urban waste at the end of its operating life.

The product must be taken to centers specialized in separate collection of electric and electronic equipment or to a dealer, when a similar product is purchased.

The machine holder shall be responsible for bringing it to collection centers.

For further information about collection systems, address the local waste disposal service.

Correct disposal of the equipment, which is no longer used, prevents negative consequences for the environment and human health.

Abuses will be prosecuted by law.

INSTRUCTIONS FOR THE USER

ASSEMBLING THE DEVICE

ELECTRICAL CONNECTION

Connect the electrical cables to the transformer.

Insert the electrical plug at the base of the battery charger stand and insert the power plug in the 220 V socket.

The green LED on the stand will light up.

MOUNTING THE BATTERY

Take the hand piece and insert the battery as shown in the illustration, without forcing it in.

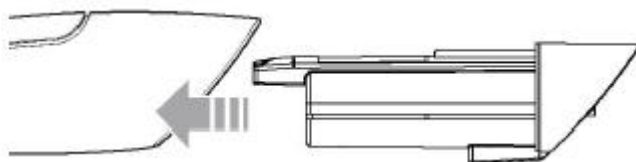
Fig. 2

MOUNTING THE OPTICAL FIBRE

Slowly insert the optical fibre in the hand piece connection, as shown in the drawing, and push the optical fibre fully home.

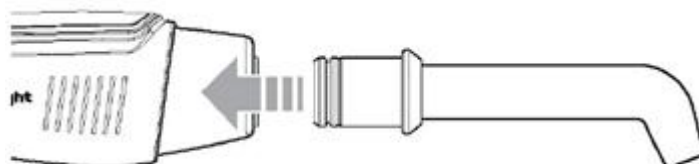
Fig. 3

MOUNTING THE PROTECTIVE SCREEN

Insert the protective screen at the base of the optical fibre to connect it.

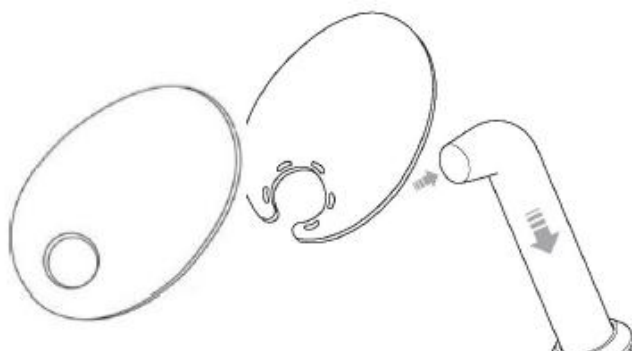
Fig. 4

CHARGING

Insert the electrical plug at the base of the battery charger stand and insert the power plug in the 220 V socket.

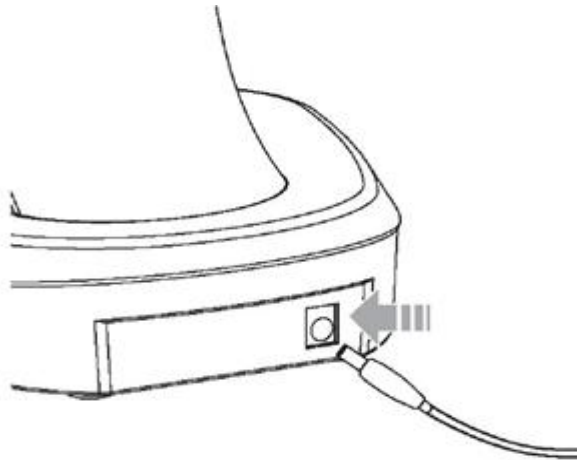
Fig. 5

Insert the hand piece in the special compartment of the battery charger stand, as shown in the illustration.

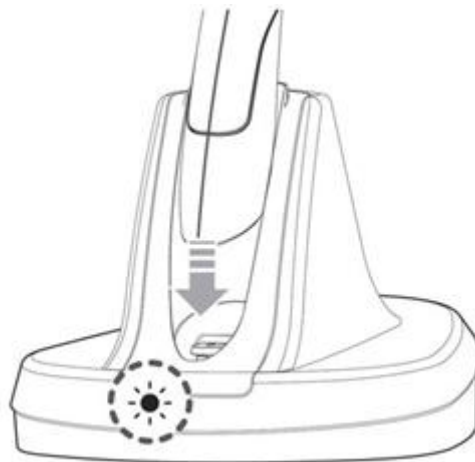
Fig. 6

The LED on the stand will light up automatically.

The LED will be red and will turn green only when the battery is fully charged

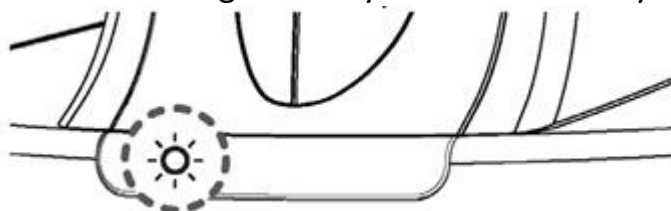
Fig. 7

The lamp emits an intense blue light radiation that interacts with the catalyst present in the polymer and hardens it. The intensity of the emitted light and the treatment time can be varied to modify the speed and degree of polymerization so allowing solidification shrinkage to be limited and the final hardness of the compound to be varied.

LAMP CONTROLS

- 1 TURBO program selection button
- 2 Treatment time selection button
- 3 Program selection button
- 4 Treatment cycle on/off button

Fig. 8

SWITCHING ON THE LAMP

TECNO GAZ DISPLAY - CYCLE DISPLAY DRAWING.

To switch on the lamp, hold the on/off button (4) down for 2 seconds.

The lamp will emit an acoustic signal, the display will light up, the words TECNO GAZ will appear and the lamp will put itself in the mode relevant to the last cycle carried out.

INFORMATION ON THE DISPLAY DRAWINGS

In the off position, the display clearly indicates all the data relevant to the chosen cycle:

- a- chosen program type (displayed for 1 second)
- b- chosen treatment time
- c- battery charge status
- d- abbreviated and fixed indication of the chosen cycle

In the on/operating position, the display clearly shows:

- a- type of program used
- b- remaining treatment time with countdown
- c- battery charge status
- d- progressive graph of the cycle

SELECTING THE PROGRAMS DRAWING

Press the program selection button (3). The programs will be scrolled in sequence. If you hold the button down for more than 1.5 seconds, you will be able to select from 2 types of display setting:

GRAPH MODE

the central part of the display will show the progressive graph, while the time will progressively elapse and the other information will stay fixed in the upper part.

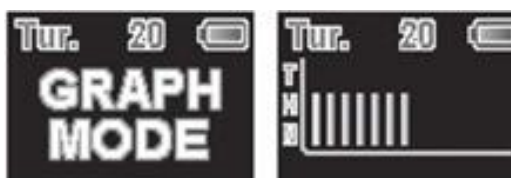
Fig. 9

TEXT MODE

the central part of the display will clearly show the time that will progressively go down, while the information will stay fixed in the upper part.

Fig. 10

BATTERY ICON INFORMATION

The battery icon shows the battery charge status, through a progressive graph that shows the remaining amount.

When the battery is about to go flat, the words LOW BATTERY will appear. Charge it as soon as possible by inserting the hand piece in the special stand already connected to the electrical mains.

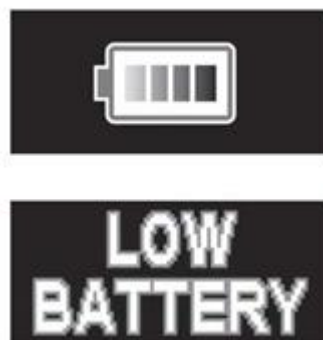
Fig. 11

AUTOMATIC SWITCH-OFF FUNCTION

If the lamp is not used for 4 minutes, it will switch itself off automatically to save battery power.

TYPES OF ACOUSTIC SIGNAL

The lamp has 3 acoustic signals:

- a- Long beep: when the lamp is switched on or off, or when the battery is connected
- b- 2 short beeps: when the treatment is started
- c- 1 short beep: when the cycle parameters are selected, every 10 seconds of use (every 5 seconds for the turbo program).

PRESENTATION OF THE PROGRAMS

The B Max lamp has 6 programs that can be selected and activated according to the operator's requirements.

TURBO MODE

This program can be selected only through the relevant button (1).

The program must be used when you need immediate maximum power.

The emitted power is about
1600mW/cm²

The time selection can be 5-10-20
seconds

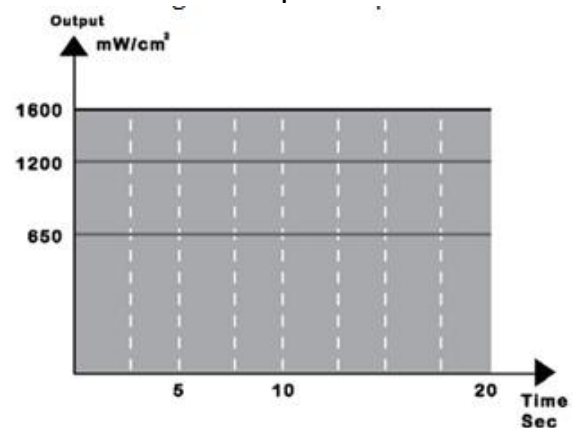
Fig. 12

NORMAL MODE

This program can be selected using the program selection button (3).

The program emits a power of about
650mW/cm²

The time selection can be 10-20-30
seconds

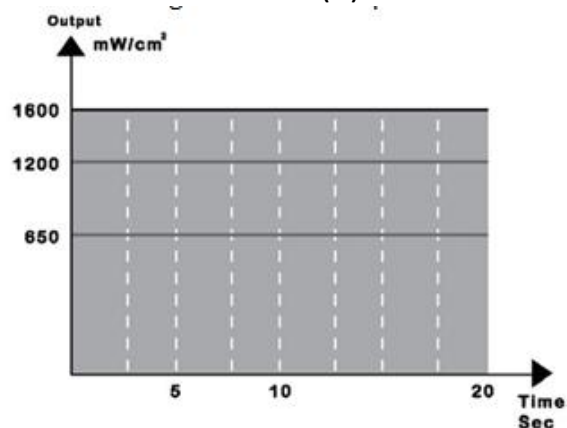
Fig. 13

HIGH MODE

This program can be selected using the program selection button (3).

The program emits a power of about 1200mW/cm²
The time selection can be 10-20-30 seconds

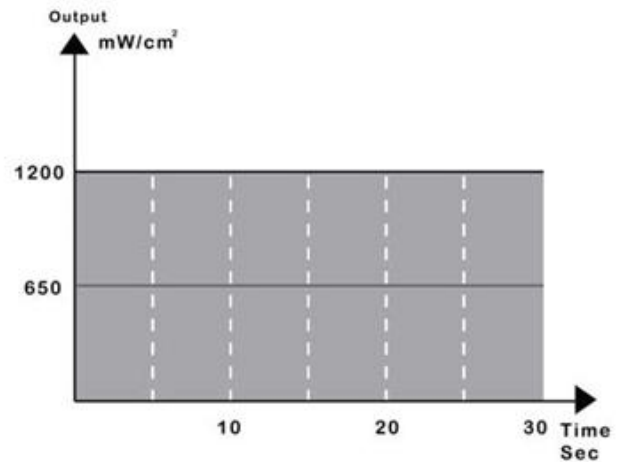
Fig. 14

SOFT START MODE

This program can be selected using the program selection button (3).

The program has a progressive course for the first 5 seconds. The power goes from 0mW up to 650 mW/cm², while during the remaining 15 seconds the power stabilizes at 1200mW/cm².
The time selection is set to 20 seconds.

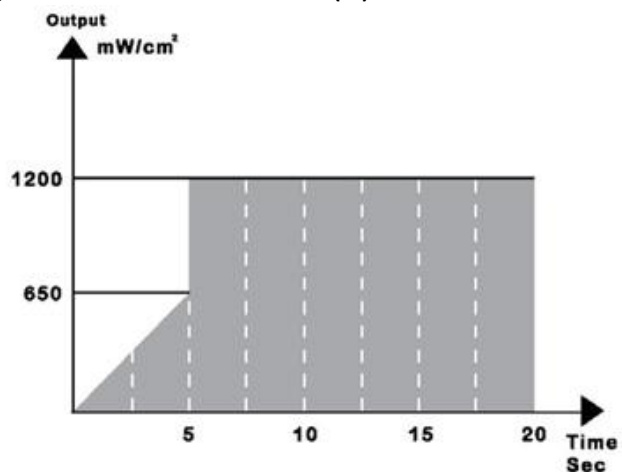
Fig. 15

PULSE SOFT MODE

This program can be selected using the program selection button (3).

The program has a progressive course for the first 5 seconds. The power goes from 150mW up to 1200mW/cm², while during the remaining 15 seconds the power stabilizes at 1200mW/cm².
The time selection is set to 20 seconds.

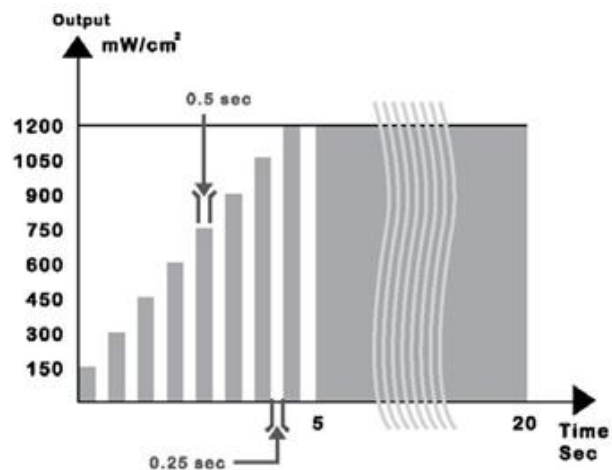
Fig. 16

PULSE MODE

This program can be selected using the program selection button (3).

The emitted power is 1200mW/cm² for a period of 0.5 seconds that is repeated every 0.25 seconds. The time selection can be 5-10-20 seconds.

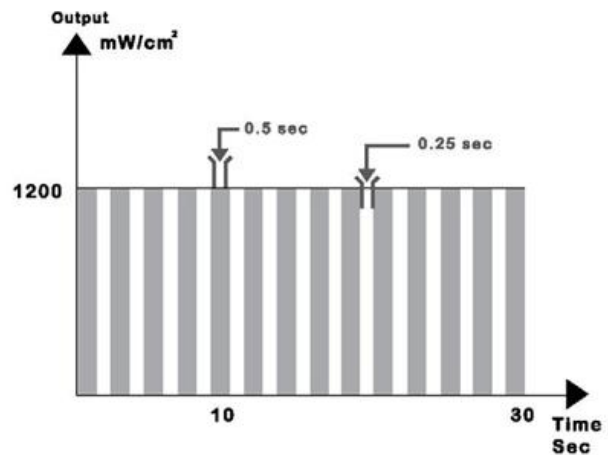
Fig. 17

ADOPTION OF PERSONAL PROTECTIVE EQUIPMENT (PPE)

INSTRUCTIONS FOR THE SAFETY OF THE OPERATOR AND THE PATIENT

The intensity of the light emitted by the polymerizing lamp can cause irreparable damage to the retina and it is therefore necessary to wear suitable P.P.E. (personal protective equipment) and observe some precautions for use:

- Never look directly at the light source.
- Never point the lamp at the patient's eyes or those of any other person.
- Do not switch on the lamp until you have positioned it in the patient's mouth.
- Do not switch on the lamp without light guide.
- Put the switch in the "OFF" position when carrying out maintenance and/or clearing work.
- The lamp must be used exclusively by trained personnel.
- Keep out of the reach of children.
- Put the protective cap on the light guide whenever you position the hand piece on the power supply unit.

In order to avoid injury to people and damage to property, the instructions contained in this booklet must be followed scrupulously.

MAINTENANCE AND TROUBLESHOOTING

ROUTINE MAINTENANCE AND CLEANING:

Routine maintenance consists of normal cleaning operations to be carried out when required.

Before starting to clean the lamp, you must unplug it from the power supply mains. To avoid the risk of an electric shock, remove the battery from the hand piece.

- Do not touch the electrical contacts of the charging station or the hand piece with metal or greasy parts.
- Do not use non-original charging devices; they could damage the accumulator cells.
- Do not immerse the hand piece in water or any other liquid.

Clean the seat of the light guide gently and carefully using only a soft dry cloth.

Do not insert objects or materials in the seat of the light guide because they may damage the LED.

CLEANING THE LIGHT GUIDE:

- Clean the optical fibre regularly with a soft cloth. Remove the patina from the ends, in particular before and after steam sterilization.
 - Remove the polymerized compounds with alcohol and, if necessary, use a plastic spatula. To avoid scratching the surface, do not use sharp or pointed objects.
- Before inserting the light guide, make sure the ferrule and its end are perfectly clean and dry.

Immediately replace the optical fibre if, upon visual examination, it is found to be scratched or damaged since the light intensity will be greatly reduced.

CLEANING THE CHARGING STAND AND THE HAND PIECE:

- To disinfect all parts of the instrument, spray disinfectant on a cloth and wipe it over the instrument. Do not allow the disinfectant to seep into the instrument.
- Wipe the handpiece and charger with a standard aldehyde-free disinfecting solution. Do not clean with highly aggressive solutions (e.g. solutions containing orange oil or solutions with an ethanol content of more than 50%), solvents (e.g. acetone), or pointed instruments, which damage or scratch the plastic.
- **WARNING!** Abrasive or harsh solvents or detergents should never be used, since these substances damage surfaces.

EXTRAORDINARY MAINTENANCE AND REPAIRS

B-MAX does not require extraordinary or periodic maintenance; it does not have mechanical parts or gaskets subject to wear.

Any other type of maintenance work that is not explicitly envisaged is to be carried out only by TECNO-GAZ or by authorized personnel.

Unauthorized maintenance work is considered tampering and will automatically terminate the guarantee and the non-liability of TECNO-GAZ for damaging events consequent to the said maintenance work.

DIAGNOSTICS AND TROUBLESHOOTING

The CONTROL button (3) is not working properly.	If the button is pressed too forcefully, the lamp could switch itself off and not work properly.	To solve the problem, switch the lamp off and then on again to reset it. If the fault situation continues, recharge the battery. Press the buttons gently.
The display does not come on.	The battery may be flat or there may be problems with the battery connection.	Recharge the battery or take it out, clean the contacts and put it back in its compartment.
The lamp switches itself off during use.	The lamp is not charged, or the start button (4) has been pressed accidentally for more than 2 seconds.	
The intensity of radiation is reduced.	The optical fibre has become opaque, or has deteriorated and black streaks can be seen. The battery is very flat.	Replace the optical fibre. Recharge the battery.

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TECNO-GAZ S.p.A.