Istruzioni per l'uso
Instructions for use
Mode d'emploi
Instrucciones para el uso

NEGATIVOSCOPIO

PAR. 01

DICHIARAZIONE DI CONFORMITÀ – CONFORMITY STATEMENT DECLARATION DE CONFORMITE - DECLARACIÓN DE CONFORMIDAD

- La sottoscritta - The undersigned - La soussignée - La que suscribe:
TECNO-GAZ S.p.A., Strada Cavalli n. 4, 43038 Sala Baganza, Parma, Italia.
- Dichiaro sotto la propria responsabilità che il NEGATIVOSCOPIO 30X40
- Declares under its own responsibility that the NEGATOSCOPE 30X40
- Déclare sous sa propre responsabilité que le NEGATIVOSCOPE 30X40
- Declara bajo su propia responsabilidad que el NEGATIVOSCOPIO 30X40

Modello – Model – Modèle - Modelo: **CAMPO VISIVO 30X40 – FIELD OF VISION 30X40
CHAMP VISUEL 30X40 – CAMPO VISUAL 30X40**

Articolo – Item – Article - Artículo: **7085/N**

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WARRANTY

This equipment is covered by warranty for twelve months, with the exception of electric components, which are covered by warranty for six months.

This warranty starts on delivery of the equipment to the customer, as proved by the warranty coupon which has to be returned duly filled, stamped and signed by the seller.

In case of controversy, the date shown on the sales note will be applied. Warranty repair or replacement of a part is at the manufacturer's unquestionable discretion and does not include personnel's transfer and package and transport freight. Lamps, fuses, glassware and damages or breakdowns resulting from poor maintenance, negligence, unskilful use or other causes not due to the seller will be excluded from the warranty.

Components subject to normal wear and tear and fittings are excluded from the warranty.

The manufacturer does not acknowledge the right of replacement of the complete equipment.

This warranty does not imply any compensation for direct or indirect damages of any kind to people or objects due to equipment inefficiency. No compensation will be paid due to machine outage.

This warranty will automatically void if the equipment is tampered with, repaired or altered by the buyer or any unauthorised third parties. When servicing is required, the buyer shall only apply to the seller or customer services indicated by the seller. Warranty replaced components shall be returned to Tecno Gaz free of carriage.

If components are not returned, their cost will be charged to the applicant.

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TABLE OF SECTIONS AND ILLUSTRATIONS

01 CONFORMITY STATEMENT.....	1
02 MARKS	2
03 WARRANTY	3
04 TECHNICAL FEATURES.....	14
05 INSTRUCTIONS FOR PACKAGING, TRANSPORT AND STORAGE	14
06 USE OF EQUIPMENT	15
07 DESCRIPTION OF EQUIPMENT.....	15
08 INSTRUCTIONS FOR USE	16
09 STANDARD MAINTENANCE	17
10 FIGURE No. 5 “Exploded”	17
11 FIGURE No. 6 “Electric system”	18
12 LIST OF SPARE PARTS AND FITTINGS	19
13 INSTRUCTIONS FOR SCRAPPING	19

WARNING

TECNO GAZ reserves the right to make technical changes to improve the machine without giving any previous notice thereof.

TECNO GAZ reserves the property of this manual and forbids its use or distribution to third parties without its consent.

PAR. 04**TECHNICAL FEATURES**

MECHANICAL FEATURES	
WIDTH	460 mm
HEIGHT	55÷95 mm
LENGTH	380 mm
WEIGHT	4 kg
CASING MATERIAL	ABS
LOWER PLANE MATERIAL	AISI 304 STAINLESS STEEL

ELECTRICAL FEATURES	
SUPPLY VOLTAGE	220-230 V AC
FREQUENCY	50 HZ
PHASES	1 + NEUTRAL
FUSE CURRENT	1 A
TYPE OF PROTECTION	PE
FEEDING CABLE	L=2m, 2x1 sqmm+earthing
CONNECTOR	10÷16A+earthing

PAR. 05**INSTRUCTIONS FOR PACKAGING, TRANSPORT AND STORAGE**

The equipment has a light weight and moderate volume, so it does not need any special package.

It is protected by a cellophane bag, then wrapped up in a polystyrene foam, placed into a corrugated cardboard box and sealed with adhesive tape.

Do not shake or bump the package during transport and do not expose it to moisture.

WARNING

Do not throw the packaging material into the environment, recover recyclable materials and give the residues for disposal to a specialised firm.

The packaged products shall be stored in dry places at a temperature between -10°C and +40°C.

DO NOT STACK MORE THAN 5 UNITS ON EACH OTHER.

PAR. 06

USE OF EQUIPMENT

The equipment art. 7085/N can be used only in dentists' offices and dental mechanic laboratories.


Dentists will use the negatoscope to check X-rays or other related negatives.

TECNO GAZ is not responsible for improper use of the equipment and for uses other than those above-mentioned.

PAR. 07

DESCRIPTION OF EQUIPMENT

7085/N negatoscope 30x40 is an elegant equipment, made of shock-resistant plastic material, with 300mmx400mm 5,5°-inclined actual visual area.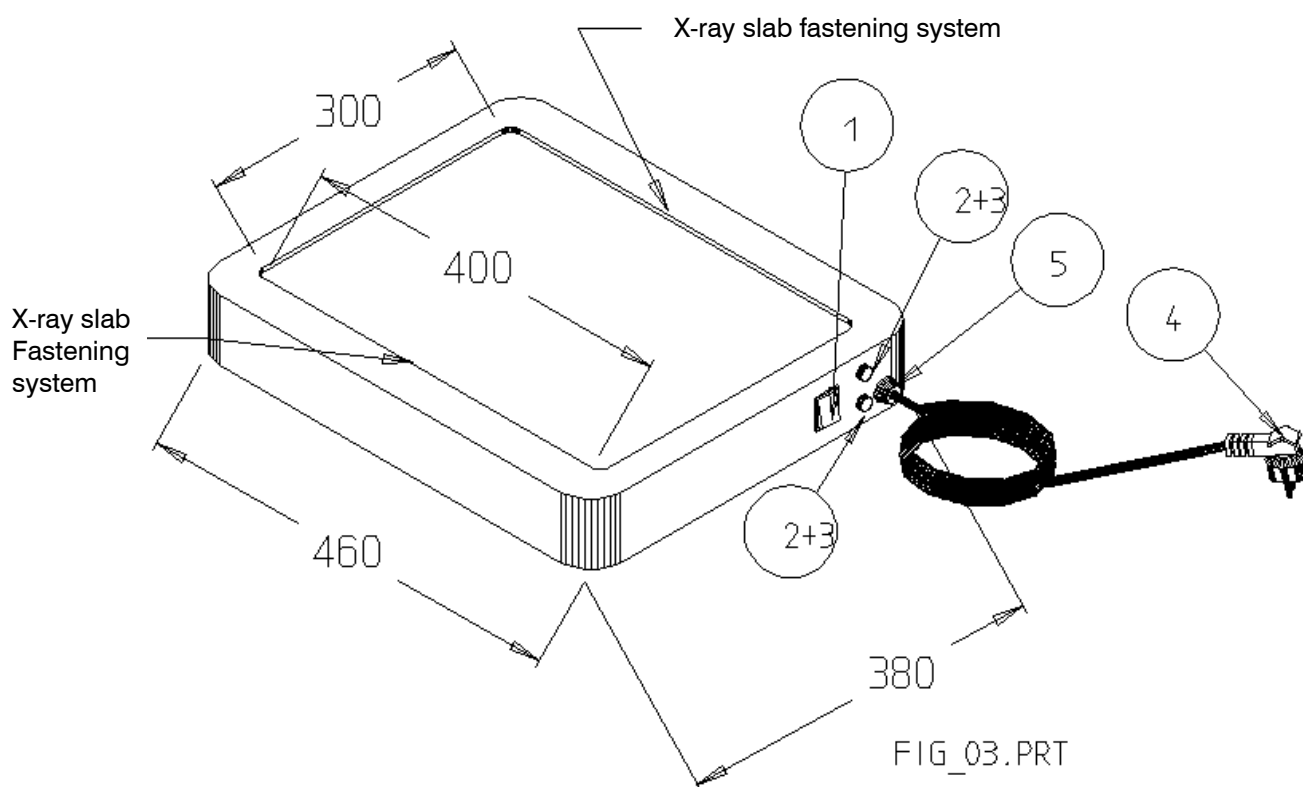

POS.1-FIG.03-05 luminous general switch

POS.2-FIG.03-05 fuse carrier

POS.3-FIG.03-05 fuses

POS.4-FIG.03-05 feeding cable with vulcanised plug

POS.5-FIG.05 cable presser

PAR. 08


INSTRUCTIONS FOR USE

7085/N negatoscope 30x40 shall be connected to a single-phase electric socket with voltage $220V \pm 10\%$, frequency 50 Hz.

The socket shall be provided with earthing protection, magneto-thermal circuit breaker with differential (life protection).

Do not place extension, interfaces, etc. between the equipment plug and the electric system socket, as these alterations can adversely affect equipment safety characteristics.

7085/N negatoscope 30x40 can be used on desk top or hanged on walls. On desk tops, it rests on four rubber feet provided; to be hanged on walls, the operator shall use two wall nogs with more than 4 kg capacity load (weight of equipment) to be fixed on the wall at the distance between centers, as shown in fig. 04 below.

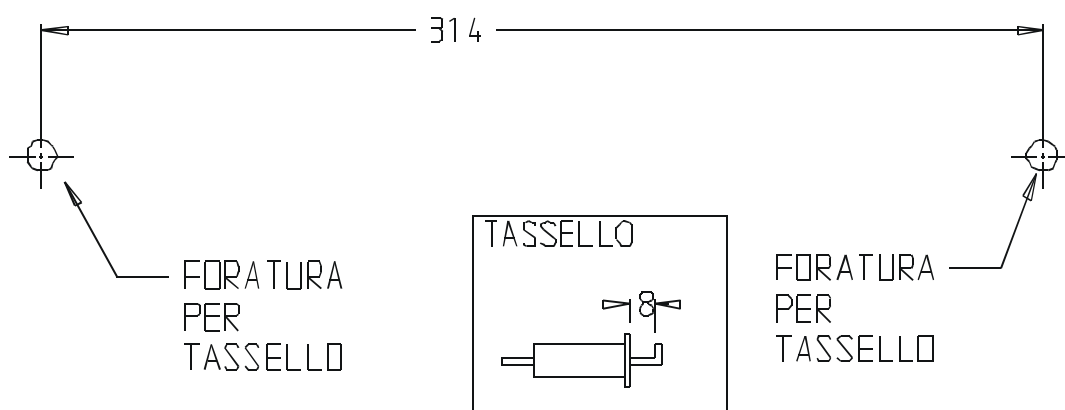FIG_04.PRT

The negatoscope is provided with two holes in its lower plane with distance between centers like those shown in the previous figure to be hanged to the nogs.

7085/N negatoscope 30x40 is provided with negative fastening systems, placed in the two longer sides, as shown in figure 1.

Always switch off the negatoscope after use.

PAR. 09


STANDARD MAINTENANCE

Do not use solvents to clean the equipment, use a damp tissue with equipment and plug switched off.

Maintenance have to be performed by TECNO GAZ qualified technicians, with negatoscope switched off and unplugged.


PAR. 10

FIGURE No. 5 “Exploded”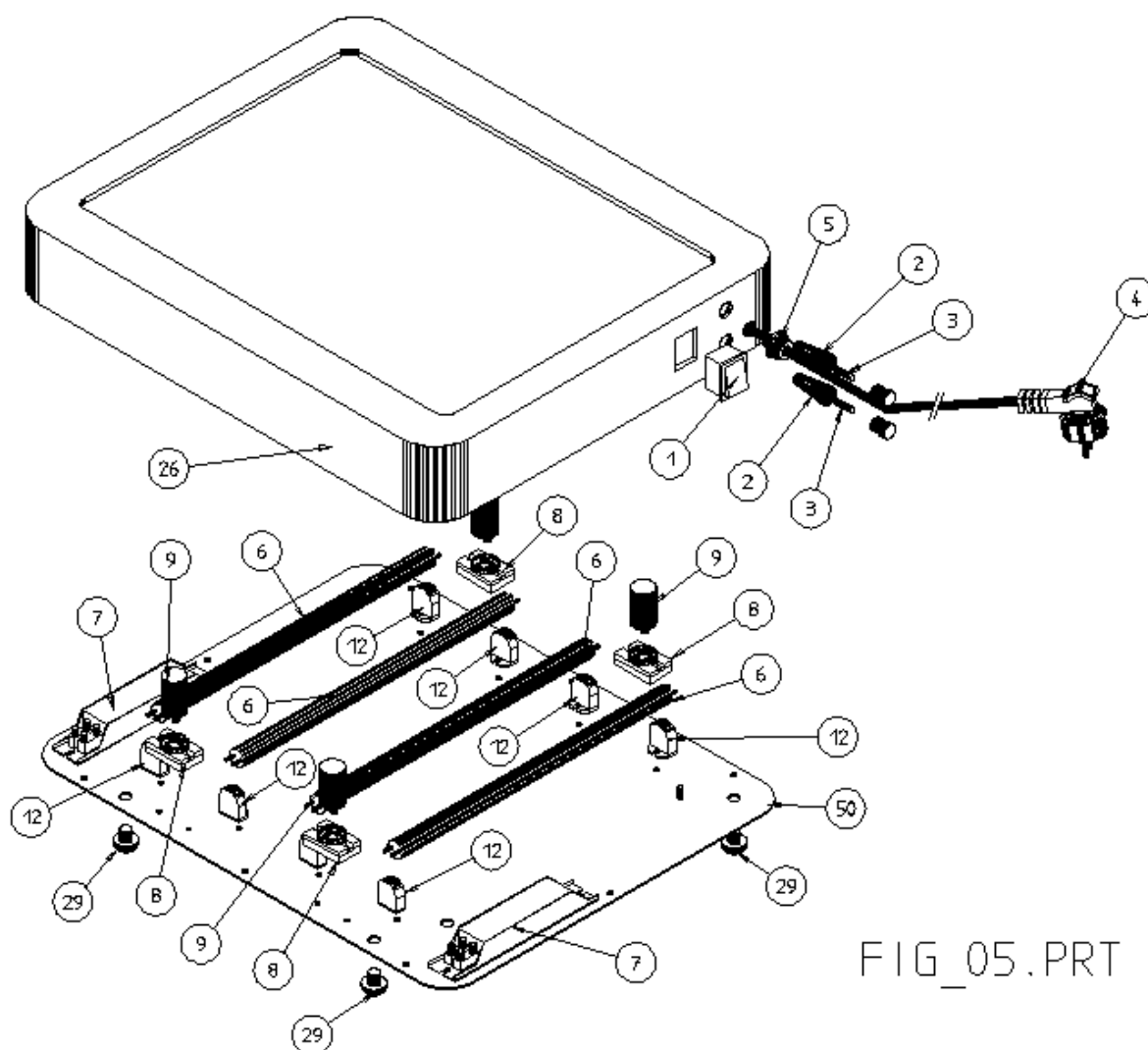

PAR. 11

FIGURE No. 6 “Electric system”

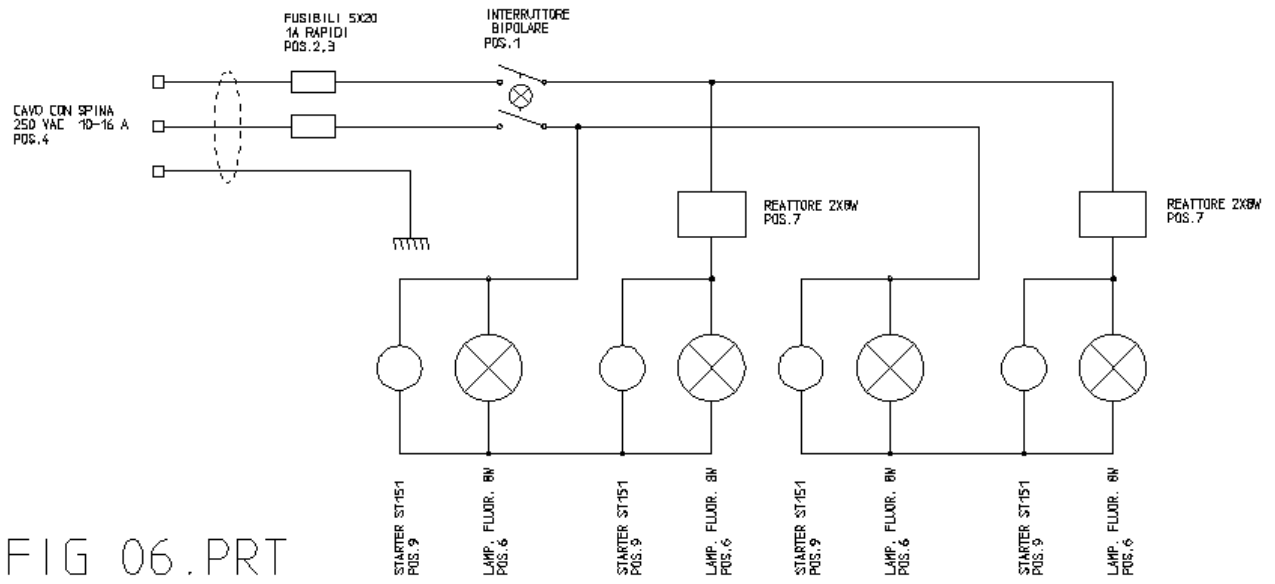PAR. 12

LIST OF SPARE PARTS AND FITTINGS

POS.	Q.ty	ITEM	CODE
1	1	Bipolar switch	CEQ0002
2	2	Fuse carrier	CEF0004
3	2	Fast fuses 1A 5x20	CEF0005
4	1	Cable with vulcanised plug	CE06003
5	1	Cable presser	CE02022
6	4	8W Tubular fluorescent lamps	CEV0005
7	2	2x9W Reactor	CEL0006
8	4	Lamp carrier / Starter	CEE0008
9	4	Starter	CEL0004
12	8	Small lamp carriers	CEE0006
26	1	Complete casing	SNEA051
29	4	Rubber feet	CM70003

REQUESTS FOR SPARE PARTS OR FITTINGS SHALL BE TRANSMITTED TO TECNO GAZ DISTRIBUTORS.

Always indicate on your application for spare parts:

- 1. NAME OF EQUIPMENT**
- 2. SALE CODE**
- 3. SERIAL NUMBER**
- 4. AMOUNT REQUIRED**
- 5. DESCRIPTION OF PART**
- 6. CODE (SEE MANUAL FOR REFERENCE)**
- 7. INSTRUCTIONS FOR SHIPMENT**

PAR. 13

INSTRUCTIONS FOR SCRAPPING

The equipment is made of recyclable ferrous materials, plastics and electronic components to be delivered to authorised companies for disposal.

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