


## USE CLEAN IOT SERVICE


*The following document replaces section 12 of the user manual.*

Activate the product guarantee or perform the installation via the 'TECNO-GAZ' mobile app.

Log on to the web page: **s.tecnogaz.com/clean/** via the browser address bar.

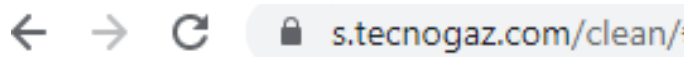

The following screen will then open:

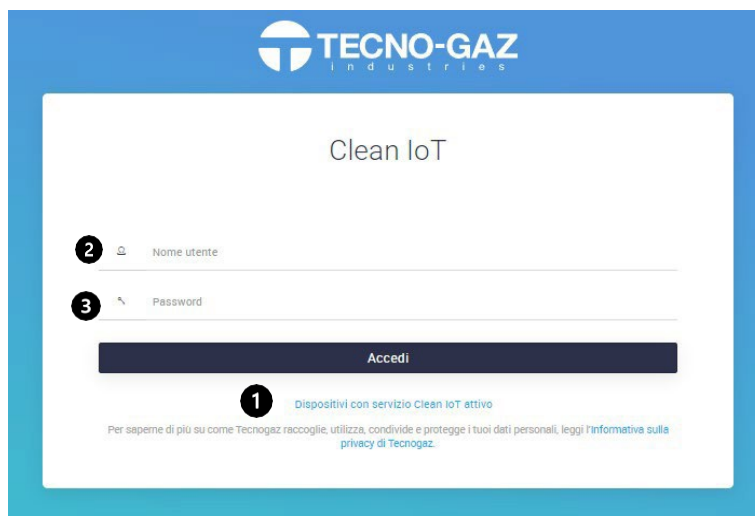

### Clean IoT service to activate

If the Clean IoT service is not activated, click “Request activation of Clean IoT service” (❶) and follow the instructions.

When all the documentation is approved and TECNO-GAZ receives what is required, the Clean IoT service will be activated.


The service cannot be activated without the completion of the documentation to be accepted and uploaded to the portal.




## Clean IoT activated

Enter Username (2) and password (3)

Once the service has been activated, the following screen will appear after logging in.

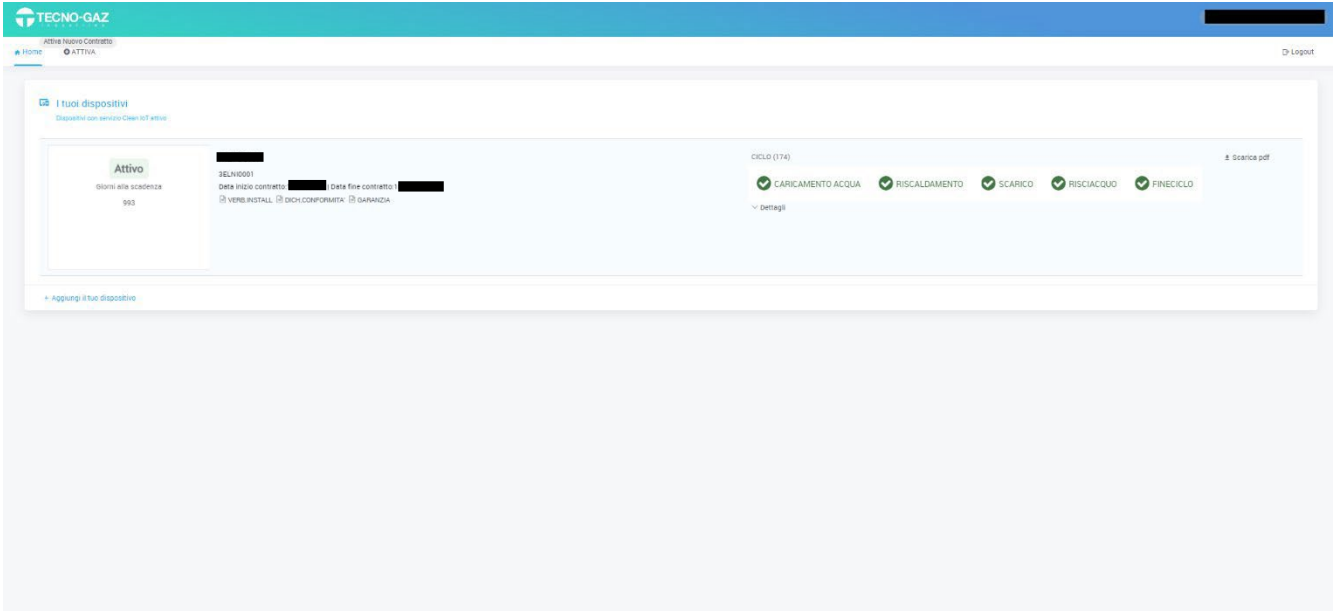

Select "Calendar" to view cycles performed.

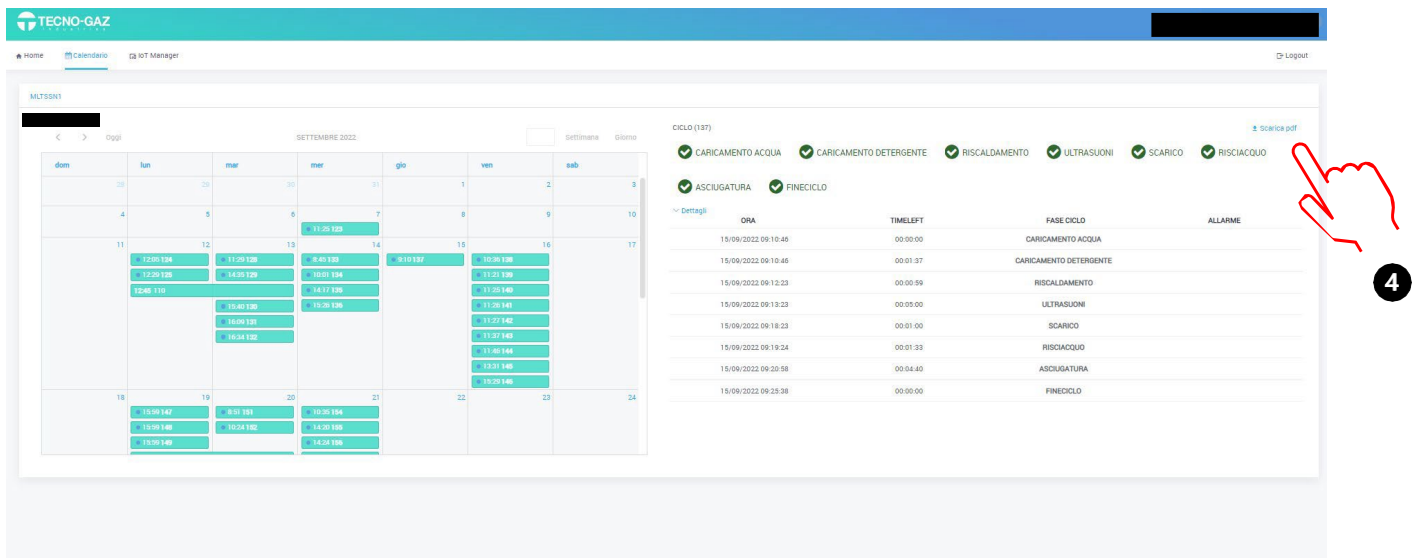

By selecting a certain cycle within a certain day, you can view the course of the characteristic quantities on the right-hand side. By selecting 'Download PDF', you can download the cycle file in PDF format (4).