*MANUALE UTENTE
USER MANUAL
MANUEL DE L'UTILISATEUR
ANWENDERHANDBUCH
MANUAL DEL USUARIO*

PEARL

Art. 4450KS

ITALIANO

ENGLISH

FRANÇAIS

DEUTCH

ESPAÑOL

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| | |
|---|----|
| Declaration of conformity | 2 |
| Warranty condition | 3 |
| Loss of warranty | 4 |
| Delivery terms | 5 |
| General warnings and safety rules | 6 |
| Useful contacts and addresses | 6 |
| Symbols | 6 |
| Product description | 8 |
| Constructive specifications | 10 |
| Positioning | 11 |
| Installation | 11 |
| Description of machine operation | 13 |
| Adoption of personal protective equipment (PPE) | 15 |
| Instructions for the user | 15 |
| Demolition and dismantling | 17 |
| Maintenance and troubleshooting | 17 |
| Commercial components, spare parts and relevant documentation | 19 |
| APPENDIX: Basic whitening procedures | 20 |

DECLARATION OF CONFORMITY

(according to ISO/IEC 17050-1)

This device compliance to Directive 93/42/EEC .

The original declaration of confirmity is provided in attached to the manual.

WARRANTY CONDITION

- 1) **DURATION:** the product is covered by a 12-month (twelve) guarantee.
- 2) **EXCLUSION:** the guarantee excludes:
 - a) faults originating from non-performance of scheduled maintenance and due to negligence or improper use of the product by the User;
 - b) periodic checks and maintenance;
 - c) repair or replacement of parts subject to wear that are fragile or have an unpredictable life, unless they prove faulty at the time of their delivery;
 - d) faults caused by personnel or originating from technical personnel transfer and transportation;
 - e) faults or damage originating from improper use or operating errors;
 - f) faults or damage originating from contamination in the water and air infeed systems, extraordinary chemical agents or electrical events;
 - g) faults or damage originating from the use of detergents, disinfecting agents, sterilizing fluids or processes that are not described explicitly in the instruction manual for operation and maintenance;
 - h) natural color change of plastic components.
- 3) **INSTALLATION AND TESTING REPORT:** an essential prerequisite for guarantee validity is the return of the installation and testing report after all its sections have been filled in by the User and the Fitter. The report must be returned within 15 days after installation, otherwise the guarantee becomes null and void.
- 4) **LIMITS:** the guarantee entitles its holder to repairs or replacement of faulty components free of charge. The right to replace the entire machine is excluded. As for components either applied or added to TECNO-GAZ products or by third parties, having their own guarantee certificate, the conditions, limits and exclusions indicated in the relevant certificates apply.

Under no circumstances shall carrying out of one or more than one repair during the guarantee term change the guarantee expiry date.
- 5) **CLAIMS:** in the event of the Purchaser making a claim relating to the application of the guarantee and the quality or conditions of the delivered equipment, the said Purchaser shall in no way suspend and/or delay payments.
- 6) **DISPUTES:** if a dispute arises on the application and interpretation of this Certificate of guarantee, the Court of PARMA (Italy) shall be the court of jurisdiction, regardless of the location in which the purchase agreement was stipulated.

7) DEROGATIONS: any special derogations from the guarantee terms described in this section shall not imply the granting to the Purchaser of any right and shall be considered as granted for a specific case only.

8) MISCELLANEOUS: for any other issue, which is not covered by this Certificate of Guarantee, the rules described in the Civil Code and existing relevant Laws in the Republic of Italy apply.

LOSS OF WARRANTY

The guarantee becomes null and void in the event that:

- a) the equipment presents damage caused by a fall, exposure to flames, liquid spillage, natural events, bad weather conditions or by any other cause that is not due to manufacturing defects;
- b) installation has not been performed in line with TECNO-GAZ instructions and by authorized personnel;
- c) the equipment is repaired, modified or tampered with by the Purchaser or by non-authorized third parties;
- d) when asking for an intervention under guarantee, the product serial number has been removed, deleted, counterfeited, etc.;
- e) the installation and testing report is not returned suitably filled in and signed within 15 days from installation;
- f) the Purchaser suspends and/or delays payment of amounts due, for any reason whatsoever, relating to the purchase and/or maintenance of the equipment;
- g) scheduled periodic maintenance or any other instruction envisaged in the manual for operation and maintenance is not carried out.

DELIVERY TERMS

1. All goods delivered must be packaged in their original packaging. If the goods are delivered in a non-genuine packaging, a charge will be applied to restore the original packaging.
2. All goods must be delivered CARRIAGE PAID.
3. The client shall ask a written authorization to TECNO-GAZ sales dept. before returning the goods by means of the "Form for Client's Returned Goods Management" (CM-M-002) in compliance with the requirements of TECNO-GAZ Quality Management System (ref. CM-P-003, CM-I-009).
4. The document accompanying the returned goods shall show:
 - the TECNO-GAZ product code/item no.;
 - the number of the TECNO-GAZ purchase invoice or shipment document;
 - the product serial number and batch number;
 - a description of the goods returned;
 - the number of the Form for Client's Returned Goods Management" (CM-M-002) or a copy of the same attached.
5. Returned goods are considered as accepted only after they have been examined. If they prove non-conforming, they may be sent back to the consignor at TECNO-GAZ full discretion.
6. Charges will be applied to restore and test the returned product.
7. Parts replaced under guarantee must be returned, carriage paid, to TECNO-GAZ.
8. If the parts are not returned, their cost will automatically be debited to the recipient.
9. TECNO-GAZ does not accept goods returns from end users.
10. Goods sent to TECNO-GAZ for repair are managed as returned goods.

GENERAL WARNINGS AND SAFETY RULES

-  Make sure that the equipment is supplied with the correct voltage as shown in the plate.
-  Do not remove the plate.
-  Make sure that the machine is equipped with a grounding system.
-  Use a dry cloth to clean the machine.
-  Before any operation is performed, disconnect the power cable from the socket.
-  Use original spare parts only.

Non-compliance with the rules above relieves TECNO-GAZ of all responsibility.

USEFUL CONTACTS AND ADDRESSES

For technical service please apply to the dealer that invoiced the device, or directly to:

TECNO-GAZ SpA

Str. Cavalli n°4 – Sala Baganza – Parma – ITALIA.

Tel. +39 521 8380 - Fax +39 521 833391

e-mail : info@tecnogaz.com

<http://www.tecnogaz.com>

SYMBOLS

Pursuant to DIRECTIVE 2002/96/EC, this symbol shows that the product must not be disposed of as urban waste at the end of its operating life.

General warning and information for the user.

Pay attention to the instructions provided alongside this symbol.

Conformity with the Directive 2004/108/CE and Directive 2006/95/CE and further amendments.

PE External protection.

Protection GROUNDING SYSTEM.

(Operating) GROUNDING SYSTEM.

Alternate current.

Warning: high voltage. Risk of electrocution!

Read the instruction manual carefully.

POS. Position (number identifying a component in the figures or diagrams).

Fg. Figure

Ω Ohm (Electric Resistance unit).

s Seconds (time unit).

W Watt (Power unit).

Hz Hertz (Frequency unit).

mm Millimeters (Length unit).

A Ampere (Electric Current unit).

V Volt (Electric Voltage unit).

Pa Pascal (Atmospheric Pressure unit).

°C Degrees Centigrade (Temperature unit).

kg Kilograms (Weight unit).

PRODUCT DESCRIPTIONClassification of medical devices pursuant to Directive 93/42/EEC (Annex IX)

- Duration: Temporary
- Invasive devices: Non-invasive
- Active medical device: Active

Due to the above characteristics the device falls under class I (in accordance with Rule 1)

Classification in accordance with general regulations (standard CEI EN 60601)

- According to the type of protection against electrical hazards: Class I equipment
- According to the type of protection against ingress of water: Ordinary equipment
- According to the degree of safety in use in the presence of a flammable anaesthetic mixture with air or oxygen or nitrous oxide: Device not suitable for use with what is described above.

The dental whitening lamp is for use in the beauty and dental sector.

Dental whitening is the only and sole purpose of the whitening lamp.

The lamp emits an intense light radiation.

The operator who uses the whitening lamp device must have the necessary qualifications and experience.

The operator must have received the necessary training in its correct use and must know the position of all the controls:

- for positioning the lamp
- for switching the lamp on and off and the time setting functions.

Read the instructions for use carefully before installing and using the device.

Illegitimate use of the device and operator negligence can cause damage to persons, the environment and property; the manufacturer cannot be held responsible for said events.

The operator who uses the device must have been correctly trained.

The dental whitening lamp consists of (fg.1):

- base with 4 castors, which is a useful support for moving and positioning the device (A).
- A central plastic body that is ergonomic and easy to clean (B).
 - Highly visible LCD screen (C).
 - Intuitive, visible and accessible button unit (D).
 - Anodized aluminium support (E).
- Plastic lamp on steel shelf (F)
- anodized aluminium arm, provided with articulated joints for easy and precise positioning (G).
- 2 pairs of protective glasses (for operator and patient).

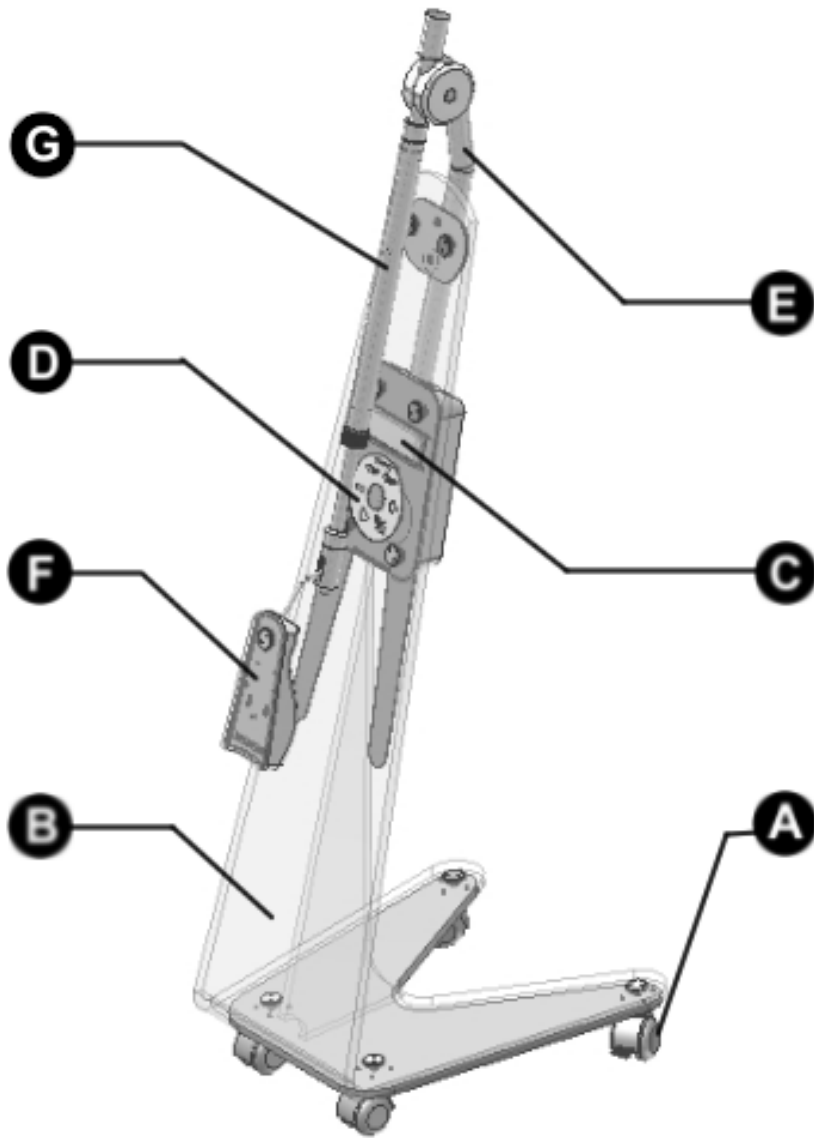
fig.1

CONSTRUCTIVE SPECIFICATION

| | |
|-------------------------|---------------------|
| Sizes (X,Y,Z) | 1350 x 420 x 430 mm |
| Weight | 15 kg |
| Packaging sizes (X,Y,Z) | 1400 x 500 x 500 mm |
| Supply voltage | 230V~ ± 10% |
| Number of phases | 1 |
| Supply frequency | 50 Hz |
| Maximum absorbed power | 50 W |

| | |
|----------------------|--------|
| Fuse | F 1 A |
| Radiation wavelength | 495 nm |
| Output power | 30 W |

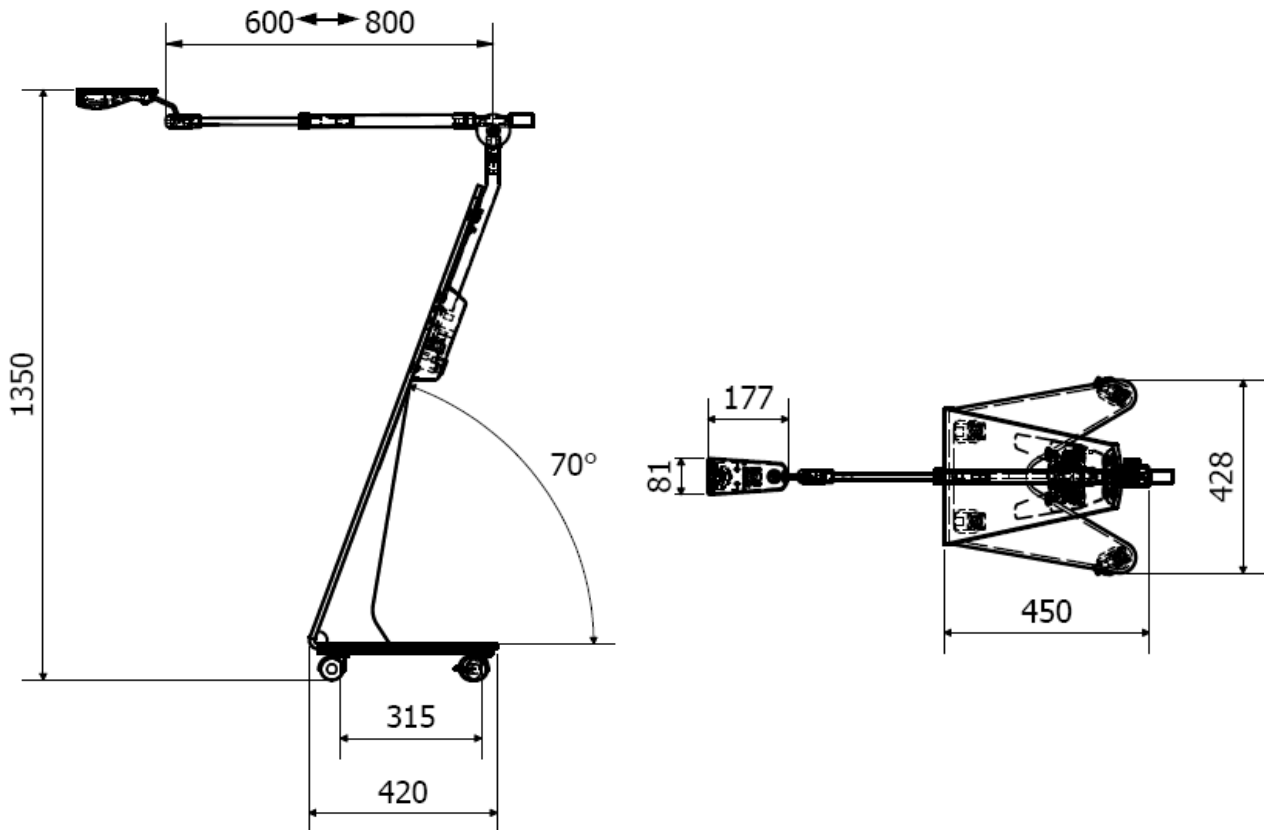
fig.2

POSITIONING

| | |
|-------------|-----------------|
| Temperature | - 20 to + 45 °C |
| Humidity | 10 to 100 % |
| Pressure | 500 to 1060 hPa |

Before installing, check the integrity of the device; if there are signs of transit damage, please notify the dealer. **WARNING!** Do not switch on the device.

The lamp must be placed far away from instruments that generate strong heat emissions and electromagnetic radiations that may damage the installed electronics.

Failure to comply with the above will terminate the guarantee.

Disposal of the packaging components must be carried out in compliance with the current regulations in the country where the device is used.

INSTALLATION (FOUNDATION)

Because of the characteristics of the device is important to follow certain precautions:

The device should not be used in potentially explosive atmospheres.

**Remove packaging, and check the status of the unit.
If any signs of damage to vehicles, alert the manufacturer.**

**Packaging must be preserved throughout the warranty period.
The manufacturer does not accept returns without original packaging.**

The lamp is put in a cellophane bag and then arranged in a corrugated cardboard box and prevented from moving about with polystyrene foam.

It must be transported following the rules for the transportation of fragile goods.

The original packaging should be kept for the duration of the guarantee period, since only in this container is it possible to avoid damage in transit.


Place the unit on a horizontal surface, away from heat sources.

Place the device so that the dashboard controls and the LCD screen is clearly visible from the operator.

Connect the power cord through the special connector on top on the rear lamp fig. 4.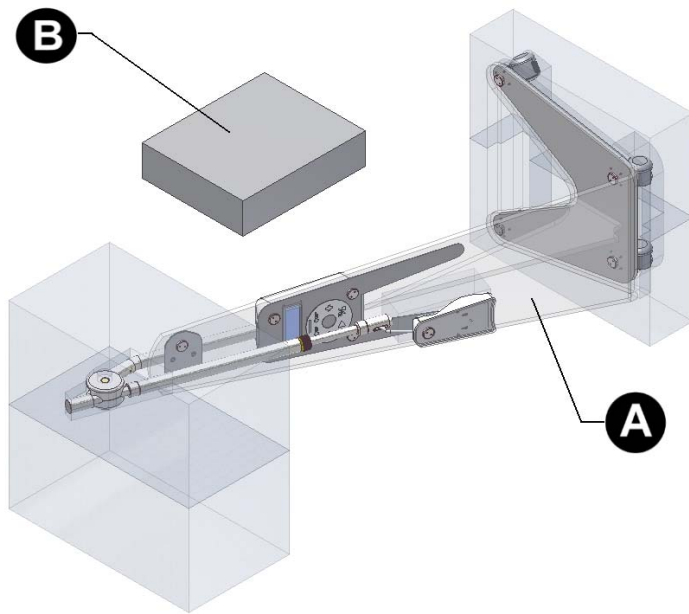

- A. SLSA093 Complete lamp
(body, arm, instruction manual, installation accessories)
- B. 4450-S Kit accessories
(large and small mouth opener, 2 protective glasses, Basickit, Desens)

fig. 3

Connection and putting into service:

The electrical connection must be made in accordance with current technical regulations.

Make sure the measured mains voltage value corresponds to that written on the machine's identification plate.

Make sure the electrical systems are efficiently earthed.

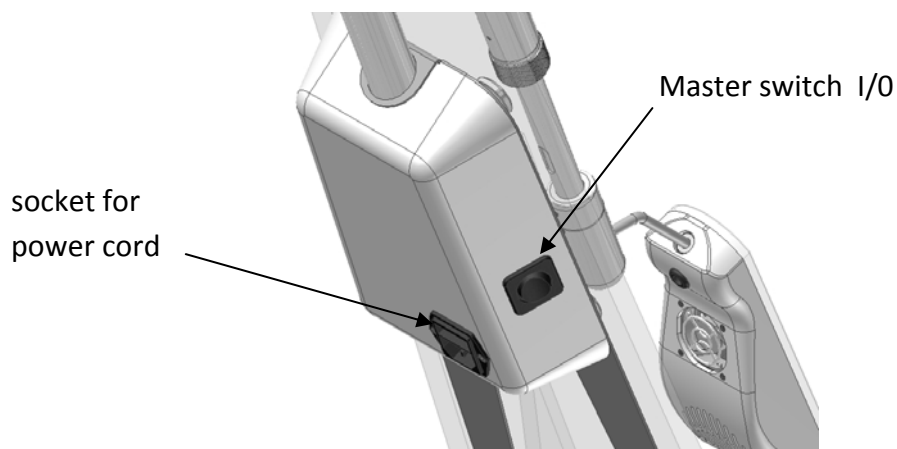
fig.4


DESCRIPTION OF MACHINE OPERATION

The lamp is a dental whitening device to be used with manufacturer certified whitening products.

In the whitening treatment, it is used to polymerize the protective dam and in combination with a whitening product.

Other uses are not envisaged and are not allowed.

- A. Bleaching program
- B. Polymerization program
- C. Decrease period
- D. Increase period
- E. Execution program
- F. Pause/Stop program

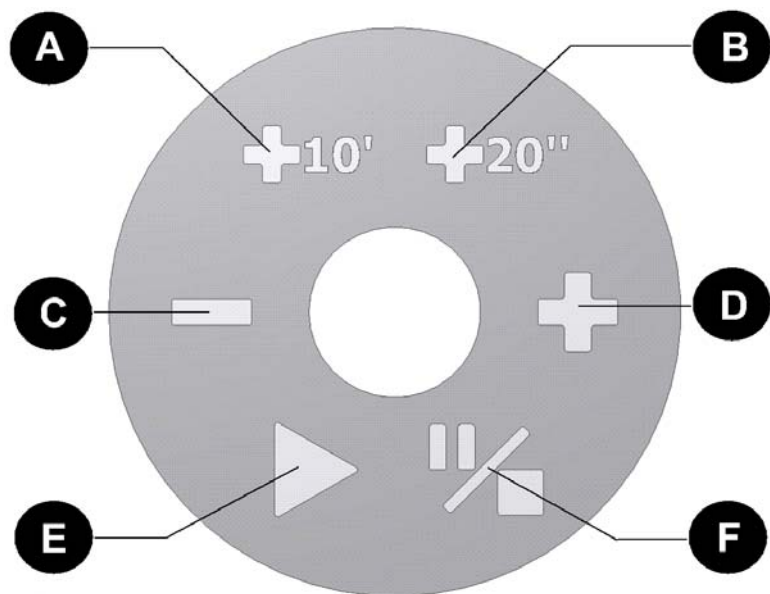fg.5

The lamp is a dental whitening device and its innovative LED technology solves the problems of heat emission, light power variations and lamp deterioration.

To obtain maximum efficiency, the light is brought to the outside of the lamp by optical fibres.

The device is easy to use and position and therefore ideal for any work situation.

Position the lamp near the patient's mouth and adjust its height and distance.

The lamp should be placed at 2-10 cm from the patient's mouth.

Put the master switch on the side, in position “I” (fig.4), the device is now ready to be used.


1. Set the type of program through the button on the dashboard:

- Preliminary polymerization  20^m pos.B fig.5.

On display are the second required for the polymerization of protective dam. At the end, the display shows the time of standard treatment of bleaching (10').

- Treatment of bleaching  10^m pos.A fig.5.

On display are 10' (standard step) for the first phase of treatment

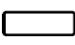2. Varying the period of treatment using the keys  and  pos.C and D fig.5

- Step in seconds for the program polymerization
- Step in minutes for the program bleaching.

3. Turn on the lamp through the key  pos.E fig.5.

4. If you want to stop treatment, press  Pause/Stop pos.F fig 5.

5. If you want to stop finally treatment, press Pause/Stop.

6. If nothing is the lamp turns off when the cycle ends and begins a set period of further cooling led off.

Operating and environmental limits:

| | | |
|-----------------------|------------|------|
| Operating temperature | 5 ÷ 30 | °C |
| Humidity | 10 ÷ 90 | % |
| Pressure | 500 ÷ 1060 | hPa. |

ADOPTION OF PERSONAL PROTECTIVE EQUIPMENT (PPE)

Before switching on the bleaching lamp, the operator, patient, and in general all persons present during the treatment, must put on the special glasses (PPE - personal protective equipment) supplied with the device.

The intensity of the emitted light can cause irreparable damage to the retina and it is therefore necessary to use special PPE (personal protective equipment).

Check that the lamp is positioned correctly.

INSTRUCTIONS FOR THE USER**LAMP POSITIONING:**

Correct positioning of the lamp is essential for proper use.

The distance between the lamp and the patient's mouth should be about 2-10 cm.

PEARL is provided with three articulated joints:

- main articulated joint **S1**
- telescopic joint **S2**
- secondary articulated joint **S3**

The joints give different ways of positioning:

JOINT S1 Supporting the arm lamp through the joint S3.

Turn (45/90°) counter-clockwise the handwheel V (fg.6 pos. V), obtained the ideal positioning, blocking the joint rotating clockwise.

- **NOT ROTATE THE SNODO WITHOUT UNSCREW THE HANDWHEEL V**

1A

Movement rotation that allows you to raise / lower the lamp (+30 ° / -30 °).

2A

Movement rotation, can rotate his arm lamp on the vertical axis. (Independent from handwheel V).

JOINT S2 The hardness of the joint is determined by tightening the ring G (fig. 6 pos. G).

- NOT FORCING TELESCOPIC JOINT; LOOSEN THE SCREW G

1B

Scrolling allows lengthen and shorten the distance between lamp and body.

2B

Movement rotation, can rotate his arm lamp on its longitudinal axis.

JOINT S3 The hardness of the joint is determined by the tightening of wheat T (fig. 6 pos.T).

1C

Movement rotation, allows you to the lamp upwards or downwards (+40° / -40°)

2C

Movement rotation, allows you to move the lamp to the right or left

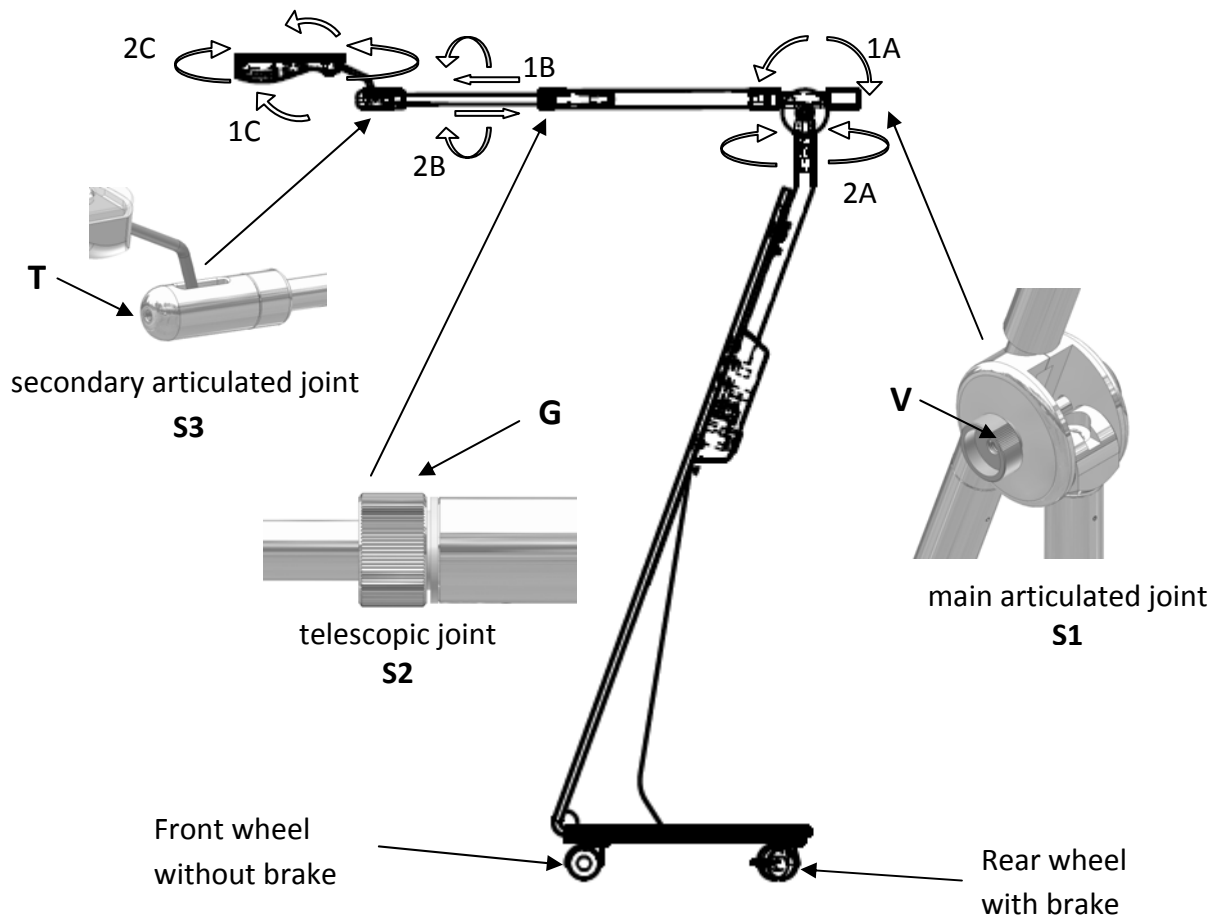
fig. 6

DEMOLITION AND ELIMINATION


Directive on waste from electrical and electronic equipment (WEEE)

Pursuant to DIRECTIVE 2002/96/EC, this symbol shows that the product must not be disposed of as urban waste at the end of its operating life.

The product must be taken to centers specialized in separate collection of electric and electronic equipment or to a dealer, when a similar product is purchased.

The machine holder shall be responsible for bringing it to collection centers.

For further information about collection systems, address the local waste disposal service.

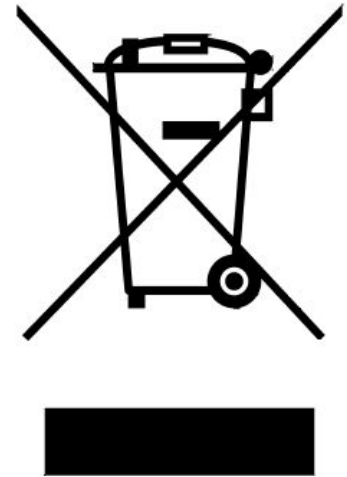Correct disposal of the equipment, which is no longer used, prevents negative consequences for the environment and human health.

Abuses will be prosecuted by law.

MAINTENANCE AND TROUBLESHOOTING

The device requires no maintenance, except regular cleaning.

For cleaning, please refer to the health and hygiene regulations.

- Switch off the machine and pull the plug out of the electrical socket.
- If the machine has just been used, waiting for the cooling: shutdown fan after using
- Use a sponge dampened with soapy water and dry with a clean dry cloth.
- Do not use harsh detergents, which could irremediably alter the appearance of the plastic parts of the machine.
- Do not spray directly onto the machine, as this would risk damaging the electrical part inside it.

DO NOT DO MAINTENANCE OR INSIDE REPAIR TO THE DEVICE.
DISMANTLE THE DEVICE MEANS BREAKING IT.

| | Defect | Cause | Intervention |
|---|---|--|---|
| 1 | Pushing START button the device does not work | Lack Power | Control plug fit correctly in sockets |
| | | Fuse/s burned | Replace the fuse/s FA1 as tag |
| 2 | The head of the lamp is not in the desired location | The screw T joint of secondary S3 (fg. 6) is not tight enough. | Once torqued elements G and V (fg. 6), taking the head lamp, tighten gradually screw T verifying that the head remains in place and can move simultaneously under the action of |

Diagnostics and troubleshooting

If the device does not work properly or not at all, contact the manufacturer directly or an authorized service centre.

It is not possible to work directly on the device.

The only work allowed is the following:

- Cleaning and disinfection (maintenance and troubleshooting)
- Checking the efficiency of the LEDs: Reduction in the light power, as a consequence of LEDs that are not working or power supply problems, reduces the whitening process.

COMMERCIAL COMPONENTS, SPARE PARTS AND RELEVANT DOCUMENTATION

The commercial components that can be supplied directly to the customer are the following:

| | |
|---|-------------------------------------|
| <p style="text-align: center;">BASICKIT</p> <p>n.4 Seringues 4:1 bi-composant de 2,5 ml. de Gel de concentration de peroxyde d'hydrogène 38%</p> <p style="text-align: center;">n.8 bec pour seringues 4:1</p> <p style="text-align: center;">Seringues n.2 monocomposant 3 cc/3.5 gr</p> <p>n.6 bec des barrages seringue monocomposant</p> | <p>Code 4411S2</p> |
| <p style="text-align: center;">DESENS</p> <p>Solution acide de nitrate de potassium</p> <p style="text-align: center;">10 flacons de 0.5 ml (monodoses)</p> <p style="text-align: center;">10 petits pinceaux</p> | <p>Code 4412S2</p> |
| <p>PROTECTIVE GLASSES</p> | <p>Code CM61046 (2 pcs)</p> |
| <p>LARGE MOUTH OPENER</p> | <p>Code 4420-S (10 pcs)</p> |
| <p>SMALL MOUTH OPENER</p> | <p>Code 4422-S (10 pcs)</p> |

APPENDIX: BASIC WHITENING PROCEDURES

Every dental practice that uses the professional whitening system must adopt an operating protocol that is suitable for guaranteeing a safe, satisfactory, controlled result.

INFORMATION FOR THE CUSTOMER

- Presentation of the technique, procedures, results and all other operating and behaviour aspects that the technique requires.
- Possible supply of informative material (WHITENING SPECIAL).

It should be mentioned that the result is subjective and can vary from patient to patient for obvious physiological and morphological reasons.

DIAGNOSIS AND MEDICAL AND DENTAL HISTORY

The dentist carries out an examination and a medical-dental assessment to assess the feasibility of the treatment and the presence of possible contraindications.

BEGINNING OF TREATMENT:**PREPARATION OF THE PATIENT**

1. Clean the teeth properly with a preventive treatment.

Wash the teeth using the syringe water, and then dry them with the syringe air.

Apply Vaseline or suntan cream to the lips, and then apply the supplied mouth opener that is designed to keep the patient's tongue lowered so it cannot accidentally come into contact with the whitening gel.

2. Precisely apply the supplied photopolymerizing liquid dam (LIQUIDMASK) to the gums and mucous membranes. This is to isolate the mucous membranes and prevent the whitening gel from coming into contact with the gum flaps.

Using the bleaching lamp, polymerize with a 20-second irradiation treatment.

3. Apply a layer of whitening gel (BGEL) to the teeth to be treated. Spread it on evenly and avoid using more than necessary.

Warning:

when you use the syringes with nozzle, the presence of air trapped in the syringe could cause the gel to come out in an uncontrolled manner.

(To make this procedure easier, rotate the plunger as you measure out the required quantity).

Be very careful to move the plunger very gradually, or place the gel on a suitable surface and then spread it on with a special brush. In this way, you can spread the gel correctly and handle the material better.

4. Both patient and operator must put on the special glasses provided, and then arrange the lamp so that the LED head is next to the patient's mouth at a distance of not more than 10 cm.

Use the buttons situated on the control panel to set the treatment time and start the treatment (for treatment times, see next chapter).

During the treatment, the patient should be checked and assisted to verify his/her comfort and state of well-being.

5. At the end of treatment, the whitening gel must be removed by the dentist with the help of an assistant, who must take care to avoid letting it fall on the patient's exposed mucous membranes.

Remove the polymerized dam and make the patient rinse his/her mouth.

6. If the patient has a marked dental sensitivity, spread a phial of DESENS (potassium nitrate) over the treated teeth with the special brush.

7. At the end, the results are assessed and the patient should be given information on post-treatment behaviour (avoid drinking coffee, chocolate, substances containing tannin and eating hot food or drinking hot drinks for at least 72 hours).

The maximum achievement of whitening should be assessed 10-15 days after the treatment.

TREATMENT TIMES

The equipment has a system of LEDs with optical fibre diffuser that allows concentration of the light and a reduction in treatment times.

The treatment times must be customized for each individual patient, according to his/her condition (yellow teeth - grey teeth - marks caused by the use of antibiotics, etc.). However, we provide you with some general reference instructions below.

WHITENING IN A SINGLE SITTING

Carry out a whitening treatment of 3 STEPS (point 4) lasting 10 minutes each. Between the steps, redistribute or add the whitening gel (BGEL).

WHITENING IN 2 SITTINGS

Carry out a whitening treatment of 2 STEPS (point 4) lasting 10 minutes each. Between the steps, redistribute or add the whitening gel (BGEL).

Wait 15 days and repeat a treatment in the same way.

WARNING

Use only and exclusively manufacturer's products.

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TECNO-GAZ S.p.A.