Skylab • Nybra

Premessa

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Introduction

The sandblaster is one of the most used instruments in the dental laboratory and this is why it is among the primary pieces of equipment on which every operator should carefully invest.

The sandblaster must be suitable and technologically evolved, otherwise it will not comply with Italian legislative decree DLGS 626/94 on worker protection and may create environmental pollution that is harmful to operators.

The main features of a modern sandblaster are: ergonomics, efficiency, operating versatility and compliance with regulations. It is important to opt for models that are rich in accessories, safety devices and cutting edge solutions, in order to guarantee good operativeness, easy working, compliance with current regulations, personal protection and protection of co-workers.

Tecno-Gaz S.p.A. has made 2 sandblaster models that are unequalled on the world market. Their features and the solutions adopted are the very best technically, technologically and as regards compliance with regulations.

SKYLAB and NYBRA are unique and exclusive models.

Check your sandblaster and assess whether it meets all the safety requirements.

- Does your sandblaster have a fume and dust extraction system?
- Does your sandblaster have a safety system that permits sand to be sprayed only with the door hermetically closed?
- Check its declaration of conformity and CE marking.

Safety and compliance with the regulations are important factors as regards both personal and legal aspects.

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With the Skylab and Nybra sandblasters, Tecno-Gaz has aimed to create flexible and reliable instruments with a careful eye on operator safety. High productivity, coupled with a very full range of accessories supplied as standard, make Tecno-Gaz sandblasters the best on the market.

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Venturi system: no clogging problems

Conventional sandblasters have sand containers under air pressure and this causes the sand to clump together, which often leads to clogging and obstruction.

SKYLAB and NYBRA have eliminated this problem by adopting a sand suction system using the VENTURI method. This solution allows you to avoid having pressurized sand tanks and to never have mixing of air and sand except at the tip of the hand piece. **This method guarantees maximum efficiency, avoids blockages and clogging and guarantees a very low sand consumption.**

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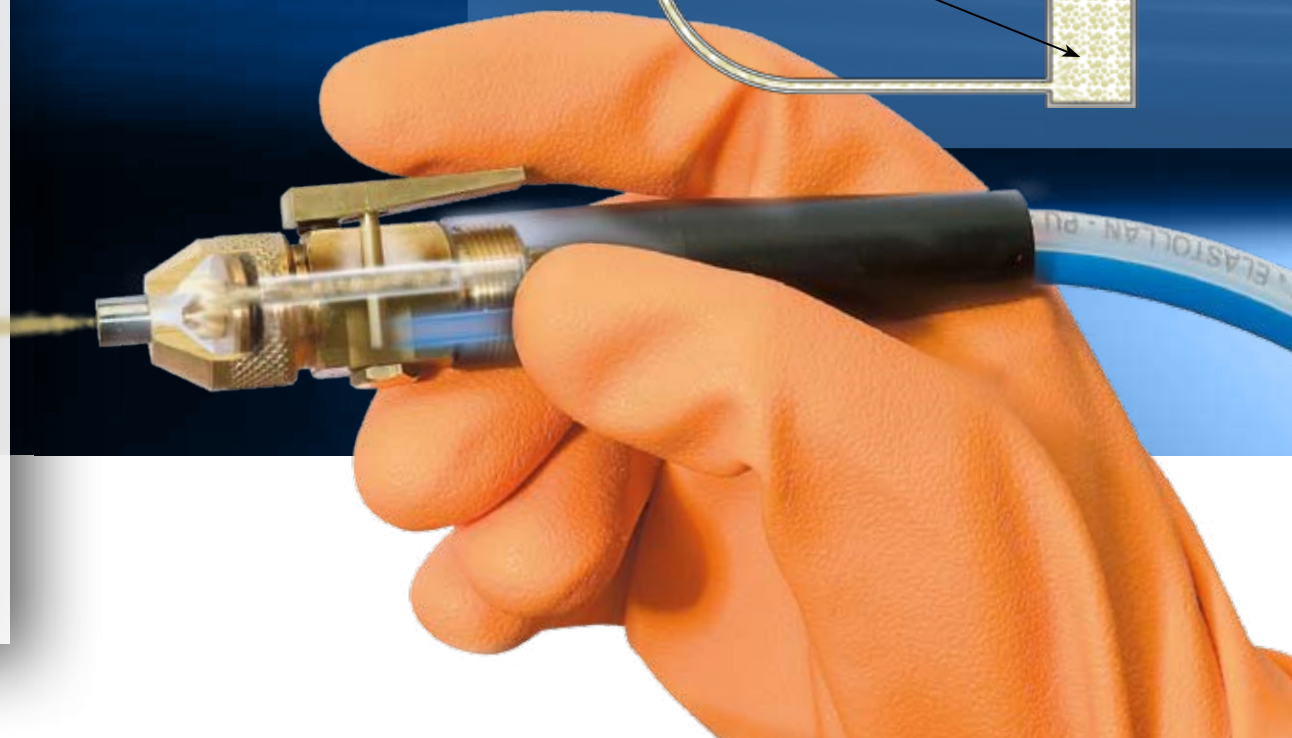
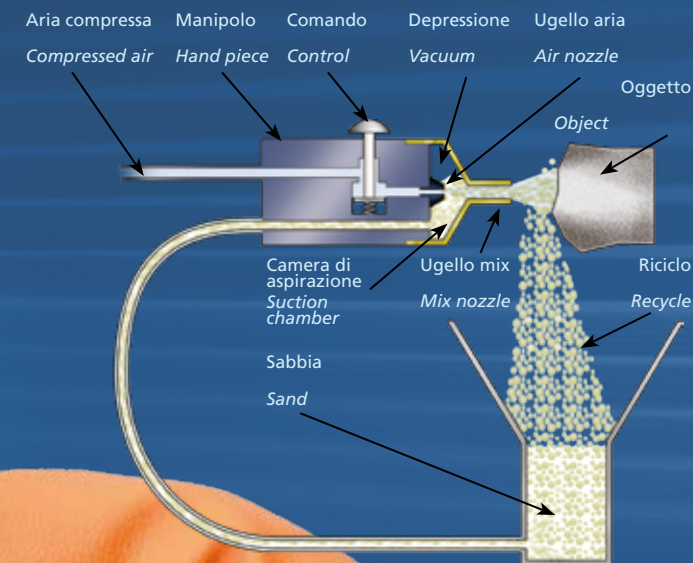
Special hand pieces for sand spraying

Each sandblaster cabinet is equipped with a hand piece for sand spraying. Each hand piece can be adapted to all types of sand: dioxides, balls, aluminas.

Each hand piece is equipped with:

- adjustable ring nut that allows the tip to be calibrated according to the grain size of the sand used
- lever for operating the sand spraying
- comfortable handgrip

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Art. 4516-S (12 pz.)

Glass panels with protection

To prevent the glass panels from being damaged by the abrasive action of the sand, they are protected with removable, easy-to-replace adhesive acetate film.

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Art. CM61001

Very easy glove replacement

The gloves can be replaced easily and quickly, since the retaining ring nuts are quick fitting. This solution avoids complicated operations and the use of tools.

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Art. SSKA228

Sealing gaskets

The gaskets fitted between glass and cabinet structure are produced with special rubber foam, in a single piece to guarantee a perfect seal and containment of all the micropowders.

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Anti glare lights

The lights inside each cabinet are insulated and protected and are fitted with a special anti glare parabolic reflector that safeguards against "dazzling" the operators.

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Active safety devices for operators

The work cabinet is equipped with a special safety system that allows the sand to be sprayed only with the glass panel perfectly closed. This guarantees maximum safety for workers and environmental protection in compliance with Italian legislative decree DLGS 626/94.

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SKYLAB is the maximum expression of sandblaster technology in dental mechanics. No other equipment can rely on such advanced technological solutions.

SKYLAB was created for those who have to sandblast prosthetic products and have a preference for rationality, convenience and precision.

SKYLAB has been designed to adapt to the operator's needs, through many transformations and personalizations.

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Three sandblasters in one

When a company makes a piece of technical equipment, it has to know the operating needs of those who use it daily.

Today, the dental technician uses many types of sand and it is therefore not conceivable to use "noble" polishing sands in a cabinet where heavy aluminas are used. Pollution and promiscuity are sure to create problems during the various sandblasting stages, which is why **Tecno-Gaz** decided to make the first sandblaster with 3 independent cabinets. **THREE cabinets give you the possibility of being able to use aluminas, balls, dioxides and other sands and reserve a specific cabinet for each individual sand.** In this way, you can be certain of the sand's purity and avoid any kind of pollution.

Each cabinet has totally independent controls:

- air pressure adjustment
- display of pressure by means of manometer
- activation of suction
- self-contained lighting
- self-contained sand spraying

Inside each cabinet there is:

- air gun for cleaning the manufactured products
- hand piece for sand spraying
- sieve for filtering the sand
- plug for dividing pure sand and used sand
- steel object holder grill

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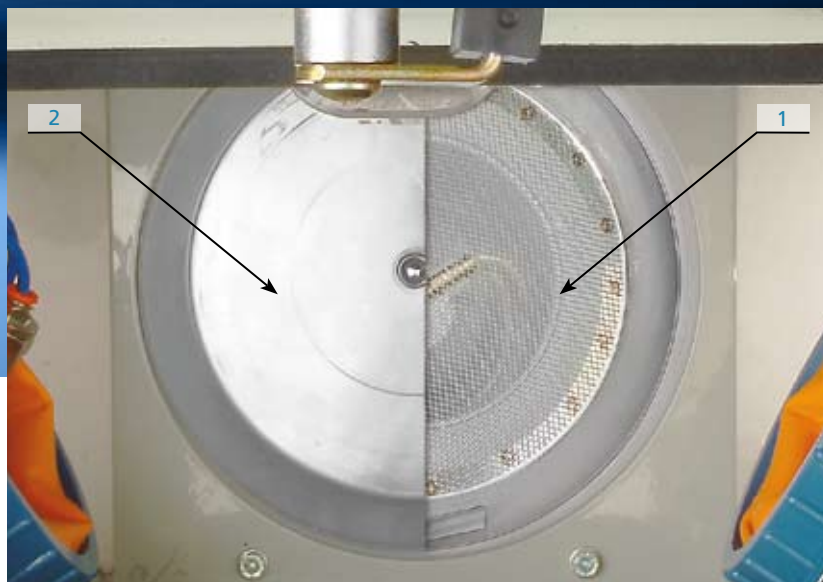
Disposable sand or automatic recycling sand

The cabinets can be prepared according to the individual requirement of each operator, who can "organize" each cabinet:

With automatic recycling of the sand (using the special sieve filter 1). The used sand is automatically filtered and put into the sand container for prompt reuse.

Disposable sand (using the special plug 2). In this way there is a distinction between pure and used sand. The used sand can then be sucked up with the lance supplied with the machine. Each cabinet can in any case be adapted each time to suit the requirements of the moment. A truly extraordinary solution that allows maximum operativeness and ease of use.

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Accensione luce
Switching on of
light

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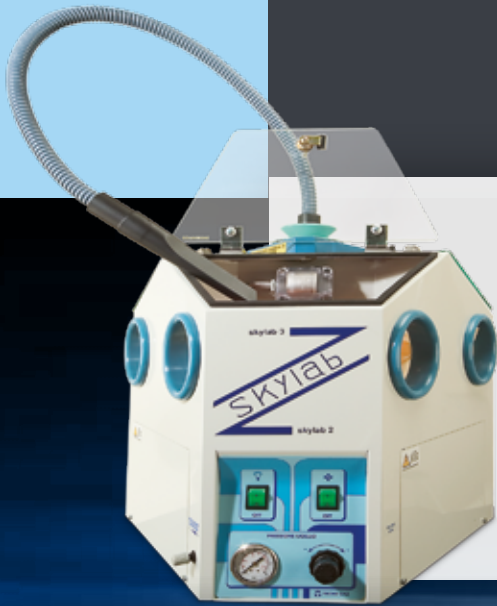
Integrated dual function aspirator

The sandblaster is equipped with integrated dual function aspirator, with electronic control that allows it to be adjusted and controlled.

The aspirator has been designed to fulfil two functions:

- suction of fumes and dust produced during the sandblasting process, to protect the operator from inhaling harmful fumes and to comply with the requirements of worker protection directives;
- suction and removal of the disposable sand accumulated in the cabinets. This function can be actuated using the lance with pipe supplied with the machine.

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Skylab revolves through 240°

SKYLAB can be placed in the centre of the room and used by three workers at the same time, or it can be placed on any dental laboratory bench and used by one operator. **All the cabinets are usable** since SKYLAB is equipped with a system that allows the equipment to be rotated with stop on each individual position. **The equipment is equipped with a positioning support, specially designed to allow it to be placed on all dental laboratory benches.**

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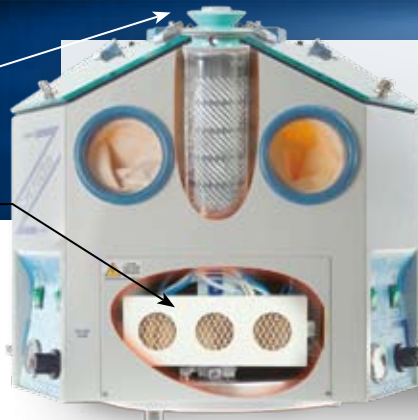
Air filters

SKYLAB is provided with 2 filters:

- primary filter used to contain the dust sucked out during sandblasting and collect the sucked out sand. Code 4517/S
- secondary filter used to filter the air being discharged, which allows the discharged air to be cleaned. Code SSKA230.

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Sandblaster mod. NYBRA is an innovative and technological 2-sand sandblaster.

NYBRA was created for everyone who has to sandblast prosthetic products and has a preference for rationality, convenience and precision.

NYBRA has been designed to adapt to the operator's needs, through transformations and personalizations.

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Integrated dual function aspirator

The sandblaster is equipped with integrated dual function aspirator, with electronic control that allows it to be adjusted and controlled.

The aspirator has been designed to fulfil two functions:

- suction of fumes and dust produced during the sandblasting process, to protect the operator from inhaling harmful fumes and to comply with the requirements of worker protection directives.
- suction and removal of the disposable sand accumulated in the cabinets. This function can be actuated using the lance with pipe supplied with the machine.

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Nybra: sandblaster with 2 independent sand units

NYBRA is a 2-sand sandblaster with completely independent controls and adjustments.

A simple intuitive board controls all the functions:

- air pressure adjustment, hand piece 1 and hand piece 2
- display of pressure by means of manometer
- switching on and adjustment of suction system
- switching on of cabinet lighting

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The cabinet contains:

- air gun for cleaning the manufactured products
- 2 hand pieces for sand spraying
- stainless steel object holder grills

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Air filters

NYBRA is supplied with 2 filters:

- primary filter (1) used to contain the dust sucked out during sandblasting and collect the sucked out sand. Art. 4517/S
- secondary filter (2) used to filter the air being discharged, which allows the discharged air to be cleaned. Art. SNYA230

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All rights reserved. Variations can be done without notice. Tecno-Gaz S.p.A. is to be considered not responsible for damages caused by the lack or the wrongness of the information here mentioned.