ORMA

Art. 2900S

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Declaration of Conformity	2
General precautions and safety	2
Product description	3
Manufacturing features	4
Description of use	5
Accessories	12
Demolition and disposal	13
Precautions	13
Troubleshooting	14
Procedures for service and assistance	15

DECLARATION OF CONFORMITY

The undersigned TECNO-GAZ S.p.A.,
located in Strada Cavalli, 4 - CAP 43038 Sala Baganza (Parma) ITALY;
declares under its sole responsibility that the ORMA device is manufactured
in accordance with the laws transposing the directives in force.

The original declaration of conformity is enclosed with the manual.

Model: **ORMA**

Item Code: **2900S**

GENERAL PRECAUTIONS AND SAFETY

Make sure the appliance is positioned in stable on the ground or on the accessory bearing.

Do not remove the label

Use original parts only

Failure to comply with the points mentioned above removes any liability from the manufacturer.



PRODUCT DESCRIPTION

Welcome to use of Orma. With this helper, it is not necessary to worry about required clean environment to be polluted by the sole. This instruction will give you operation details and make you quickly get familiar with it and create the fun of using shoe covers.

MANUFACTURING FEATURES

Model	Orma
Dimension	800x445x330 mm
Level over ground	20 mm
Net Weight	21 Kg
Film-feeding time	4.0 s
Shaping time	2.0 s
Maximum power	1300W
Start preheating time	3min
Voltage/Frequency	220±5%V 50Hz
Hot standby power	75W
Highest temperature of heater	250°C

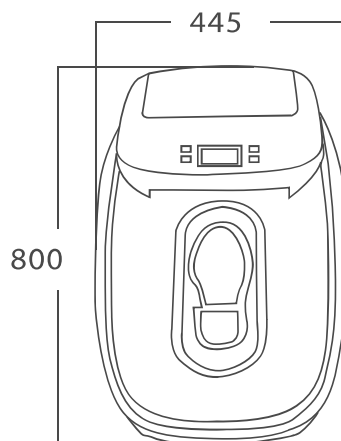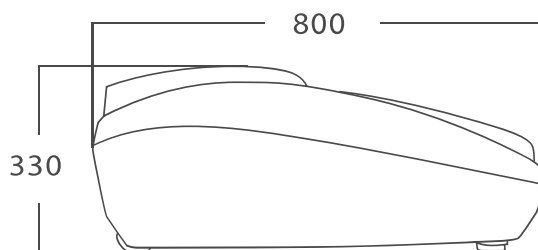USING THE DEVICE

The Orma machine shall be used in a clean environment without much high-conductivity fiber. The ambient temperature is 5°C-40°C, and the humidity is not more than 87%. Film used in Orma machine is made of PVC with certain static electricity. If it is used in the location with the requirement of static electricity, the application scope must be measured.

In order to ensure the safe usage, please strictly follow the following points:

- Before using Orma , please read this instruction carefully and follow the operating procedure.
- Before power on, please make sure the input voltage meet the 220v-240v.
- In order to avoid chine failure and safety problem, it is prohibited to place the machine in the open or flush the machine with water for cleaning.
- It is prohibited to cover bare foot with film in the machine.
- It is prohibited to put any foreign matter into the machine.
- The attention of anti-skid shall be paid, the shoes covered by films are prohibited to step on a place with water or watered blanket
- During maintenance on the machine, it's necessary to disconnect the power
- Children are prohibited to use or play the machine.

The power supply must have reliably earthed, or it may cause serious personal safety or equipment damage problem.

The shoe cover could be used by visitors or only in case of by hospital operators.

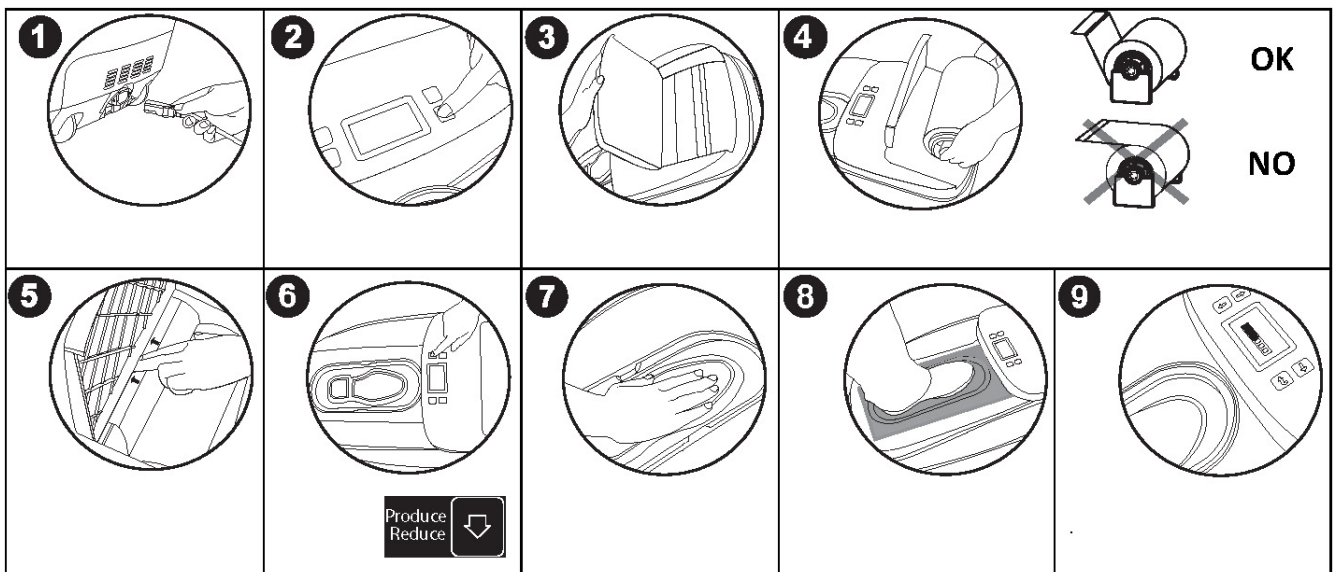
The shoe cover, mustn't be used in environment or activity where there are electrostatic charge as the same shoes that generate lead to dangerous situations for users.

Follow instructions step by step for a correctly usage:



- Connect to the power: insert both ends of wire into the indoors socket and the machine socket repectively.
- Press  to open the machine.
- Open the top cover on the machine, and clean up the head of coiled film to remove burrs and other impurities affecting the installing quality to ensure the smooth coil.


- Locate the coiled film into the case of the machine horizontally.
- Press the film head with one hand and push it into the film outlet medially.
- At the same time, hit  with another hand and send the film into the machine until the film is positioned.
- Press the footboard once, the film will be automatically sent out to working location. If need to adjust remain of the coiled film, please see: "Reset Remain". If it does not work, press  to turn back the film and repeat the above steps.
- When the coiled film is located in the working position, the machine can be used.
- Step on the position with "PRESS" mark.
- When five digits on the screen are lit and a long "beep" sound is produced, the shoe cover is shaped.
- Please leave your foot from the footboard, by the time, the film will automatically stretch out for next usage.
- If necessary press  to close the machine.

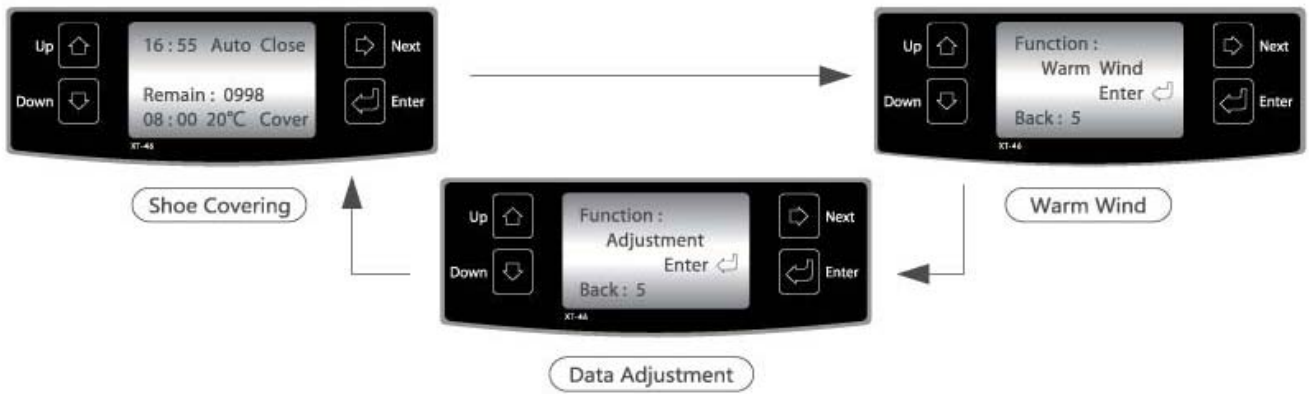
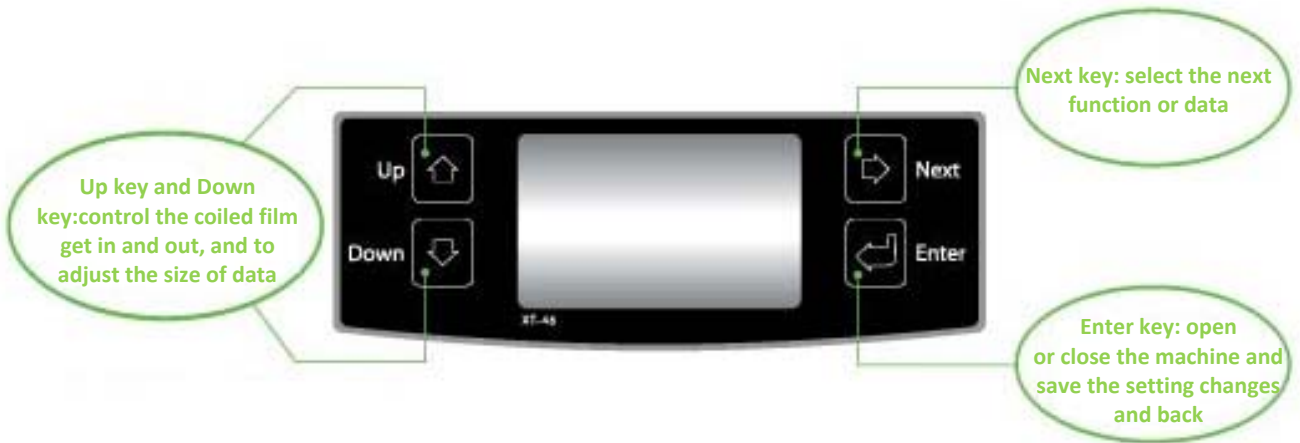
About the set up of the machine refers to LCD screen and four function keys. Through the function keys to implement three basic functions: "Shoe Covering", "Warm Wind"

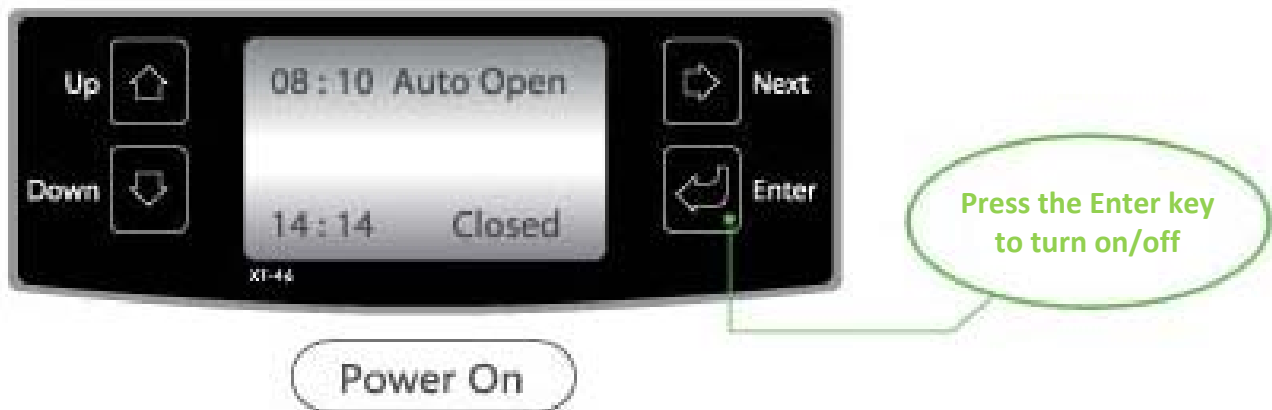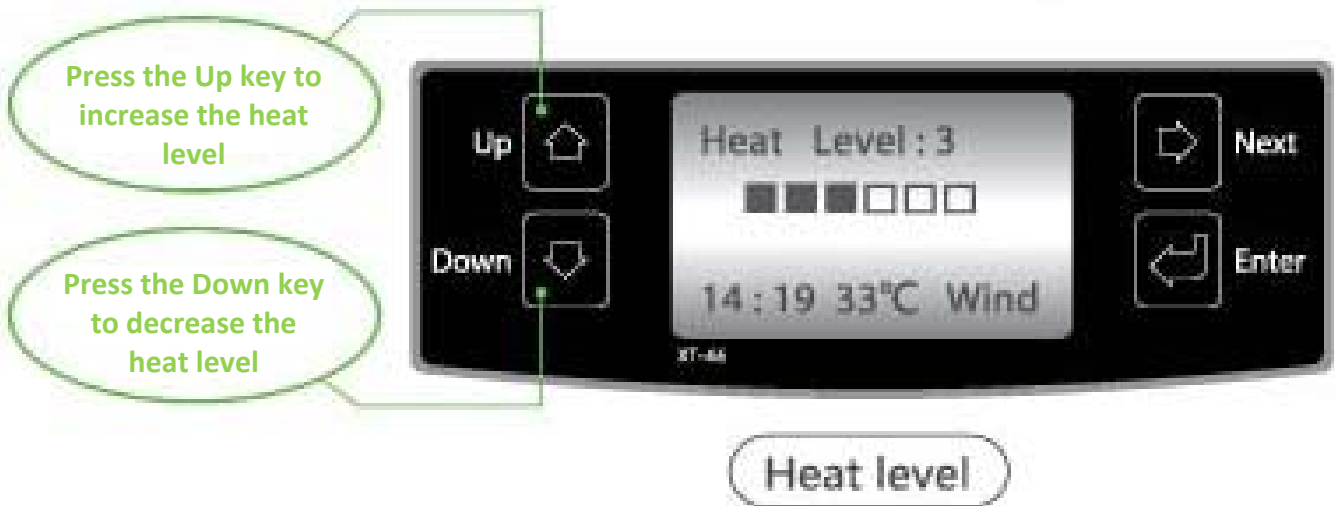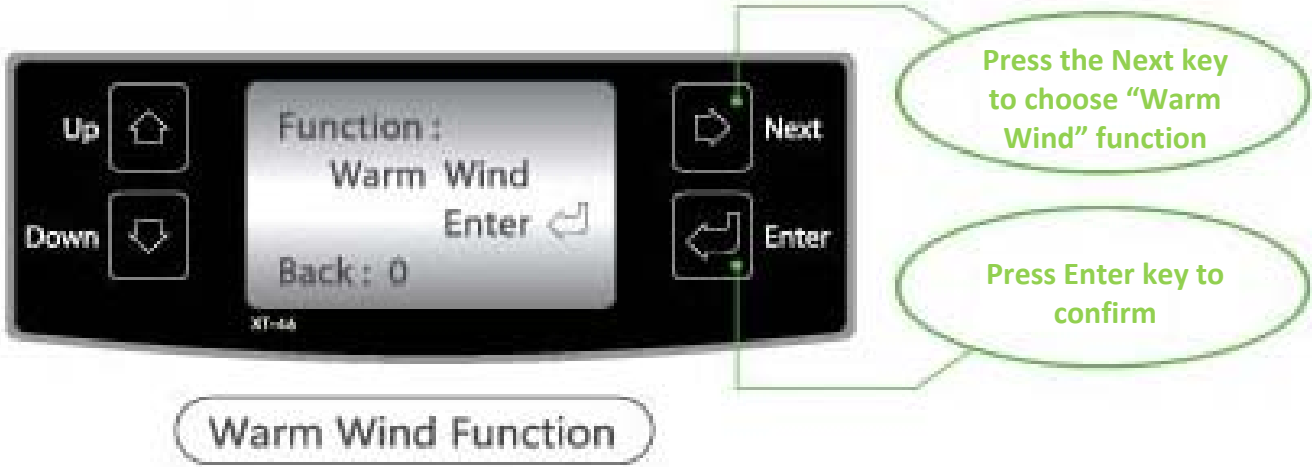and “Data Adjustment”, these functions can be looping shown by pressing  key. See the following figures.


- This function is used to warm or dry, and the heater is in continuous working state.
- Before using this function, please check that there is not film on the footboard. The footboard should be pressed.
- According to the ambient temperature or the requirement, the temperature of the warm wind can be adjusted.
- If need to exist this function, please press and the machine will be closed; press , the machine will be open and in the shoe covering function.

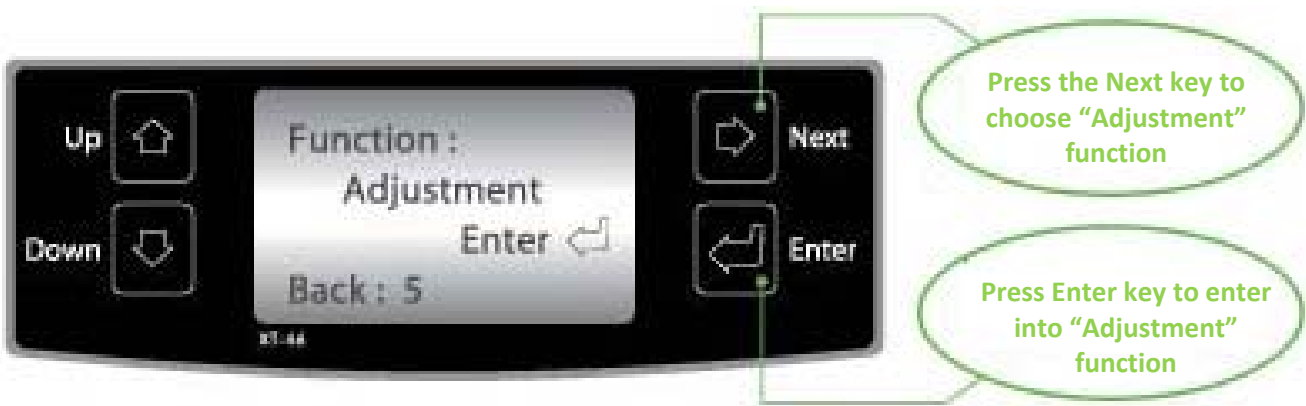As shown in the down figure, press  to choose the adjustment function and press  to enter into the looping submenu of the adjustment function, then press  to select the five sub-function: "Reset Remain", "Temp. Control", "Reset Time", "Working Time", "Cancel Timing".

Reset remain:

According to the number on the coiled film or estimated value of remain, reset remain of the film.

Temperature Control:

There are 9 levels of the temperature that is from 1 to 9, and the default level is 2. According to personal preference, the level of the temperature can be micro-adjusted.

Reset time:

According to the down figure, adjust the current time.

Working time:

According to the down figure, reset the open and close time.

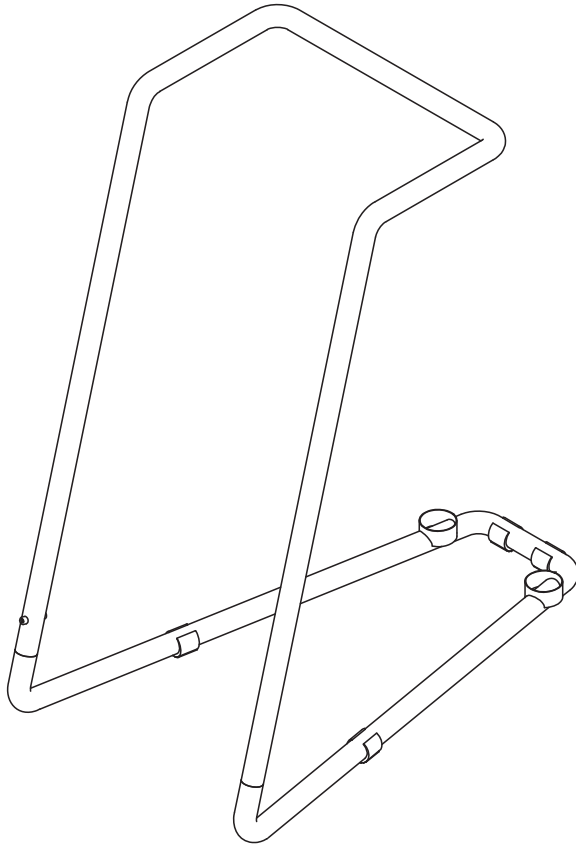
Cancel timing:


ACCESSORIES E PARTS

The device ORMA could be combined to the accesory “Maniglia”, easy to assemble and use for its support functionality of the device during use.

Maniglia: 2910S

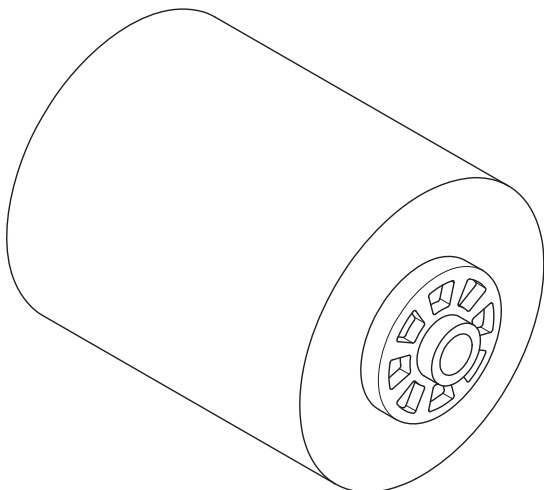About spare parts we made reference to film coils.

Film coil: 2912S

DEMOLITION AND DISPOSAL

The device ORMA has classified as RAEE it response to directive on waste from electrical and electronic equipment (WEEE)

Pursuant to DIRECTIVE 2002/96/EC, this symbol shows that the product must not be disposed of as urban waste at the end of its operating life.

The product must be taken to centers specialized in separate collection of electric and electronic equipment or to a dealer, when a similar product is purchased.

The machine holder shall be responsible for bringing it to collection centers.

For further information about collection systems, address the local waste disposal service.

Correct disposal of the equipment, which is no longer used, prevents negative consequences for the environment and human health.

Abuses will be prosecuted by law.

For the management of demolition and disposal outside the Italian market, apply the existing laws in each country to regulate the WEEE training.

PRECAUZIONI

- Keep the film outlet unhindered.
- Do not press the footboard when the film feeding.
- Prevent child to use it or put hand into the footboard.
- Before moving the machine, unplug the power wire from the machine.
- Keep the bottom ventilated and do not block ventilation slot at the bottom with foreign matter.

TROUBLESHOOTING

FAILURE	REASON	TROUBLESHOOTING METHOD
Blocking	External blocking	Pull out the film from the outlet for next usage or withdraw it for another installation.
	Internal blocking	Contact professional personnel for repair
Non-separated film	Serious blocking	Withdraw it for another installation
	Non-functional	Contact professional personnel for repair
Delivered film bias	The film is not normally located	Re-install it according to the page 6-7; pull the film in place until the film position is correct
		Open the top cover to check the rotation of the coil. If interference is found, get rid of it.
Delivered direction bias	Natural wind too high	Place the machine in the leeward location
	The air outlet loosened or bias	Contact professional personnel for repair
Poor shaping effect	Shearing problem	Contact professional personnel for repair
	Bottom outlet for ventilation blocked	Check the machine bottom and get rid for it

PROCEDURES FOR SERVICE AND ASSISTANCE

In case of failure or revision, please contact **TECNO-GAZ S.p.A.** helpline

PHONE	+39 0521 8380
FAX	+39 0521 833391
@	service@tecnogaz.com

The Service Department will evaluate whether the machine must be sent back to the manufacturer or the intervention of a technician is required and, after inspecting the machine, an estimate will be drawn up and forwarded to the distributor, who will then transmit it to the end customer for acknowledgement and signature.

After receiving the estimate signed for acceptance, the device will be processed and shipped within the time indicated in the estimate.

The following steps are mandatory if the machine must be shipped to the company:

- Use the original packaging box. If you no longer have it, use adequate packaging. Transport risks are sender's charge.
- Ship only the device (do not add any component of the accessory kit).
- Thoroughly clean the device, before shipping it. Should it come dirty, you will be charged for the cost of cleaning and disinfection.
- Please include a document which is clearly specifying the anomaly or the service you are requesting.
- Always send with carriage paid, otherwise you will be charged for the transport costs incurred.

All non-original packaging that we are receiving will be disposed of.

The device will be send with in a new original packaging box (you will be charged for the packaging cost) through your shipping agent.

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En cas de panne ou de révision, contacter directement le service d'assistance téléphonique **TECNO-GAZ S.p.A.**