Ekstraplus

Art. PT002TTM

Art. PT004TTM

<i>Istruzioni per l'uso</i>	<i>ITALIANO</i>
<i>Instructions for use</i>	<i>ENGLISH</i>
<i>Mode d'emploi</i>	<i>FRANÇAIS</i>
<i>Bedienungsanleitung</i>	<i>DEUTSCH</i>
<i>Instrucciones de uso</i>	<i>ESPAÑOL</i>

0TTMI0003
26.09.2013

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
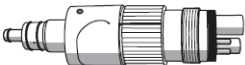
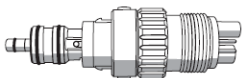
TABLE OF CONTENTS

- 1. APPLICATION AND INTENDED USE**
- 2. SAFETY**
 - 2.1 SAFETY DEVICES
 - 2.2 DISPOSAL
- 3. SPECIFICATIONS**
- 4. PACKAGING**
 - 4.1 PACKAGING
 - 4.2 TRANSPORT
 - 4.3 STORAGE
 - 4.4 SUPPLY
- 5. DESCRIPTION OF THE DEVICE**
- 6. INSTALLATION**
 - 6.1 CONNECTIONS
 - 6.2 FILLING THE DISINFECTANT/DETERGENT AND LUBRICANT TANKS
- 7. OPERATION**
 - 7.1 OPENING AND CLOSING THE DOOR
 - 7.2 DESCRIPTION OF THE CONTROL PANEL
 - 7.3 DESCRIPTION OF THE SOFTWARE
 - 7.4 DESCRIPTION OF THE MENU
 - 7.5 CONNECTION AND TYPE OF INSTRUMENTS
 - 7.6 DISPLAY DURING THE CYCLE
 - 7.7 ERRORS: CAUSES AND SOLUTIONS
- 8. SAFETY DURING OPERATION**
- 9. CYCLE ALARM MESSAGES**
- 10. EMISSION OF HARMFUL SUBSTANCES**
 - 10.1 SIGNALS
 - 10.2 ERRORS
 - 10.4 GENERAL PROBLEMS
- 11. END OF CYCLE**
- 12. MAINTENANCE**
- 13. PROCEDURES FOR SERVICE AND ASSISTANCE**
- 14. ACCESSORIES**

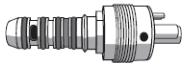General instructions and information for the recipient.

Pay attention to the indications on the side of this symbol.

1

APPLICATION AND INTENDED USE

EKSTRA PLUS is a unit intended exclusively for the Dental Industry for the maintenance of dental handpieces and turbines, according to the instructions in this manual, which forms an integral part of the device.

The device performs a default maintenance cycle to clean and lubricate handpieces and turbines. Maintenance is performed inside the air and water ducts of the turbines and handpieces as well as in the compartments of the moving mechanical parts.

This is performed automatically according to the mode selected by the user, without the risks that derive from handling contaminated instruments.

The device can perform 2 types of preset maintenance: COMPLETE or just LUBRICATION.

Another program EKSTRA doubles the phases of group A: CLEANING, DISINFECTION and BLEED

PHASE		DESCRIPTION	COMPL.	LUB.
BLEED		Air injection in all the ducts of the handpiece and/or turbine to remove the liquid or solid particles that penetrate the instrument during use.	15 s station	15 s station
A	CLEAN	This phase reduces over 90% of the amount of microbial contamination. It removes the residual organic material from the previous procedure.	MAX 15 s station	N.A.
	DISINFECT	Chemical disinfection is only performed if suitable liquid disinfectant is used in the percentages specified by the manufacturer.	300 s (adjustable from 1' to 20')	N.A.
	BLEED	Air injection in all the ducts of the handpiece and/or turbine to expel any residue of the cleaning-disinfection process and to dry the treated parts.	120 s station	N.A.
B	LUBRICATE	Lubrication of the moving mechanical parts with specific oil (nebulised for a preset volume) in just the supply duct of the turbine and the handpiece gear compartment.	MAX 15 s station	MAX 15 s station
	BLEED	Air injection in all the ducts of the handpiece and/or turbine to remove the oil particles in excess	180 s station	180 s station

2

SAFETY

The unit must only be used to perform maintenance on instruments whose manufacturer declares them to be compatible with an automatic maintenance system and from a general aspect, solely for the uses intended by the manufacturer of the EKSTRA PLUS device.

The device must not be used in potentially explosive atmospheres.

Place the device away from sources of heat or electromagnetic radiation.

Place the unit with the control panel and the screen clearly visible by the operator.

The electrical system must comply with regulations in force. Verify that the system is adequate for the characteristics of the device.

Check the integrity of the power cord.

In accordance with the regulations in force, the manufacturer is responsible for the product placed on the market. Responsibility ceases the moment when operations are performed on the device by unqualified personnel or with non-original spare parts.

DO NOT OPEN THE DEVICE.

Disconnect the power cord in the event of a fault or risk of liquid penetrating the device and/or when maintenance and cleaning operations must be performed on the device.

Disconnect the power cord from the socket before performing any operation.

Follow the instructions on the safety data sheet of the liquid used.

Wear suitable PPE.

Do not remove the label.

2.1 Safety devices

The unit is supplied with the following safety devices:

- Electromagnetic lock to prevent the door from being opened during the cycle.
- Chamber aerosol suction (Para. 10).

2.2 Disposal

This product is subject to Directive 2002/96/EC of the European Parliament and of the Council of the European Union on waste electrical equipment (WEEE). In jurisdictions adopting this Directive, the product was launched on the market after 13 August 2005 and must not be disposed of as non-recyclable household waste. Use the relative local WEEE collection facilities for the disposal of this product or comply with the regulations in force.

For further information concerning collection systems, contact your local waste disposal service.


Adequate disposal of equipment no longer used prevents negative effects on the environment and human health. Any violation will be prosecuted according to law.

The machine is constructed with ferrous materials and thermoplastic and electromechanical components. The electronic board consists of commonly used electronic components as well as a mini battery. When disposing of the components of the machine, contact the local recovery and disposal facilities, in accordance with the laws in force in the country where the machine is used.

Do not leave the machine in unattended areas or in an unstable or dangerous position.

3

SPECIFICATIONS

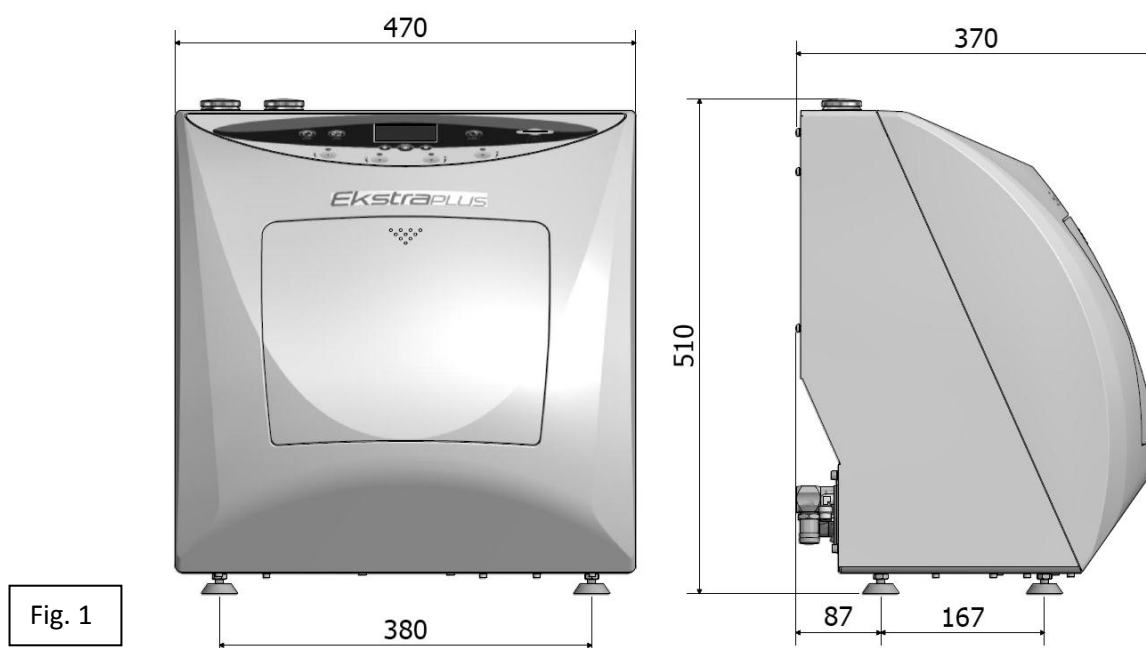MECHANICAL	Operating temperature	+5°C to +30°C
	MAX Relative humidity at 30°C	80%
	Overall dimensions (L x H x P) [mm]	470 x 510 x 370 mm
	Packaging dimensions (L x H x P)[mm]	580 x 540 x 565 mm
	Dimensions with door open (P)[mm]	600 mm
	Weight (empty)	24 kg.
ELECTRICAL	Power supply voltage	230 VAC +/-10 % single phase
	MAX Absorbed power	30 W
	Frequency	50 / 60 Hz
	External earth protection	PE
	Power cord	2 + 1 x 1 mm ²
	Fuses	2 x 2A (Ø5 x 20 mm)
PNEUMATIC	Non-lubricated dry air pressure	4-8 bar
	Consumption	70 NI/min
	Supply pipe with quick coupling	Ø 8 mm
TANKS	Volume	1 l
	Oil consumption	0.25 ml
	Detergent consumption	2 ml
	Material	Polyethylene

4

PACKAGING

Never lift the device from the front guard. This incorrect operation could damage the unit.

Do not use devices that have been visibly damaged during transport.

The packaging must be kept throughout the warranty period. The manufacturer does not accept returns without the original packaging.

Once unpacked, check the condition of the device.

4.1

Packaging

The machine is closed in a corrugated cardboard box and stabilised with packaging material.

This cardboard box is secured with packing tape and strapped to a pallet.

The serial number is marked on the packaging of the machine.

4.2

Transport

Transportation must be carried:

- a) with no jolts and collisions.
- b) with no overturning.
- c) with no exposure to moisture.

The machine (24 kg) must be lifted out of the packaging manually by 2 persons with the relative straps.

4.3

Storage

The machine must be kept in a dry place and at a temperature that ranges between -5 °C and +40 °C. **It is forbidden to stack machines.**

4.4

Supply

Besides the device, the supply includes the following:

USER MANUAL: To be read carefully and stored in a place that is accessible to all operators assigned to operate the device. IT FORMS AN INTEGRAL PART OF THE DEVICE.

DECLARATION OF CONFORMITY: to be kept.

INSTALLATION - TESTING REPORT AND WARRANTY TERMS AND CONDITIONS: To be completed according to the instructions in the form when the machine is installed.

QUICK START GUIDE: To be kept near the machine.

Recommended lubricant (See par. 15. Accessories)

Recommended detergent (See par. 15. Accessories)

PNEUMATIC SUPPLY PIPE

Use to install the pneumatic supply

Supply provided
3 m – Code CP01009

Fig. 2

EXTENDABLE DRAIN PIPE

Use to install the drain at the back of the device

Supply provided
1 pc – Code CM61032

Fig. 3

POWER CORD

Use to install the power supply

Supply provided
1 pc – Code 3ELEE0001

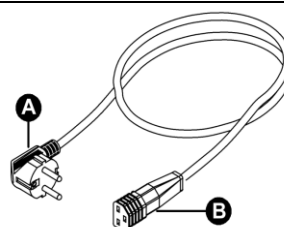
Fig. 4

MEMORY CARD

Use to store cycles. (It is preferable to always use the original memory card for perfect compatibility with the machine).

IMPORTANT: The SOFTWARE version is updated via the card (Para. 7.4)

Supply provided
1 pc – SPFA243

Fig. 5

5**DESCRIPTION OF THE DEVICE**

EKSTRA PLUS comes as a device that is to be used on a counter. The front part is made of thermoplastic material and the rear in painted sheet metal.


Fig. 6

- The control panel at the front has 4 buttons to select the stations and 6 buttons to control the process; a display for displaying the functions of the machine.
- The front part includes the compartment for maintenance to be performed on the instruments in 4 stations, for the turbines and for the handpieces. This compartment is closed with a door that is opened and closed manually.
- The cabinet contains all the electric, pneumatic and hydraulic components for machine operation:
 - A 1-litre tank for the disinfectant with a filler inlet (Fig. 8), a drain outlet (Fig. 9) and a level indicator.
 - A 1-litre tank for the lubricant with a filler inlet (Fig. 8), a drain outlet (Fig. 9) and a level indicator.
 - A device for the internal rotation of the handpieces.


OPERATION: Maintenance is performed inside the air and water ducts of the turbines and handpieces as well as in the compartments of the moving mechanical parts.

This is performed automatically according to the mode selected by the user: COMPLETE or just LUBRICATION.

An additional EKSTRA cycle performs a complete treatment with double detergent cleansing and drainage, if the user deems it necessary.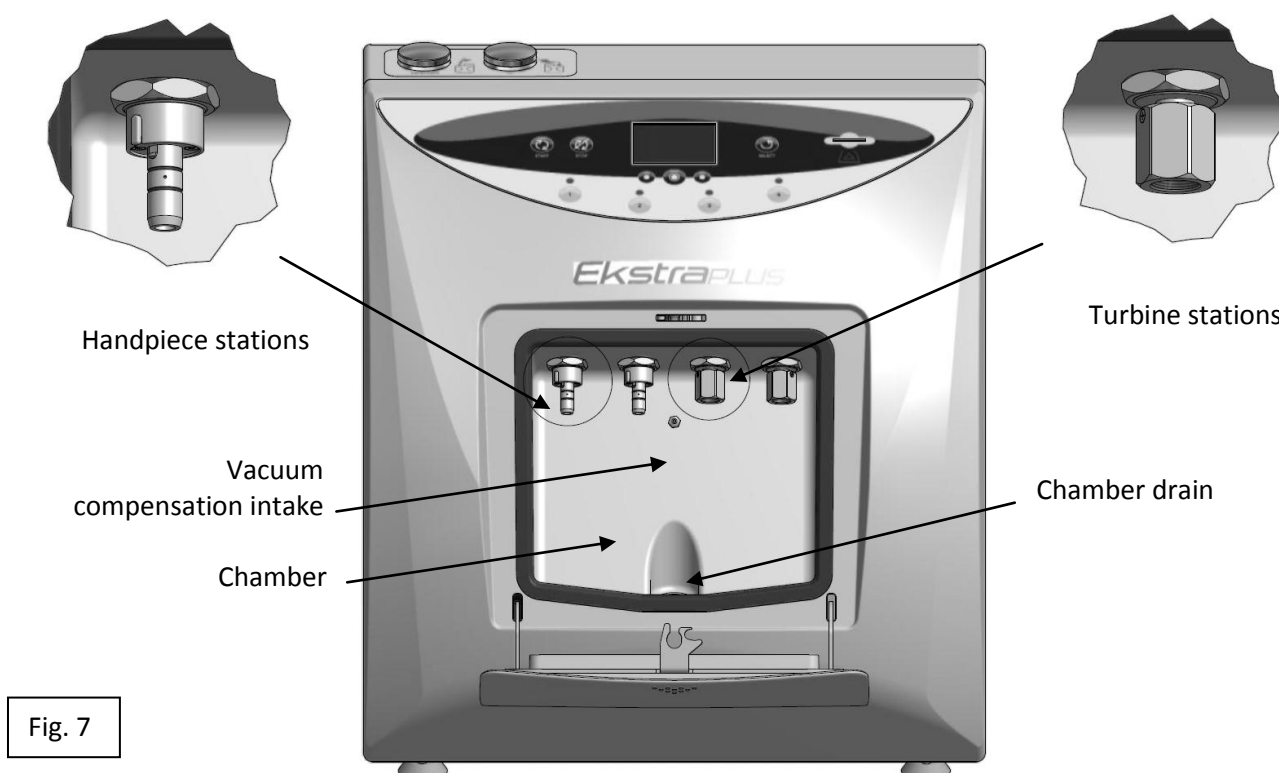

There are nozzles in the upper part for loading the liquid used for the treatment of the instruments.

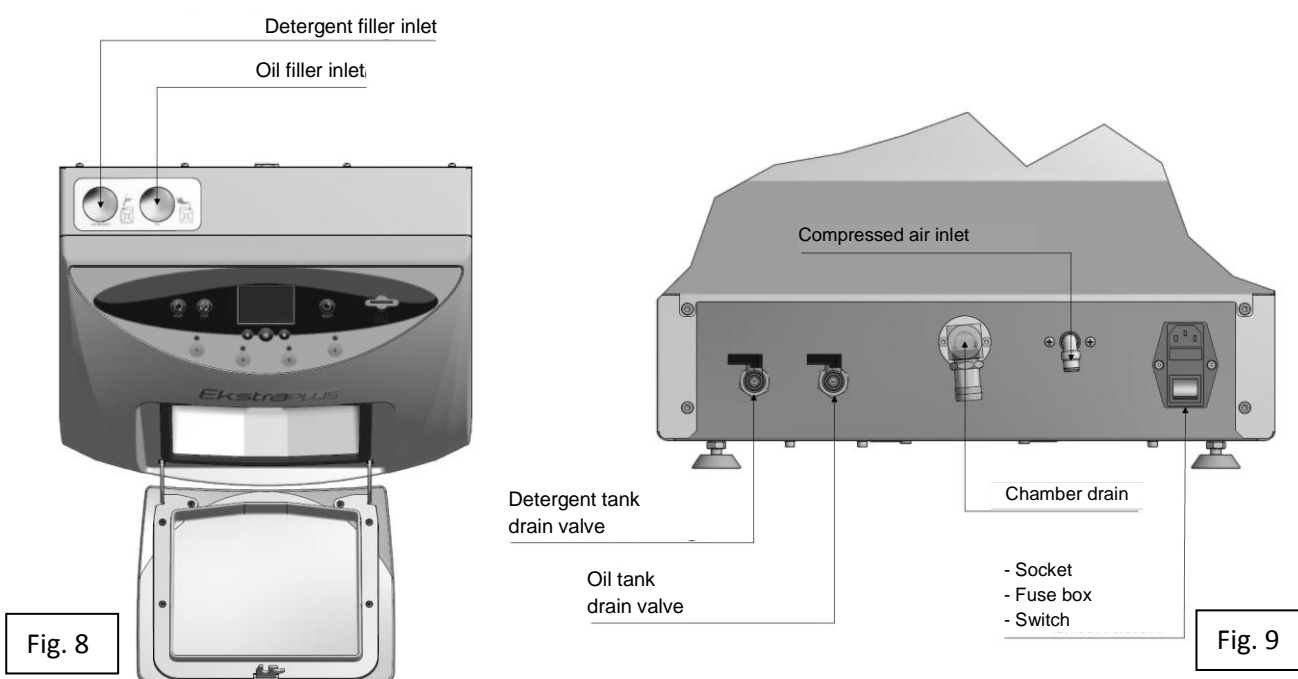The following are found in the rear part:

- The valves to drain the tanks that contain the liquids,
- The outlet to drain the liquids from the maintenance compartment,
- The pneumatic supply connector,
- The power supply connector with fuses and a switch.

6**INSTALLATION**

The machine must be **installed** by authorised personnel

The device must not be used in potentially explosive atmospheres.

The device must be placed away from sources of heat or electromagnetic radiation.

The electrical system must comply with regulations in force. Verify that the system is adequate for the characteristics of the device.

The machine must be removed from the packaging by the technician who not only installs the machine but must also verify the condition of the goods.

The criteria and the method with which the machine is installed are also of significant interest for the user.

The packaging components must be kept throughout the warranty period and must be used if the machine is to be shipped.

It is advisable not to set the machine in a tight space in order to be able to move freely during use and facilitate access to various parts.

The space above the machine must not have wall cabinets or shelves that could prevent access to the liquid filler inlets.

The lighting system of the installation area must comply with current regulations.

Before placing the machine, calculate the height of the counter according to the height of the operator.

Before positioning the machine, calculate the correct height of the worktop. The operator must be able to easily see the display and access the filler inlets.

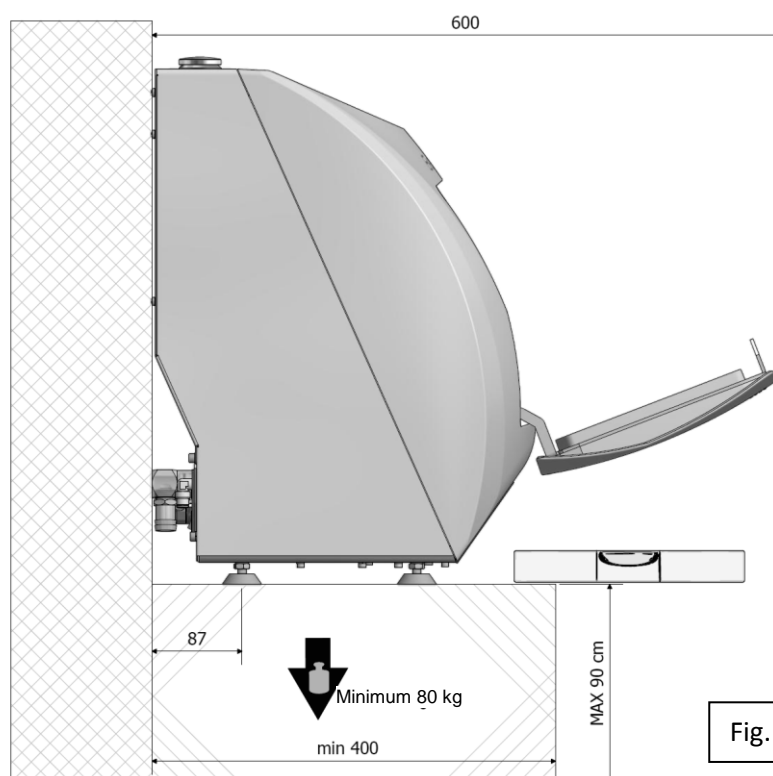
Fig. 10

Place the machine on a counter that is at least **90 cm** high and **perfectly level**.

Place the device so as to allow the operator to easily inspect and clean the device. The operator must be able to access the back of the device

Place on a perfectly level surface, which is stable and can withstand loads of at least **80 kg**.

The supports of the device can be slightly adjusted.

AN INCORRECT GRADIENT CAUSES RESIDUE TO DEPOSIT AT THE BOTTOM OF THE CHAMBER


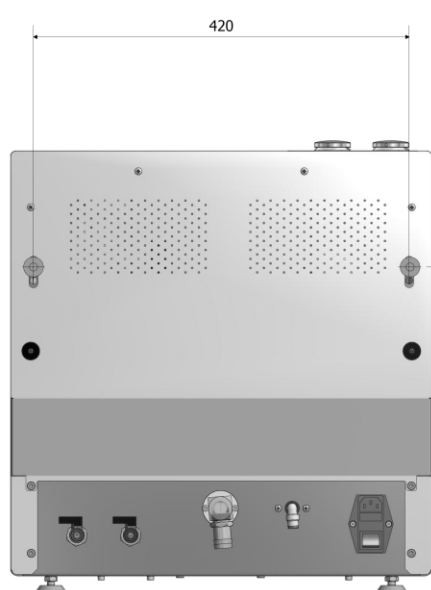
Fig. 11

6.1 Connections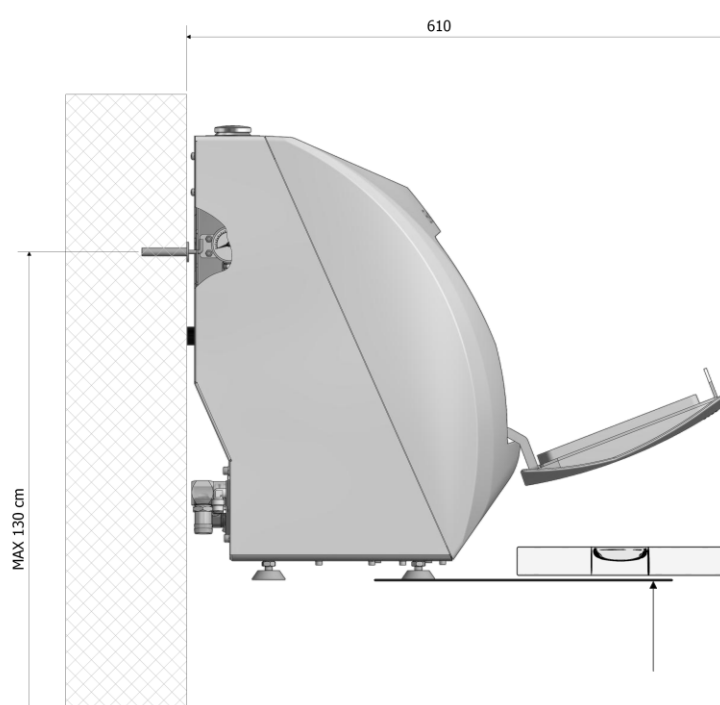

The position of the cables and pipes must not pose a tripping hazard for the operator and the personnel.


Check the integrity of the power cord.

1) Insert the power supply pin (Fig. 12); insert the other end into a socket with the following characteristics:

- A) 230V single phase + earth
- B) Frequency 50 Hz
- C) 10 A minimum current with a circuit breaker and RCD.

The socket must be at the side of the machine, which is reached by the supplied cable.

2) Connect the blue pipe, supplied with the swivel coupling Fig. 12.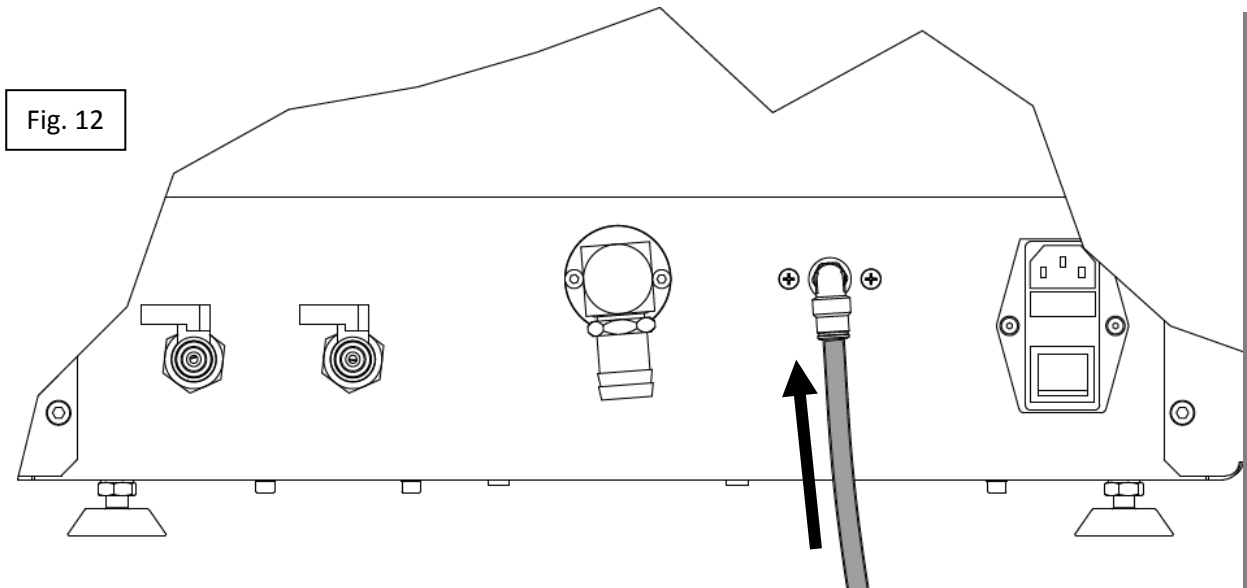


then to a source of compressed air with the following characteristics:

- A) dry and non-lubricated compressed air
- B) pressure between 6 and 8 Bar
- C) minimum flow rate of 100 l/min

If, after switching the machine on, the message “**LOW PRESSURE**” appears (Para. 9 Screen A1), it means that the air pressure is less than 4 Bar; restore the pneumatic supply.

3) Connect the spent fluid drain pipe to the swivel hose connection of the drain of the device (Fig. 13).

Connect the other end to the sewage drain, making sure the inlet of the pipe is sealed well (Fig. 14).

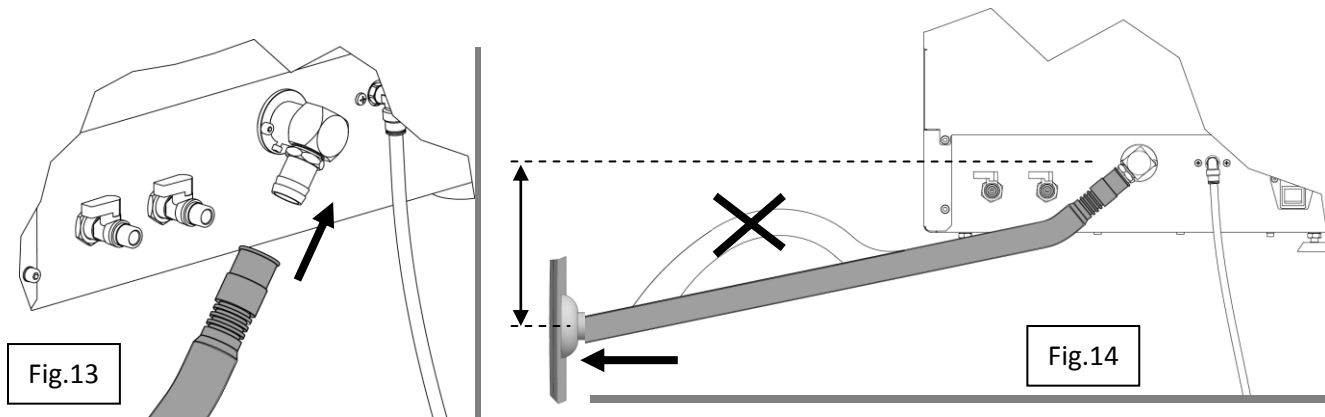The device drains by gravity and therefore, the drain pipe must descend with respect to the machine.

Prevent any upward bend in the drain pipe (siphons) (Fig. 14) that could generate stagnation. There must be no non-return valves.


ATTENTION: The sewage drain must be lower than the drain outlet of the machine (Fig. 14) and the connection pipe must not be bent below the sewage drain.

In the absence of a sewer line, in a ventilated area, you can use a tank accumulation:

- Do not immerse the exhaust chamber in the liquid accumulated in the tank;
- The tank should not be sent to pressure or sealed.

6.2 Filling the disinfectant/detergent and lubricant tanks:

The tanks can contain over 1 l; however, do not exceed this amount when filling the tanks.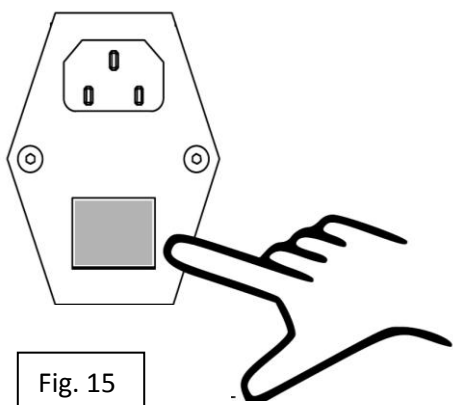

After connecting the above, **move the main switch behind the device** (Fig. 15) from **0** to **I**;

The display lights up and the manufacturer logo appears for a few seconds (screen A) and immediately after message indicating a low level in the tank (Para. 9: Screen A1).


ALWAYS SWITCH THE MACHINE ON BEFORE FILLING THE TANKS


MAKE SURE THE TANK DRAIN VALVES (Fig. 9) ARE CLOSED.

The device must be switched for the filling operations and the next option must be displayed: **'CYCLE SELECTION'** (Para. 7.3 Screen B).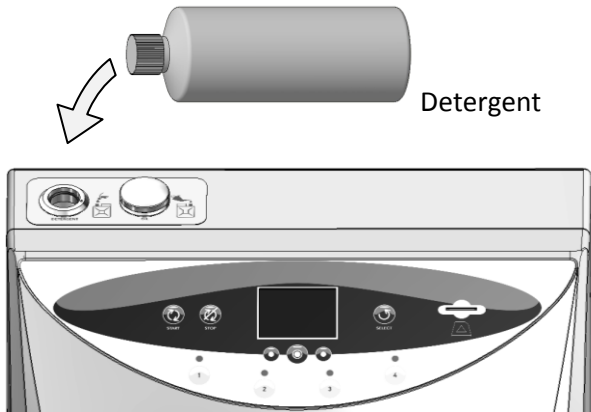

Detergent

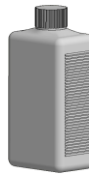Oil

Lift the cap of the liquid that is to be topped-up;
Pour the liquid (preferably using a funnel) until a beep and visual signal (on the display) indicate that the tank is full.

Fig. 16

7

OPERATION

Once the connections are made and the maintenance liquids have been poured, the device is ready for operation.

7.1 Opening and closing the door

The door lock snaps open without a handle.

- The door is closed by simply pushing it upwards until the latch enters the relative outlet (Fig. 17a);
- Pressing the part near latch of the door will snap the lock closed; refer to (Fig. 17b). Once the door islet go, it remains closed against the guard.

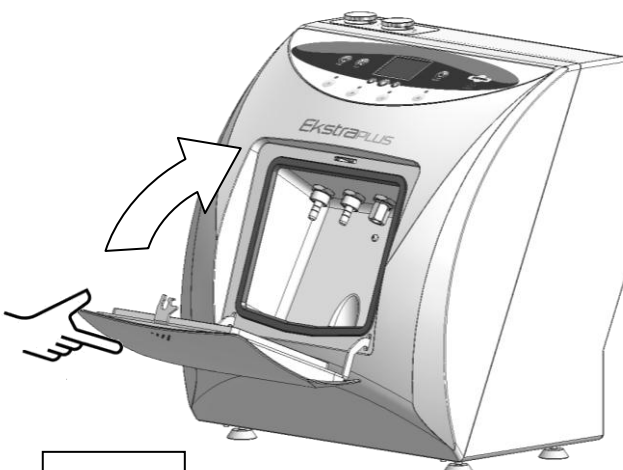
Fig.17a

Fig.17b

- The door is opened by pressing the upper middle part of it (the same area that is pressed to close it) until the lock snaps; once the door is let go, it pivots downwards autonomously thanks to the return springs (Fig. 17b; Fig. 17a).

The door cannot be opened during the cycle as it is locked and is only unlocked at the end of the cycle or if the STOP command is activated.

7.2 Description of the control panel

The control panel consists of:

Fig. 18

- A display that shows the instrument maintenance process, the functions available for it to be controlled (MENU), the alarms or errors during operation and the functions reserved for the maintenance technicians (TECHNICAL MENU).
- The buttons below the display are used to control the screens; their functions are described in the lower area (D) of the screens, in line with the buttons themselves.
- A START button to launch the maintenance.
- A STOP button to stop the maintenance.
- A SELECT button to select the maintenance that is to be performed. COMPLETE and LUBRICATION alternate each time the button is pressed.
- 4 buttons with LEDs to select the stations that are to be activated.

It is possible to inhibit the treatment of water/spray channels, for the treatment of tools that do not have connections for water/spray cooling channels integrated in the quick connection

The inhibition occurs through prolonged pressing (3 seconds) of the button of the machine to which it is connected (Par.7.6)


- A slot for the SD Card.

7.3 Description of the software

Each area of the display shows figures and/or descriptions that represent the condition of the machine during the process, or messages with which the user chooses or sets the various functions available.

SCREEN A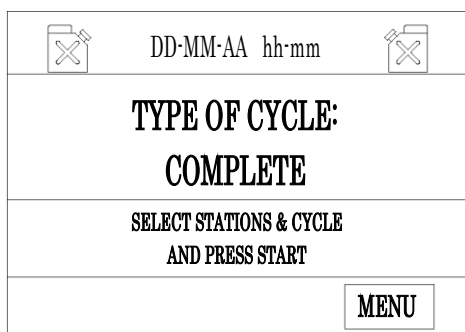

SCREEN B = LAST CYCLE RUN

Operate the main switch at the back of the machine (Fig. 15) for it to be powered and "Screen A", which displays the manufacturer logo and the version of the software, appears for a few seconds on the display.

The main screen (Screen B) that appears immediately after the manufacturer logo is divided into four areas: A, B, C and D.

A

B

C

D

- The date and time appear at the top, in area A.

To the right and left of the date and time, the symbols of the tanks that contain oil and detergent indicate the relative status (full, empty or intermediate).

The last cycle performed appears in area B. The alternatives cycle is retrieved by pressing the SELECT button (LUBRICATION and EKSTRA).

The instructions to set the maintenance are displayed in area C.

As described in Para. 7.4, area D is reserved for the description of the functions of the buttons below;


- On the left side, the DOWN ARROW moves the cursor down and the NEXT ARROW moves the cursor to the right,
- At the centre, OK confirms the selection,
- On the right, EXIT returns to the main menu (screen B).

Only the MENU function is activated in screen B, with which the various options contained therein, can be accessed.

Maintenance begins once the instructions in the previous paragraphs are carried out, the selection described in area C is made and lastly START is pressed.

7.4 Description of the menu

The menu structure is shown in the screen sequence in the figure below.

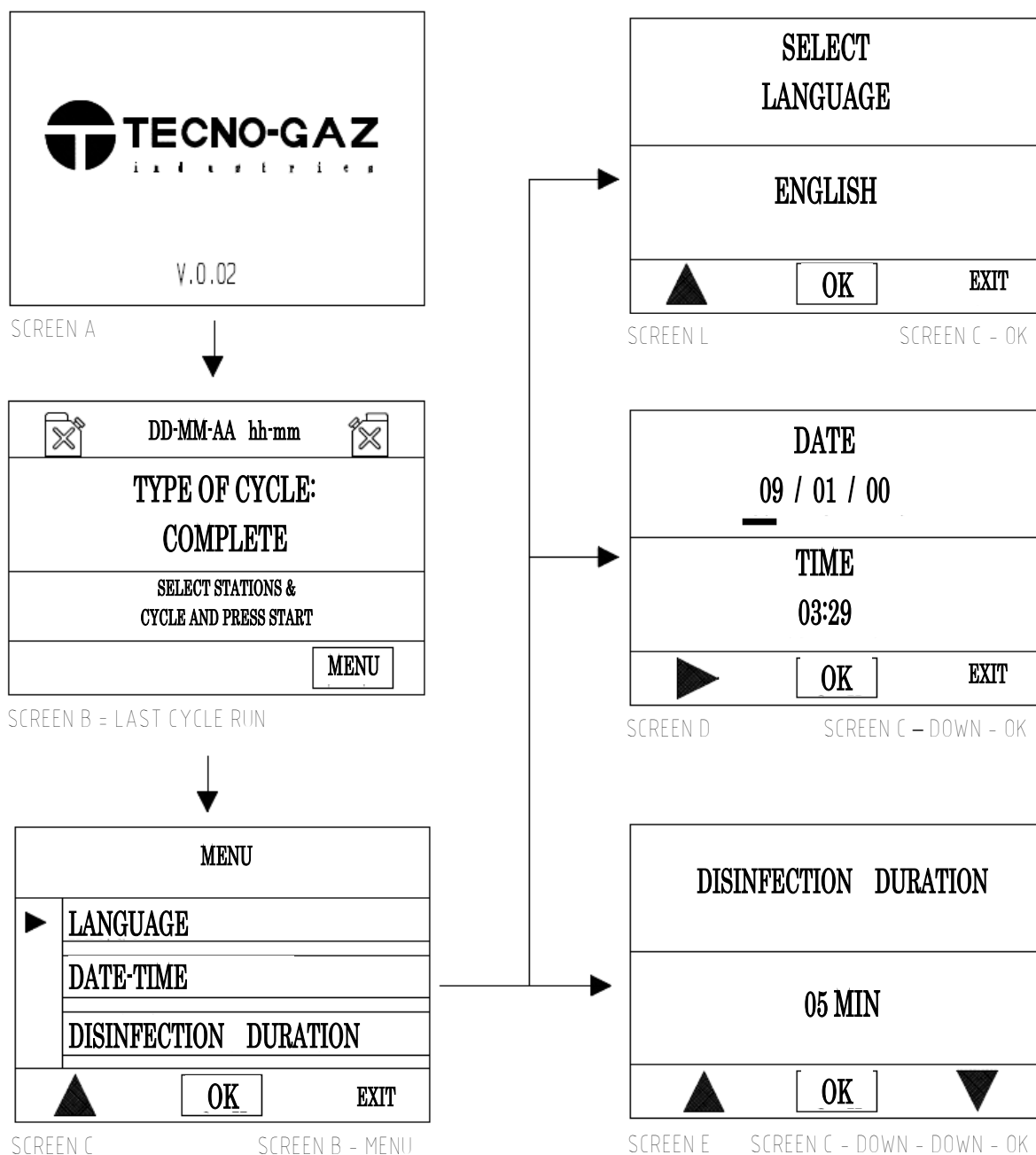As previously mentioned, the menu is accessed by pressing the button below MENU, which only appears in screen B.

The display shows screen C, in which 4 options are given:

- 1) Setting the desired language
- 2) Setting the date and time
- 3) Setting the duration disinfection
- 4) Technical menu reserved for assistance technicians, which is password-protected.

1) SETTING THE LANGUAGE

MENU		
▶	LANGUAGE	
	DATE - TIME	
	DISINFECTION DURATION	
▲	OK	EXIT

SCREEN C

SCREEN B - MENU

When in screen C, move the cursor to LANGUAGE and press OK to confirm for screen L to be displayed.

SELECT LANGUAGE		
ENGLISH		
▲	OK	EXIT

SCREEN L

SCREEN C - OK

Use the DOWN command to retrieve all the languages available: ITALIAN, FRENCH, GERMAN, ENGLISH and SPANISH.

Once the desired language is identified, press OK to confirm.

All the text on the display will immediately appear in the selected language.

Press the command below EXIT to return to screen C.

2) DATE AND TIME

DATE		
<u>09</u> / 01 / 00		
TIME		
03:29		
▶	OK	EXIT

SCREEN D

SCREEN C - DOWN

When in screen C, move the cursor to DATE AND TIME and press OK to confirm and for screen D to appear.

The first digits of the date are underlined, indicating the day.


If these are to be changed, press OK to confirm and screen D1 appears.

FLASHES ↘	DATE	
	<u>09</u> / 01 / 00	
	TIME	
	03:29	
▲	OK	EXIT

SCREEN D1

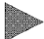SCREEN D - OK

The UP command increases the day progressively;

<div data-bbox="301 248 699 533"> <div>DATE</div> <div>23 / 01 / 00</div> <div>TIME</div> <div>03:29</div> <div>  <div>OK</div> <div>EXIT</div> </div> </div> <div>SCREEN D2</div> <div>SCREEN D1 - OK</div>	<p>Once the correct day is identified, press OK to confirm and screen D2 appears.</p>
<div data-bbox="301 604 699 887"> <div>DATE</div> <div>23 / <u>01</u> / 00</div> <div>TIME</div> <div>03:29</div> <div>  <div>OK</div> <div>EXIT</div> </div> </div> <div>SCREEN D3</div> <div>SCREEN D2 - NEXT</div>	<p>The month will be underlined, as shown in screen D3, when the NEXT button is pressed.</p>
<div data-bbox="301 958 699 1240"> <div>DATE</div> <div>23 / <u>01</u> / 00</div> <div>FLASHES</div> <div>TIME</div> <div>03:29</div> <div>  <div>OK</div> <div>EXIT</div> </div> </div> <div>SCREEN D4</div> <div>SCREEN D3 - OK</div>	<p>Screen D4 appears by pressing OK and the underlining flashes.</p> <p>The UP command increases the months progressively;</p>
<div data-bbox="301 1312 699 1594"> <div>DATE</div> <div>09 / <u>12</u> / 13</div> <div>TIME</div> <div>03:29</div> <div>  <div>OK</div> <div>EXIT</div> </div> </div> <div>SCREEN D5</div> <div>SCREEN D4 - OK</div>	<p>Once the correct month is identified, press OK to confirm and screen D5 appears.</p>
<div data-bbox="301 1664 699 1946"> <div>DATE</div> <div>09 / 12 / <u>13</u></div> <div>TIME</div> <div>03:29</div> <div>  <div>OK</div> <div>EXIT</div> </div> </div> <div>SCREEN D6</div> <div>SCREEN D5 - NEXT - OK - OK</div>	<p>If the year must also be changed, proceed in the same way: NEXT will move the underlining to the year and the relative digits will flash when OK is pressed.</p> <p>The year is changed by pressing the UP command and then press OK to confirm and for screen D6 to appear.</p>

<div> <div>DATE</div> <div>09 / 12 / 13</div> </div> <div> <div>TIME</div> <div>03:29</div> </div> <div> <div>▶</div> <div>OK</div> <div>EXIT</div> </div>	<p>The time is changed by moving the underlining to the hour with the NEXT command.</p> <p>The hour and minutes are set in the same way as the date.</p>
--	--

SCREEN D7

SCREEN D6 - OK

Press the button below EXIT at any point while modifying the data and the changes are saved before returning to MENU in screen C (Para. 7.4).

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<div> <div>DISINFECTION PERIOD</div> <div>05 MIN</div> </div> <div> <div>▲</div> <div>OK</div> <div>▼</div> </div>	<p>From video C, moving the cursor to the "DISINFECTION DURATION" position and confirming by clicking on "OK" makes video "E" appear.</p> <p>With the command "UP"/"DOWN", the minutes needed for disinfection can be increased/decreased;</p> <p>Once the time period is identified, confirming by clicking on "OK" makes video C appear.</p>
--	--

SCREEN E

SCREEN C - DOWN - DOWN - OK

4) TECHNICAL MENU

<div> <div>PASSWORD</div> <div>0 - 0 - 0 - 0</div> </div> <div> <div>▶</div> <div>OK</div> <div>EXIT</div> </div>	<p>With the cursor on TECHNICAL MENU, press OK to confirm and the password field is highlighted</p> <p>The user cannot access this menu.</p>
---	---

SCREEN T

SCREEN C - DOWN - DOWN - DOWN - OK

5) LABORATORY DATA SETTINGS

```

TECNO-GAZ INDUSTRIES
EKSTRA S/N PFxxxxxxx
VERSION: xxx

CLINIC:xxxxxxxx
ADDRESS:xxxxxxx
POSTCODE:xxxxxxx
CITY: xxxxxx
PROVINCE:xxxxxx
DATE:xxxxxx

COMPLETE CYCLE
START:xxxxxxx
END:xxxxxxx
STATIONS: 1 2 3 4
CYCLE OK
  
```

As mentioned, the user can store all the cycles performed on the SD card.




- Device data
- Part of the customised text report on the user's computer. The user can enter the clinic's data and/or other text with notes that are deemed relevant:

Create/complete the text file in the SD card and name it **SET_UP.TXT**.





- Cycle data:
 - Type of cycle selected;
 - Cycle start/end;
 - Stations concerned;
 - Cycle result or alarm message.

7.5 Connection and type of instruments

Maintenance is performed on handpieces and turbines inside the chamber. They must be connected correctly as described below.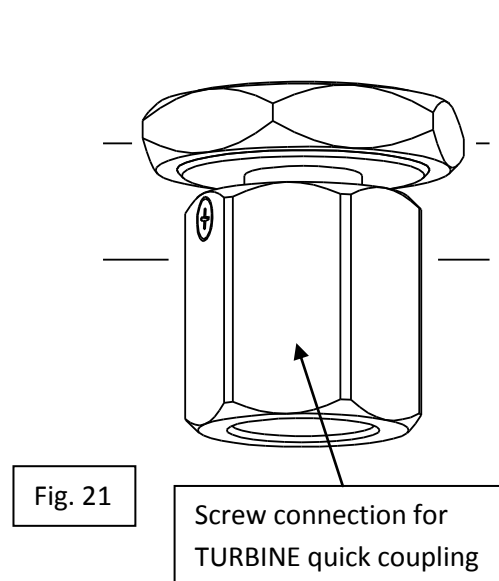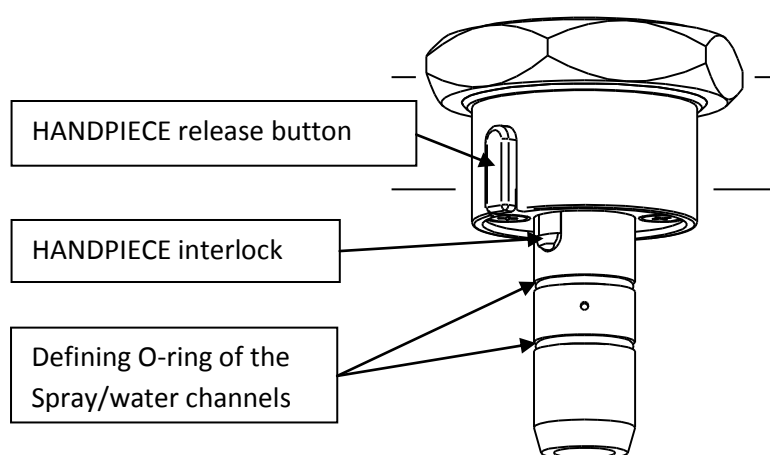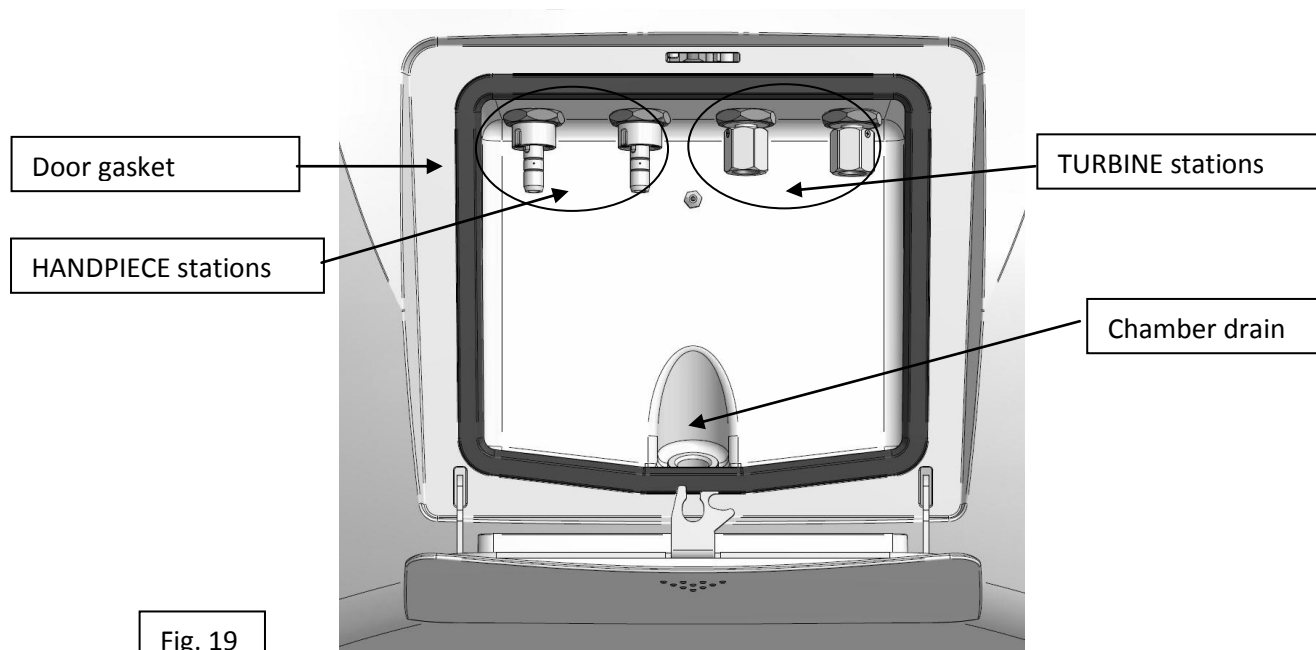

The device can host:

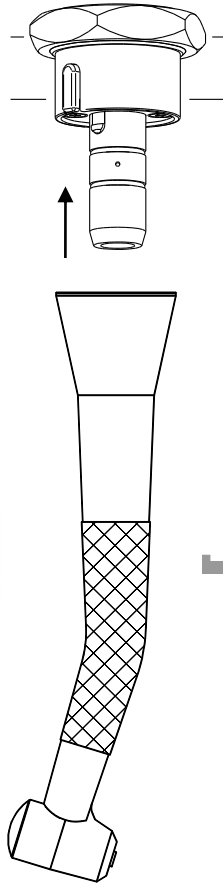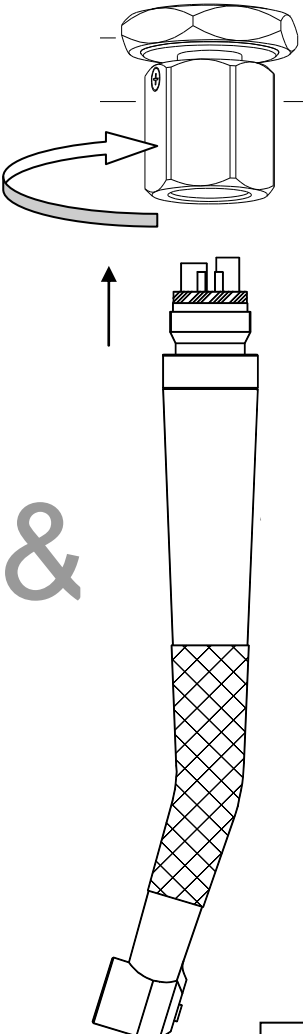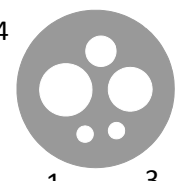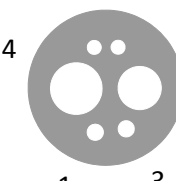Stations 1 and 2 Handpiece (ISO 3964 E-Type coupling)	Stations 3 and 4 Turbine with MIDWEST-type quick adapter (ISO 9168 Type B/C)
 <p>Fig. 21</p>	 <div style="display: flex; justify-content: space-around; align-items: flex-start;"> <div style="text-align: center;">  <p>ISO 9168 Type B</p> </div> <div style="text-align: center;">  <p>ISO 9168 Type C</p> </div> </div> <p>Channels:</p> <ul style="list-style-type: none"> 1) Water 2) Actuating air 3) Spray air 4) Drain 5) Fibre optic 6) Electrical connection <p>Fig. 22</p>
The LUBRICATION phase involves:	
Handpiece central channel	2. Turbine actuating air 4. Drain
The DISINFECTION/CLEANING phase involves:	
Spray/water channel Handpiece central channel	1. Water 2. Actuating air 3. Spray air 4. Drain

7.6 Display during the cycle

As previously mentioned, after performing the following steps:

- Inserting the instruments
- Closing the door
- Selecting the stations and type of cycle

The cycle is launched by pressing the START button.

Since the treatment, as previously shown in Fig.20 and Fig.22, is applied to individual channel types

it is possible to inhibit the treatment of water/spray channels, for the treatment of tools that do not have connections for water/spray cooling channels integrated in the quick connection:

DO YOU WANT TO EXCLUDE
THE TREATMENT FOR
WATER AND SPRAY PIPES
P2 ?

OK

EXIT

VIDEO J


VIDEO B x P2 (3 s)

Prolonged pressing (3 seconds) of the button of the machine to which the tool is connected generates the message of video "J" (e.g. Machine 2);

- it is possible to exit without changing the setting or you can confirm your choice with OK;
- If the change is confirmed, the LED of the machine flashes to indicate the different setting
- Pressing the button of the modified machine resets the settings


COMPLETE / EKSTRA CYCLE:

The cycle is performed according to the sequence of the steps listed below and shown on the display.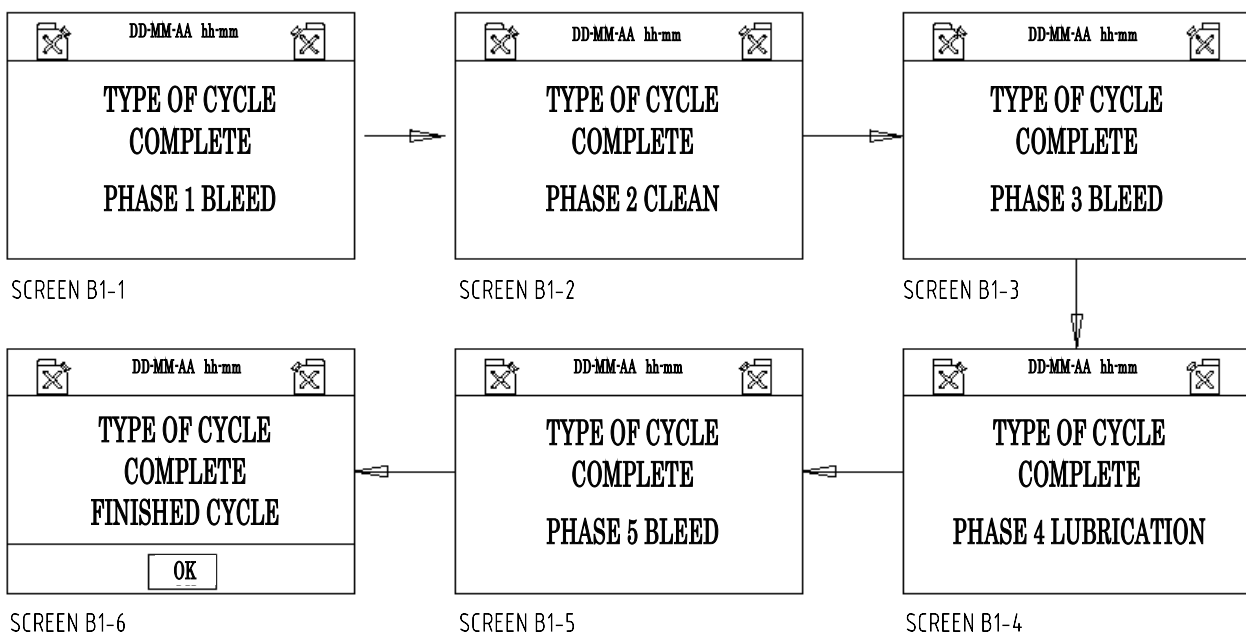




LUBRICATION CYCLE:





The lubrication cycle is performed according to the following steps: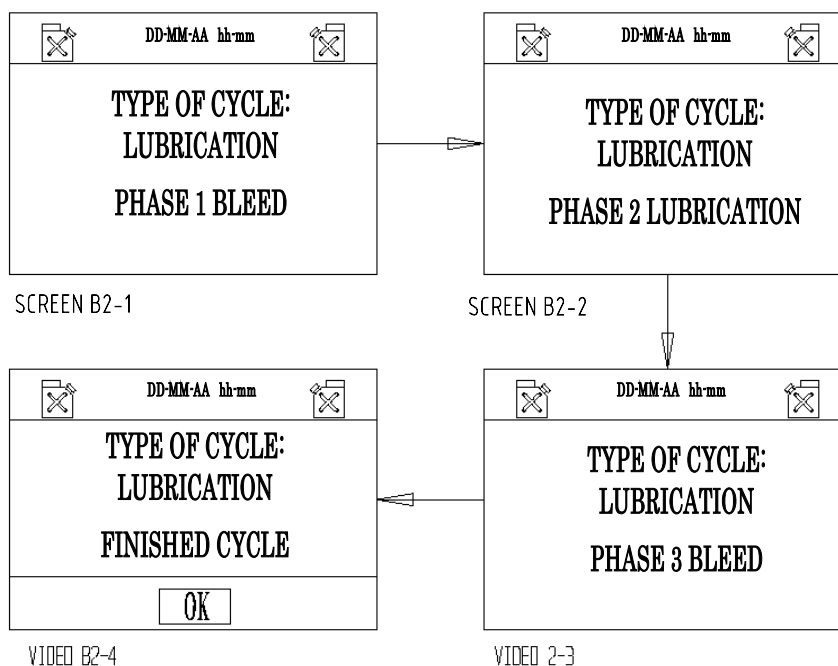

7.7 Errors: causes and solutions

Situations that may occur prior to operation and prevent the cycle from being launched:

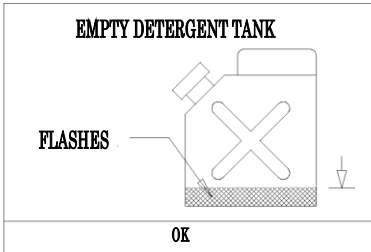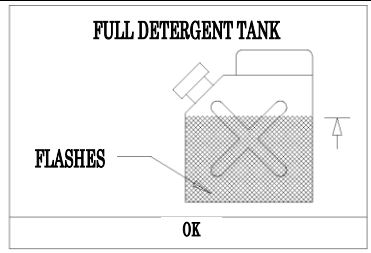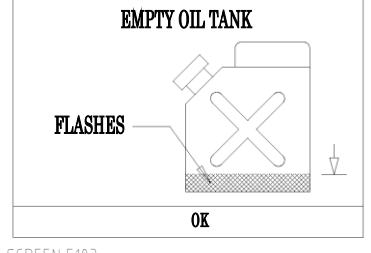 <p>SCREEN E101</p>	<p>CAUSES: This is displayed when the user presses START and the level of detergent is insufficient to perform a cycle.</p> <p>SOLUTIONS: Pour detergent into the tank.</p>
 <p>SCREEN E102</p>	<p>CAUSES: This is displayed when the level reaches the maximum limit while filling the tank, accompanied by a beep.</p> <p>SOLUTIONS: Suspend the filling operations and put the cap on the filler inlet of the tank.</p>
 <p>SCREEN E103</p>	<p>CAUSES: This is displayed when the user presses START and the level of oil is insufficient to perform a cycle.</p> <p>SOLUTIONS: Pour oil into the tank.</p>

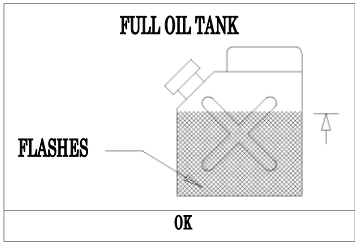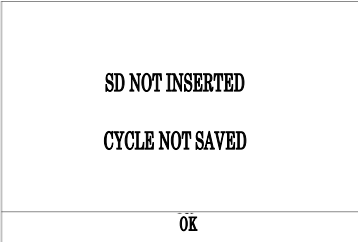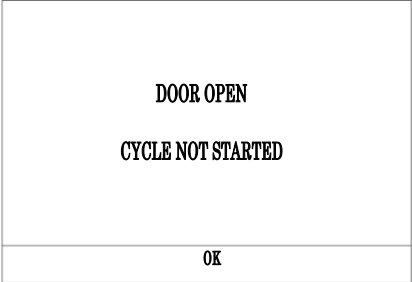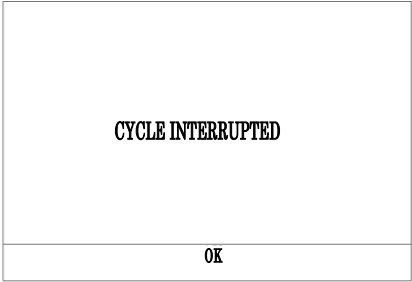 <p>SCREEN E104</p>	<p>CAUSES: This is displayed when the level reaches the maximum limit while filling the tank, accompanied by a beep.</p> <p>SOLUTIONS: Suspend the filling operations and put the cap on the filler inlet of the tank.</p>
 <p>SCREEN E105</p>	<p>CAUSES: Screen E105 appears if START is pressed and no SD is inserted.</p> <p>SOLUTIONS: The cycle is launched and not saved if OK is pressed to confirm. Insert the SD for the cycle to be launched and saved.</p>
 <p>SCREEN E106</p> <p>MICRO M2 NOT ACTIVE</p>	<p>CAUSES: This is displayed when the cycle is launched with the door open or when the closing device does not work.</p> <p>SOLUTIONS: Press OK to confirm, close the door and restart the cycle. If the screen with the door closed means a faulty lock and require a technical intervention.</p>
 <p>SCREEN E107</p> <p>MICRO M2 NOT ACTIVE</p>	<p>CAUSES:</p> <ol style="list-style-type: none"> 1) If STOP is pressed during operation, the cycle is interrupted and screen E107 appears. 2) If a fault occurs in the lock during the cycle, the cycle is interrupted. <p>SOLUTIONS:</p> <ol style="list-style-type: none"> 1) Press OK and screen B appears. 2) Request technical support.

8

SAFETY DURING OPERATION

The cycle can be launched if the door is closed.

The door is locked by the safety device upon the START command and remains blocked until the cycle is completed.

The door cannot be opened during the cycle.

The cycle is launched from the beginning when START is pressed.

The STOP command interrupts the cycle, resets the functions performed and unlocks the door.

When there is no disinfectant or oil in the tanks, START is inhibited and this is indicated on the display. When this signal is given during the cycle, the latter is not interrupted.

When there is no air, START is inhibited and the cycle is interrupted, thereby resetting the selections.

Safety during the cycle is controlled by software that displays the errors and anomalies during operation and stores them on the SD card.

<p>ALARM 201</p> <p>LOW AIR PRESSURE</p>
<p>OK</p>

SCREEN A1 LESS THAN 4 BAR

<p>ALARM 202</p> <p>STATION 1</p>
<p>OK</p>

SCREEN A2 DISPENS. DETERGENT

<p>ALARM 203</p> <p>STATION 2</p>
<p>OK</p>

SCREEN A3 DISPENS. DETERGENT

<p>ALARM 204</p> <p>STATION 3</p>
<p>OK</p>

SCREEN A4 DISPENS. DETERGENT

<p>ALARM 205</p> <p>STATION 4</p>
<p>OK</p>

SCREEN A5 DISPENS. DETERGENT

<p>ALARM 206</p> <p>STATION 1</p>
<p>OK</p>

SCREEN A6 DISPENS. OIL

<p>ALARM 207</p> <p>STATION 2</p>
<p>OK</p>

SCREEN A7 DISPENS. OIL

<p>ALARM 208</p> <p>STATION 3</p>
<p>OK</p>

SCREEN A8 DISPENS. OIL

<p>ALARM 209</p> <p>STATION 4</p>
<p>OK</p>

SCREEN A9 DISPENS. OIL

<p>ALARM 210</p>
<p>OK</p>

SCREEN A10 LEVEL L1 + L2

<p>ALARM 211</p>
<p>OK</p>

SCREEN A11 LEVEL L3 + L4

<p>ALARM 212</p>
<p>OK</p>

SCREEN A12 DETERGENT PUMP

<p>ALARM 213</p>
<p>OK</p>

SCREEN A13 OIL PUMP

The display of the above alarms are faults or malfunctions that the user cannot always resolve:

- Press OK, restore the conditions of use and repeat the cycle.
- Unexplained recurrence of these alarms prompts the user to contact technical support by following the instructions in Para. 13.

10

EMISSION OF HARMFUL SUBSTANCES

Toxic mists are produced by the liquids used to perform maintenance on the instruments. The door seal is fundamental for safety in the working environment and hence, for the operator. Periodically make sure that the door seal is perfectly efficient.

The liquids used, together with the relative mists, if any, are extracted through the chamber drain:

- at the end of the cycle
- if the cycle is interrupted (STOP button)
- if an error occurs that interrupts the cycle

TECNO-GAZ CANNOT BE HELD LIABLE FOR ANY DAMAGE DERIVING FROM IMPROPER USE OF THE MACHINE.

11

END OF CYCLE

The set end of cycle is indicated by a beep and FINISHED CYCLE appearing on the screen.

- Press OK to proceed according to the instructions on the screen
- Press the door to open it

Disconnect the instruments from the stations where maintenance has been performed

DRY THE INSTRUMENTS WITH A DRY CLOTH

DRY THE CHAMBER WITH A DRY CLOTH

The maintenance cycle is complete.

12**MAINTENANCE****1) ROUTINE MAINTENANCE**

The machine does not require any special maintenance.

The components that are most exposed to wear and dirt are:

- a) the seal of the door (Fig. 19 Code 1TTMA0072);
- b) The seals of the handpiece couplings (Fig. 20 Code 3MECM0017);

The recommended disinfectant can be used for the periodic cleaning of the maintenance chamber and the cabinet in general.

2) SPECIAL MAINTENANCE

EXTRAORDINARY MAINTENANCE AND REPAIRS MUST BE PERFORMED BY AUTHORISED PERSONNEL.

3) SPARE PARTS

All accessories that are supplied and provided on request are provided as spare parts.

The mechanical, electrical and electronic components, such as spare parts, are only available for repairs carried out by authorised personnel.

As previously mentioned any unauthorised intervention is considered as tampering and subject to the relative consequences (refer to the warranty).

13**PROCEDURES FOR SERVICE AND ASSISTANCE**

Contact the Assistance Department directly in the event of a fault, review and/or validation:

MEDILINE ITALIA s.r.l.

PHONE	+39 0522 94.29.96
FAX	+39 0522 94.47.98
@	service@tecnogaz.com

The Assistance Department will evaluate whether the machine must be returned or whether a technician's intervention is required, and once the machine is inspected, an estimate will be forwarded to the distributor who will forward it to the customer for acknowledgement and signature.

After receiving the signed quotation, the device will be processed and will be shipped according to the schedules indicated on the quotation.


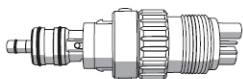
The following steps are mandatory if the machine must be shipped to the company:

	Use the original packaging. If you no longer have this, use adequate packaging. The transportation risks are borne by the sender.
	Send <u>only</u> the device (do not put any component of the accessory kit).
	Thoroughly clean the chamber and the entire device before sending it. If the device is received dirty and with debris, it will be returned without being repaired or cleaned and disinfected, which shall be charged to the customer.
	Always drain the tanks through the valve couplings at the back of the device (<i>Fig. 9</i>).
	Indicate the anomaly in writing, and place a document inside the package clearly indicating the anomaly noted or the service you are requesting.
	Always send with carriage paid, otherwise the shipping charges incurred will be charged.

All non-original packaging that we receive are disposed of.

The device will be returned in original packaging (the cost of the packaging will be charged) via the customer's shipping agent.

Turbine quick adapters of major brands can be requested in order to obtain greater operation flexibility of the device and the recommended liquids.

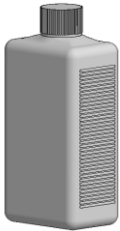     	Sirona B	Code 3MEDQ0003
	Multiflex (KaVo)	Code 3MEDQ0004
	Roto quick (W&H)	Code 3MEDQ0002
	Unifix (Bien Air)	Code 3MEDQ0005
	MachLite/Platelus (NSK)	Code 3MEDQ0001
	QDJ (NSK)	Code 3MEDQ0006
	Oil 500 ml	Code 3LIQB0002
	Disinfectant 750 ml	Code 3LIQA0001

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TECNO-GAZ S.p.A.
MEDILINE ITALIA S.r.l.