I

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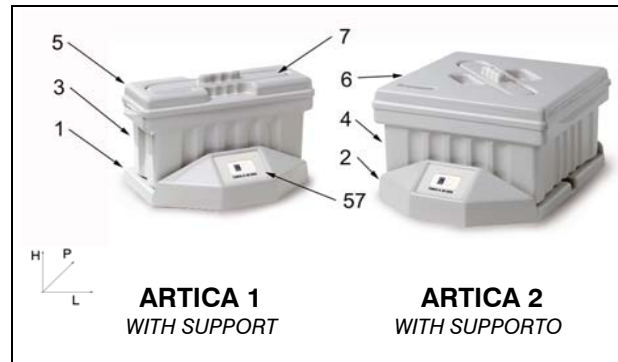
GB

## USE AND MAINTENANCE INSTRUCTIONS

ARTICA 1  
WITH SUPPORTARTICA 2  
WITH SUPPORTO

## SIZE - CAPACITY

H 210; L 335; P 240	2 LITRES	H 215; L 335; P 500	6 LITRES
---------------------	----------	---------------------	----------

## COMPONENTS

TANK, LID, LID LOCK PLUG ABS TRAY, SUPPORT			TANK, LID, LID LOCK PLUG vABS TRAY, SUPPORT		
Pos.3	Pos.5	Pos.7	Pos.4	Pos.6	Pos.7
TANK	LID	LID LOCK PLUG	TANK	LID	LID LOCK PLUG
SAIA103	SAIA105	SAIA107	SAIA103	SAIA105	SAIA107
SAIA104	SAIA106	SAIA109	SAIA104	SAIA106	SAIA109

## INSTRUCTIONS

POSITION THE SUPPORT ON A PLANE SURFACE SO THAT THE ANTISLIP FEET ARE FIRMLY POSITIONED.  
FIT THE TANK ONTO THE SUPPORT BY PAYING ATTENTION TO FIX IT PROPERLY.  
POUR INTO THE TANKS SOME DISINFECTING LIQUID SO AS TO REACH A LEVEL ENABLING TO IMMERSE MOST OF THE TRAY WITH HOLES.  
CLOSE WITH THE SUITABLE LID (POS.5,6) AND USE THE LID LOCK PLUG (POS.7) TO INSERT THE TOOLS.  
REMOVE THE LID ONLY TO REPLACE THE DISINFECTING LIQUID AND TO REMOVE THE TRAY WITH HOLES.  
SET THE TIMER TO BE SURE THE INSTRUMENTS UNDERGO A MINIMUM DISINFECTION TIME: ROTATE 360° CLOCKWISE AND THEN SET BACK TO THE DESIRED TIME.

## WARNINGS:

- TO LIFT THE TANK USE THE HANDLES LOCATED ON THE SIDES UNDER THE RIMS OF THE TANKS THEMSELVES.
- MAKE SURE THAT THE TANKS ARE CORRECTLY FITTED INTO THE SUPPORT FIXING IN.
- NEVER POUR INTO THE TANKS LIQUIDS HAVING A TEMPERATURE EXCEEDING 40°C (THE CONTAINER COULD BUCKLE).
- DO NOT USE: ORGANIC SOLVENTS, ALCOHOL, OIL, ACETIC ACID, VEGETAL OILS.
- PERIODICALLY REPLACE THE DISINFECTING LIQUID WITH A LIQUID HAVING THE SAME CHARACTERISTICS.

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ARTICA 1  
ARTICA 2

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GB USE AND MAINTENANCE INSTRUCTIONS

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ARTICA is covered by a warranty period of 12 months. The warranty begins the date ARTICA is delivered to the client, such date must appear on the warranty coupon, which must be properly filled in, stamped and signed by retailer and then sent back to TECNO-GAZ. In case of complains it will be considered valid the date appearing on the delivered note. Repairs or replacement under warranty conditions of any components will be exclusively carried out by TECNO-GAZ; the warranty does not cover travel expenses of personnel, neither packaging nor transport expenses. The warranty does not the timer and the failure or damages caused by wrong maintenance, negligence, unskillfulness or any other cause the manufacturer is not to be held responsible for. Those components and equipment subjected to regular wearing out are not covered by the warranty. TECNO-GAZ does not recognize the right to have the whole ARTICA replaced. The warranty does not meet any request of indemnity for direct or indirect damages caused to goods and persons, provoked by the inefficiency of the ARTICA. TECNO-GAZ does not recognize any indemnity in case the ARTICA is not used. The warranty will be automatically declined in case the ARTICA is tampered, repaired or modified by the client or by third parties not been authorized by TECNO-GAZ. The purchaser is compelled to contact the retailer or after-sales services centre indicated by TECNO-GAZ for the repairing. The components repaired during warranty period must be sent back to TECNO-GAZ carriage paid. In case the replaced components are not been sent back to TECNO-GAZ the purchaser will be charged for them.