Istruzioni per l'uso
Instructions for use
Mode d'emploi
Bedienungsanleitung
Instrucciones para el uso

RAY

I

GB

F

I

GARANZIA

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GB

WARRANTY

RAY is covered by a warranty period of 12 months. The warranty begins the date RAY is delivered to the client, such date must appear on the warranty coupon, which must be properly filled in, stamped and signed by retailer and then sent back to TECNO-GAZ .In case of complaints it will be considered valid the date appearing on the delivered note. Repairs or replacement under warranty conditions of any components will be exclusively carried out by TECNO-GAZ ; the warranty does not cover the staff's travel expenses , neither packaging nor transport expenses. The warranty does not cover the timer and the failure or damages caused by wrong maintenance , negligence, lack of ability or any other cause the manufacturer is not to be hold responsible for. Those components and equipment subjected to regular wearing out are not covered by the warranty. TECNO-GAZ does not recognize the right to have the whole RAY replaced. The warranty does not meet any request of indemnity for direct or indirect damages caused to goods and persons, provoked by the inefficiency of the RAY. TECNO-GAZ does not recognize any indemnity in case the RAY is not used. The warranty will be automatically declined in case the RAY is tampered, repaired or modified by the client or by third parties not been authorized by TECNO-GAZ.

The purchaser is compelled to contact the retailer or after-sales services centre indicated by TECNO-GAZ for the repairing. The components repaired during warranty period must be sent back to TECNO-GAZ carriage paid and properly wrapped (you are suggested to use the original packing) . In case the replaced components are not been sent back to TECNO-GAZ the purchaser will be charged for them.

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ENGLISH**WARNING**

- DON'T DRAW IT NEAR HEAT SOURCES
- DON'T EXPOSE IT TO INTENSE LIGHT (ex. : NEAR THE WINDOW)
- IN CASE OF TRANSPORT, BE CAREFUL THE FOUR JARS ARE CLOSED
- HANDLE CAREFULLY THE LIQUIDS FOR DEVELOPMENT AND FIXAGE FOLLOWING THE MANUFACTURER'S ISTRUCTIONS
- DON'T FILL UP THE JARS
- ALWAYS PUT THE WATER BRFORE THE PRODUCT THAT MUST BE DILUTED, THEN MIX CAUTIOUSLY AND SCREW DOWN THE BUNG OF THE CONTAINER
- IN ORDER TO AVOID DAMAGING THE RADIOGRAPHS DURING THE DEVELOPMENT, THE HANDS HAVE TO BE ALWAYS IN THE SPECIAL BUSHINGS
- REPLACE THE WATER EVERY DAY IN ORDER TO AVOID CONTAMINATION AND TO PROTRACT THE TIME OF THE BATHS OF DEVELOPMENT AND FIXAGE
- THE LIQUIDS OF DEVELOPMENT AND FIXAGE HAVE TO BE CARRIED OFF AS SPECIAL RUBBISH, IN COMPLIANCE WITH THE REGULATIONS IN FORCE IN THE COUNTRY OF DESTINATION
- THE CLEANING HAS TO BE ACHIEVED INSIDE AND OUTSIDE THE RAY WITH A DAMP PATCH WITHOUT SOLVENTS

TECHNICAL FEATURES

SIZE	HEIGHT (H)	350 mm
	WIDTH (W)	555 mm
	DEPTH (D)	340 mm
WEIGHT WITH PACKAGING		Kg 6.5
MATERIAL		ABS

EQUIPMENT

QUANTITY	CODE	DESCRIPTION
1	SRYA001	BASE OF ABS
1	SRYA002	COVER OF ABS
1	SRYA042	TRANSPARENT DOOR COMPLETE
2	SPRA055	BUSHING
2	SRYA010	RING NUT
2	SRYA011	MUFF
2	CM61043	PLIERS
4	CM61013	BUNG 'S CONTAINERS
1	SRYA016	DEVELOPMENT 'S ADHESIVE
1	SRYA017	FIXAGE 'S ADHESIVE
2	SRYA018	WATER 'S ADHESIVE
1	SRYA019	TIME DEVELOPMENT 'S ADHESIVE

INSTALLATION

Remove the packaging of the mini-camera RAY and check the integrity of its parts .
 Place the camera away from heat and light sources, on a plane, properly strong, not rickety surface that has to be sheltered from impacts.
 Read all this book before preceding .

DESCRIPTION

RAY is a mini-camera created for developing Rx intra-oral films, with the least lost of time.
 It is easy to use, it doesn't need electrical light and can be used in daylight.
 It is easy to transport it and it is the best solution for dental cabinets.
 Equipped with 4 containers :
 2 for washing and 2 for developing and fixing, all equipped with bung.
 Equipped with 2 radiograph-holder pliers.

INSTRUCTIONS

JARS' REFILLING:

Once you have placed the equipment following the above mentioned cautions, start filling the jars up :

- Remove the transparent panel : turn the handgrip at 180° (if necessary, remove also the cover)
- Open the jars unscrewing the bungs
- Dilute the concentrated solutions as indicated by the manufacturing companies

Fill the jars up:

- WATER with water at about 25 °C
- DEVELOPMENT with proper liquid at about 25 °C
- FIXAGE with proper liquid at about 25 °C

Place again (the cover) the transparent panel.

Put the hands in the special bushings and start developing .

DEVELOPMENT:

N.B. respect the time indicated in the table on the cover and copied out in the development summary.

- Open the radiograph by using the pliers and start the **DEVELOPMENT** inserting the film in the special jar.
- Carry out an **intermediate washing** and then the **FIXAGE**, always dipping in the homonymous jar.
- Start the **final washing** and then read the radiograph.

FILING:

For files under 6 months wash for 3 minutes in water and let dry at room temperature.

For files over 6 months wash for 15 minutes in water and let dry at room temperature.

N.B.

These instructions are generic, proceeding by integrating them with the instructions given by the radiographs' films and developing liquids' manufacturing companies is important.

DEVELOPMENT SUMMARY:

- 5. DEVELOPMENT
- 6. INTERMEDIATE WASHING
- 7. FIXAGE
- 8. FINAL WASHING

<i>TEMPERATURE</i>		<i>DEVELOPMENT</i>	<i>FIXAGE</i>
<i>°C</i>	<i>°F</i>	<i>Sec</i>	<i>Sec</i>
18	64	30	60
20	68	23	45
22	72	16	30
25	76	12	25
27	80	9	20
29	84	8	20

REPLACEMENT OF THE DEVELOPMENT AND FIXAGE LIQUIDS:

The replacement of liquids, apart from water that must be replaced every day, is at the user's discretion and depends on the frequency of the device's use.

The quality of development depends on the correct maintenance, so you are advised to replace the liquids every week.

DEMOLITION

The mini-camera RAY is made of ABS plastic materials. Don't leave the device in unguarded places, entrust an authorized company of its carrying off.

AS FAR AS RUBBISH, DEMOLITION AND CARRYING OFF OF PLASTIC MATERIALS, COMPLY WITH THE REGULATIONS IN FORCE IN THE COUNTRY YOU USE IT .

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TECNO-GAZ S.p.A.