# *Multisteril*

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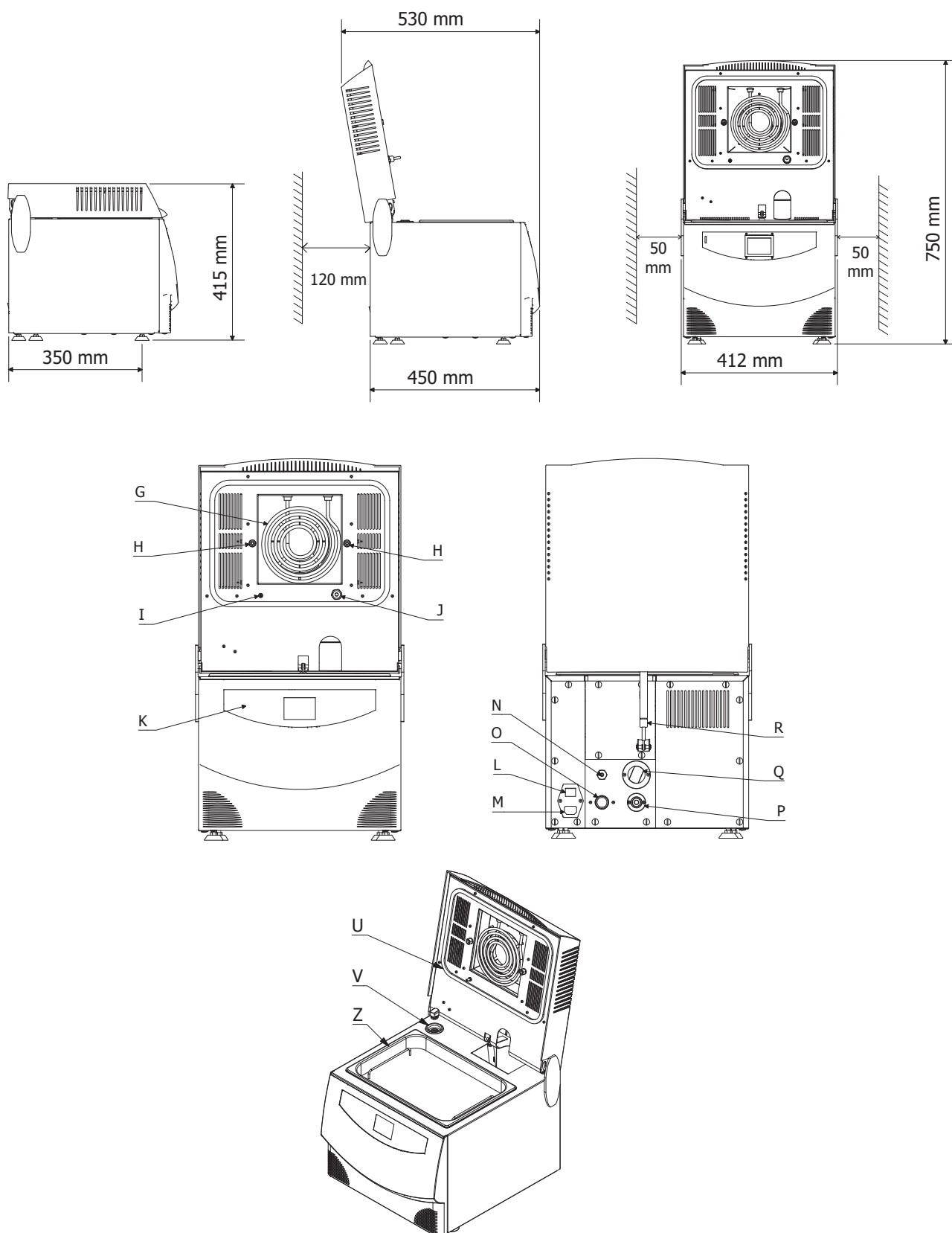
## INDICE

- 1. UTILISATION AND INTENDED USE**
- 2. SAFETY**
  - 2.1 SAFETY MARKING
  - 2.2 SAFETY DEVICES
  - 2.3 DISPOSAL
- 3. TECHNICAL DATA**
- 4. UNPACKING**
- 5. COMPONENTS**
- 6. INSTALLATION**
  - 6.1 CONNECTIONS
  - 6.2 FILLING OF THE DETERGENT RESERVOIR
- 7. OPERATION OF THE APPLIANCE**
  - 7.1 CONTROL PANEL
  - 7.2 LANGUAGE SELECTION
  - 7.3 CYCLE SELECTION
  - 7.4 CYCLES STANDARD SETUP
  - 7.5 CYCLE CHANGES
  - 7.6 DELAYED START SETUP
- 8. USE INSTRUCTIONS**
- 9. MAINTENANCE**
  - 9.1 CLEANING
  - 9.2 TANK CLEANING
  - 9.3 TANK FILTER CLEANING
  - 9.4 DRAIN FILTER CLEANING
  - 9.5 WATER INFED FILTER CLEANING
  - 9.6 CLEANING THE LEVEL PROBE
  - 9.7 PLANNED MAINTENANCE
- 10. ERROR ALARMS**
  - 10.1 ALARMS
  - 10.2 ERRORS
  - 10.4 GENERAL PROBLEMS
- 11. PROCEDURES FOR AFTERSALE SERVICE**
  - A. ACCESSORIES**
  - B. DRAWINGS**

1

**UTILISATION AND INTENDED USE**

<b>INTENDED USE</b>	<b>Multifunctional tank has been designed to remove organic and inorganic residue and dry instruments used in medical, dental, veterinary and beauty field.</b>
---------------------	---

The appliance uses ultrasound cleaning to clean areas which cannot be easily reached through the manual clearing and without the risks caused by the handling of contaminated tools.

The appliance can store up to 3 cycles which can be set by the operator.

The main steps are:

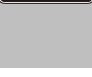
<b>CLEANING</b>	This step reduces by more than 90% the microbial contamination. Such step removes the organic and inorganic residues.
<b>RINSE</b>	This step removes any residue of detergent from inside the tank.
<b>DRYING</b>	After the rinsing the appliance can be dried to remove water residues which might jeopardize the next sterilization cycle


**The multifunction tank shall be used exclusively to clean tools and materials compatible with the ultrasound cleaning system and in general for uses intended by the manufacturer.**

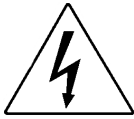2

SAFETY

	The appliance shall never be used in potentially explosive atmospheres.
	Place the appliance away from heat sources or electromagnetic radiation.
	Place the tool so that the control panel and the display are clearly visible by the operator.
	The electric system shall comply with the regulation in force, make sure the system suits the appliance characteristics.
	Check for the power supply cord integrity.
	Do not use flammable detergents
	Do not inject acids (ex HCl) nor chlorides into the tank
	Do not put any object in contact with the tank bottom
	The manufacturer is responsible for the sold product pursuant to the regulation in force. Such <u>responsibility voids</u> in case some operations on the appliance or part of it are performed by <u>unqualified staff or using non original spare parts</u> .
	<b>DO NOT OPEN THE APPLIANCE.</b> In case of damage or risk of fluid infiltration into the appliance for the appliance maintenance or cleaning unplug the power cord.
	Do not remove the plate
	Before any intervention unplug the power cord from power socket.
	Follow indications reported on the safety sheet of the used detergent.
	In some cases ultrasounds can cause unpleasant noise. Wear suitable PPE.

## 2.1 Safety marking

	DANGEROUS VOLTAGE
 <b>ATTENZIONE</b> <b>ATTENTION</b> <b>ATTENTION</b> <b>ACHTUNG</b> ALTA TEMPERATURA HIGH TEMPERATURES TEMPERATURES ELEVÉES HOHE TEMPERATUR	HIGH TEMPERATURE
	EARTHING

## 2.2 Safety devices

- ) Door opening sensor
- ) Safety thermostat
- ) Water loading check system

## 2.3 Disposal

*This product falls within directive 2002/96/EC of the European Parliament and of the Council of European Union on waste electric and electronic equipment (WEEE). For jurisdictions adopting such directive the product was launched in the market in a date following the 13 of August 2005 and shall not be disposed of as non recyclable household waste. Apply to specialized WEEE facilities for local collection for the disposal of this product or comply with the provisions in force.*

## 3

## TECHNICAL DATA

MECHANIC			
Working temperature	+5°C ÷ +30°C		
MAX relative humidity at 30°C	80%		
Overall dimensions (LxHxP)	415 x 415 x 530 mm		
Open lid height	750 mm		
Open lid depth	530 mm		
Weight (empty reservoir)	20 kg		
ELECTRIC			
Power voltage	230 V a.c. +/-10 % single phase		
MAX absorbed power	1 kW		
Frequency	50 / 60 Hz		
Power cord	2 + 1 x 1mm2		
Fuses	6.3 A (5 x 20)		
ULTRASOUNDS			
N° of transducers	4		
Frequency	39 KHz		
WATER SUPPLY			
Water pressure	1.5 ÷ 2 Bar		
MAX Temperature	+30°C		
Power hose dimension	3/8 G / length 3 m		
TANK			
Tank volume	9 l (S/S AISI 316)		
Water load volume	6 l		
DETERGENT RESERVOIR			
Volume	1.5 l + 0.5 l ( reserve )		
Available cycles with full reservoir		Selected percentage	Available cycles*
		1%	≈25
		2%	≈12
		3%	≈8
		4%	≈6
		5%	≈5
		6%	≈4
Material	Polyethylene		
FILTERS			
Tank filter	S/S AISI 304		
Discharge filter	S/S AISI 304 19 x 40 mm		

04

UNPACKING

The packing shall not undergo any shock and shall be handled with care avoiding letting it roll or fall.

The device is supported by a wooden pallet wrapped in corrugated cardboard and reinforced inside with cardboard pieces.

Open the corrugated cardboard, remove the reinforcement pieces and extract the appliance using exclusively the belts coming with the equipment.

The appliance shall be handled exclusively by means of the belts. Never lift the appliance by seizing it by the control panel.  
Such incorrect handling could damage the tool.

The packaging shall be stored for the whole warranty period.  
The manufacturer does not accept returns deprived of their original packing.

Do not use appliances with clear damages due to the transport.

Remove the packaging and check the appliance condition.

CONTENTS:

- **MULTI-FUNCTION TANK**
- **USE MANUAL:** to be carefully read and stored in a place accessible to all operators in charge of the device operation.
- **DECLARATION OF CONFORMITY :** to be stored.
- **INSTALLATION REPORT – TESTING AND WARRANTY CONDITIONS:** to be filled out upon appliance installation following indications reported on the form.
- **QUICK GUIDELINES FOR USE:** to be stored close to the appliance.
- Water supply hose
- Discharge hose
- Detergent drain hose
- Power cord
- Standard tray
- Dental burs tray
- Package of suggested detergent

**05****DESCRIPTION OF COMPONENTS**

See Annex B – DRAWINGS

<b>G</b>	<b>Drying resistance</b>
<b>H</b>	<b>Water nozzles</b>
<b>I</b>	<b>Detergent nozzle</b>
<b>J</b>	<b>Probe</b>
<b>K</b>	<b>Keyboard and LCD</b>
<b>L</b>	<b>Main switch</b>
<b>M</b>	<b>Power plug with fuse holder</b>
<b>N</b>	<b>Detergent reservoir drain with overflow device</b>
<b>O</b>	<b>Water feeding (3/8 G)</b>
<b>P</b>	<b>Drain fitting</b>
<b>Q</b>	<b>Filter</b>
<b>R</b>	<b>Gas spring</b>
<b>U</b>	<b>Lead gasket</b>
<b>V</b>	<b>Detergent manifold</b>
<b>Z</b>	<b>S/S tank</b>

06

INSTALLATION

The appliance shall never be used in potentially explosive atmospheres.

Place the appliance away from heat sources or electromagnetic radiation.

Place the tool so that the control panel and the display are clearly visible by the operator.

The electric system shall comply with the regulation in force, make sure the system suits the appliance characteristics.

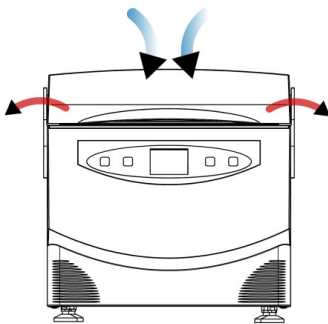
Open the lid and extract the accessories from the tank.

Position on a perfectly horizontal surface able to support loads of at least **35 kg** . The appliance supports are adjustable and allow for minor adjustment;

A wrong slope causes the deposit of a film of water on the tank bottom with associated drying problems


Position the appliance so as to allow the operator to easily inspect and clean the tank; the operator shall have access to the appliance rear panel for the filter cleaning.


For the drying phase air is taken from the rear side and hot air is discharged from the device sides.

**Let at least 50 mm clearance on both sides.**
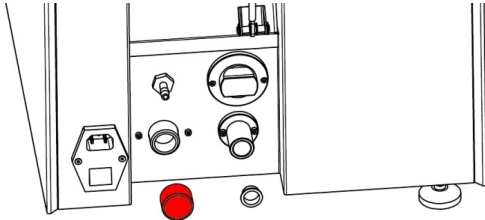
## 6.1 Connections


**Remove safety caps.**
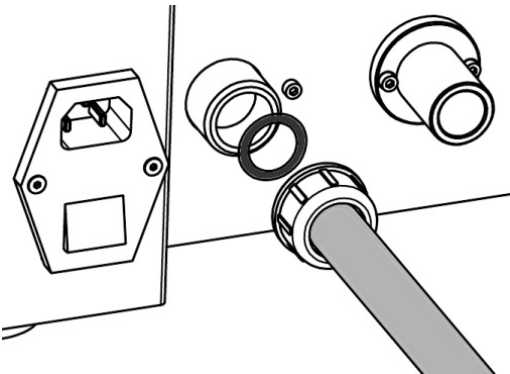
### **Water feeding hose.**

Connect the hose to the appliance (O) fitting also the rubber gasket. Connect the other side to the feeding cock.

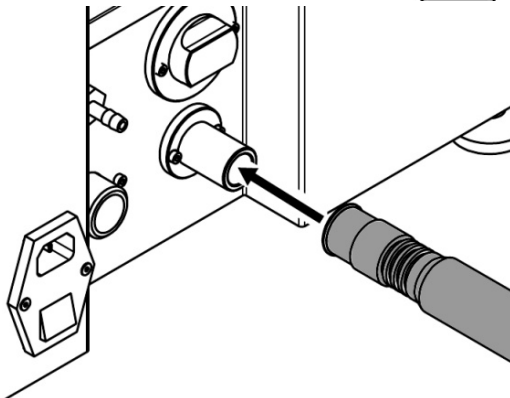OPTIONAL: emergency water block system (WATER BLOCK). Such device shall be vertically mounted immediately downstream the feeding cock. See the device instructions for further information on the tool installation and use.

### **Water drain hose.**

Remove the safety cap. Connect the hose to the device (P). Connect the other end to the discharge. The device discharges by fall consequently the drain hose has to be lower than the device. The hose has not to be bent upward (siphons). Non-return valves must not be present.

### **Detergent reservoir drain hose**

Take the transparent drain hose to a visible point at the same height of the appliance leaning plane or under it. Do not obstruct nor keep it immersed into the fluid. The drain hose has also an overflow function, when the detergent reservoir is full, if some more detergent is injected the drain hose discharges it.

## Power cord

Check the integrity of the power cord.

Connect the power cord to the socket on the rear side of the appliance (M).

Do not use extensions, reductions or adapters for connection

Switch on the device through the switch (L) .

Open the water feeding cock and make sure the feeding hose has no leak (if leaks are present, clamp couplers and make sure gaskets are fitted). Make sure the tank filter is present and that it is clean.

### Check of correct drain

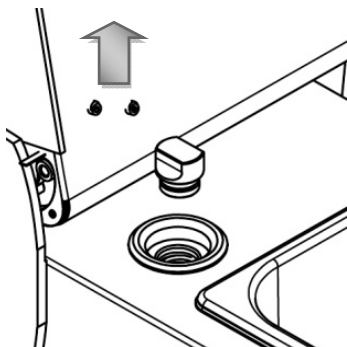
The appliance On, inject 1 liter of water into the tank and make sure the drain takes about 1 min.

## 6.2 Filling of the detergent reservoir:


The tank has a capacity of about 2 l , 0.5 of which as reserve.  
During the filling the device has to be ON and display 'CYCLE SELECTION'.

**ALWAYS SWITCH ON THE DEVICE BEFORE FILLING THE DETERGENT TANK**

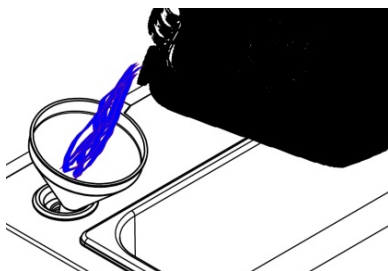Open the tank lid and lift the cap (V) on the left side of the tank.


Feed the detergent (use the funnel coming with the appliance) until and acoustic/visual signal (on the display) warns that the reservoir is full

**Use exclusively fluids recommended by the manufacturer.**

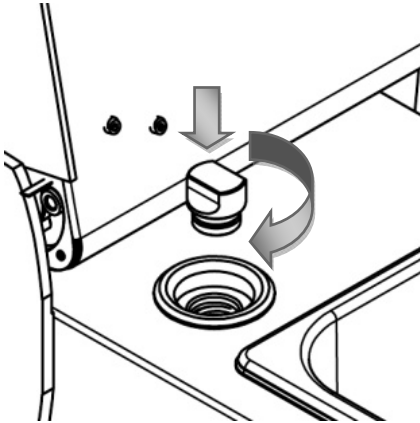**Do not mix fluids inside the reservoir since deriving chemical reactions can cause relevant damages to the device.**

Slightly press and rotate to close the detergent feeding cap.

**Close the detergent feeding cap to avoid detergent crystallization which could cause obstructions of and damages to some components.**

## 7

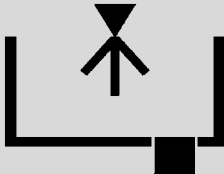
## OPERATION OF THE APPLIANCE

The multifunctional appliance has a lifting lid to allow access to the washing tank.


The tray, equipped with rubber supports and filled with items to be cleaned, has to be positioned inside the tank.



The multifunctional appliance has an internal reservoir suitable to hold a detergent which is dosed and injected automatically into the washing tank during the cycle.

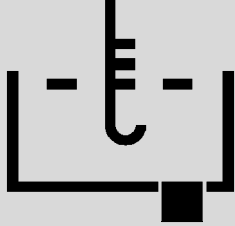A complete washing cycle includes the following steps.

PHA SE	SYMBOL	DESCRIPTION	TIME																																			
1		<b>Water loading</b> Water enters the tank until exceeding the tray edge by approx 2 mm. The total volume is of approx 6 l.	<b>1.5 min</b> (P infeed = 2.5 bar)																																			
2		<b>Detergent loading</b> A pump doses automatically and transfers the preset amount of detergent from the internal reservoir to the tank: <table><tr><th>Selected percentage</th><th>Volume (ml)</th><th>Available cycles*</th></tr><tr><td>1%</td><td>60</td><td>≈25</td></tr><tr><td>2%</td><td>120</td><td>≈12</td></tr><tr><td>3</td><td>180</td><td>≈8</td></tr><tr><td>4%</td><td>240</td><td>≈6</td></tr><tr><td>5%</td><td>300</td><td>≈5</td></tr><tr><td>6%</td><td>360</td><td>≈4</td></tr></table> <i>*With full reservoir (1500 ml)</i>	Selected percentage	Volume (ml)	Available cycles*	1%	60	≈25	2%	120	≈12	3	180	≈8	4%	240	≈6	5%	300	≈5	6%	360	≈4	<table><tr><th>%</th><th>sec</th></tr><tr><td>1</td><td>80</td></tr><tr><td>2</td><td>160</td></tr><tr><td>3</td><td>240</td></tr><tr><td>4</td><td>320</td></tr><tr><td>5</td><td>400</td></tr><tr><td>6</td><td>480</td></tr></table>	%	sec	1	80	2	160	3	240	4	320	5	400	6	480
Selected percentage	Volume (ml)	Available cycles*																																				
1%	60	≈25																																				
2%	120	≈12																																				
3	180	≈8																																				
4%	240	≈6																																				
5%	300	≈5																																				
6%	360	≈4																																				
%	sec																																					
1	80																																					
2	160																																					
3	240																																					
4	320																																					
5	400																																					
6	480																																					

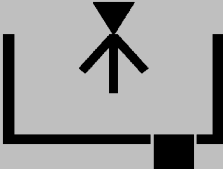
3		<b>Heating</b> The device performs a heating phase lasting 5 minutes and a first decontamination of tools. If temperature is not achieved, heating continues also during the washing phase.	5 min <sup>1</sup>
4	 LAMPEGGIANTE	<b>Degas (DEGAS)</b> The device activates intermittent ultrasounds for 5 minutes. This process reduces gases in the fluid and improves effectiveness of the next cleaning phase.	5 min <sup>1</sup>
4S	PRESS START TO CONTINUE	<b>STANDBY</b> <b>This new phase allows operating important process as:</b> <ul style="list-style-type: none"> <li>- Keeps the instruments in the washing liquid (SOAKING)</li> <li>- Add instruments (ATTENTION: STEEL MAX 1 KG.)</li> </ul> TO EXIT THE STANDBY PHASE AND END AUTOMATICALLY THE CYCLE IS NECESSARY TO PRESS START.	
5		<b>Ultrasound cleaning</b> Cleaning phase by means of ultrasonic cavitation.  <b>SWEEP</b> Ultrasounds undergo frequency modulation thus reducing the negative effects of stationary waves and increasing distribution of ultrasonic energy into the cleaning fluid.  Cleaning is more effective (cavitation is improved)	1 min to 30 min
6		<b>Draining</b> The tank drains automatically	≈ 8 min

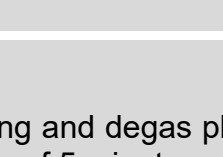7		<b>Second water loading</b> The tank is filled again to rinse tools	<b>1.5 min</b> (P infeed = 2.5 bar)
8		<b>Ultrasound rinse</b> Short activation of ultrasounds, rinse by cavitation.	<b>1 min</b>
9		<b>Draining</b> The tank drains automatically	<b>≈ 8 min</b>
10		<b>Drying</b> To dry tools, the appliance generates a hot air flow touching items and removing humidity as vapour.	<b>15 min</b>

**Notes:**

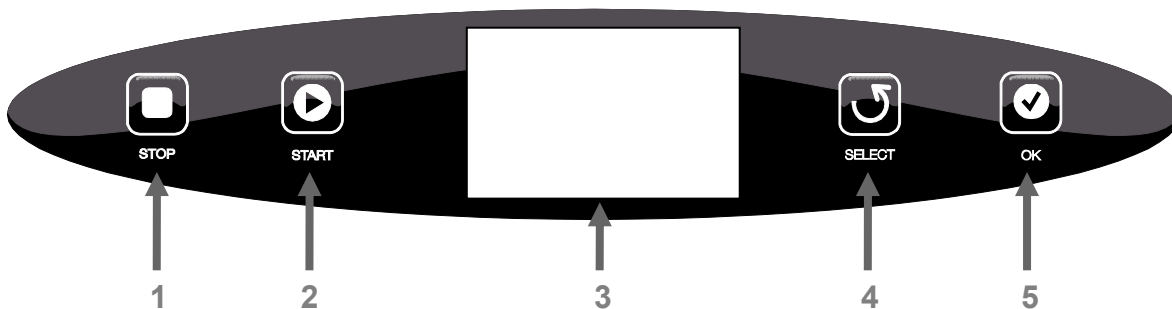1 Heating and degas phases , if they are both preset, are carried out simultaneously for a total time of 5 minutes.

**7.1 Control panel**

**CYCLE START/STOP KEYS**

1	<b>STOP</b> – IT STOPS THE CURRENT CYCLE
2	<b>START</b> – START CYCLE

**PROGRAMMING KEYS**

4	<b>SELECT</b> – NAVIGATION BUTTON
5	<b>OK</b> – CONFIRM SELECTION AND GO TO THE NEXT SECTION
3	<b>DISPLAY</b>

## 7.2 Language selection

Press **STOP** for 8 seconds.

The display shows the preset language.

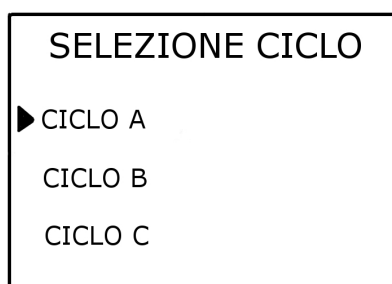
Press **SELECT** to scroll available languages.

Press **OK** to confirm the selected language.

## 7.3 Cycle selection

Upon the appliance switch ON the manufacturer's logo is displayed together with the firmware version.

After 3 seconds have elapsed the display shows:

Press **SELECT** to scroll programs.

Press **START** to start the selected cycle.

## 7.4 Cycles standard setup

The appliance comes with the following default parameters:

	<b>A</b>	<b>B</b>	<b>C</b>
Standby	ON	OFF	OFF
Detergent infeed	1 %	1 %	1 %
Heating	ON	OFF	ON
Degas function	SI	SI	SI
SWEEP function	NO	SI	NO
Ultrasound cleaning	20 min	20 min	5 min
Drying	SI	SI	SI

Parameters can be modified for all cycles.

## 7.5 Cycle changes

Select the cycle to be modified.

Press **OK** to display the MENU.

Available parameters:

Description	Parameters	Step
Standby	ON - OFF	-
Detergent infeed	0 ÷ 6 %	1%
Heating	ON - OFF	-
Degas function	SI - NO	-
SWEEP function	SI - NO	-
Ultrasound cleaning	0 ÷ 30 min	1 min
Drying	SI - NO	-

Press **OK** to scroll the cycle parameters.

Press **SELECT** to modify the value of the selected parameter.

**The menu of the parameters and setup scrolls circularly.**

Once the setup is completed select EXIT and press **SELECT** to quit the MENU.

**Changes are saved.**

## 7.6 DELAYED START setup:

Hold down **START** for 8 seconds.

Press **SELECT** to scroll values from 1 to 9 hours.

Press **START** to start the delayed start.

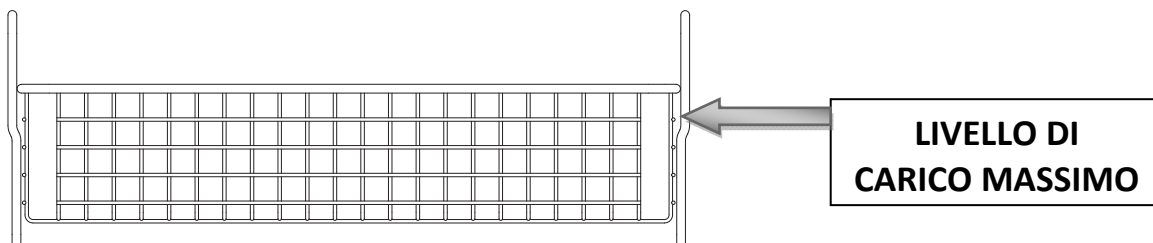
During the waiting time a timer is displayed showing the time remaining before the cycle start.

**WARNING: In case voltage is missing the delayed start is cancelled.**

# 8

## USE INSTRUCTIONS

Open the water feeding cock. Check the presence of the filter in the tank and make sure it is clean. Position the item to be cleaned into the TRAY in case of small parts use the DENTAL BURS TRAY.

Items to be cleaned have to be distributed uniformly and must not touch the tank walls or bottom.

**Fit the tray inside the tank, be sure there is no contact between the tank walls and the tray.**

**Cements, alginates, and composites must be treated with specific fluids.**

In case of rough dirt on the items, such dirt has to be removed before loading the tools inside the tray to avoid the drain filter obstruction.

To prevent contact between the steel tray feet and the steel cleaning tank rubber rings are used. In case rings are missing or damaged replace them with new ones. The wrong use can cause damages to the tank. Switch on the machine through the switch (L) .

Start the selected cycle pressing **START** the appliance shows the current cycle phase and the countdown of the time (estimated time) necessary to complete the cycle.

If the cycle provides STANDBY phase you will have to press **START** a second time (after the loading water, detergent input, degas and heating phases) to go to the washing step and finish the whole cycle.

At the end of the cycle the message ' **END CYCLE OK** ' is displayed accompanied by an acoustic alarm. Press OK to go to the cycle selection.

**If the door is opened while the device is ON, the current process is immediately stopped.**

**When the lid is closed the cycle will restart exactly from the point it has been interrupted.**

**At the end of the day close the cock mounted upstream the device.**

**Do not touch items, the lid or the resistance during drying. Risk of burns.**

**Drain the reservoir completely before injecting a detergent different from the one previously held**

### **Use of detergent other than that contained into the reservoir:**

Detergent other than that contained in the reservoir can be used.

In the menu of the cycle to be performed enter 'NO' as reference to the detergent automatic infeed.

Start the cycle, after some minutes (at the end of the water drain phase) the device sends out an acoustic signal, it is now possible to open the lid and inject into the tank the correct detergent percentage (1% - 60ml).

Close the lid to complete the cycle.

### **Detergent reservoir draining:**

During the appliance use, when injection of detergent is required, in order to drain completely the detergent reservoir, press and hold down button **SELECT** and at the same time hold down button **OK**.

After 5 seconds the peristaltic pump starts, during this phase an intermittent acoustic signal is sent out which stops only when the phase is over, the detergent is drained directly into the tank through the nozzle on the lid (I).

After about 10 min slightly open the lid (without letting the lid opening sensor intervene) and make sure no detergent comes out of the nozzle.

Once the draining is completed, stop the process using button **STOP**.

9

MAINTENANCE

9.1 Cleaning

The cleaning shall be carried out after the cooling of the device and the unplugging of the power cord.

Use non aggressive detergents suitable for plastics

Use detergents suitable for s/s steel

9.2 Tank cleaning

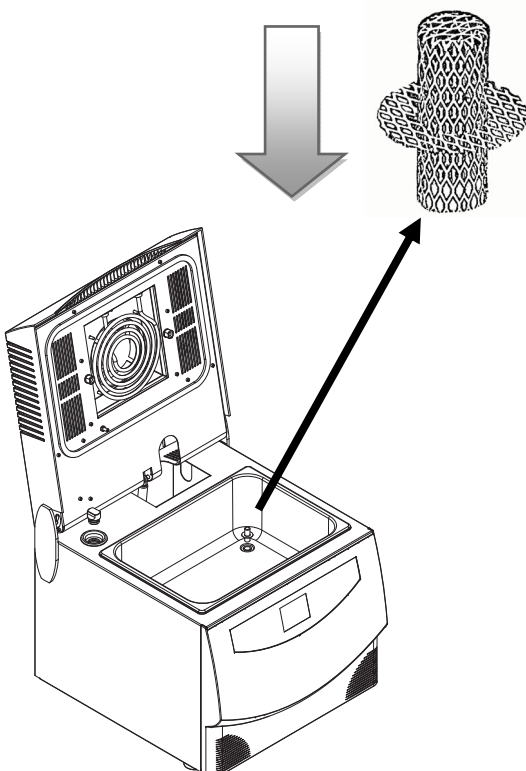
Check the cleaning of the tank at the end of each cycle

Remove deposits or debris to avoid injection of materials into the drain circuit and cause obstruction.

9.3 Tank filter cleaning

Check the cleaning of the tank filter at the end of each cycle

Lift the device lid;

Remove residues from the tank.

Extract the metal filter in the tank and pull it upward, as shown in the picture;


Wash the filter under tap water to remove impurities.

Re-place the filter in its position: the filter open side has to be inside the drain coupler while the close one has to be seen from inside the tank.

## 9.4 Drain filter cleaning

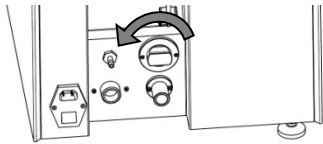**To be checked periodically.**

Make sure no water is present into the tank.


**The drain filter is located on the rear side of the device (P).**

Unscrew the filter from its housing (the filter might contain water, before extracting it place a sponge under the filter).

Carefully extract the filter to avoid any damage.

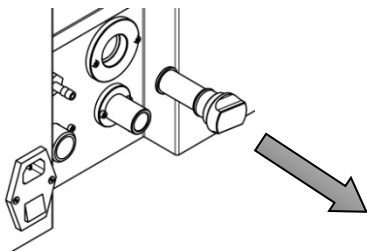
Rinse the filter under tap water to remove impurities.

**Clean the filter housing**


Carefully fit the cleaned filter into its housing.

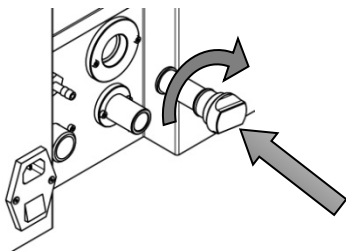Completely screw the filter to avoid water leaks.

## 9.5 Cleaning of the water infeed filter

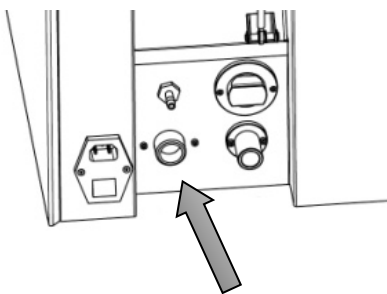**To be checked periodically**


Close water supply, remove the water feeding hose and make sure the filter (O) is clean.

On the contrary extract it using pliers without forcing and wash with water.

Remount the filter and the feeding hose.

## 9.6 Cleaning the level probe

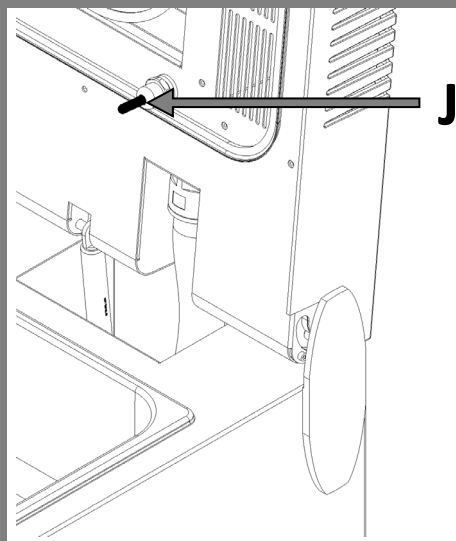Check periodically the PROBE J.  
Clean the metal with products suitable for stainless steel or a stripper.  
Use a sponge (slightly abrasive).

## 9.7 Planned maintenance

*Use exclusively original spare parts*

Description	Interval	Operator
Replacement of the peristaltic pump hose.	Yearly	Authorized technicians
Drying fan check	Yearly	Authorized technicians
Check/Replacement of the Drain valve	Yearly	Authorized engineer
Tank filter	Whenever necessary	User
Drain filter	Whenever necessary	User
Drain filter gasket	Whenever necessary	User
Detergent cap gasket	Whenever necessary	User
Lid gasket	Whenever necessary	Authorized technicians

**Maintenance and repair** operations shall be carried out after unplugging the machine from power supply.

A correct maintenance of the device ensures its correct functioning as well as time and service cost saving.

## 10

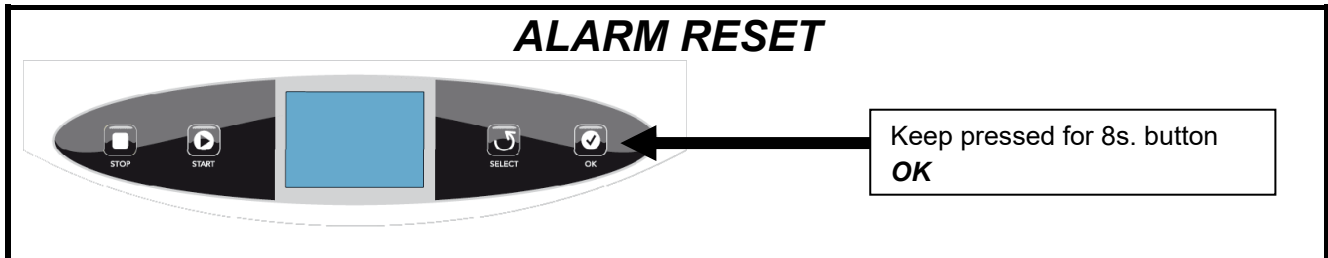
## ERROR ALARMS

## 10.1 Alarms


Display	Cause	Solution
	The lid is open.	Close the lid.
	Water inside the tank.	Wait for tank drain and restart the cycle
	Insufficient amount of detergent inside the reservoir ( $\leq 0.5$ l)	Refill the detergent reservoir and restart the cycle or set a cycle without detergent
	Detergent reservoir full.	Stop the filling of the detergent reservoir. If filling goes on detergent will come out from the rear overflow hose.

## 10.2 Errors

In case of error alarm the device has to be reset to start a new cycle.

Display	Cause	Solution
<p><b>ERROR 005</b></p> <p><b>ERROR 012</b></p>	<p><b>Water infeed alarm</b></p> <ul style="list-style-type: none"> <li>- the tap is closed</li> <li>- filters clogged</li> <li>- pressure of the water system not normal</li> </ul>	<ul style="list-style-type: none"> <li>- Make sure the tap is open.</li> <li>- Clean tank filter, drain filter, and water infeed filter.</li> <li>- Make sure water pressure ranges are within the indicated values. If the pressure is insufficient, contact a plumber.</li> <li>- Make sure the Water Block (OPTIONAL) is not locked.</li> <li>- Make sure there is no water into the tank and start a new cycle</li> </ul>
<p><b>ERROR 010</b></p>	<p><b>Water level alarm</b></p> <ul style="list-style-type: none"> <li>- the device has been moved or tilted during the cleaning.</li> <li>- too many instruments have been added after water infeed.</li> </ul>	<ul style="list-style-type: none"> <li>- Make sure there is no water into the tank and start a new cycle.</li> <li>- Do not add instruments during the cycle.</li> </ul>
<p><b>ERROR 008</b></p>	<p><b>Water drain alarm</b></p> <ul style="list-style-type: none"> <li>- filters clogged</li> <li>- drain hose in wrong position.</li> <li>- Wrong installation: anti-return valve on the drain function</li> </ul>	<ul style="list-style-type: none"> <li>- Clean the tank filter and the drain filter.</li> <li>- Position the drain hose correctly, see installation.</li> <li>- Make sure there is no water into the tank and start a new cycle.</li> </ul>

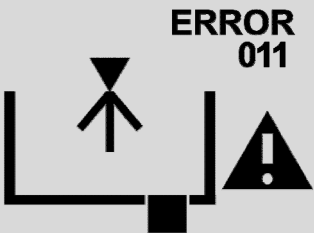Display	Causes	Solutions
	<p><b>Water infeed alarm</b></p> <ul style="list-style-type: none"> <li>- the tap is closed</li> <li>- filters clogged</li> <li>- pressure of the water system not normal</li> <li>.</li> </ul>	<ul style="list-style-type: none"> <li>- Make sure that the tap is open.</li> <li>- Clean tank filter, drain filter, and water infeed filter.</li> <li>- Make sure water pressure ranges are within the indicated values. If the pressure is insufficient, contact a plumber.</li> <li>- Make sure the Water Block (OPTIONAL) is not locked.</li> <li>- Make sure there is no water into the tank and start a new cycle</li> </ul>

#### 10.4 General problems

Problems	Possible causes	Solution
<b>Poor cleaning</b>	- Calcareous water, white halos on the tools	- Install a water softener before the device. - Some detergents might reduce such effect
	- Traces of elements which have not been pre-treated (cements, alginates)	- Pre-treat with suitable detergents.
	- Wrong cycle parameters	- Modify parameters increasing the cleaning time .
	- The ultrasound noise is not heard during the cleaning phase	- Contact the technical service
<b>Poor drying</b>	- Load excessive, drops on tools	- Reduce the load
	- Wrong installation, rests of water on the tank bottom after the draining.	- Position the device correctly, modify the height of the feet
	- No air flow from the lid side slots	- Contact the technical service
<b>Some instruments get damaged</b>	- Instruments that cannot be treated with ultrasounds cleaning (e.g. Mirrors)	- Do not clean with ultrasounds instruments that cannot be treated with ultrasounds treatment
<b>Hum upon device switch on or during the draining phase</b>	- Drain solenoid valve hum	- The solenoid valve works correctly, the noise is present for short periods only.

**11****PROCEDURES FOR AFTERSALE SERVICE**

In case of damage or for overhaul contact directly the phone service.

<b>PHONE</b>	<b>+39 0521 8380</b>
<b>FAX</b>	<b>+39 0521 833391</b>
<b>@</b>	<b><u><a href="mailto:service@tecnogaz.com">service@tecnogaz.com</a></u></b>

The service staff shall decide whether the return to the manufacturer or the intervention of a technician is necessary; after checking the device the service staff will draw up a quote to be submitted to the distributor which, on its turn, will submit it to the final customer for subscription.

Upon reception of the subscribed quote the device will be repaired and then shipped back within the times indicated on the quote form.

In case of shipment of the device to the manufacturer follow the compulsory indications below:

- Use the original packaging, in case you do not have it anymore, use a suitable packaging. The goods shipment risk is charged to the sender.
- Ship only the multifunction tank (do not introduce any other component of the accessories kit).
- Carefully clean the whole tank before shipping it. If the received device is dirty or has residues the tank shall be shipped back without having been repaired or will be cleaned or disinfected.
- Always drain detergent from the reservoir.
- Write down and include into the package a document describing accurately the anomaly or the service required.
- Free port shipment is required, otherwise the supported transport costs will be charged.

All non original packages we will receive shall be disposed of.

The machine will be shipped back in its original packaging (the cost for the packaging will be charged to you) through the customer's forwarder.


## A

## ACCESSORIES

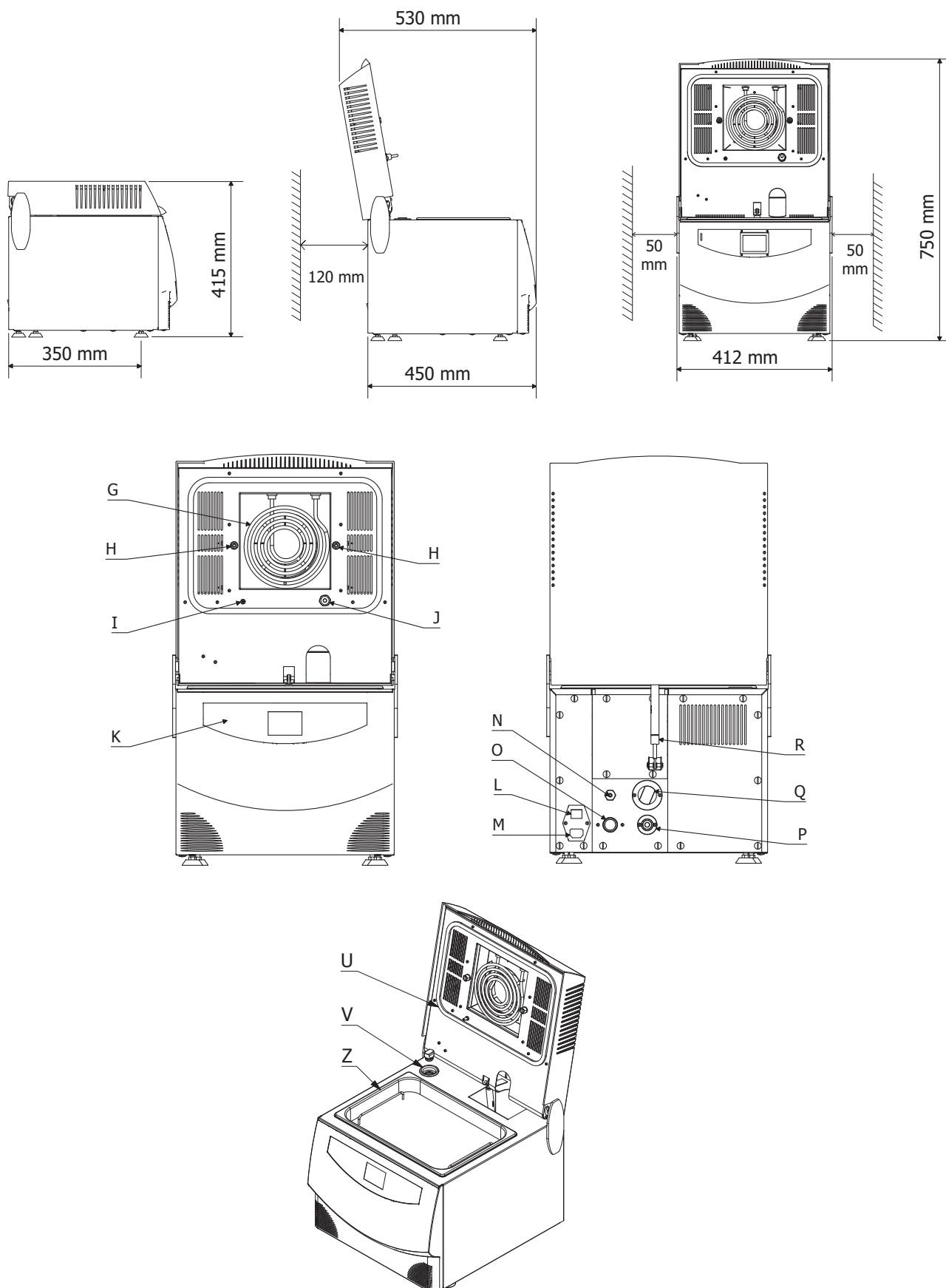	DESCRIPTION	CODE
	<b>TRAY</b> (180 mm X 280 mm H60 mm)	<b>DVMA069</b>
	<b>FEET GASKET</b>	<b>DFAA493</b>
	<b>DENTAL BURS TRAY</b> ( D 70 mm H 60 mm)	<b>DVMA072</b>
	<b>FEEDING HOSE (L 3 mt)</b>	<b>CPLA002</b>
	<b>DRAINING HOSE (L 0.8-2.7 mt)</b>	<b>CPLA001</b>
	<b>POWER CORD (L 2.5 mt)</b>	<b>CECG006</b>
	<b>FUNNEL</b>	<b>CPMG001</b>
	<b>WATER BLOCK (OPTIONAL)</b>	<b>DVMA092</b>
	<b>TANK FILTER</b>	<b>DVMA115</b>
	<b>TRAY HOLDER (OPTIONAL)</b>	<b>VM801ZVM</b>
	<b>N°1 TRAY (OPTIONAL)</b>	<b>DVMA070</b>
	<b>LIQUID DETERGENT/DISINFECTANT</b> <b>DILUTION PERCENTAGE</b> <b>FOR ULTRASONIC WASH: 1%</b> <b>1,0 L BOTTLE</b> <b>6 BOTTLES PACK</b>	<b>DM401ALP</b>

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**TECNO-GAZ S.p.A**