KYRI

<i>Istruzioni per l'uso</i>	<i>ITALIANO</i>
<i>Instructions for use</i>	<i>ENGLISH</i>
<i>Mode d'emploi</i>	<i>FRANCAIS</i>
<i>Instrucciones de uso</i>	<i>ESPAÑOL</i>
<i>Bedienungsanleitung</i>	<i>DEUTSCH</i>
<i>Instruções para o uso</i>	<i>PORTUGUÊS</i>
<i>Kullanım Kılavuzu</i>	<i>TÜRKÇE</i>

INDICE

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INTRODUCTION

Dear Customer,

Thank you for choosing the KYRI multifunctional chair and for your trust in our brand. We aim to offer you all our help and assistance in order to meet your expectations in full.

Before attempting to use the chair, take the time to read this operating manual with care and keep it in a safe place for future reference.

Please contact only our authorised technicians to request installation, maintenance and assistance. We ask that you use only original spare parts.

2

INTENDED USE


Kyri is a multifunctional dental chair.

3

SAFETY

3.1

Safety marking

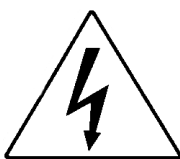DANGEROUS
VOLTAGE

GROUND CONNECTION

3.2

Safety devices

The chair is supplied with the following safety devices:

- Automatic release of the motor on the back
- Sensor under the leg guard
- Anti-crushing sensors on the rear part of the footboard
- Anti-crushing sensor on the tube holder arm (on request) or Mayo table holder arm (on request).

The chair also has an automatic clamping input for the use of dynamic instruments (only in combination with Kyriscart).

3.3

Safety devices

- In accordance with the regulations in force, the manufacturer is responsible for the product placed on the market. **Responsibility ceases** the moment when operations are performed on the device by unqualified personnel or with non-original spare parts.
- The chair must not be installed in a room where there is potentially explosive or flammable atmosphere.
- The chair must be installed in an environment that is compliant with the legislative requirements in force.

3.4

Disposal

This product is subject to Directive 2002/96/EC of the European Parliament and of the Council of the European Union on waste electrical equipment (WEEE). In jurisdictions adopting this Directive, the product was launched on the market after 13 August 2005 and must not be disposed of as non-recyclable household waste. Use local WEEE collection facilities for the disposal of this product, or comply with the regulations in force.

The maximum allowable weight on the chair is 180 Kg. This value must not be exceeded.

Do not sit on the tip of the chair when this is tilted upwards.

Do not rest against or sit on the leg guard when it is in table configuration. The maximum weight capacity is 30 kg.

To avoid the risk of electric shock, connect this unit only to a power supply network with protective earthing.

Make sure that the voltage of the electrical system is stable and always remains within the legal range: 230Vac +/- 10% , 50/60 Hz.

Unplug the unit before leaving the surgery.

The unit is not protected against the penetration of liquids.

To avoid contamination, it is advisable to cover all the upholstery with a paper or cotton sheet to offer protection and prevent promiscuous contact with it, and to replace the cover after each patient.

The unit is not suitable for use in the presence of a mix of anaesthetic gas that can catch fire in contact with oxygen or nitrous oxide.

The equipment was designed to ensure continuous operation with intermittent loads. The recommended operating times and pauses are specified below:
Operation: 1 min / Pause: 10 min.

Tecno-gaz guarantees the safety, reliability and performance of the equipment.

The terms of the guarantee are as follows:

observance of the conditions on the guarantee certificate; the equipment must be used only in accordance with the instructions in this booklet;

the electrical system in the room where the equipment is installed must be compliant with the rules in force. Assembly, repairs, additions to the unit, calibration, and all operations in general that involve opening the guards, must always be carried out by the authorised technicians.

Attention! Use only original spare parts.

4

SPECIFICATIONS

Power supply voltage	230VAC-50HZ
Low voltage Linak motor	24 Vdc
Power of lifting motor	6000 N fast running
Power of back motor	6000 N
Power of seat motor	6000 N
Power of single leg guard motor	2500 N
Max lifting capacity	200 Kg
Power consumption	120 VA
Weight of chair	120 Kg
Length of base	800 mm
Maximum height of leg guard	1200 mm

AMBIENT CONDITIONS

The equipment must be installed and/or stored in a room where the conditions are as follows:

- a) temperature between 10 and 40°C.
- b) relative humidity between 30 and 75%.
- c) atmospheric pressure between 700 and 1060 hPa.

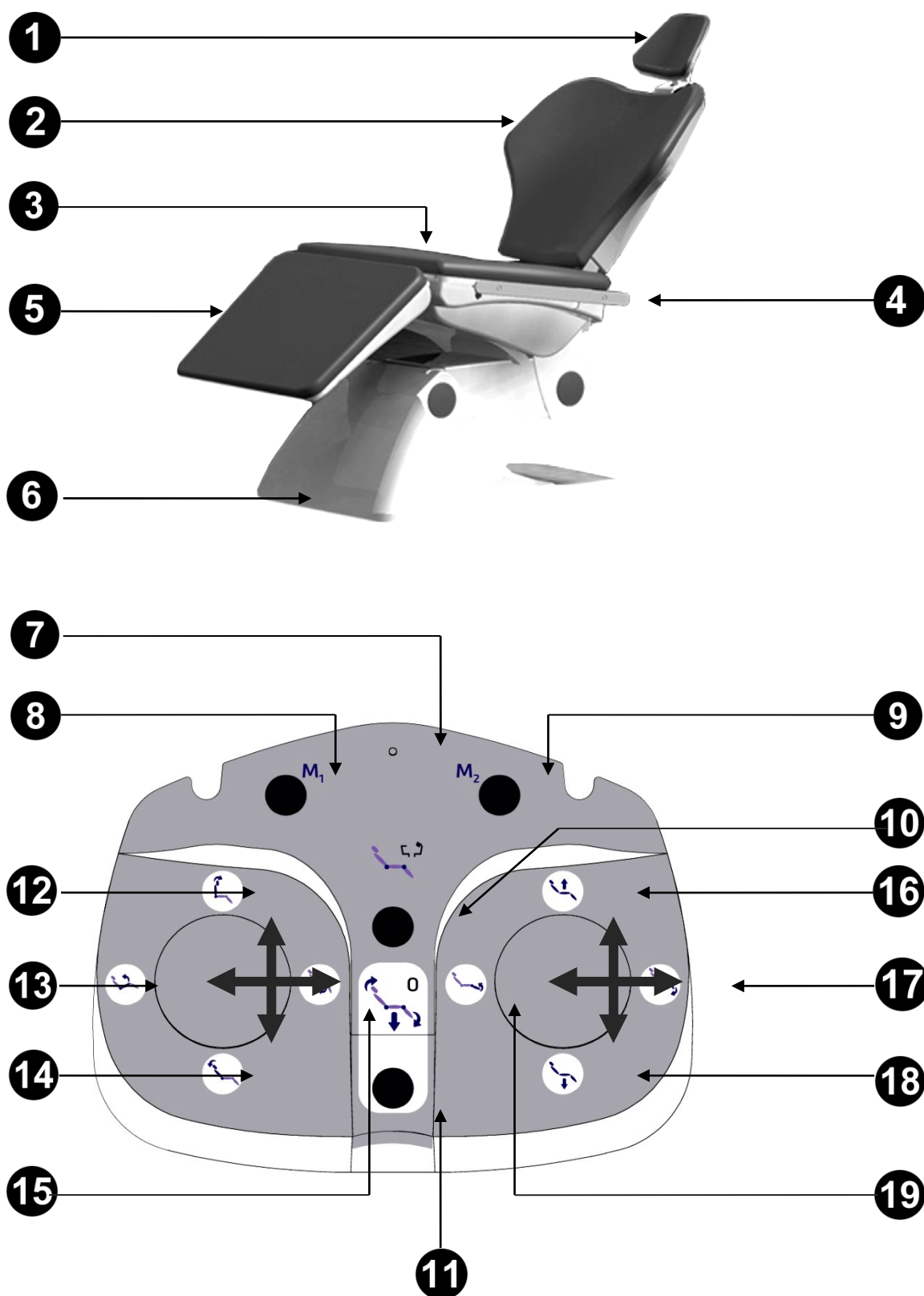
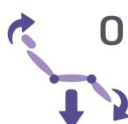
Fig. 1 Description of parts and controls

1	Headrest	
2	Back	
3	Seat	
4	Rail	
5	Leg guard	
6	Base	
7	Pedal control LED During use: Flashing: battery low During charging: Slow flashing: battery charging Steady: battery full	
8	M₁	Position 1 memory
9	M₂	Position 2 memory
10		Rinsing position + last position memory

11		Automatic return to zero position
12		Increase tilt of back
13		Increase tilt of seat
14		Decrease tilt of back
15		Decrease tilt of seat
16		Raise chair
17		Decrease tilt of leg guard
18		Lower chair
19		Increase tilt of leg guard

06**ACCESSORIES ON REQUEST****6.1 Suction tube support**

The module is attached to the KYRI chair by means of an articulated arm that can be used to move the module into the most convenient position for the operator.

Suction tubes

The suction device is operated by removing the tube from its support.

To adjust the suction power, move the cursor on the handle of the tube holder terminal.

Disassembly: Remove the suction tubes from the couplings of the conveyor by turning and pulling the tube attachment. Separate the suction tubes from the tube holder terminals by turning and pulling the tube attachment.

NB: Never attempt to do the above by holding the suction tube directly.

SUCTION TUBES: use only the tubes provided and replace these with original ones.

The suction tubes must comply with the Biocompatibility Standard EN 30993-1.

Maintenance.

Regularly lubricate the suction tubes and the cursors for closing the tube holder terminals, using silicone spray.

Cleaning the tubes.

Since different suction systems can be fitted (liquid ring, wet loop, air), it is advisable to always disinfect the suction system observing the instructions of the manufacturer of the suction system concerning the product and the times and methods of use.

Sterilisation.

- Tube holder terminals: steam autoclave.
- Suction tubes: cold water sterilisation.

6.2 Armrest

Can be fitted on both the rails.

6.3 Infusion pole


The pole comes with a clamp and can be fitted on both the rails. Maximum load: 3 kg. Complete with height-adjustable support for Vital monitor.

6.4 Mayo table supporting arm

The arm comes with a clamp and can be fitted on both the rails.

07

INSTALLATION

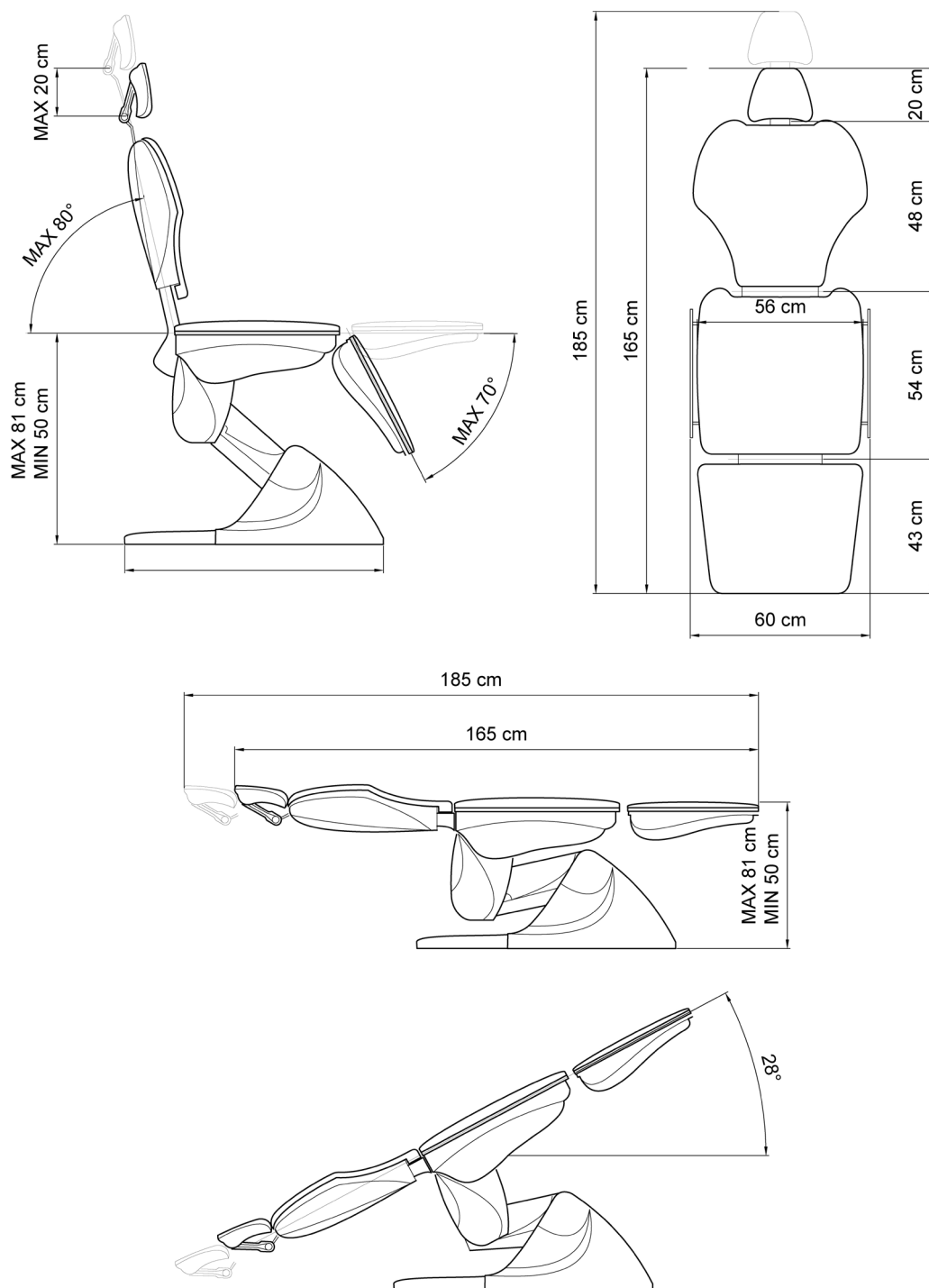Lift the chair from the packaging bed without tilting it, and put it down on the ground. Check that all the cables are undamaged and that none of the accessories are missing.

Installation must be carried out by qualified personnel using the template and installation diagrams provided with the product.

8.1 Electrical movements and memories

The chair can be moved in 4 ways by means of 4 independent electrical actuators:

Chair up and down movement

Press button **18** or **16** on the footboard to move the chair down or up.

Back movement

Press button **12** or **14** to move the back up or down.

Seat movement

Press button **13** or **15** to move the seat down or up at an angle of up to 20°.

Leg guard movement

Press button **17** or **19** to move the leg guard down or up level with the seat.

Memories

The chair has two positions that can be chosen at will and saved as **M1 and M2**, and a **RINSING** position that can also be saved.

To save a position, simply perform the required movement and then press and hold the position of the required memory for a minimum of two seconds. Two beeps confirm that the position has been saved. To select a memory, simply press M1, M2 or Rinsing; a beep indicates the saved movement has been completed.

ATTENTION! To stop the chair before it reaches the saved position, simply press one of the buttons on the footboard.

Return to zero memory

The chair also has a memory for automatic return to the zero position (the chair all the way down and the back fully upright) to allow the patient to climb off the chair easily. A beep indicates the saved movement has been completed.

N.B. In order to guarantee the precision of the memories over time, it is advisable to use the return to zero memory function as often as possible during the day.

N.B. In the case of a WIRELESS pedal, the LED will flash when the battery is low; to recharge the battery, simply connect the battery charger provided with the footboard and press any button; the LED starts flashing slowly and remains steady when the battery is fully charged.

8.2 Wireless footboard (on request)

The Wireless footboard (on request) has a rechargeable battery.

The LED (Pos _ _ fig.1) starts flashing when the battery is low; to recharge the battery, simply connect the battery charger provided with the footboard and press any button; the LED starts flashing slowly and remains steady when the battery is fully charged; the time for recharging depends on how low the battery is.

N.B. It is advisable to recharge the footboard at the end of every working day.

Make sure to connect the battery the right way round

9**MAINTENANCE****CLEANING THE CHAIR.**

The equipment is made from materials that can be cleaned and disinfected as indicated in these instructions.

The many types of pharmaceuticals and chemical products used in the surgery can damage the painted surfaces and the parts made from plastic.

The tests and research carried out revealed that the surfaces cannot be fully protected against the aggression of all the products available on the market.

The effects of these products essentially depend on the length of time these remain on the surfaces.

It is important to remove these substances immediately from the surfaces of the unit, using a damp cloth.

If using detergents, spray these onto a cloth and not directly on the unit.

ATTENTION!

Do not use products containing alcohol, ammonia, abrasive substances or benzol. Do not exceed the maximum allowable concentrations shown below.

Quaternary ammonium: maximum 2% Glutaraldehyde: maximum 2%

TECNO-GAZ CANNOT BE HELD LIABLE FOR DAMAGE DUE TO THE USE OF DISINFECTANTS CONTAINING ACTIVE SUBSTANCES OTHER THAN THOSE LISTED ABOVE.

In the case of faults, please contact the assistance service:

PHONE	+39 0522 94.29.96
FAX	+39 0522 94.47.98
@	service@tecnogaz.com

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TECNO-GAZ S.p.A.