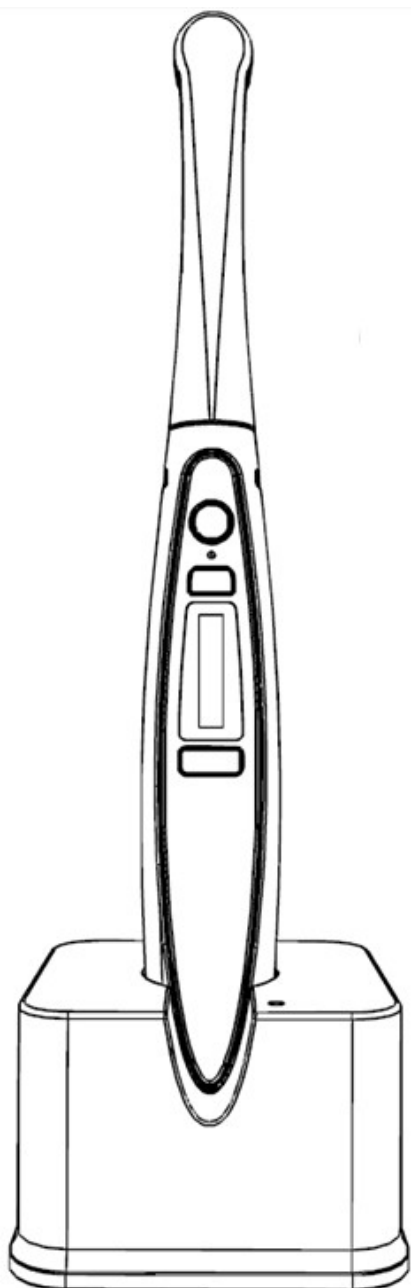
B-Max^{Plus}

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11

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TABLE

- 1. INTRODUCTION**
- 2. INTENDED FOR USE AND CAUTION IN OPERATION**
- 3. COMPONENTS**
- 4. UNPACKING AND CAUTION BEFORE UNPACKING THE BOX**
- 5. INSTALLATION**
- 6. OPERATION**
- 7. PROGRAM INFORMATION**
- 8. MAINTENANCE AND CLEANING**
- 9. SPECIFICATION**
- 10. GENERAL INFORMATION**
- 11. DEMOLITION AND DISMANTLING**

1

INTRODUCTION

The B-Max Plus LED curing light emits extremely high light waves and must only be used as indicated in this manual.

If using the B-Max Plus LED curing light in the Standard and high-power mode and in close proximity of the gingival tissue. Do not expose tissue for more than 20 second. If longer curing time is required, consider a dual-cure product (composite or adhesive). In X-tra power mode, Do not expose soft oral tissue for more than 9 seconds. The X-tra power mode has a 2 second safety delay to limit oral tissue heating during consecutive curing. If a longer cure is needed. Consider a dual cure product (composite and adhesive).

Read all instruction before operating this unit. Do not look directly into the light output Patient, clinician and assistants should wear UV orange eye protection when this device is in use. Do not expose soft oral tissues at close proximity. Maintain a safe distance between the lens and soft tissue.

2 INTENDED FOR USE AND CAUTION IN OPERATION

1) INDICATION FOR USE

The source of illumination for curing photo-activated dental restorative materials and adhesive.

2) WARNING – CAUTION

B-Max Plus will not operate if batteries are put in backwards.

If B-Max Plus does not turn on when fresh batteries are inserted, check the battery well fixed on body of handpiece

Do not insert fingers, instruments or other objects into the battery compartment

STORAGE AND TRAVEL

If storing the B-Max Plus for periods longer than 2 weeks or packing it for travel, always remove the batteries.

If batteries are left in the unit for long periods of time without recharging they may become nonfunctional or un-chargeable.

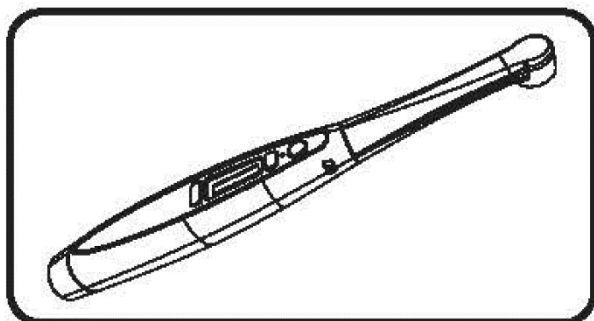
CAUTION

- Do not mix rechargeable batteries with non-rechargeable batteries
- Do not attempt to charge non rechargeable batteries.
- Use only safe rechargeable Lithium ion batteries or non-rechargeable batteries.
- Do not store batteries in temperatures over 60°C or in direct sunlight.
- Do not autoclave batteries, charger, AC power adaptor or hand-piece..


3

COMPONENTS


1) COMPONENTS.


Cordless curing light wand

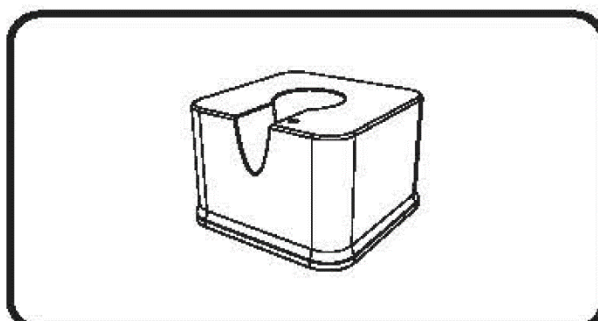
Charger base

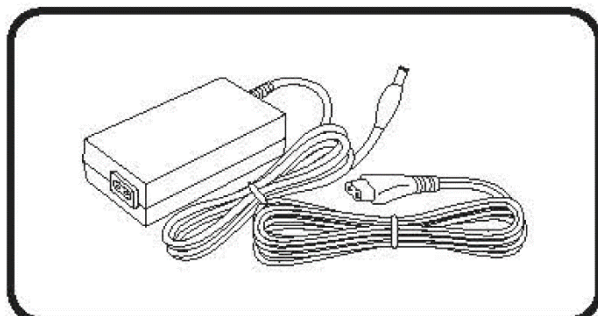
Medical grade 5 VDC AC power adaptor

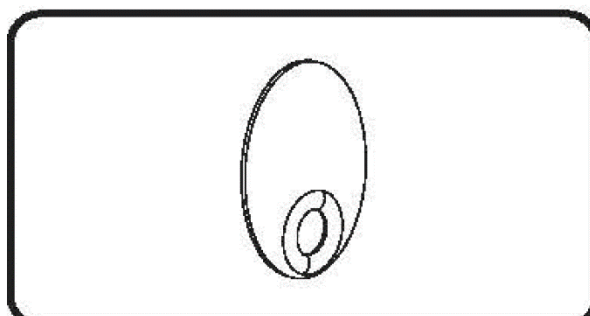Light shield

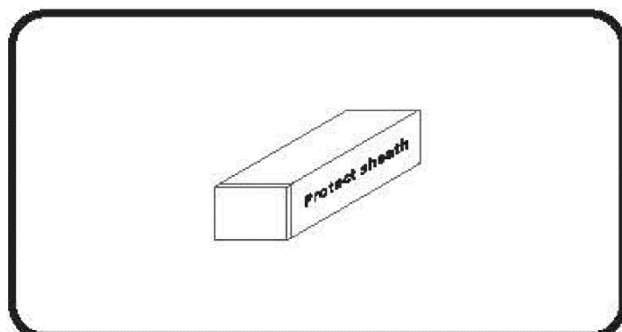
Cordless barrier sleeves

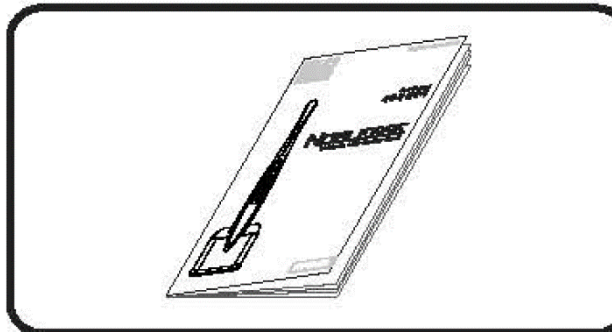Manual

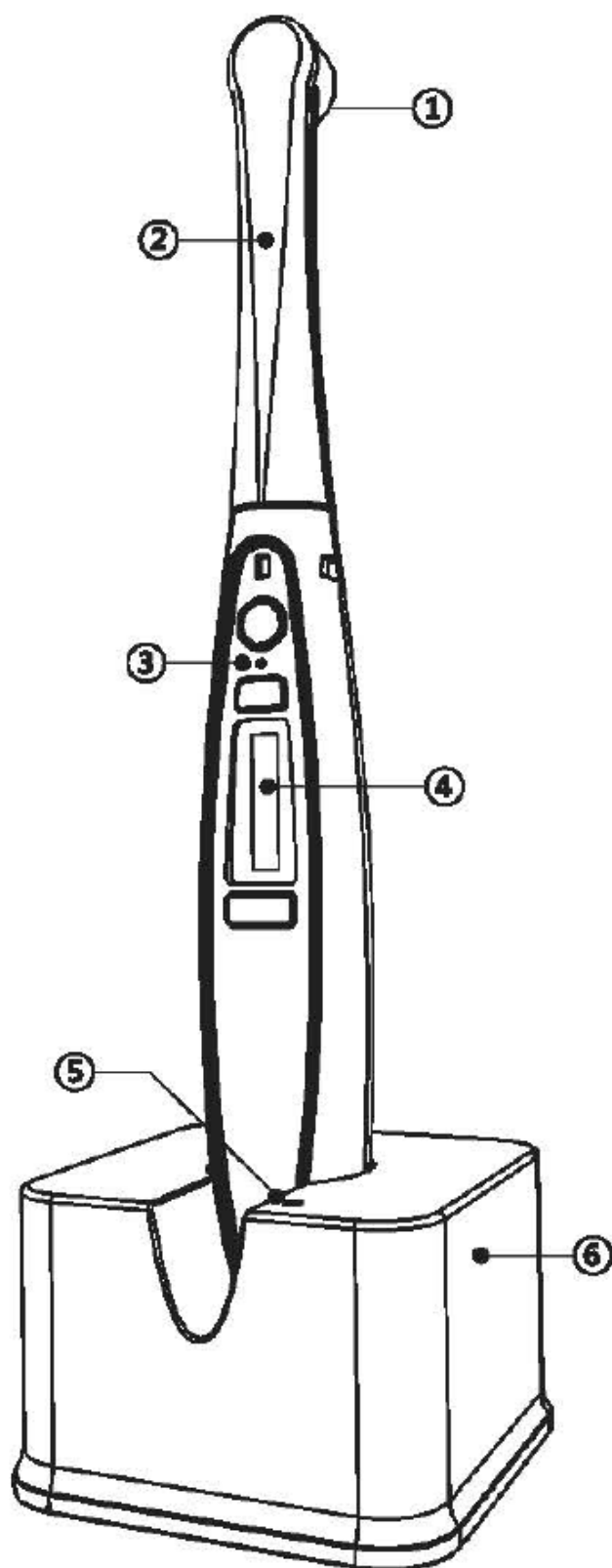2) PRODUCT DESCRIPTION.

1) LED CURING LIGHT EMITS PART
Emission part from light source.

2) SWIVEL HEAD PART
Rotating head part to 180°
counterclockwise direction.

3) CONTROL PLATE
Controlling operation by
button.

4) OLED DISPLAY
Displaying indicator of device.

5) CHARGING INDICATOR LED
Indication battery charging.

6) CHARGER BASE
Mounting and charging the
cordless curing light.

04

UNPACKING AND CAUTION BEFORE UNPACKING THE BOX

1) UNPACKING.

- (1) Unpack package box and check all components
- (2) Do not throw away any components in package box.
- (3) Keep the package box for upgrade and maintenance.
- (4) Manage no need of package material by the related law.

2) CHECK BEFORE USE

- Check program change by M button.
- Check light on of white LED by R button.
- Check Blue LED emission by S button.
- Check charging when you mount light want on charger base.

CAUTION

- Do not directly exposure device in water.
- Do not look directly into light output.**
- Do not use it in other industry except for dentistry.

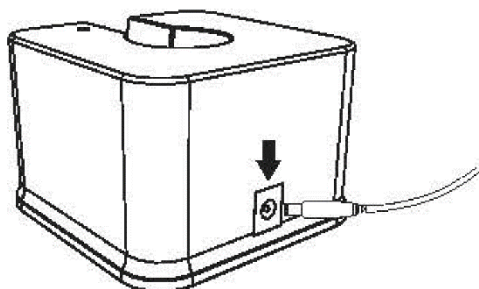
STORAGE CONDITION

- Do not store the device in high moisture and temperature place.
- Do not store the device in place directly exposed by sunshine.
- Do not store the device in place nearby chemical and gas material.
- Storage condition:

Air pressure : 70 kPa - 106 kPa


Temperature: 10°C - 40°C

Moisture : 30% - 85%


05**INSTALLATION****1) CONNECTION CHARGER BASE WITH POWER ADAPTOR**

Put charger base on flat place without moisture and connect with power adaptor.

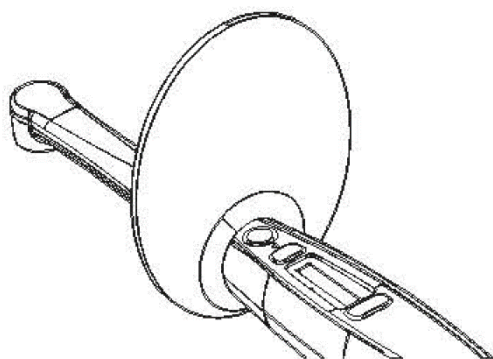X Power adaptor supplied by MaxDental should be only used.


2) CONNECTION LIGHT SHIELD WITH LIGHT WAND

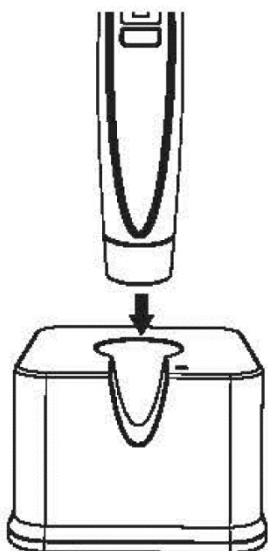Put light shield into front of S button.

3) CHARGING


In process of charging: Amber LED on
In completion of charging : Green LED on

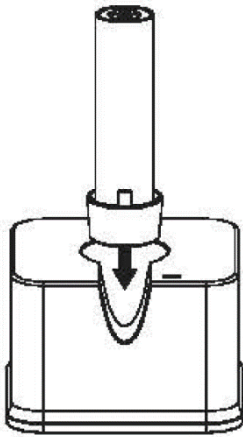4) HOW TO CHARGE BATTERIES.


- Insert batteries into charger regardless of direction and green LEDs indicate batteries are ready for use.
- Batteries will take about 1-3 hours to charge.
- Leave batteries in charger until ready for use.

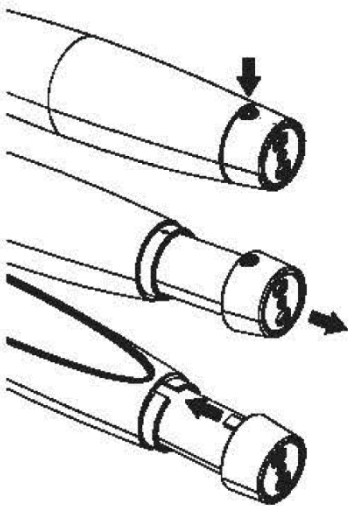5) HOW TO CHANGE THE BATTERY

Screw nut by using the tool and change the battery.

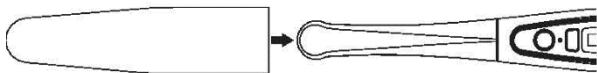6) INSTALLING HYGIENIC BARRIER SLEEVES


The hygienic barrier sleeve prevents cross contamination and helps keep dental composite material adhering to the surface of the lens and wand body.

06

OPERATING

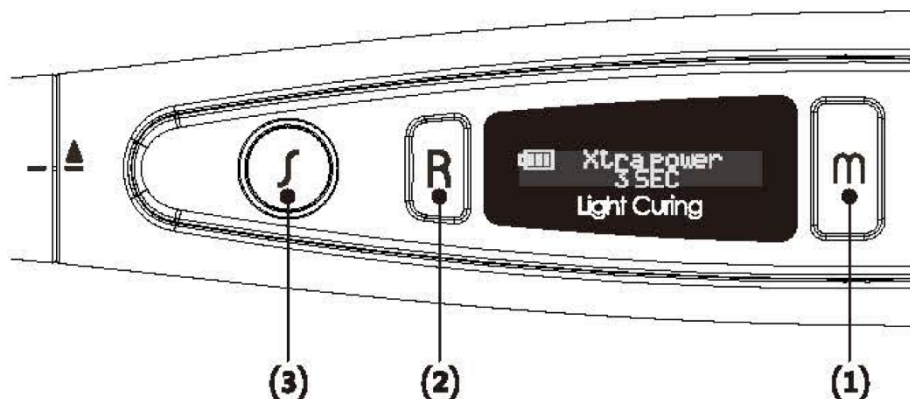1) DESCRIPTION OF BUTTON

- (1) **Mode** Switching button for program mode.
- (2) **Ready** Shooting button for White LED with using illumination in intra oral
- (3) **Shoot** Shooting button for Dental Blue LED emission

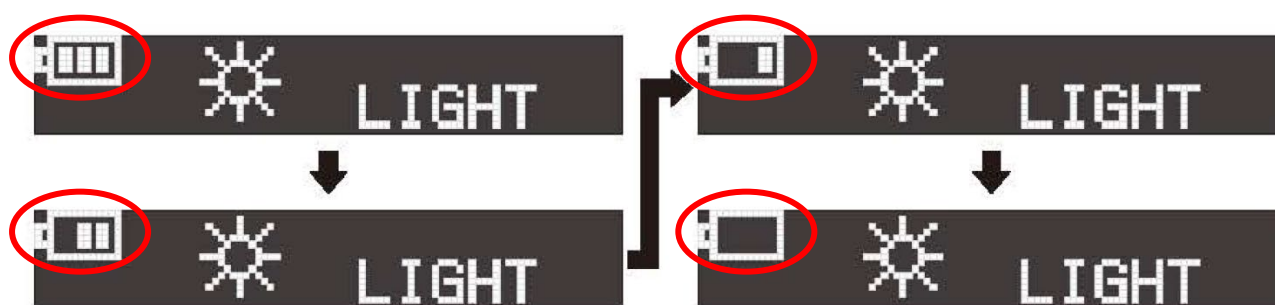
2) POWER ON

Turning power in by any button pressing with beep sound

3) OLED Display

(1) Battery indicator

Low battery is indicated with flickering of battery symbol on display
 Batteries will take about 1-3 hours to charge.
 Leave batteries in charger until ready for use

(2) Mode information

1. Curing mode

Working Blue LED with S button and on the way operation,
it can be stop by S button

2. Illumination mode

This mode is working by R button and it is benefit to light on deep side of mouth

3. Cool down.

As the protection mode from heat up by LED emission, controlling by temperature sensor

4) SLEEP MODE

2 minutes' inactive of device will goto sleep mode with beep sound. it can wake up by any button pressing.

5) CURING PROGRAM

Setting with 13 kinds of curing program and different curing time for each program.

(1) Selection mode

Selection mode by pressing and holding of M button.

Selection mode

(2) Customizing of program in selection mode

1. Program changing by M button
2. Program selecting by R button
3. Coming out from selection mode to operating mode by M button.

※ The customized program is not erased after power off

Mode Enable

☀	Normal
	10 SEC
☀	Normal
	20 SEC
☀	Soft Start
	15 SEC
☀	Soft Start-P
	5 SEC
☀	Soft Start-T
	20 SEC
☀	Pulse
	5 SEC
☀	Pulse
	10 SEC
☀	High power
	5 SEC
☀	Xtra power
	1 SEC
☀	Xtra power
	3 SEC
☀	Orthodontics
	3X6
☀	Orthodontics
	3X10
☀	Bleaching
	180 SEC

Mode Disable



○	Normal
	10 SEC
○	Normal
	20 SEC
○	Soft Start
	15 SEC
○	Soft Start-P
	5 SEC
○	Soft Start-T
	20 SEC
○	Pulse
	5 SEC
○	Pulse
	10 SEC
○	High power
	5 SEC
○	Xtra power
	1 SEC
○	Xtra power
	3 SEC
○	Orthodontics
	3X6
○	Orthodontics
	3X10
○	Bleaching
	180 SEC

6) BEEP SOUND

- (1) Short beep sound: Mode change, staring curing mode and illuminating mode, terminating of sleep mode.
- (2) Twice beep sound: Terminating of curing mode and illuminating mode.
- (3) Long beep sound: Sleep mode by S button pressing and holding

7) HEAD ROTATING

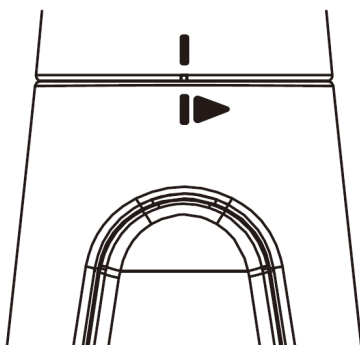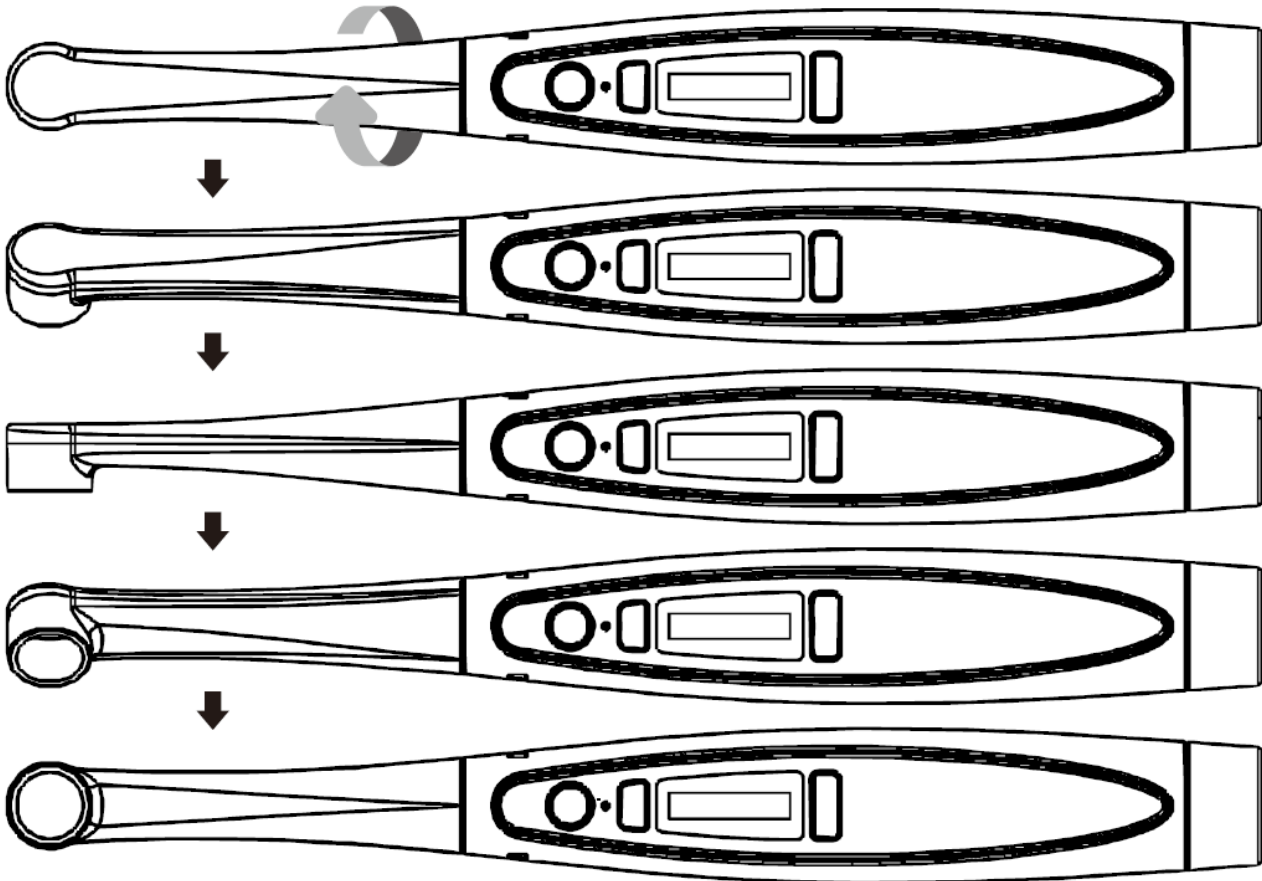Turn the head part by rotating counterclockwise direction.

Make head turn around as per mark on light curing wand

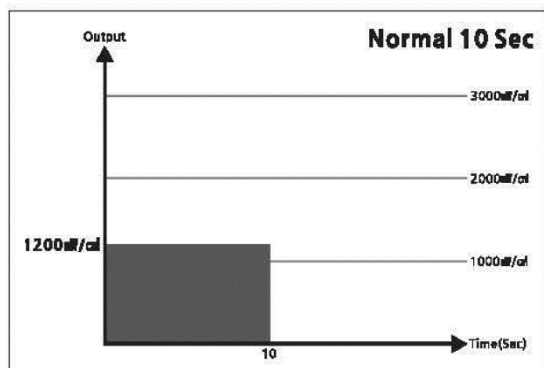
7

CURING MODE DESCRIPTION


1) NORMAL MODE

Curing time: 10 sec, 20 sec

Normal
10 SEC


Normal
20 SEC

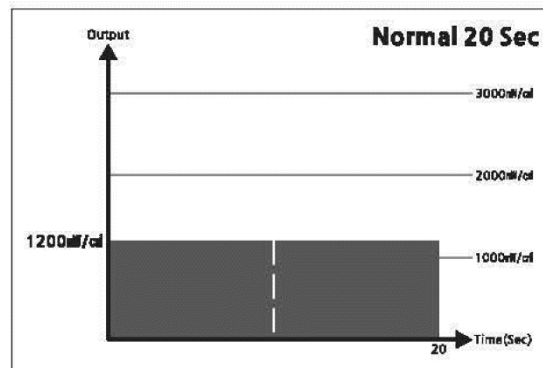2) SOFT START MODE


Curing time: 5 sec start 10 sec peak

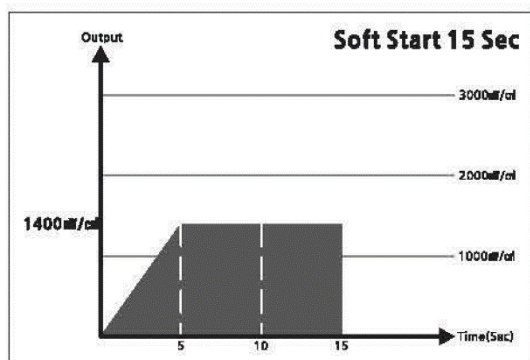Soft start
15 SEC


3) SOFT START-P MODE

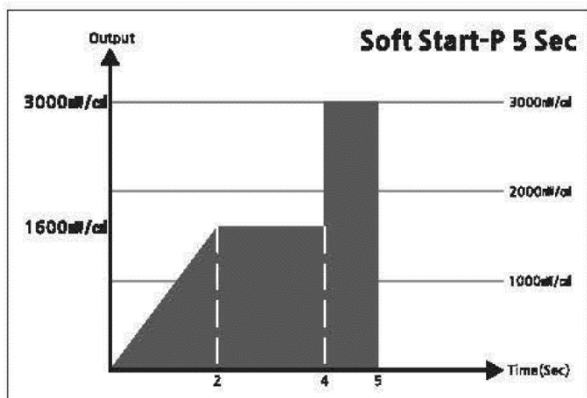Curing time: 2 sec start 2 sec maintain 1 sec peak



4) SOFT START-T MODE

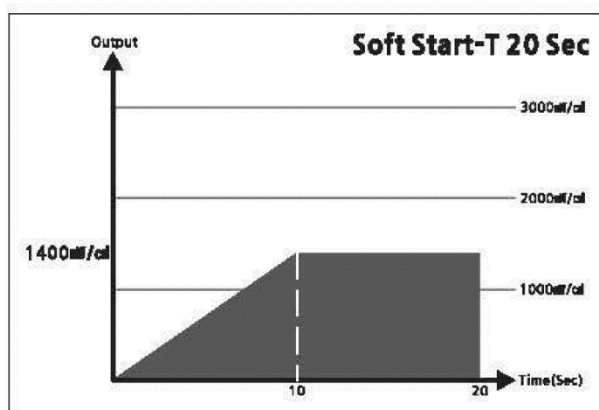Curing time: 10 sec start 10 sec peak


5) PULSE MODE

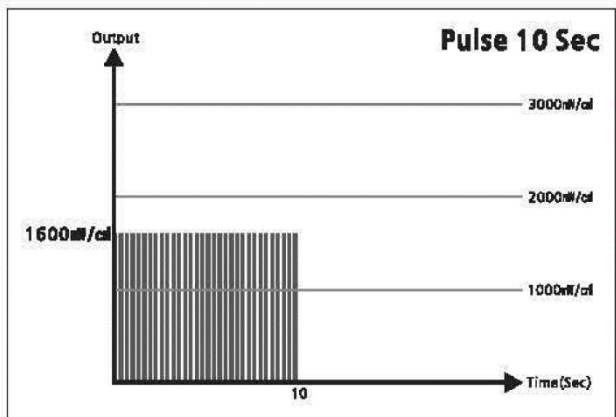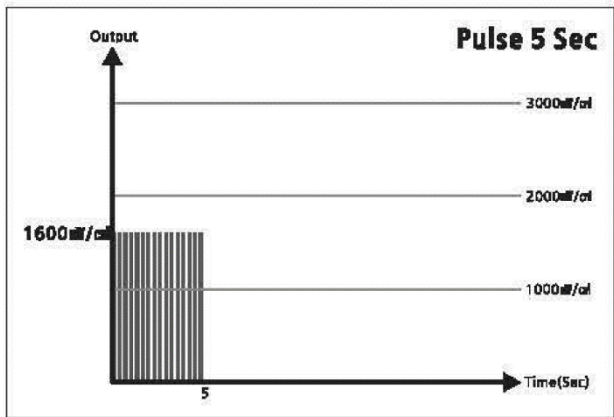Curing time : 5 sec. 10 sec



6) HIGH POWER MODE

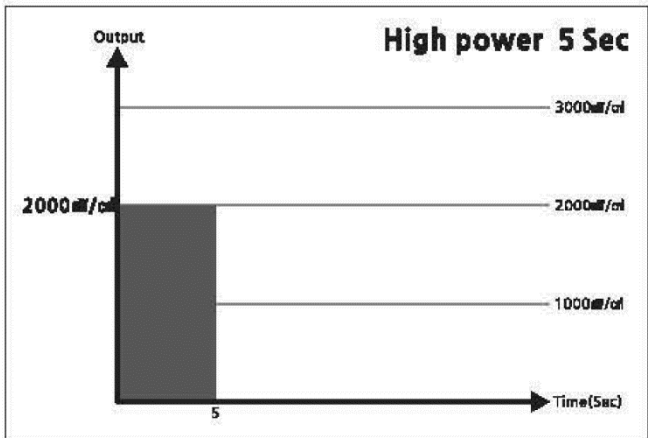Curing time : 5 sec



7) XTRA POWER MODE

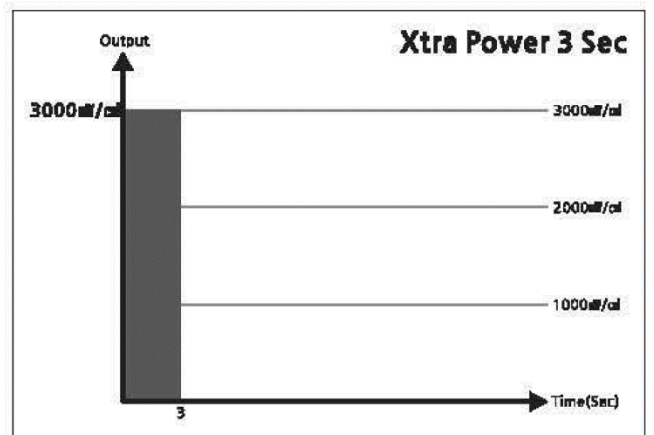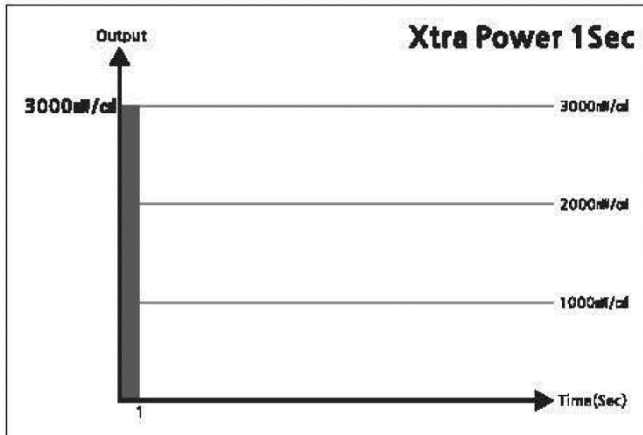Curing time : 1 sec. 3 sec


8) ORTHODONTICS MODE

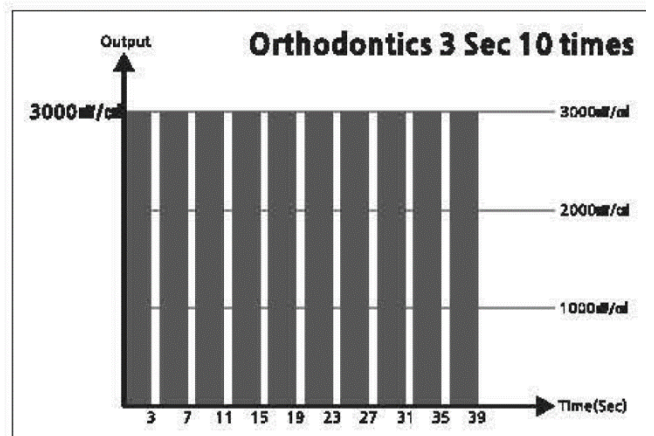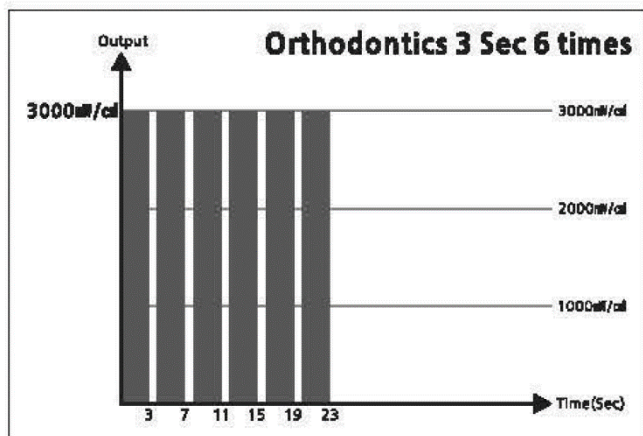Curing time: 3 sec On/1 sec Off (6 times continuous operation)
3 sec On/1 sec Off (10 times continuous operation)

9) BLEACHING MODE

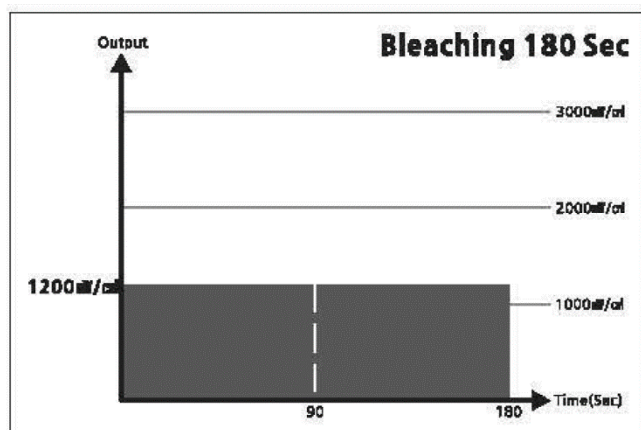Curing time: 180 sec

 Bleaching
180 SEC

The B-Max Plus curing light is plated with metal. After each use, moisten a gauze or soft cloth with an anti-microbial surface disinfectant and wipe the surface and lens.

Periodically check the lens for cured dental resins. If necessary, use a non-diamond instrument to carefully remove any adhered resin.

The barrier sleeve prevents cross contamination and helps keep dental composite material from adhering to the surface of the lens and wand body.

CAUTION

- Do not autoclave batteries, charger, power adapter or B-Max Wand.
- Do not insert fingers, instruments or other objects into the battery compartment of the Wand.
- Do not attempt to clean the gold contacts or any part of the battery compartment.
- Do not immerse in any kind of ultrasonic bath or any liquids.
- Do not wipe down B-Max Plus curing light with caustic or abrasive cleaners.

9

SPECIFICATION

Operating voltage	5V DC
Battery	3.7V Lithium-ion 2600mAh
Curing Light Range	430nm~490nm (Peak 40 nm)
Curing light source	Various Power Intensity 1200~3000mw/cm ² (tolerance +/-10%)
Dental Light Source	Chip White LED
Dental Light Output	More than 20,000 lux
Dimensions of Handpiece	228 x 28 x 26 (m m}
Dimensions of Charging Cradle	73 x 73 x 48 (m m}
Weight of Handpiece	170g
Weight of Charging Cradle	140g

**WARNING AND PRECAUTIONS.**

Read all instruction before operating this unit. The manufacturer accepts no liability for any damage resulting from the importer use of this unit and/ or for any purpose other than those covered by these instructions.

10

GENERAL INFORMATION

1) MANUFACTURED BY

Max Dental Co., Ltd.
 #203-605, 397, Seokcheon-ro,
 Ojeong-gu, Bucheon-si, Gyeonggi-do, Korea
 Tel: +82-32-624-3277
 Fax: +82-32-624-3278

2) NUMBER AND DATE OF MANUFACTURE

Mentioned in the product label (Mentioned when the product is manufactured)

3) PRODUCT NAME, TYPE NAME (MODEL NAME)

(1) Nome del modello: NOBLESSE

4) WEIGHT OR PACKING UNIT

(1) Weight: Handpiece- 170g, Charging Cradle- 140g
 (2) Packing unit: set

5) OTHER FEATURES OF THE MEDICAL INSTRUMENT AND MATTERS ON TECHNOLOGICAL INFORMATION

(1) Electrical rating
 - Batteria: 3,7 V, litio, 2600 mAh
 - Adattatore: 100V-240V, 50Hz- 60Hz, SV DC
 (2) Other matters to be mentioned
 Refer to user's manual and attached documents.

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