- I** **MANUALE OPERATORE**
- GB** **OPERATOR'S MANUAL**
- F** **MANUEL OPERATEUR**
- D** **BEDIENUNGSANLEITUNG**
- E** **MANUAL DEL OPERADOR**

ATTENZIONE:

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WARRANTY

This equipment is covered by warranty for twelve months, except for **moving membranes**, which are covered by warranty for six months.

This warranty starts on date of delivery of the equipment to the customer, as proved by the warranty coupon which must be returned duly filled, stamped and signed by the seller.

In case of controversy, the date shown on the sales note will be applied.

Warranty repair or replacement of a part is at the manufacturer's unquestionable discretion and does not include personnel's transfer and package and transport freight.

Breakdowns resulting from poor maintenance, negligence, unskilful use or other causes not depending on the manufacturer will be excluded from warranty.

Wear parts and fittings are excluded from warranty.

The manufacturer does not acknowledge the right of replacement of the complete equipment.

This warranty does not imply any compensation for direct or indirect damages of any kind to people or property due to equipment inefficiency.

This warranty will automatically void if the equipment is tampered with, repaired or altered by the buyer or any unauthorised third party.

When servicing is required, the buyer shall only apply to the seller or services indicated by the manufacturer.

Components replaced under warranty shall be returned to Tecno Gaz free of carriage.

If components are not returned, their cost will be charged to the applicant.

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USE

The bag resuscitator is a medical equipment for resuscitation and has hence to be used by duly trained staff.

Before and after its use, clean, disinfect or sterilize the bag (see PAR.08 "Routine Maintenance"), seal it in a sack suitable to maintain the obtained level of cleanness, disinfection and sterilization and pack it into the original package ready for the use.

All the TECNO-GAZ bag resuscitators are conceived and manufactured for the connection to reservoirs, oxygen tubes, PEEP valves, Face masks, intubation fittings; in compliance with the EN 1281-1 Technical standard.

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PAR.04

PARAGRAPHS INDEX & PICTURES

01 DECLARATION OF CONFORMITY	3
02 WARRANTY	4
04 USE	7
04 PARAGRAPHS INDEX AND PICTURES	12
05 WARNINGS.....	13
06 INSTRUCTIONS FOR PACKAGING, STORAGE, AND TRANSPORT	13
07 ROUTINE MAINTENANCE	14
08 PICTURES	24 - 25
09 SPARTE PARTS.....	26 - 27
10 ACCESSORIES	28

PAR.06

WARNINGS

- **BEFORE ANY CLEANING INTERVENTION REMOVE FROM THE BAG RESUSCITATOR COMPONENTS RESIDUES, IF ANY .**
- **DECONTAMINATE THE RESUSCITATOR BAG BEFORE AND AFTER THE USE AS DESCRIBED IN PAR.08**
- This manual has to accompany the bag resuscitator to ensure the product traceability in compliance with the 93/42/EC directive.
- The reading of this Manual does not imply the ability to use the bag resuscitator; the manual gives instructions on the equipment maintenance and on its technical features.
- The bag resuscitator, like other resuscitation tools, has to be used by trained staff after attending a special course or school.
- Connect the bag resuscitator only to accessories in compliance with the EN 1281-1 standard
- Use the bag resuscitator at a temperature range of +50 to -5 °C
- In case of storage do not expose the resuscitator to sun rays
- During the use, do not aim the bag resuscitator suction mouth on dust or other substances that could pollute the air delivered to the patient.
- **DO NOT** replace the package, not even with one of a bag of the same type.
- **DO NOT** use the package to pack another bag resuscitator.

PAR.07

INSTRUCTIONS FOR PACKAGING, STORAGE, AND TRANSPORT

The bag resuscitator is shipped **NON STERILE**, bagged in a cellophane sack packaged in a cardboard box .

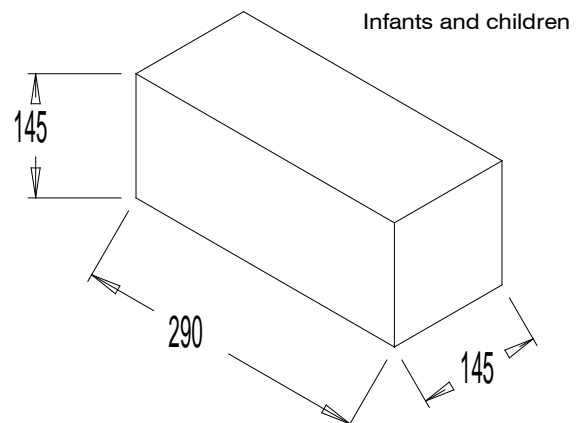
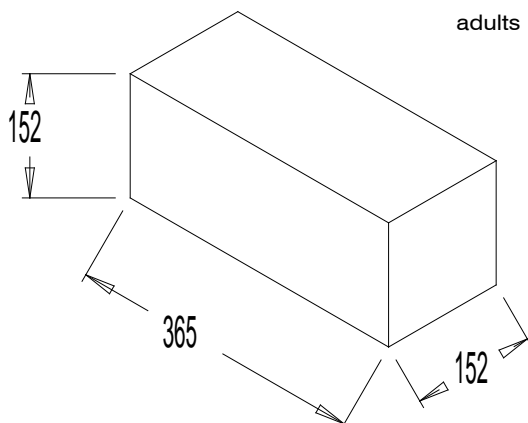
It is allowed to stack up to **5 MAX** packed bag resuscitator.

The package is integrating part of the instructions manual of the bag since it reads THE LOT for the product traceability, in compliance with the 93/42/EC directive.

DO NOT replace the package, not even with one of a bag of the same type.

DO NOT use the package to pack another bag resuscitator.

After cleaning, disinfection or sterilization interventions the operator has to perform before and after the use, the bag has to be sealed in a sack suitable to maintain the obtained level of cleanness, disinfection and sterilization and packed into the original package ready for the use.

PAR.08

ROUTINE MAINTENANCE

a) Manual washing

After use, dismount the bag resuscitator, **ELIMINATE RESIDUES, IF ANY**, clean components with hot water without exceeding 60°C and adding some detergent compatible to the bag resuscitator construction materials – see the enclosed product Technical Sheet.

Rinse with clean water to eliminate completely detergent residues, drain and carefully dry all components.

When dried, re-assemble the bag resuscitator components, seal it in a sack suitable to maintain the obtained level of cleanness, then place it in its box ready for use.

b) Disinfection with Chemicals

After use, dismount the bag resuscitator, **ELIMINATE RESIDUES, IF ANY**, clean components with hot water – see Point a) Manual washing.

After drying all components disinfect them with a product compatible to the bag resuscitator construction materials - see the enclosed product Technical Sheet.

Use the disinfectant following the producer instructions.

For disinfection, do not exceed the times suggested by the producer.

After disinfection rinse components with abundant water to eliminate chemical residues.

Drain and carefully dry all components.

When dried, re-assemble the bag resuscitator components, seal it in a sack suitable to maintain the obtained level of cleanness, then place it in its box ready for use.

c) Water steam sterilization

After use, dismount the bag resuscitator, **ELIMINATE RESIDUES, IF ANY**, clean components with hot water – see Point a) Manual washing.

After drying all components use the cycle **121°C NOT BAGGED** for sterilization; up to 3 sterilization cycles per day are ensured.

Lay down each bag resuscitator component on the central tray, if possible.

Do not overlap bag resuscitator components.

When the sterilization cycle is over, extract the bag resuscitator components, let them cool, re-assemble the bag and seal it in a sack suitable to maintain the obtained level of cleanness, then place it in its box ready for use.

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